# RAISED TOILET SEAT MODEL NOS. 1391P (WITHOUT ARMS) 1391AP (WITH ARMS)

## Assembly, Installation and Operating Instructions

### SAVE THESE INSTRUCTIONS

*NOTE: Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact Carrier/Dealer for further instruction.*

### SAFETY SUMMARY

The following recommendations are made for the safe installation of the Raised Toilet Seat:

#### WARNING

DO NOT install this equipment without first reading and understanding this instruction sheet. If you are unable to understand these Warnings, Cautions and Instructions, contact a healthcare professional, dealer or technical personnel if applicable before attempting to install this equipment - otherwise, injury or damage may occur.

Use the arm supports ONLY for assistance. DO NOT attempt to use the arms to support full body weight.

Always check to make sure that the Raised Toilet Seat is properly and securely locked in place before using.

Make certain that the knob that secures the Raised Toilet Seat to the commode is tight at all times.

Center your weight on the Raised Toilet Seat. The seat may tip if you lean too far forward or to either side.

Users with limited physical capabilities should be supervised or assisted when using the Raised Toilet Seat.

The Raised Toilet Seat has a weight limitation of 250 lbs. (114 kg.).

DO NOT Use Raised Toilet Seat if the toilet lip measurement exceeds 1-3/4-inches (FIGURE 1). The Raised Toilet Seat will not hold and may cause injury.

DO NOT use tools. Hand tighten only. DO NOT overtighten knob. The threads of the adjustment bracket may strip. Check seat for proper installation and realign/retighten if necessary.

#### CAUTION

DO NOT FORCE knob as this may damage the knob and/or the Raised Toilet Seat.

### INSTALLING RAISED TOILET SEAT (FIGURE 1)

*NOTE: Refer to the WARNINGS and CAUTIONS in the SAFETY SUMMARY of this instruction sheet.*

1. Remove contents from the carton.
2. Rotate the knob on the front of the Raised Toilet Seat counterclockwise enough so the adjustment bracket will fit inside of the toilet bowl.

*NOTE: The existing toilet seat/lid can either be removed or flipped-up out of the way depending on usage of the Raised Toilet Seat.*

3. Place the Raised Toilet Seat on the toilet bowl.

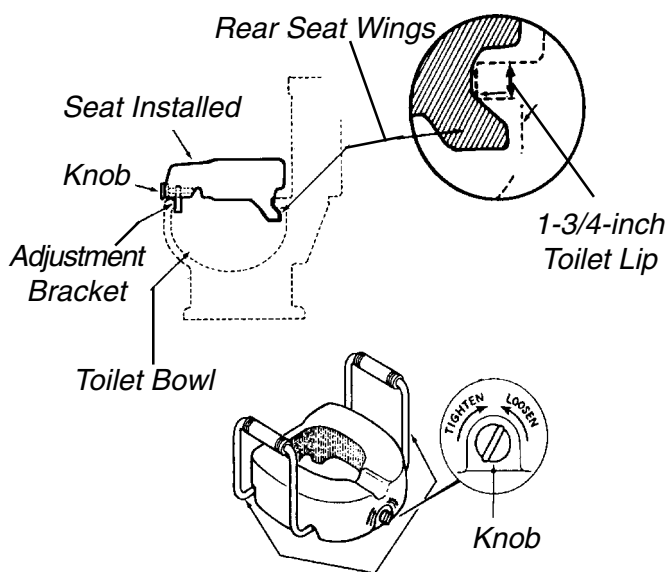
FIGURE 1-INSTALLING/REMOVING RAISED TOILET SEAT

*NOTE: The Raised Toilet Seat MUST be level, straight, and the adjustment bracket MUST be inside the toilet bowl. The rear seat wings MUST be positioned under the rear lip of the toilet bowl (FIGURE 1).*

4. After the Raised Toilet Seat is properly placed onto the toilet, turn the knob clockwise until the adjustment bracket is securely fastened to the front of the toilet bowl.

## REMOVING RAISED TOILET SEAT (FIGURE 1)

*NOTE: Refer to the WARNINGS and CAUTIONS in the SAFETY SUMMARY of this instruction sheet.*

1. Rotate the knob counterclockwise until seat is loose.
2. Lift-up the front of the Raised Toilet Seat until the adjustment bracket clears the lip of the toilet bowl.
3. Pull the seat towards you at an angle to remove the seat from the toilet bowl.

## CLEANING PROCEDURES

### WARNING

**DO NOT** use abrasive detergents, cleaners or cloths to clean/dry the Raised Toilet Seat.

1. Use a non-abrasive detergent or non-abrasive cleaner with warm water.
2. Rinse with warm water and wipe with a non-abrasive cloth.

### LIMITED WARRANTY

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants its product to be free from defects in materials and workmanship for the lifetime of the product for the original purchaser. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back cover. Provide dealer's name, address, model number, and the date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

Invacare Corporation will issue a return authorization. The defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty (30) days of return authorization date. DO NOT return products to our factory without our prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO: MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN AUTHORIZED INVACARE DEALER, SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE.

THE FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE.

IT SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESSED WARRANTY PROVIDED HEREIN AND THE REMEDY FOR VIOLATIONS OF ANY IMPLIED WARRANTY SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

# ASIENTO ELEVADO PARA RETRETE MODELOS NOS. 1391P (SIN BRAZOS) 1391AP (CON BRAZOS)

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