User Manual

DEALER: This manual MUST be given to the user of the product.

USER: BEFORE using this product, read this manual and save for future reference.

Yes, you can.

Suction Cup Grab Bars

Model 218 - 18 inch

Model 222 - 22 inch

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I General

Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact carrier/dealer for further instruction.

I.1 Symbols

Signal words are used in this manual and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.

Gives useful tips, recommendations and information for efficient, trouble-free use.

WARNING

Warning indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

CAUTION

Caution indicates a potentially hazardous situation which, if not avoided, may result in property damage or minor injury or both.


2 Safety

2.1 Weight Limitation

WARNING

The grab bar provides support and increases stability for an individual. The grab bar is not designed to support the full weight of an individual. The maximum loads are shown in the figure:

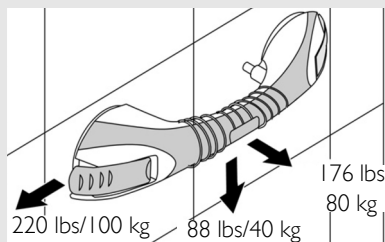3 General Warnings

WARNING

DO NOT USE THIS PRODUCT OR ANY AVAILABLE OPTIONAL EQUIPMENT WITHOUT FIRST COMPLETELY READING AND UNDERSTANDING THESE INSTRUCTIONS AND ANY ADDITIONAL INSTRUCTIONAL MATERIAL SUCH AS OWNER'S MANUALS, SERVICE MANUALS OR INSTRUCTION SHEETS SUPPLIED WITH THIS PRODUCT OR OPTIONAL EQUIPMENT. IF YOU ARE UNABLE TO UNDERSTAND THE WARNINGS, CAUTIONS OR INSTRUCTIONS, CONTACT A HEALTHCARE PROFESSIONAL, DEALER OR TECHNICAL PERSONNEL BEFORE ATTEMPTING TO USE THIS EQUIPMENT - OTHERWISE, INJURY OR DAMAGE MAY OCCUR.

ACCESSORIES WARNING

Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

WARNING

Grab bars add security for bath and commode areas when attached as recommended. Users with limited physical capabilities should be supervised or assisted when using grab bars.



4 Usage

4.1 Installing the Grab Bar

WARNING:

Risk of Falling.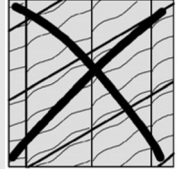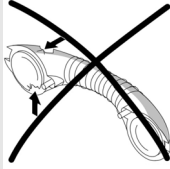


Damaged grab bars may not adhere to the wall properly.


- DO NOT install grab bars if damage is found.

Grab bars may not get adequate suction to adhere to the wall.


- DO NOT install grab bars directly onto grout lines or gaps in the wall surface.
- DO NOT install grab bars onto dirty or wet walls.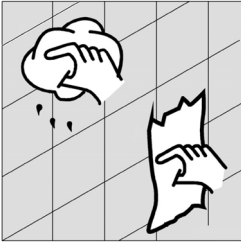


1. Clean and dry the grab bar. Refer to [Cleaning Grab Bars](#) on page 4.
2. Clean and dry the wall.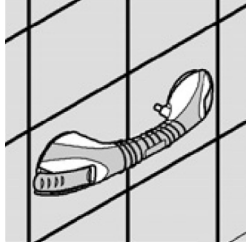



3. Position the grab bar so that it is not overlapping any grout lines or gaps in the wall surface.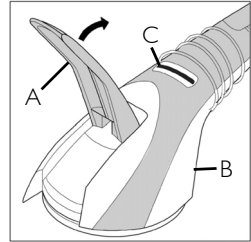

4. Lift the lever (A) on each end of the grab bar (B) until the window (C) turns RED.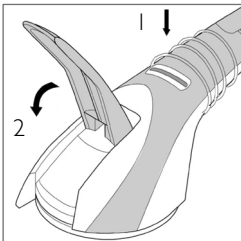

5. Press the grab bar against the wall.
6. Look at the window (1). Perform one of the following:
 - The window turns GREEN - The grab bar is properly secured to the wall. Push the levers all the way down (2).
 - The window turns RED - Reposition the grab bar on the wall, ensuring that the grab bar is not overlapping any grout lines or gaps in the wall surface, and repeat STEPS 5 and 6.

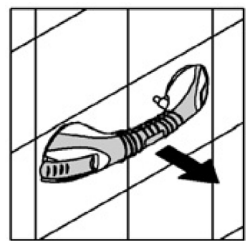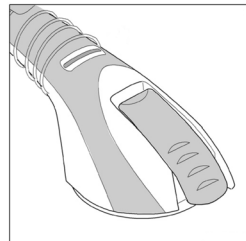
7. Check that the grab bar is secured to the wall before use.

4.2 Using the Grab Bar

WARNING:

Risk of Falling.

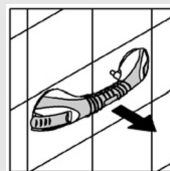- DO NOT use as sports equipment
- DO NOT use as a climbing aid.
- ALWAYS check that the grab bar is secured to the wall before use.
- Use EXTREME caution on wet tub or floor surfaces.
- NEVER use the grab bar if any looseness is detected. Grab bars are only secure as the wall they adhere to.

CAUTION:

Risk of Property Damage.

Use of suction cup grab bars may cause loose tiles or pieces of damaged walls and tub surrounds to fall off.

Use the grab bar for support when entering or exiting the bathing area.


5 Care and Maintenance

5.1 Cleaning Grab Bars

CAUTION:

Risk of Product Damage.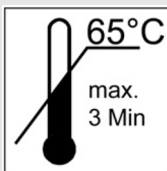

- DO NOT use an abrasive detergent, cleaner or cloth to clean or dry the grab bars.
- Clean grab bars by hand.
- Maximum cleaning temperature is 65°C.
- Regular cleaning and inspection is recommended.

1. To clean the grab bar, wipe the entire surface with a damp cloth and mild soap or non-abrasive cleaner.

6 Limited Warranty - North America

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state. Invacare warrants this product to be free from defects in materials and workmanship for three years for the original purchaser. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back cover. Provide dealer's name, address, model number, the date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

Invacare Corporation will issue a return authorization. The defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty (30) days of return authorization date. DO NOT return products to our factory without our prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO: MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN INVACARE DEALER, SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE.

THE FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. IT SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESSED WARRANTY PROVIDED HEREIN AND THE REMEDY FOR VIOLATIONS OF ANY IMPLIED WARRANTY SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER. THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

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© 2010 Invacare Corporation. All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by [™] and [®]. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.