Shower Chair

Assembly, Installation and Operating Instructions

Model Nos. 9680P, 9680 (w/o Backrest), 9681P, 9681 (w/ Backrest)

Yes, you can.™

SAVE THESE INSTRUCTIONS

NOTE: Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact Carrier/ Dealer for further instruction.

SAFETY SUMMARY

To ensure the safe installation of the Shower Chair, these instructions **MUST** be followed:

GENERAL WARNINGS

DO NOT install this equipment without first reading and understanding this instruction sheet. If you are unable to understand these **Warnings, Cautions and Instructions**, contact a healthcare professional, dealer or technical personnel if applicable before attempting to install this equipment - otherwise, injury or damage may occur.

Ensure that the snap buttons fully protrude through the same respective adjustment hole of each leg extension. This ensures that the adjustable legs are securely locked in position and an even height adjustment is achieved.

Always inspect the Shower Chair to ensure that it is properly positioned and is stable before using.

Ensure that **ALL** hardware is tight at **ALL** times.

Check rubber tips for rips, tears, cracks or wear. If any of these conditions exist, replace rubber tips immediately.

Rubber tips **MUST** be in flush contact with the floor/tub floor at all times.

DO NOT use with tub floors less than 16-inches (40.64 cm) wide.

Users with limited physical capabilities should be supervised or assisted when using the Shower Chair.

Ensure user weight is evenly distributed when seated. Improper loading may result in shower chair tipping over.

For models with backrest, ensure the compression buttons on the backrest are fully visible through notch on back of seat before using.

The Shower Chair has a weight limitation of 250 lbs (114 kg).

Shower Chair is **NOT** to be used as a transfer bench or transfer device.

ASSEMBLING THE SHOWER CHAIR (FIGURE 1)

NOTE: Refer to the WARNINGS in the SAFETY SUMMARY of this instruction sheet.

1. Remove contents from carton.
2. Place seat upside down on ground/floor.
3. Position top leg cutout over bottom leg cutout, as shown in FIGURE 1, aligning center holes of each tube.

NOTE: Top leg must be positioned on top of bottom leg to ensure a proper and safe fit.

4. Insert the phillips head screw and flat washer through the top center hole. Thread through both legs and secure with lock washer and wing nut.
5. Align leg sections with underside of seat while seat is still upside down on ground/floor.
6. Install one (1) slotted hex head screw through the leg tube and into seat. Loosely tighten. Repeat for remaining three (3) slotted hex head screws.
7. Once all components are assembled, tighten all hardware until a resistance is felt and all slack is removed. Continue to tighten 1/4 to 1/2 of a turn to securely fasten.

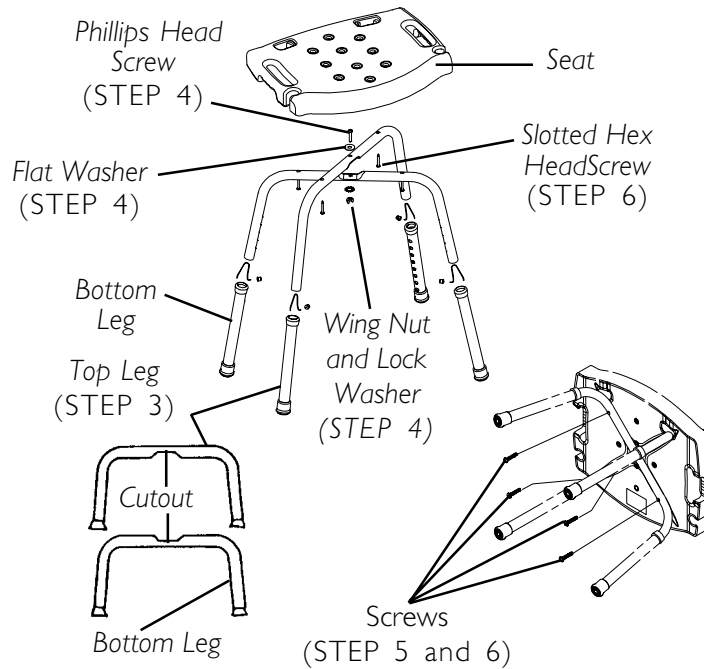
FIGURE I - ASSEMBLING THE SHOWER CHAIR

NOTE: For models with backs, refer to INSTALLING/REMOVING BACKREST in this instruction sheet.

INSTALLING/REMOVING THE BACKREST ON MODELS 968IP AND 968I (FIGURE 2)

NOTE: Refer to the GENERAL WARNINGS in the SAFETY SUMMARY of this instruction sheet.

1. Position backrest in slots provided on seat.
2. Push down on back until backrest is secured.
3. Ensure the compression buttons on the backrest are fully visible through notch on back of seat before using.
4. To remove backrest, place one (1) knee on the seat.
5. Hold back handles and lift up.

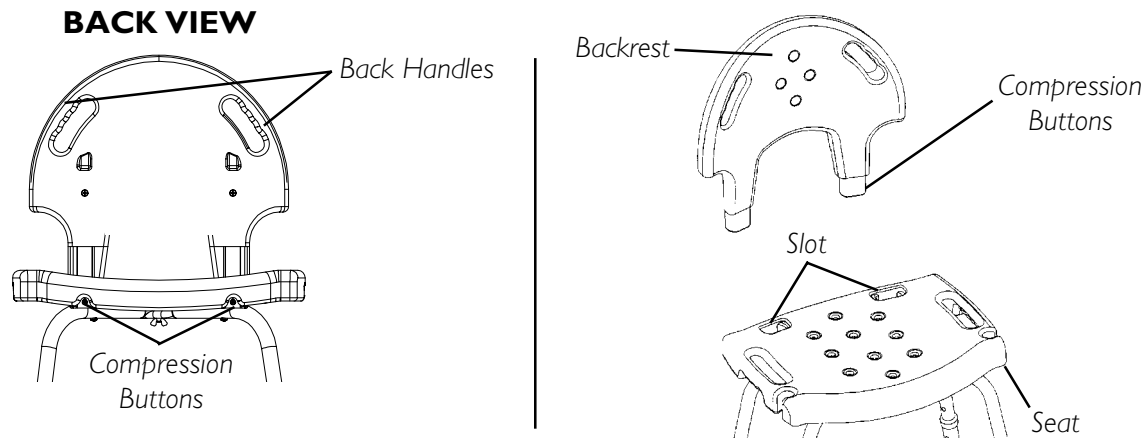
FIGURE 2 - INSTALLING/REMOVING THE BACKREST ON MODELS 9681P AND 9681

ADJUSTING THE LEG EXTENSIONS (FIGURE 3)

NOTE: Refer to the GENERAL WARNINGS in the SAFETY SUMMARY of this instruction sheet.

1. Select one (1) of the eight (8) adjustment holes located on the lower leg extension.
2. Depress the snap button and slide leg extension up or down to the desired height.
3. Ensure that snap button protrudes fully through the adjustment hole of the leg frame. Repeat for the other legs.
4. Ensure all four (4) legs are adjusted to the same height.

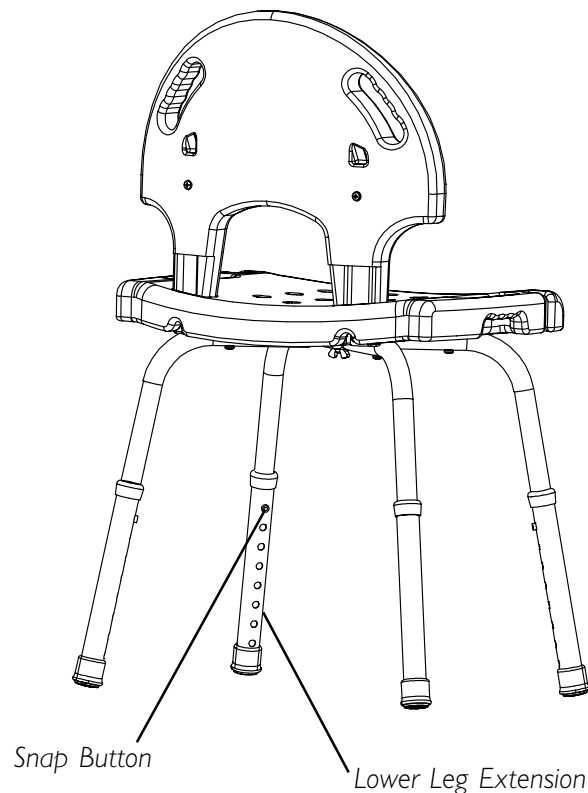
FIGURE 3 - ADJUSTING THE LEG EXTENSIONS

LIMITED WARRANTY

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants its product to be free from defects in materials and workmanship for the lifetime of the product for the original purchaser. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back cover. Provide dealer's name, address, model number, and the date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

Invacare Corporation will issue a return authorization. The defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty (30) days of return authorization date. **DO NOT** return products to our factory without our prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO: MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS: PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN AUTHORIZED INVACARE DEALER, SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE.

THE FOREGOING EXPRESS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES WHATSOEVER, WHETHER EXPRESS OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

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© 2003 Invacare Corporation
Invacare is a registered trademark of
Yes, you can. is a trademark of Invacare
Invacare Corporation www.invacare.com