

# Quad Cane Camlock Replacement Kit

## Assembly, Installation and Operating Instructions

Model Nos. 1055948



Yes, you can.

### SAVE THESE INSTRUCTIONS

*NOTE: Check ALL parts for shipping damage. In case of shipping damage, DO NOT use. Contact Carrier/Dealer for further instructions.*

## Safety Summary

To ensure the safe and proper installation of the Quad Cane Camlock, these instructions Must be followed:

### **⚠ WARNING**

**DO NOT** use this equipment without first reading and understanding this instruction sheet. If you are unable to understand the warnings, cautions and instructions, contact a healthcare professional, dealer or technical personnel if applicable, before attempting to install this equipment - otherwise, injury or damage may occur.

Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

Camlock **MUST** be properly engaged (locked) during use.

Double-button **MUST** be fully engaged and protruding through height adjustment hole on cane handle to ensure a positive lock.

**ALWAYS** test to see that the cane handle is properly and securely locked in place before using. This will ensure that camlock is secure.

Quad Canes are designed to provide ambulatory assistance to an individual weighing up to 250 lbs. (113.6 kg.). The replacement camlock does not in any way change the weight limitation of your quad cane.

## Description of Kit

ITEM	DESCRIPTION	QTY
1048085	Cam Lever	1
1048086	Cam Housing	1

## Assembling the Camlock

*NOTE: For this procedure, refer to Detail "A" of FIGURE 1 on page 2.*

1. Soak camlock housing in warm water for approximately 10 minutes.

*NOTE: Soaking will allow the housing to soften adequately to aid in installation.*

2. Remove housing from water.

3. Insert cam lever into camlock housing, handle first, through cutout in camlock housing.


4. Rotate cam lever 90° until the rear (round) portion of the cam lever is positioned into the recess of the cam housing with large tab on rear of cam lever facing down.

**For more information regarding Invacare products, parts, and services, please visit [www.invacare.com](http://www.invacare.com)**

# Installing the Camlock

*NOTE: For this procedure, refer to Detail "B" of FIGURE 1.*

1. Remove old camlock assembly from upper cane handle.
2. Slide camlock assembly onto bottom of upper cane handle ensuring small tab on top front of camlock housing snaps into small notch on the bottom front of cane handle.
3. Operate cam lever up and down to ensure it opens and closes properly.

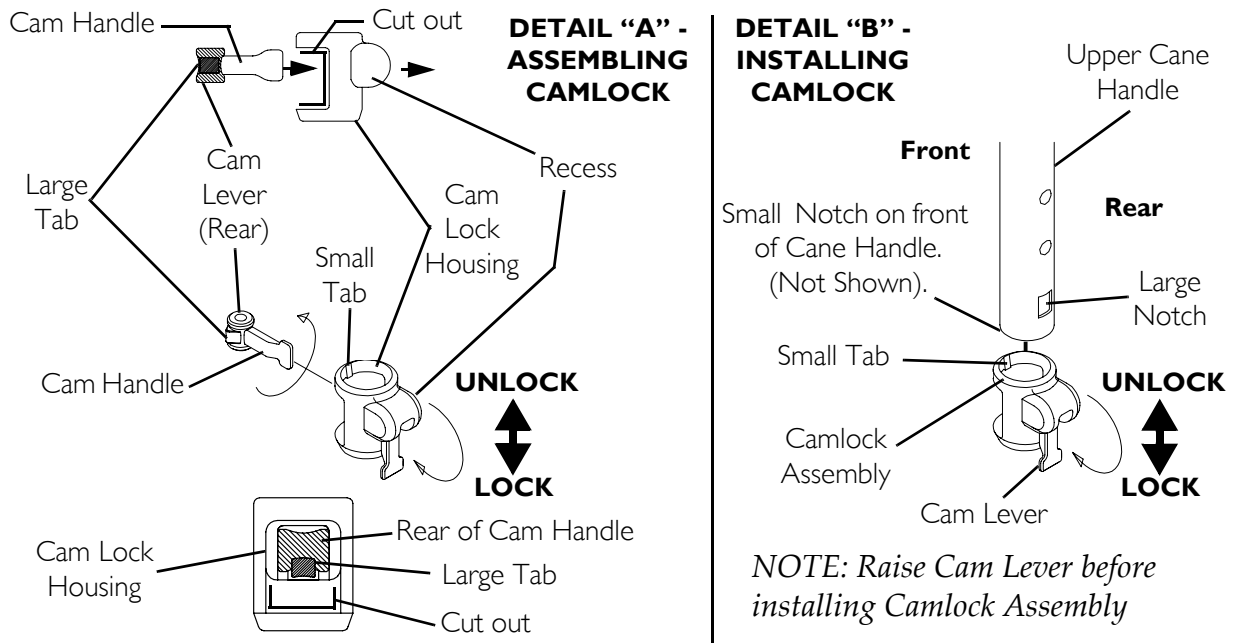

**FIGURE 1** Assembling the Camlock


# Reassembling the Quad Cane

*NOTE: For this procedure, refer to FIGURE 2.*

1. Raise cam lever before installing upper cane handle onto base.
2. Re-install cane handle with new camlock assembly onto cane base.
3. Push cam lever down.

*NOTE: This will engage the large tab on bottom rear of cam lever into large notch on rear of cane handle.*

4. Test cane to ensure that cane and camlock assembly are properly secured, i.e., no rattle or looseness of the cane base and upper cane handle.

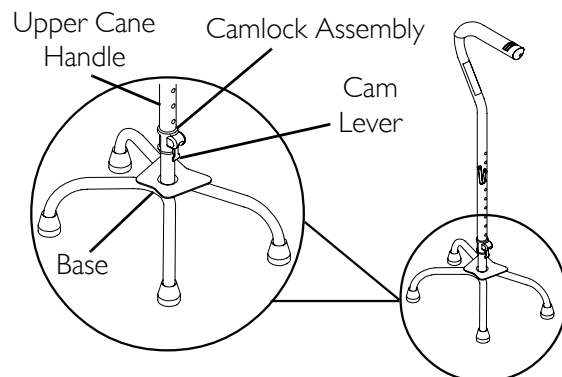

*NOTE: Push Cam Lever down after re-installing upper cane handle to base.*

**FIGURE 2** Reassembling the Quad Cane

Invacare Corporation [www.invacare.com](http://www.invacare.com)



**Yes, you can.®**

**USA**  
One Invacare Way  
Elyria, Ohio USA  
44036-2125  
800-333-6900

**Canada**  
570 Matheson Blvd E Unit 8  
Mississauga Ontario  
L4Z 4G4 Canada  
800-668-5324

Invacare, Yes, you can. and the Medallion Design are registered trademarks of Invacare Corporation.  
© 2006 Invacare Corporation

Part No 1055947

Rev B - 10/12/06

## Free Manuals Download Website

<http://myh66.com>

<http://usermanuals.us>

<http://www.somanuals.com>

<http://www.4manuals.cc>

<http://www.manual-lib.com>

<http://www.404manual.com>

<http://www.luxmanual.com>

<http://aubethermostatmanual.com>

Golf course search by state

<http://golfingnear.com>

Email search by domain

<http://emailbydomain.com>

Auto manuals search

<http://auto.somanuals.com>

TV manuals search

<http://tv.somanuals.com>