

Back Cane Replacement Kits for 9000 Series Wheelchairs



Yes, you can.

Assembly, Installation and Operating Instructions

Kit Nos. I048919, I048920, I048921, and I048922

SAVE THESE INSTRUCTIONS

NOTE: These instructions apply to the 9000 XT/XDT and the 9000 SL with adjustable back angle option.

NOTE: Check all parts for shipping damage. In case of damage, DO NOT use. Contact Carrier/Invacare for further instructions.

Safety Summary

⚠ WARNING

DO NOT install this equipment without first reading and understanding this instruction sheet. If you are unable to understand the warnings, cautions and instructions, contact a healthcare professional, dealer or technical personnel if applicable, before attempting to install this equipment - otherwise, injury or damage may occur.

Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

After ANY adjustments, repair or service and BEFORE use, make sure that all attaching component parts are secure.

NOTE: For this procedure, refer to FIGURE 1.

1. Remove the two mounting screws and washers that secure the back upholstery to the back canes (Detail "A").
2. Remove the two mounting screws, locknuts, and (for wheelchairs manufactured after 11/1/2005) spacers that secure one back cane to the wheelchair frame (Detail "C").
3. Pull the loose back cane out of the back upholstery.
4. Slide a new back cane through the back upholstery.
5. Secure the new back cane to the wheelchair frame at the desired back height with the two mounting screws, locknuts, and (for wheelchairs manufactured after 11/1/2005) spacers (Detail "B").

NOTE: For wheelchairs manufactured after 11/1/2005, the spacers must be on the outside of the wheelchair in order for the mounting screws to be properly tightened (Detail "C").

**For more information regarding Invacare products, parts, and services,
please visit www.invacare.com**

6. Repeat STEPS 2-5 for the other back cane, ensuring that the back height selected in STEP 5 is the same for both back canes.
7. Secure the back upholstery to the new back canes with the existing mounting screws and washers (Detail "A").

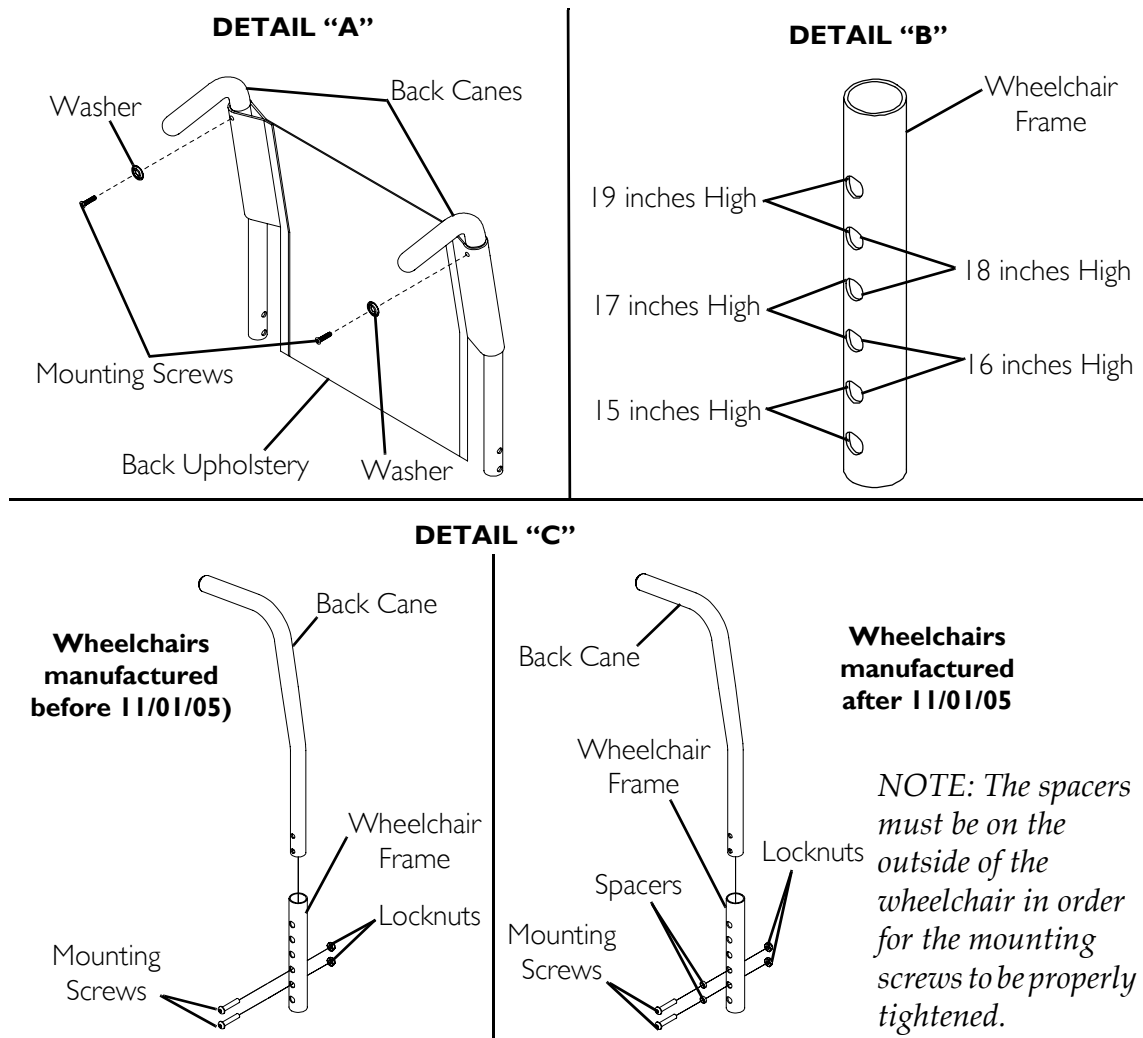


FIGURE 1 Replacing Back Canes



Yes, you can.®

Invacare Corporation

www.invacare.com

USA

One Invacare Way
Elyria, Ohio USA
44036-2125
800-333-6900

Canada

570 Matheson Blvd. E Unit 8
Mississauga Ontario
L4Z 4G4 Canada
800-668-5324

Invacare, the Medallion Design, and Yes, you can. are registered trademarks of Invacare Corporation.

© 2005 Invacare Corporation

Part No 1046525

Rev C - 12/6/05

MADE IN MEXICO

Free Manuals Download Website

<http://myh66.com>

<http://usermanuals.us>

<http://www.somanuals.com>

<http://www.4manuals.cc>

<http://www.manual-lib.com>

<http://www.404manual.com>

<http://www.luxmanual.com>

<http://aubethermostatmanual.com>

Golf course search by state

<http://golfingnear.com>

Email search by domain

<http://emailbydomain.com>

Auto manuals search

<http://auto.somanuals.com>

TV manuals search

<http://tv.somanuals.com>