

### SAFETY SUMMARY

Check all parts for shipping damage before using. In case of damage, do not use. Contact the dealer for further instructions.

#### NOTICE

The information contained in this document is subject to change without notice.

#### WARNING

DO NOT install, maintain or operate this equipment without reading, understanding, and following the proper Invacare instructions, otherwise injury or damage may result.

Your therapist and physician should be consulted if you have any questions regarding individual limitations and needs.

## HEADREST REINFORCEMENT PLATE INSTALLATION FOR THE 12, 14, 16, 17, 18 AND 20-INCH PERSONAL BACK

*NOTE: 13, 15 and 19-inch Personal Backs have an existing reinforcement plate.*

1. Remove the Personal Back from the back canes.
2. Remove the zippered cover from the Personal Back.
3. Mount the Personal Back in bench vise with the front facing you.
4. Push the foam of the Personal Back downward.
5. With hole closest to top edge in the middle of reinforcement plate (marked "A" - Refer to FIGURE 2) align Personal Back (FIGURE 1) dimples on inside with **TOP** two (2) corner holes of the reinforcement plate. Clamp plate to Personal Back.
6. Drill (**from the inside of the Personal Back**) the **TOP** two (2) corner holes with a 3/16-inch drill bit.

7. Unclamp the reinforcement plate and turn the Personal Back around in the vise.
8. Clamp the reinforcement plate to the outside of the Personal Back, aligning holes on plate with holes drilled in Personal Back.
9. Drill (**from the outside of the Personal Back**) the **BOTTOM** two (2) corner holes with a 3/16-inch drill bit.
10. Drill mounting holes in Personal Back for the selected headrest. Refer to FIGURE 2.
11. Unclamp the reinforcement plate and turn Personal Back around in vise.
12. Mount reinforcement plate on inside of Personal Back (FIGURE 1).

*NOTE: Position heads of button head screws on the inside of the backrest.*

13. Secure reinforcement plate to Personal Back with button head screws and hexnuts.

*NOTE: Use appropriate holes in reinforcement plate for mounting selected headrest.*

Kit No. 1049522 includes the following:

ITEM NO.	DESCRIPTION	QUANTITY
1.	Headrest Reinforcement Plate	1
2.	Screws 10-32 x 1/2-inch	4
3.	Locknuts	4
4.	Instruction Sheet	1

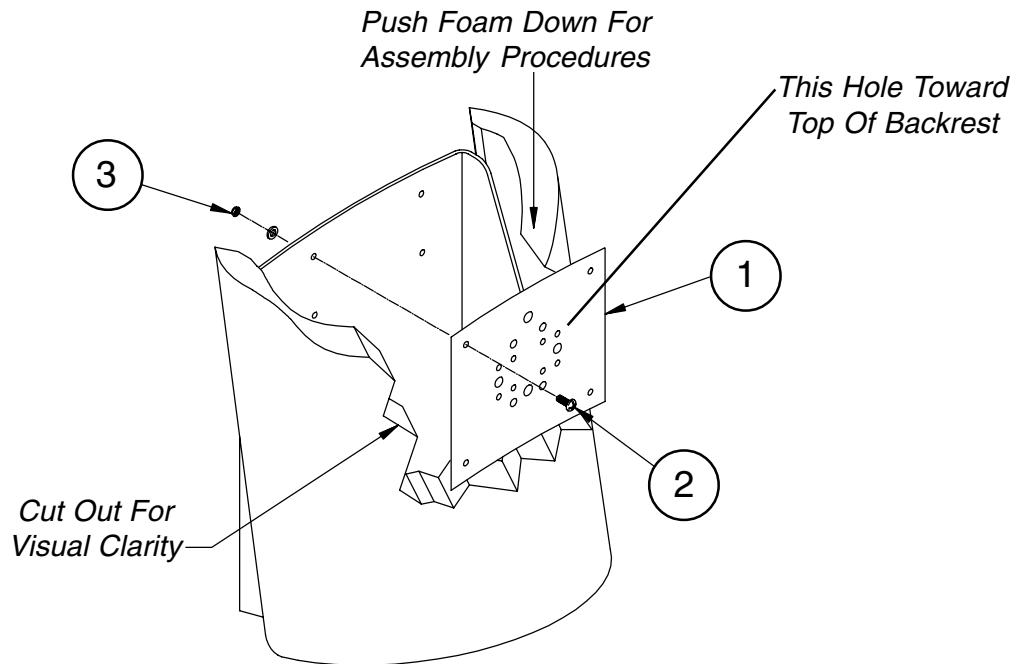


FIGURE 1 - HEADREST REINFORCEMENT PLATE INSTALLATION

**HOLE APPLICATIONS**

- A = Canyon
- B = Whitmeyer
- C = Otto Bock
- D = Otto Bock
- E = Freedom
- F = Mounting Hole
- G = Pindot

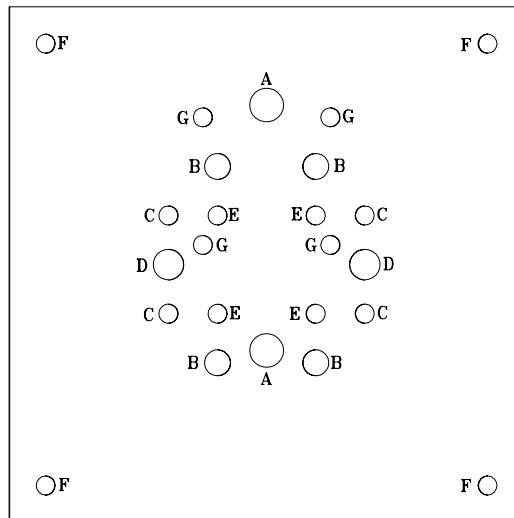


FIGURE 2 - HOLE APPLICATIONS

Invacare Corporation [www.invacare.com](http://www.invacare.com)



**Yes, you can.™**

**USA**  
 One Invacare Way  
 Elyria, Ohio USA  
 44036-2125  
 800-333-6900

**Canada**  
 5970 Chedworth Way  
 Mississauga, Ontario  
 L5R 3T9, Canada  
 800-668-5324

Invacare is a registered trademark of Invacare Corporation.  
 © 1999 Invacare Corporation

Form No. 94-148 Part No. 1049521 Rev F (1) 7/00

## Free Manuals Download Website

<http://myh66.com>

<http://usermanuals.us>

<http://www.somanuals.com>

<http://www.4manuals.cc>

<http://www.manual-lib.com>

<http://www.404manual.com>

<http://www.luxmanual.com>

<http://aubethermostatmanual.com>

Golf course search by state

<http://golfingnear.com>

Email search by domain

<http://emailbydomain.com>

Auto manuals search

<http://auto.somanuals.com>

TV manuals search

<http://tv.somanuals.com>