### SAVE THESE INSTRUCTIONS

## Safety Summary

To ensure the safe and proper installation of the WARNING/CAUTION labels, these instructions **MUST** be followed:

### **⚠ WARNING**

**DO NOT** install or use this kit without first reading and understanding this instruction sheet. If you are unable to understand the Warnings, Cautions or Instructions, contact Invacare Technical Support before attempting to install this kit - otherwise, injury or damage may occur.

**DO NOT** use the wheelchair without installing these Warning and Caution labels.

The label kit contains the following labels:

PART NUMBER	REPLACE LABEL
1130222	1074106
1098319	60106X144
1110513	1085379
1098320	1026960

## Installing Warning/Caution Labels

1. Determine the proper location for each label. Refer to FIGURE 1.

*NOTE: The English labels are included in FIGURE 1 for ease in determining proper location.*

2. Peel one (1) label from the adhesive backing.

3. Position the appropriate warning or caution label directly over the English label at the location determined in STEP 1.

4. Repeat STEPS 1-3 for each label.

For more information regarding Invacare products, parts, and services,  
please visit [www.invacare.com](http://www.invacare.com)

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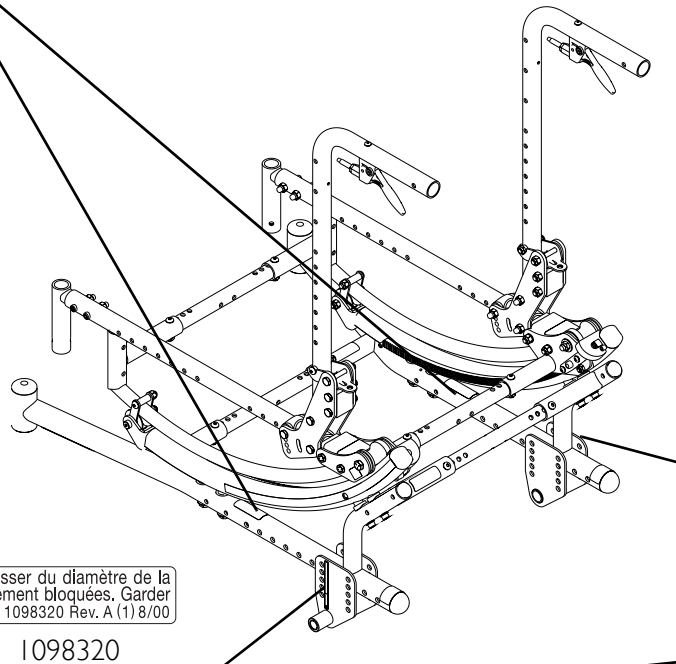
**⚠ WARNING**  
 Care must be taken so the configuration set up for the user is stable. Stability adjustments must be performed by a qualified technician.  
 1074106

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
**⚠ WARNING** Detent balls should extend beyond the diameter of the axle bushing for a positive lock. Keep detent balls clean.  
 1026960

**⚠ WARNING**  
 DO NOT OPERATE WITHOUT THE ANTI-TIP TUBES INSTALLED. P/N 60106X144  
 REV. 5/98

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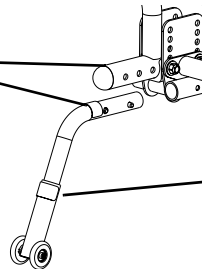**⚠ WARNING**  
 Refer to Owner's Manual for proper anti-tipper setting.  
 1085379

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Invacare is a registered trademark of Invacare Corporation.  
Yes, you can. is a trademark of Invacare Corporation.  
© 2005 Invacare Corporation

Part No 1134808

Rev A - 06/05

## Free Manuals Download Website

<http://myh66.com>

<http://usermanuals.us>

<http://www.somanuals.com>

<http://www.4manuals.cc>

<http://www.manual-lib.com>

<http://www.404manual.com>

<http://www.luxmanual.com>

<http://aubethermostatmanual.com>

Golf course search by state

<http://golfingnear.com>

Email search by domain

<http://emailbydomain.com>

Auto manuals search

<http://auto.somanuals.com>

TV manuals search

<http://tv.somanuals.com>

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**Invacare Corporation** [www.invacare.com](http://www.invacare.com)