Dual Release Walker

Assembly, Installation and Operating Instructions

Model No. 6291E - Adult, 6291E - Junior

Yes, you can:

SAVE THESE INSTRUCTIONS

NOTE: Check all parts for shipping damage before using. In case of damage, DO NOT use the equipment. Contact the Dealer/Carrier for further instructions.

SAFETY SUMMARY

To ensure the proper use of Invacare Dual Release Walkers, these instructions **MUST** be followed:

GENERAL WARNING

DO NOT install or use this equipment without first reading and understanding these instructions. If you are unable to understand the Warnings, Cautions or Instructions, contact a healthcare professional, dealer or technical personnel before attempting to install this equipment - otherwise, injury or damage may occur.

Each individual should always consult with their physician or therapist to determine proper adjustment and usage.

STABILITY WARNINGS

If walker is exposed to extreme temperature (above 100°F or below 32°F), high humidity and/or becomes wet, prior to use, ensure handgrips do not twist on side frame otherwise damage or injury may occur.

Check to make sure rubber tips and/or plastic glide tips are not ripped, worn or missing. Replace **BEFORE** using walker.

All leg extensions, must be adjusted so the walker is **LEVEL**. When using wheeled attachments, extensions **MUST** be adjusted so side frames are level.

DO NOT use rubber tips on rear leg extensions when using front wheel or brake options.

Rear legs extension with rubber tips or plastic glide tips **MUST** be in contact with floor surface at all times.

Glide tips are used on rear leg extensions **ONLY**, to allow the walker to roll easily over indoor carpeted areas without the need to lift the walker off the ground.

Glide tips on rear leg extensions should only be used with front wheel attachments.

Glide tips are not recommended for use on rough surfaces (concrete, gravel, etc.).

Glide tips are not recommended for use on all four (4) leg extensions at once.

DO NOT use glide tips and rubber tips at the same time.

Glide tips do not grip floor the way rubber tips grip flooring, therefore, caution should be observed when in use.

When using glide tips on walker, short steps should be taken to assure weight is evenly distributed and directly over walker legs. These precautions will prevent uncontrollable movement of walker.

Always use caution when using walker on wet or icy surfaces.

STABILITY WARNINGS CONTINUED

Invacare Walkers can provide ambulatory assistance for an individual weighing up to 136 kg (300lbs).

Invacare Walkers are designed to provide support, increased stability and assistance to an individual while walking; it is **NOT** intended to support the full weight of the user.

FOLDING/ASSEMBLY WARNINGS

After unfolding or assembling walker, ensure walker is securely locked in **OPEN** position and level to the ground **BEFORE** using.

DO NOT hang anything on either the left or right side frames (area that includes dual-release paddles). This will cause side frames to bend downward and prevent walker from locking into place when fully opened. This may also cause walker to tip, resulting in injury or damage.

INSTALLATION WARNINGS

Ensure all snap buttons fully protrude through adjustment holes of each leg extension. This ensures that the leg extensions are securely locked in position.

The leg extensions should be adjusted so the walker is level. If an even height cannot be achieved, adjust the leg extensions so that the rear of the walker is no more than one (1) inch lower than the front.

Always test to see that the walker and attachments are properly and securely locked in place **BEFORE** using.

OPENING THE WALKER (FIGURE 1)

1. To open the walker, pivot the left and right side frames outward.
2. Grasp the side frames and swing them outward until the release handles activate and engage. This will indicate that the walker is opened and ready to test before use.
3. To test walker, grasp the side frames and with some force, attempt to fold them inward. The walker will remain locked in the open position if it is engaged properly.

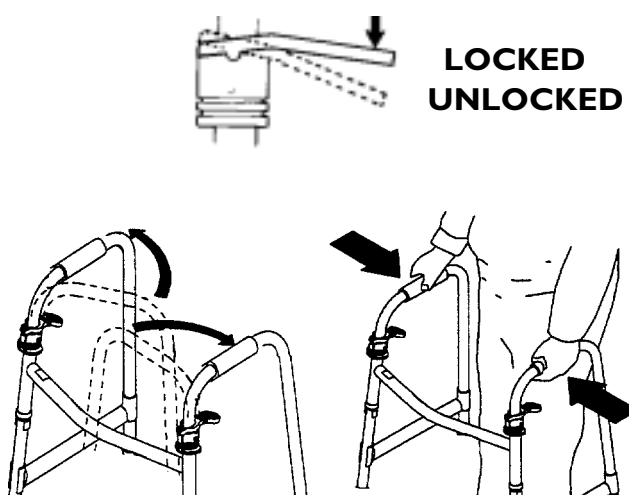
FIGURE 1 - OPENING THE WALKER

FOLDING THE WALKER (FIGURE 2)

1. To fold the walker, press down on the left or right release handle and pivot the side frame inward until it meets the front crossbrace.
2. Repeat the same procedure for the other side frame. The walker now is folded for storage or transportation.

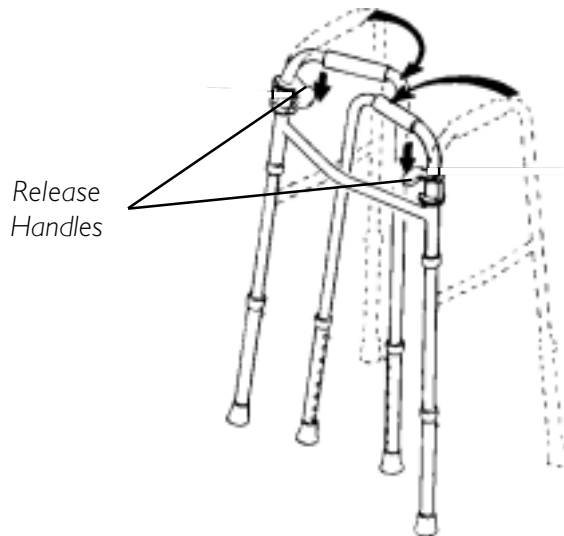
FIGURE 2 - FOLDING THE WALKER

ADJUSTING THE LEG EXTENSIONS (FIGURE 3)

WARNINGS

Ensure all snap buttons fully protrude through adjustment holes of each leg extension. This ensures that the leg extensions are securely locked in position.

The leg extensions should be adjusted so the walker is level. If an even height cannot be achieved, adjust the leg extensions so that the rear of the walker is no more than one (1) inch lower than the front.

NOTE: Each individual should always consult with their physician or therapist to determine the proper adjustment and usage.

1. Adjust the leg extensions by depressing the snap button on the lower part of the walker leg and allowing it to snap through the appropriate adjustment hole on the leg extension.

NOTE: The leg extension adjustment holes are numbered for convenience in identifying the corresponding adjustment hole to ensure proper and even height adjustment.

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© 2003 Invacare Corporation
Invacare is a registered trademark of
Yes, you can. is a trademark of
Invacare Corporation www.invacare.com
Invacare Corporation.