# ULTI-MATE™ AIR BACK INSTALLATION INSTRUCTIONS

## Assembly, Installation, and Operating Instructions

*NOTE: Check all parts for shipping damage. In case of shipping damage, DO NOT use. Contact equipment supplier for further instruction.*

*NOTE: When installing the Ulti-Mate Air Back, the cane rigidizer bar may need to be removed in order to utilize the full recline of the back.*

### SAFETY SUMMARY

#### WARNING

DO NOT use this equipment without first reading and understanding these instructions. If you are unable to understand the Warnings, Cautions, and Instructions, contact equipment supplier before attempting to use this equipment, otherwise, injury or damage may result.

The position of the back angle, rear wheels, seat depth and wheel locks, are directly related to the wheelchair's stability. Any change to one (1) or any combination of the four (4) may cause the wheelchair to decrease in stability. Use EXTREME caution when using a new seating position.

ALWAYS make sure that the wheelchair is stable BEFORE reclining the Ulti-Mate Back. Rear wheels may have to be repositioned rearward depending on the degree of recline. ALWAYS ensure stability BEFORE using maximum amount of recline. TEST wheelchair BEFORE it is occupied BY the end user to ensure safety.

To maintain the stability of the wheelchair when moving the rear wheels to ANY forward position, the amount of recline MUST be limited to prevent wheelchair from tipping onto anti-tippers.

Changing the seat depth of the wheelchair changes the stability of the wheelchair. To maintain stability, reposition the rear wheels towards the rear of the wheelchair BEFORE using the wheelchair.

The necessary back angle MUST be selected BEFORE repositioning the rear wheels forward.

Make sure that ALL hardware are tight at ALL times.

### PRODUCT DESCRIPTION

The Ulti-Mate Air Back is designed to provide the user with posture support and spinal alignment. It includes adjustments that allow the user to change the angle of recline of the back from 90°F (32°C) to 115°F (46°C) and the fit of the back without transferring out of the wheelchair.

The inflatable bladders that adjust the fit of the back can be repositioned. The cover is made from a water repellant fabric to protect the foam.


### INSTALLING ULTI-MATE AIR BACK TO UPPER BACK CANES (FIGURE 1)

1. Apply wheelchair brakes.
2. Remove back sling upholstery.
3. Reinsert any bolts required to stabilize back canes.
4. Determine right and left cane clamps and correct clamp size.

*NOTE: There are 7/8 (22mm) and 1-inch (25mm) clamps. To fit a 3/4-inch (19mm) back cane, remove the plastic washer from the 7/8-inch (22mm) clamp shown in FIGURE 1.*

5. Position right cane clamp around right back cane ensuring lever on clamp faces up and screw faces away from chair. Repeat on left side.

*NOTE: Before tightening clamps with enclosed Allen Key, ensure both clamps are level and positioned high enough on back canes to support Back. Also check that bottom locks clear any chair obstructions. Back may need to be positioned higher, lower or the obstruction removed to operate bottom locks. Clamps can be readjusted once Back is in place.*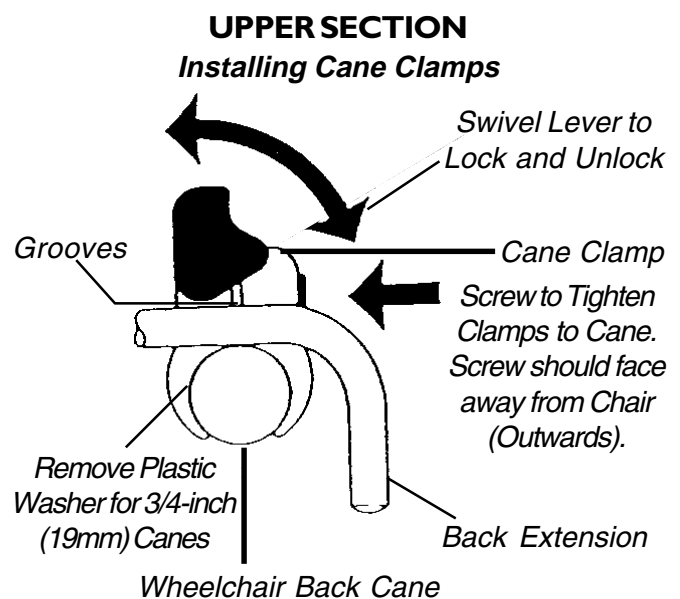

Top View - Installed on Right Wheelchair Cane

### FIGURE I-INSTALLING ULTI-MATE AIR BACK TO UPPER AND LOWER BACK CANES


## INSTALLING ULTI-MATE AIR BACK TO LOWER BACK CANES (FIGURES 1 AND 2)

1. Unlock levers on cane clamps exposing grooves completely (FIGURE 1).
2. Hold Ulti-Mate Air Back in front of canes and rest both back extensions into grooves of cane clamps (FIGURE 1).
3. To lock Ulti-Mate Air Back extensions into position, close levers on cane clamps until they cover the groove in the Back extension. If they do not, loosen the screws and align the grooves then retighten (FIGURE 1).
4. To secure lower portion of Ulti-Mate Air Back to back canes, stand behind wheelchair and locate levers with black knobs. Push levers inwards then pull them upwards until one extension on each side retracts (FIGURE 2).
5. Position Back so curve of retractable extensions hug canes (FIGURE 2).
6. Lock retractable extensions around canes by pushing black knobs down then outwards (FIGURE 2).

*NOTE: If distance from lower extensions to canes is too long or short, unzip the back cover and locate screws that hold adjustable extension plates in place. Loosen screws and slide extension plate right or left to adjust position. Retighten screws (FIGURE 2).*

7. When Ulti-Mate Air Back is securely fastened to canes, transfer client to wheelchair.

### LOWER SECTION

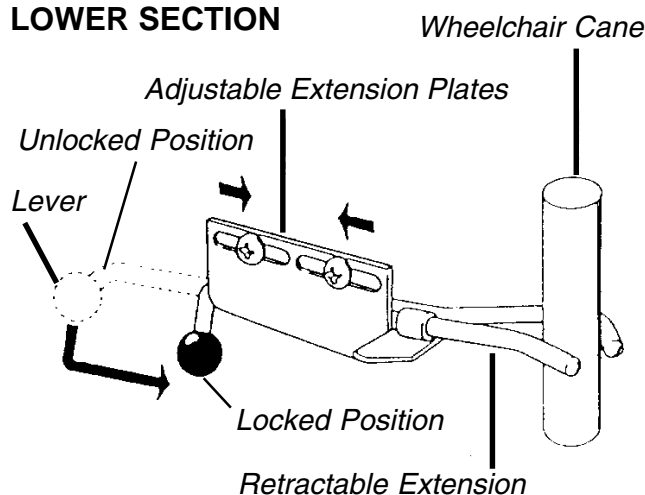**FIGURE 2 - INSTALLING ULTI-MATE AIR BACK TO LOWER BACK CANES**


## USE OF ULTI-MATE AIR BACK

### Achieving Customized Fit (FIGURE 3)

1. Ensure Back is at right height to provide client with desired support. If height adjustments are necessary, loosen cane clamps and reposition up or down.
2. To achieve a snug lateral fit, locate two (2) air pumps fastened on either side of Ulti-Mate Air Back. Squeeze pumps to inflate air pockets contained within each lateral support.

*NOTE: If your Ulti-Mate Air Back contains an optional lumbar air pocket, squeeze air pump located on middle portion of Back to inflate. DO NOT over inflate.*

3. To deflate air pockets, press small black button (release valve) located at base of each air pump.

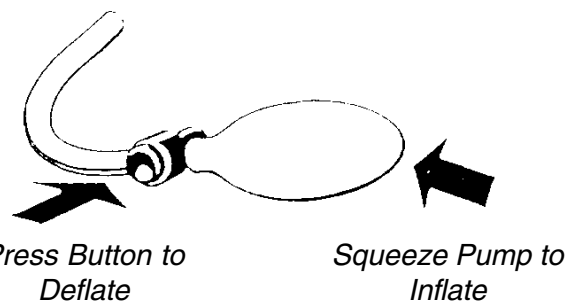**FIGURE 3 - ACHIEVING CUSTOMIZED FIT**

4. To conceal lateral air pumps from view, push them through slot in cover.

*NOTE: Conceal lumbar air pump by placing it into pocket located at top central portion of Back.*

5. You may reposition air pockets. They are attached to inner shell with hook and loop fasteners and can be repositioned up, down or sideways. To make adjustments:
  - a. Remove Ulti-Mate Air Back from wheelchair.
  - b. Unzip and remove cover (*NOTE: Cover can be removed without removing hardware.*).
  - c. Lift up foam cushion to make desired adjustments.
  - d. Reinstall cover.

## Reclining the Back

### WARNING

The position of the back angle, rear wheels, seat depth and wheel locks, are directly related to wheelchair stability. Any change to one (1) or any combination of the four (4) may cause the wheelchair to decrease in stability. Use **EXTREME** caution when using a new seating position.

**ALWAYS** make sure that the wheelchair is stable **BEFORE** reclining the Ultimate Back. Rear wheels may have to be repositioned rearward depending on the degree of recline. **ALWAYS** ensure stability **BEFORE** using maximum amount of recline. **TEST** wheelchair **BEFORE** it is occupied by the end user to ensure safety.

To maintain the stability of the wheelchair when moving the rear wheels to **ANY** forward position, the amount of recline **MUST** be limited to prevent wheelchair from tipping onto anti-tippers.

The necessary back angle **MUST** be selected **BEFORE** repositioning the rear wheels forward.

	Back Angle	Rear Wheels	Seat Depth	Wheel Locks
Back Angle	●	✓	N/A	✓
Rear Wheels	✓	●	N/A	✓
Seat Depth	N/A	✓	●	✓
Wheel Locks	✓	✓	✓	●

*NOTE: When adjusting a feature in the left hand column, follow across the shaded area and refer to the ✓ procedure to maintain the proper stability, safety and handling of the wheelchair.*

1. Locate black knob located on rear of Ulti-Mate Air Back.
2. Perform one (1) of the following:
  - a. **INCREASE RECLINE ANGLE** - Turn knob COUNTERCLOCKWISE.
  - b. **DECREASE RECLINE ANGLE** - Turn knob CLOCKWISE.

## NECK SUPPORT OR HEADREST INSTALLATION

*NOTE: When optional neck support or headrest is used, remove carrying handle.*

1. Remove cover.
2. Remove screw and carrying handle.
3. Refer to **Neck Support or Headrest Instruction Sheet**, for proper installation and adjustment.
4. Reinstall cover.

## INSPECTION AND CLEANING INSTRUCTIONS

### WARNING

Do not continue to use this product if problems are discovered. Corrective maintenance can be performed by or arranged through your equipment supplier.

Weekly visual inspection of parts including hardware, brackets, upholstery materials, foams, and plastics for deformation, corrosion, breakage, wear or compression.

Weekly check of fasteners to verify that all mechanical connections are secure.

### Cleaning Instructions

#### BACK ASSEMBLY.

Weekly cleaning with dampened cloth. Wipe entire surface.

#### COVER.

### WARNING

The cover is designed to protect the foam against a user's incontinence and to provide fire retardancy so the back must not be used without its cover. If the cover is torn, it must be replaced.

1. Can be machine washed and dried.
2. **DO NOT** use bleach.

*NOTE: We recommend cleaning the cover on a regular basis.*

#### FOAM.

If the back foam becomes contaminated because of incontinence, the foam should be replaced because the foam is absorbent and cannot be cleaned.

---

## **UNITED STATES LIMITED WARRANTY**

**PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.**

**This warranty is extended only to the original purchaser/user of our products.**

**This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.**

**Invacare warrants its product to be free from defects in materials and workmanship for two (2) years on back and 90 days on cover of use by original purchaser. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.**

**For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the next page. Provide dealer's name, address, model number, and the date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.**

**Invacare Corporation will issue a return authorization. The defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty (30) days of return authorization date. DO NOT return products to our factory without our prior consent. C.O.D. shipments will be refused; please prepay shipping charges.**

**LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTSSUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO: MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN ANAUTHORIZED INVACARE DEALER, SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE.**

**THE FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE.**

**IT SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESSED WARRANTY PROVIDED HEREIN AND THE REMEDY FOR VIOLATIONS OF ANY IMPLIED WARRANTY SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.**

**THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.**

---

## **UNITED KINGDOM LIMITED WARRANTY**

This warranty is extended only to the original purchaser of our products and does not affect statutory rights.

Invacare UK / Invacare Service Representative warrants the foam cushion to be free from defects for a period of two (2) years from the date of purchase. The cushion cover is warranted for a period of ninety (90) days from date of purchase. With regard to the original purchaser/user only, Invacare UK / Invacare Service Representative warrants its metal fabricated hardware components to be free from defects in materials and workmanship for the lifetime of the product except any wood, foam plastic or upholstered components. If within such warranty period the product shall be proven to Invacare UK's / Invacare Service Representative reasonable satisfaction to be defective, it shall be repaired or replaced, at Invacare UK's / Invacare Service Representative option ; Invacare UK's / Invacare Service Representative sole obligation, and your exclusive remedy under this warranty being limited to such repair or replacement.

For warranty service, please contact Invacare UK's Service Department / Invacare Service Representative during normal working hours. Upon receiving notice of an alleged defect in a product, Invacare UK / Invacare Service Representative will issue a serialized return authorization. The product shall then be returned to either Invacare plant or service center as specified by Invacare UK / Invacare Service Representative. Defective component part(s) must be returned for warranty inspection using the serial number as identification within thirty (30) days of return authorization date. DO NOT return products to our factory without prior consent.

**LIMITATIONS AND EXCLUSIONS: THE FOREGOING WARRANTY DOES NOT COVER NORMAL WEAR AND TEAR AND SHALL NOT APPLY TO PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE, COMMERCIAL OR INSTITUTIONAL USE, PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT (INCLUDING, BUT NOT LIMITED TO, MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS) OR TO PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE UK / INVACARE SERVICE REPRESENTATIVE.**

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