NOTE: Check all parts for shipping damages before using. In case of damage, do NOT use the equipment. Contact the Equipment Supplier for further instructions.

**SAFETY SUMMARY**

WARNING/CAUTIONS notices apply to hazards or unsafe practices which could result in minor personal injury and/or property damage.

**WARNING**

DO NOT install this equipment without first reading and understanding this instruction sheet. If you are unable to understand these instructions, contact a healthcare professional, dealer or technical personnel if applicable before attempting to install this equipment - otherwise, injury or damage may occur.

The position of the Personal Back in the three (3) different mounting bracket slots is directly related to the chair's stability. When the Personal Back is added to a TILT and/or RECLINING chair, it may cause a decrease in the chair's stability. It may be necessary to reposition the FRONT CASTERS, REAR WHEELS, BACK ANGLE, TILT-IN-SPACE, RECLINE POSITION and/or SEAT DEPTH before use. Use EXTREME caution when using a new seating position.

Your therapist and physician should be consulted if you have any questions regarding individual limitations and needs.

Working with your therapist, physician, and equipment supplier is the best way to assure that a Personal Back choice matches your individual needs.

As the needs of the individual become more complex, the Personal Back evaluation becomes more important.

**NOTICE**

Information contained in this document is subject to change without notice.

**INTRODUCTION**

The Personal Back is designed to provide the user with posture support and spinal alignment. The cover is made from a water repellant fabric to protect the foam.


**DETERMINING THE MOUNTING POSITION FOR INSTALLATION OF PERSONAL BACK (FIGURE 1)**

1. Remove back from the shipping container.
2. Remove hardware that secures the upholstery to the back canes.
3. To obtain back height required after physical assessment by therapist (A), use the following formula (FIGURE 1):

$$\text{Total Back Height Required by Therapist - (A)} = \text{Height of Personal Back (Area B)} + \text{Distance between Personal Back and Seat Cushion (Area C)}$$

$$\text{Distance between Personal Back and Seat Cushion (Area C)} = \text{Distance from Bottom of Back to Top Mounting Hardware (Area D)} + \text{Location of Mounting Bracket (Area E)}$$

4. Measure the **Total Height of the Personal Back (Area B)**.
5. Subtract the **Total Height of the Personal Back (Area B)** from the **Total Back Height Required by the Therapist (A)**. This gives you **Distance between the bottom of the Personal Back and Cushion (Area C)**.
6. Measure the **Distance from the Bottom of Back to Top Mounting Hardware (Area D)**.
7. Add **Area D (STEP 6)** and **Area C (STEP 5)** together. This will determine how much additional height is needed and **Location of Mounting Bracket (Area E)**.
8. Measure the distance from the cushion to the chair frame/back canes (**Area E, STEP 7**) and mark (**both sides**) on the inside of the chair frame/back canes.

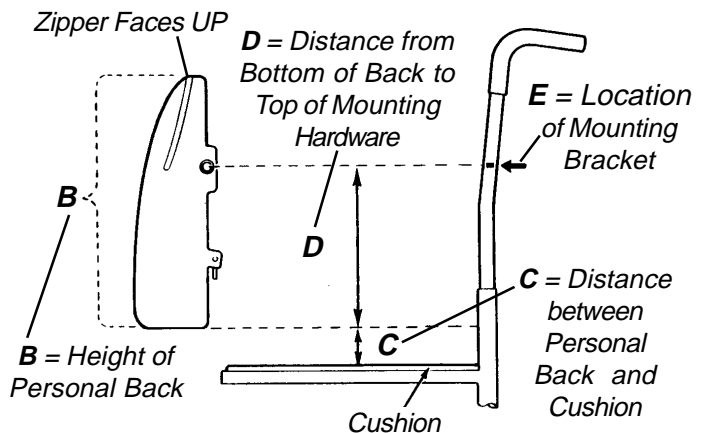**FIGURE 1 - DETERMINING THE MOUNTING POSITION FOR INSTALLATION OF PERSONAL BACK**

# PERSONAL BACK/PERSONAL BACK PLUS MOUNTING HARDWARE INSTALLATION

This kit includes the following:

DESCRIPTION	QUANTITY
Multi-Position Mounting Plates	2
Short Bolts-Phillips Head	4
Nylon Locknuts	4
Long Bolts	4
Regular Nuts	4
L-Brackets	2
Clamp Inserts	4
Half-Clamps for 1-inch (25mm) Tubes	4

*NOTE: Not all parts are required for each installation. Read instructions carefully.*

## WARNING

**DO NOT** attempt to modify the 1-inch (25mm) clamps to fit 3/4-inch (19mm) diameter tubing.

### Installation on Wheelchairs with 1-inch (25mm) Diameter Back Canes (FIGURE 2)

<b>Have the following tools available:</b>	
1/8-inch Allen Key	
3/8-inch Crescent or Box Wrench	
<b>Parts Required:</b>	<b>Quantity</b>
Multi-Position Mounting Plates	2
1-inch (25mm) Half-Clamps	4
Clamp Inserts	4
Long Bolts	4
Nylon Locknuts	4

- Determine back height. Refer to DETERMINING THE MOUNTING POSITION FOR INSTALLATION OF PERSONAL BACK in this instruction sheet.
- Insert two (2) long bolts (from the inside) through the small holes on the mounting plate, then through the half-clamps. Tighten down nuts to secure clamp.
- Perform one (1) of the following:
  - Half-clamps tight - proceed to next step.
  - Half-clamps loose - refer to INSTALLATION ON WHEELCHAIRS WITH 7/8-INCH (22MM) DIAMETER BACK CANES.

*NOTE: Clamp should dig into cane.*

- Repeat the procedure for the other side, ensuring that both sides are matched for height, and are parallel to each other.
- Check all nuts for tightness.

*NOTE: The clamp-on bracket is easily adjusted by loosening the bolt and sliding the brackets up or down on the back cane to obtain proper height adjustment.*

- Install Personal Back. Refer to PERSONAL BACK/PERSONAL BACK PLUS INSTALLATION in this instruction sheet.

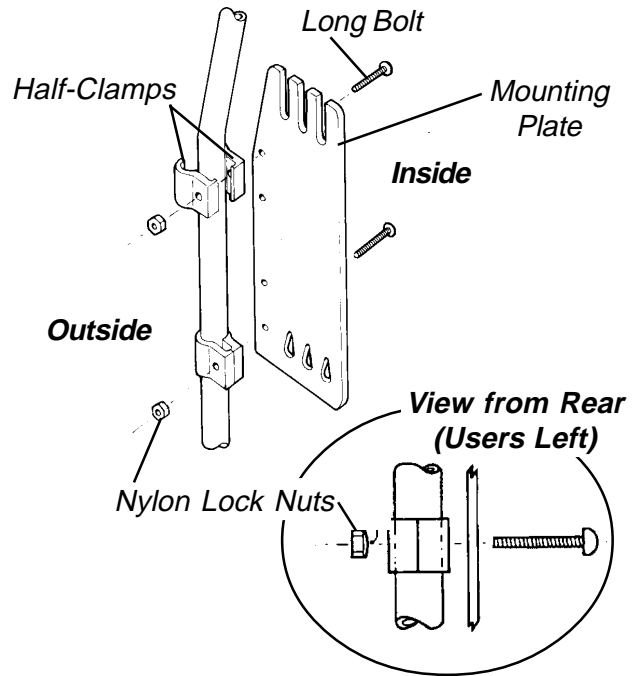
FIGURE 2 - INSTALLATION ON WHEELCHAIRS WITH 1-INCH (25MM) DIAMETER BACK CANES

### Installation on Wheelchairs with 7/8-Inch (22mm) Diameter Back Canes (FIGURE 3)

*NOTE: This procedure should be performed ONLY if cane clamps are too large for the back cane tubing.*

- Insert two (2) long bolts (from the inside) through the small holes on the mounting plate, then through the half-clamps. Refer to FIGURE 2.
- Position one (1) insert onto the back cane with the insert opening facing the back of the wheelchair.
- Slide the cane clamp over the insert until the cane clamp halves surround the insert as shown in FIGURE 3, DETAIL "A".
- Tighten down nuts to secure clamp.
- Perform one (1) of the following:
  - Half-clamps tight - proceed to next step.
  - Half-clamps loose - refer to INSTALLATION ON WHEELCHAIRS USING L-SHAPED BRACKETS.

- Repeat STEPS 1 - 4 for remaining half-clamps, ensuring that both sides are matched for height, and are parallel to each other.
- Install Personal Back. Refer to PERSONAL BACK/ PERSONAL BACK PLUS INSTALLATION in this instruction sheet.

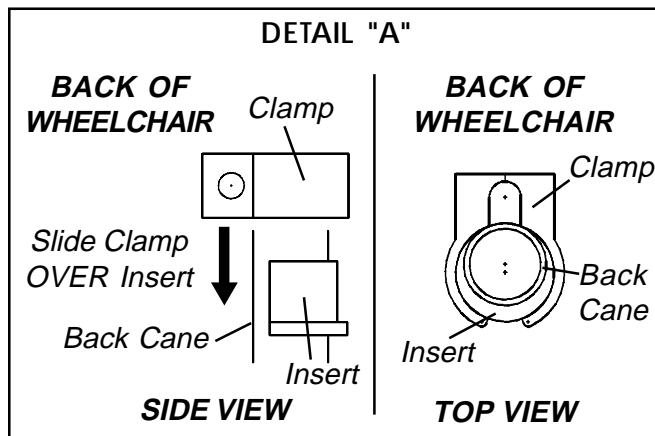
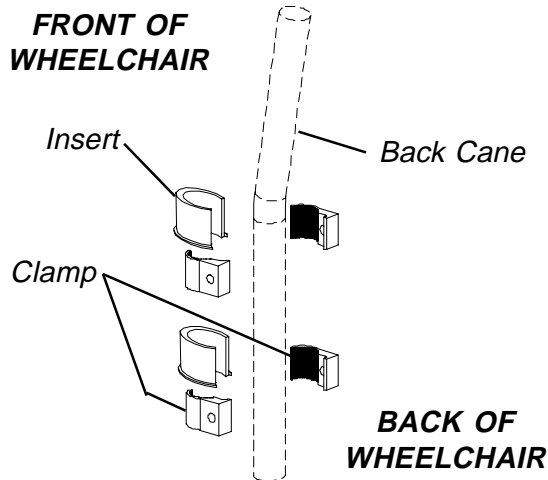
FIGURE 3 - INSTALLATION ON WHEELCHAIRS WITH 7/8-INCH (22MM) DIAMETER BACK CANES

### Installation on Wheelchairs Using L-Shaped Brackets (FIGURE 4)

<b>Have the following tools available:</b>	
Phillips Head Screwdriver	
3/8-inch Crescent/Box Wrench	
<b>Parts Required:</b>	<b>Quantity</b>
Multi-Position Plate	2
L-Brackets	2
Short Phillips Head Bolts	4
Locknuts	4

- Determine back height. Refer to DETERMINING THE MOUNTING POSITION FOR INSTALLATION OF PERSONAL BACK in this instruction sheet.

- Secure the L-shaped brackets to the back canes using existing upholstery holes and existing hardware.
- Ensure that the top of the mounting plate when installed to the L-shaped bracket is 1/2-inch (1.2 cm) higher than the mark determined in STEP 8 of DETERMINING THE MOUNTING POSITION FOR INSTALLATION OF PERSONAL BACK in this instruction sheet (DETAIL "A", FIGURE 4).
- Install two (2) short bolts (from the inside) through the small holes on mounting plate, then through L-shaped bracket. Tighten down locknuts to secure mounting plate to L-shaped bracket.
- Repeat the procedure for the other side, ensuring that both side are matched for height, and are parallel to each other.
- Install Personal Back. Refer to PERSONAL BACK/ PERSONAL BACK PLUS INSTALLATION in this instruction sheet.

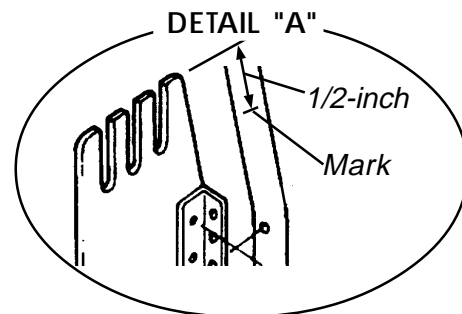
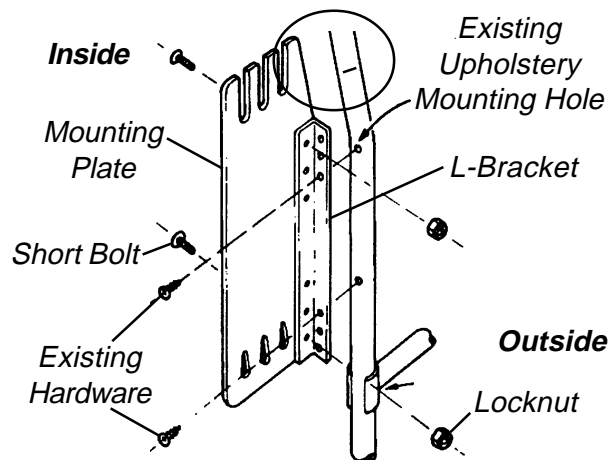
FIGURE 4 - INSTALLATION ON WHEELCHAIRS USING L-SHAPED BRACKETS

## USE OF THE PERSONAL BACK

The Personal Back includes a movable foam lumbar and left and right foam laterals attached to the back shell with hook and loop fasteners to provide additional positioning capability. To access the lumbar and the laterals, unzip the cover and push the foam forward, away from the back of the shell. The lumbar and laterals will then be accessible for making adjustments.

## PERSONAL BACK/PERSONAL BACK PLUS INSTALLATION (FIGURE 5)

### WARNING

The position of the Personal Back in the three (3) different mounting bracket slots is directly related to the chair's stability. When the Personal Back is added to a TILT and/or RECLINING chair, it may cause a decrease in the chair's stability. It may be necessary to reposition the FRONT CASTERS, REAR WHEELS, BACK ANGLE, TILT-IN-SPACE, RECLINE POSITION and/or SEAT DEPTH before use. Use EXTREME caution when using a new seating position.

1. Install the Personal Back onto the mounting plate so that the upper studs locate into the appropriate slots.
2. Turn bottom mounting pin until tear drop aligns with mounting hole.
3. Install bottom mounting pins into one (1) of the three (3) sets of tear drop holes in the bottom of the mounting bracket.
4. To lock, rotate the mounting pins to the downward position.

### WARNING

Make sure hook and loop straps are securely holding the levers. The Personal Back may come loose if the levers are not locked down.

5. Secure mounting pins from rotating with hook and loop straps on back of cover.
6. To remove the Personal Back, reverse **STEPS 1-5**.

*NOTE: Mark the upper and lower mounting positions of the Personal Back mounting bracket for reference by family or caregivers.*

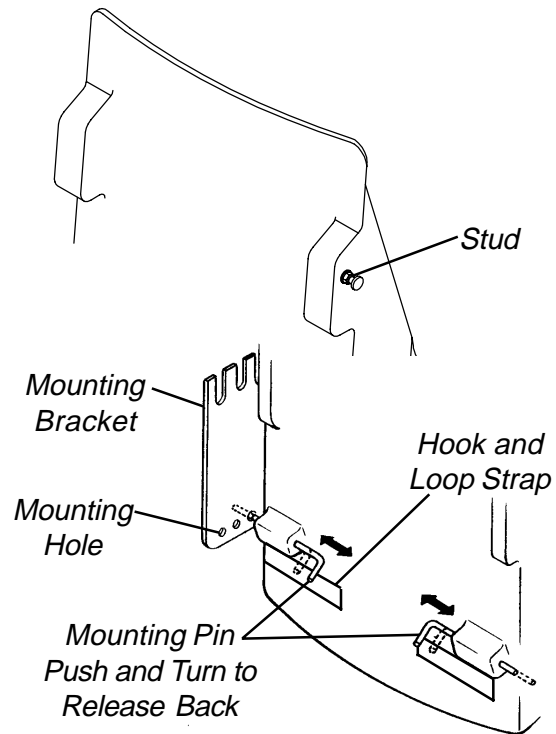
FIGURE 5 - PERSONAL BACK/PERSONAL BACK PLUS INSTALLATION

## Headrest Installation

Headrests may be installed on the Personal Back. The Headrest Reinforcement Plate **MUST** be used and is available separately.

## PERSONAL BACK MAINTENANCE

### WARNING

**DO NOT** continue to use this product if problems are discovered. Corrective maintenance can be performed at or arranged through your equipment supplier.

## Cleaning Instructions

### BACK ASSEMBLY.

Clean weekly with dampened cloth. Wipe entire surface.

Weekly visual inspection of parts including hardware, brackets, upholstery materials, foams (if accessible), and plastics, for deformation, corrosion, breakage, wear or compression.

Check ALL fasteners weekly to ensure that mechanical connections are secure.

---

**COVER.****CAUTION**

Machine wash cold. Gentle cycle. Mild detergent. NO fabric softeners or bleach. DO NOT machine dry. Air dry ONLY out of sun.

**WARNING**

The cover is designed to protect the foam against a user's incontinence and to provide fire retardancy so the cushion must not be used without its cover. If the cover is torn, it must be replaced.

**FOAM.**

If the back foam becomes contaminated because of incontinence, the back should be replaced because the foam is absorbent and cannot be cleaned. The foam cannot be replaced independently because it is permanently adhered to the back shell.

---

**UNITED KINGDOM LIMITED WARRANTY**

This warranty is extended only to the original purchaser of our products and does not affect statutory rights.

Invacare UK warrants the foam cushion to be free from defects for a period of two (2) years from the date of purchase. The cushion cover is warranted for a period of ninety (90) days from date of purchase. With regard to the original purchaser/user only, Invacare UK warrants its metal fabricated hardware components to be free from defects in materials and workmanship for the lifetime of the product except any wood, foam plastic or upholstered components. If within such warranty period the product shall be proven to Invacare UK's / Invacare Service Representative reasonable satisfaction to be defective, it shall be repaired or replaced, at Invacare UK's / Invacare Service Representative option; Invacare UK's / Invacare Service Representative sole obligation, and your exclusive remedy under this warranty being limited to such repair or replacement.

For warranty service, please contact Invacare UK's Service Department / Invacare Service Representative during normal working hours. Upon receiving notice of an alleged defect in a product, Invacare UK / Invacare Service Representative will issue a serialized return authorization. The product shall then be returned to either Invacare plant or service center as specified by Invacare UK / Invacare Service Representative. Defective component part(s) must be returned for warranty inspection using the serial number as identification within thirty (30) days of return authorization date. DO NOT return products to our factory without prior consent.

**LIMITATIONS AND EXCLUSIONS: THE FOREGOING WARRANTY DOES NOT COVER NORMAL WEAR AND TEAR AND SHALL NOT APPLY TO PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE, COMMERCIAL OR INSTITUTIONAL USE, PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT (INCLUDING, BUT NOT LIMITED TO, MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS) OR TO PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE UK / INVACARE SERVICE REPRESENTATIVE.**

## **UNITED STATES LIMITED WARRANTY**

**PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.**

**This warranty is extended only to the original purchaser/user of our products.**

**This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.**

**Invacare warrants its product to be free from defects in materials and workmanship for two (2) years on back and 90 days on cover of use by original purchaser. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.**

**For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back cover. Provide dealer's name, address, model number, date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.**

**Invacare Corporation will issue a return authorization. The defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty (30) days of return authorization date. DO NOT return products to our factory without our prior consent. C.O.D. shipments will be refused; please prepay shipping charges.**

**LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO: MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN INVACARE DEALER, SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE.**

**THE FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE.**

**IT SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESSED WARRANTY PROVIDED HEREIN AND THE REMEDY FOR VIOLATIONS OF ANY IMPLIED WARRANTY SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.**

**THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.**

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© 2001 Invacare Corporation
Invacare and "Yes, you can" are trademarks of Invacare Corporation.
**Invacare Corporation** [www.invacare.com](http://www.invacare.com)