PERSONAL SEAT AND CUSHION RIGIDIZER

Assembly, Installation and Operating Instructions

NOTE: Check all parts for shipping damage and test before use. In case of damage, DO NOT use. Contact your equipment supplier for further instruction.

SAFETY SUMMARY

The following recommendations are made for the safe installation of the Personal Seat and Cushion Rigidizer:

WARNINGS/CAUTIONS notices apply to hazards or unsafe practices which could result in personal injury and/or property damage.

WARNING

DO NOT install this equipment without first reading and understanding this instruction sheet. If you are unable to understand these instructions, contact a healthcare professional, dealer or technical personnel if applicable before attempting to install this equipment - otherwise, injury or damage may occur.

The best way to avoid problems related to pressure sores is to understand their causes and and your role in a skin management program.

Your therapist and physician should be consulted if you have questions regarding individual limitations and needs.

All cushions used for the prevention or treatment of decubitus ulcers (pressure sores) should be selected carefully. Working with your your therapist, and physician is the best way to assure that a cushion choice matches your individual needs.

As the needs of the individual become more complex, the cushion evaluation becomes more important.

Skin condition should be checked very frequently after the provision of any new cushion.

INTRODUCTION

The Personal Seat cushion is designed to provide pelvic stability, leg positioning and pressure management for the user. The cover is made from a water repellant fabric to protect the foam and the foam is coated with a water repellant coating to provide additional protection.

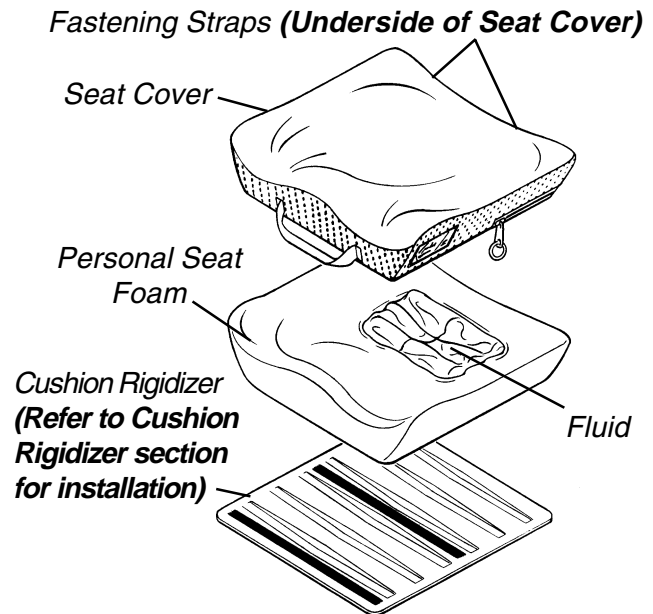
FIGURE 1 - PERSONAL SEAT

ATTACHING THE CUSHION TO THE SEATING SURFACE

The Personal Seat attaches to the wheelchair seat upholstery using the hook and loop fasteners on the bottom of the cushion cover.

NOTE: The zipper on the cushion should face rearward.

Wheelchair Seating Surface

1. Determine whether the wheelchair seating surface has hook or loop attachment strips.
2. If the wheelchair has loop attachment strips, verify that the double sided hook strips are securely attached to the loop attachment strips on the bottom of the cushion.
3. Align the front edge of the cushion with the front edge of the seating surface.
4. Secure the cushion to the seating surface, making sure that the double sided hook is securely attached to the strips of loop on the cushion cover and the chair.
5. If the wheelchair has hook attachment strips, remove the double sided hook strips from the cushion.
6. Align the front edge of the cushion with the front edge of the seating surface.
7. Secure the cushion to the seating surface, making sure that the strips of loop on the cushion cover are securely attached to the strips of hook on the chair.

8. If the wheelchair seating surface has neither hook or loop attachment strips, remove the double sided hook from the bottom of the cover.
9. Align the front edge of the cushion with the front edge of the seating surface.
10. Place the cushion on the wheelchair seating surface.

WARNING

Make sure the hook and loop fasteners are securely attached to the seating surface and the cushion before using. If the cushion is not secured to the wheelchair seating surface, exercise caution when transferring in and out of the wheelchair.

USE AND MAINTENANCE OF THE PERSONAL SEAT

Installation of the Cover

The cover is properly installed when the zipper is at the back of the cushion. The back of the cushion is where the fluid pad is located. In addition, the back surface of the cushion is marked, "BACK".

Installation of the Fluid Pad

The pad can be easily and properly installed in the recess by matching the loop strips on the bottom of the pad with the hook strips in the cushion recess. The pad is designed to be larger than the recess so it will be necessary to position the pad with accordion folds so that the entire pad will be located inside the recess.

Checking Fluid Levels:

NOTE: Regular monitoring means "daily" monitoring for high risk users, "weekly" for medium risk users and "bi-weekly" for low risk users.

Regularly monitor to determine if there is sufficient fluid to properly "float" bony prominences, trial fit the cushion for fifteen (15) minutes. Then, examine the fluid pad for areas of incomplete coverage, as well as examining the user's skin for red pressure markings. Any area from which the fluid has migrated, leaving less than 1/8 inch (3mm) coverage, may require additional fluid.

NOTE: Keep in mind that areas where there is no fluid have 3/4 inch (19mm) of soft foam underpad to help keep the bony prominence from bottoming out.

Temperature Considerations:

WARNING

Storage of the cushion in areas of extreme cold or extreme heat can cause the fluid to reach a temperature which could damage the skin of the user.

If the cushion has been stored below 41°F (5°C) or above 90°F (32°C), the cushion fluid pad should be allowed to reach normal room temperature before use. The removable fluid pad can be immersed in warm or cold water to speed the temperature change. (Water temperature should not exceed 50°F (122°C).

Temperatures in a closed automobile can exceed 201°F (94°C) when parked in direct sunlight. If your cushion is exposed to increased temperatures, check the fluid pad temperature before actual use and verify that the fluid in the pad remains uniform. Refer to **INSPECTION AND CLEANING** in this instruction sheet.

Maintaining the Cover

The Personal Seat cover is fabricated with water repellent materials. The top and sides are fabricated with a patented waterproof stretch fabric. The bottom is fabricated from a special waterproof heavyweight fabric. In the event of incontinence, some leakage may occur at the seam stitching or at the zipper, so the water repellent coating should be cleaned. The cover should be machine washed in warm water and let hang dry. Refer to **INSPECTION AND CLEANING** in this instruction sheet.

Inspection and Cleaning

1. In order to ensure that the cushion is working as designed, inspection and cleaning of the cushion on a regular basis is necessary.

WARNING

DO NOT continue to use this product if any of the following problems as described are discovered. Corrective maintenance can be performed at or arranged through your equipment supplier.

2. Visually inspect all parts weekly, including hardware, upholstery materials, foams (if accessible), and plastics for deformation, corrosion, breakage, wear and/or compression.

3. **Fluid Pad Inspection:**

NOTE: Fluid bag contents are non-toxic. The bag will not break or leak under normal use. If leakage is detected, replace bag IMMEDIATELY.

As part of the regular inspection of the cushion the fluid pad should be taken out of the recess in the cushion and inspected also. If the cushion is exposed to temperatures of 54°C (129°F) or greater, this inspection should be performed immediately. The fluid in the pad should be uniformly white in color and homogeneous. If the fluid has areas that are translucent, it has probably separated into its components and needs to be remixed. Remixing is accomplished by kneading the pad until the fluid is uniformly white in color. Reinstall the pad in the recess according to **INSTALLATION OF THE FLUID PAD** in this instruction sheet.

4. Weekly cleaning is recommended.

CAUTION

MACHINE WASH seat cover in cold water on gentle cycle using a mild detergent. **DO NOT** use fabric softeners or bleach. **DO NOT** machine dry. **Air dry ONLY.** **DO NOT** dry in sun.

5. To clean the water repellent foam coating, use a soft cloth and wipe clean with warm water and a mild soap solution. Rinse off and dry.

CAUTION

The cover is designed to protect the foam against a user's incontinence and to provide fire retardancy so the cushion must not be used without its cover. If the cover is torn, it must be replaced.

CUSHION RIGIDIZER

The Cushion Rigidizer is designed to decrease the hammocking effect of sling upholstery, providing a firm, stable base of support.

INSTALLING THE CUSHION RIGIDIZER (FIGURES 1, 2 & 3)

1. Remove seat cushion assembly from chair.
2. Open seat cover and remove the Personal Seat Foam (FIGURE 1).

NOTE: DO NOT REMOVE the protective paper for the adhesive until you have marked the placement area on the Personal Seat Foam. Be sure of placement area as the Cushion Rigidizer will not re-adhere securely if installed and removed.

3. Place the surface of the Cushion Rigidizer with adhesive tape against the bottom of Personal Seat cushion (FIGURE 2).

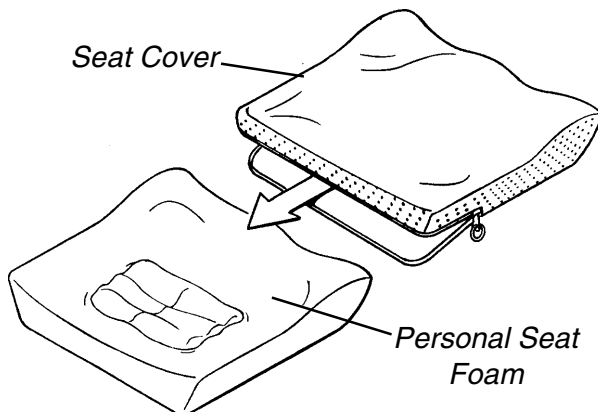
FIGURE 1 - REMOVE PERSONAL SEAT FOAM FROM SEAT COVER

WARNING

The **FRONT** edge of the Rigidizer **MUST** be parallel with the **FRONT** edge of the Personal Seat Foam and 1 inch (1.25cm) behind it so that the user's legs will not contact the front edge of the Rigidizer causing injury to the user or damage to the Rigidizer.

4. Place Cushion Rigidizer 1 inch (1.25cm) from **FRONT** edge of the Personal Seat Foam while centering it on the left and right sides (FIGURE 2).

CAUTION

DO NOT cut or rip the foam coating.

5. Outline the Personal Seat Foam with a marker, pen or pencil using the Cushion Rigidizer as your template to mark the placement area.
6. After marking the placement area, remove the protective paper from the adhesive on the the Cushion Rigidizer and install Rigidizer onto the bottom (**under-side**) of the Personal Seat Foam (FIGURE 2).
7. Press down firmly to adhere the Cushion Rigidizer to the Personal Seat Foam (FIGURE 2).

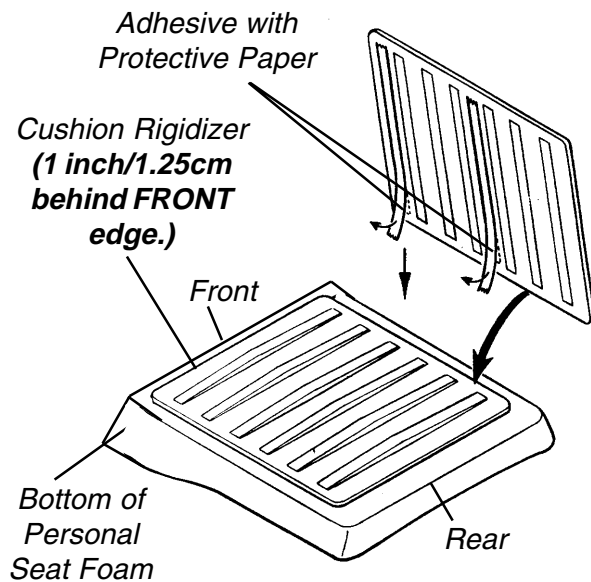
FIGURE 2 - INSTALL RIGIDIZER

8. Re-install the Personal Seat Foam into the seat cover and use the hook and loop fasteners to secure seat cover to chair.

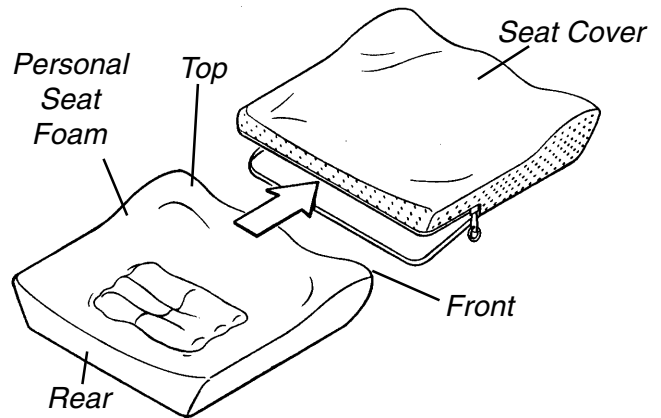
FIGURE 3 - RE-INSTALL CUSHION INTO SEAT COVER

UNITED KINGDOM LIMITED WARRANTY

This warranty is extended only to the original purchaser of our products and does not affect statutory rights.

Invacare UK warrants the foam cushion to be free from defects for a period of two (2) years from the date of purchase. The cushion cover is warranted for a period of ninety (90) days from date of purchase. With regard to the original purchaser/user only, Invacare UK warrants its metal fabricated hardware components to be free from defects in materials and workmanship for the lifetime of the product except any wood, foam plastic or upholstered components. If within such warranty period the product shall be proven to Invacare UK's / Invacare Service Representative reasonable satisfaction to be defective, it shall be repaired or replaced, at Invacare UK's / Invacare Service Representative option; Invacare UK's / Invacare Service Representative sole obligation, and your exclusive remedy under this warranty being limited to such repair or replacement.

For warranty service, please contact Invacare UK's Service Department / Invacare Service Representative during normal working hours. Upon receiving notice of an alleged defect in a product, Invacare UK / Invacare Service Representative will issue a serialized return authorization. The product shall then be returned to either Invacare plant or service center as specified by Invacare UK / Invacare Service Representative. Defective component part(s) must be returned for warranty inspection using the serial number as identification within thirty (30) days of return authorization date. DO NOT return products to our factory without prior consent.

LIMITATIONS AND EXCLUSIONS: THE FOREGOING WARRANTY DOES NOT COVER NORMAL WEAR AND TEAR AND SHALL NOT APPLY TO PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE, COMMERCIAL OR INSTITUTIONAL USE, PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT (INCLUDING, BUT NOT LIMITED TO, MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS) OR TO PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE UK / INVACARE SERVICE REPRESENTATIVE.

UNITED STATES LIMITED WARRANTY

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants its product to be free from defects in materials and workmanship for two (2) years on back and 90 days on cover of use by original purchaser. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the next page. Provide dealer's name, address, model number, and the date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

Invacare Corporation will issue a return authorization. The defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty (30) days of return authorization date. DO NOT return products to our factory without our prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECT TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO: MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN AUTHORIZED INVACARE DEALER, SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE.

THE FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE.

IT SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESSED WARRANTY PROVIDED HEREIN AND THE REMEDY FOR VIOLATIONS OF ANY IMPLIED WARRANTY SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

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© 2000 Invacare Corporation
Invacare and "Yes, you can" are trademarks of Invacare Corporation.