

### SAVE THESE INSTRUCTIONS

*NOTE: Check all parts for shipping damage before using. In case of damage, DO NOT use the equipment. Contact the Dealer/Carrier for further instructions.*

### SAFETY SUMMARY

The following recommendations are made for the safe use of the Back Extension:

#### GENERAL WARNING

DO NOT install or use this equipment without first reading and understanding these instructions. If you are unable to understand the Warnings, Cautions or Instructions, contact a healthcare professional, dealer or technical personnel before attempting to install this equipment - otherwise, injury or damage may occur.

#### INSTALLATION WARNING

After ANY adjustments, repair or service and BEFORE use, make sure that all attaching hardware is tightened securely.

This kit includes the following:

ITEM	QUANTITY
Back Extension Brackets	2
Back Extension	1
Mounting Screws	2
Washers	2

### INSTALLING THE BACK EXTENSION BRACKETS (FIGURE 1)

*NOTE: Make sure that the wheelchair is unfolded on a level surface, with the wheel locks engaged.*

1. Remove the upper two (2) mounting screws and washers securing the back upholstery. Refer to DETAIL "A" in FIGURE 1.
2. Align the upper and lower mounting holes in the back extension brackets with the upper two (2) mounting holes in the back upholstery and back canes. Refer to DETAIL "B" in FIGURE 1.

*NOTE: The back extension brackets accommodate several models of chairs, replace only the mounting screws that are removed. An extra set of screws is included for instances of loss or damage.*

3. Install the two (2) existing mounting screws and washers as shown in DETAIL "B". Tighten securely.
4. Repeat for opposite side.

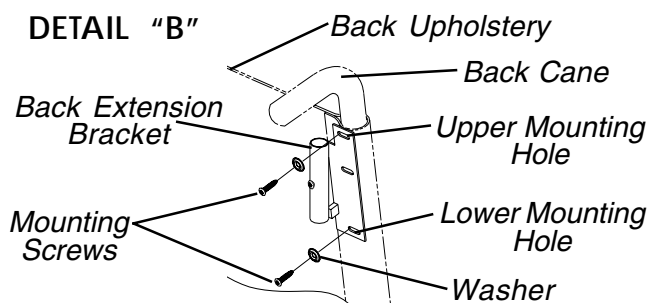
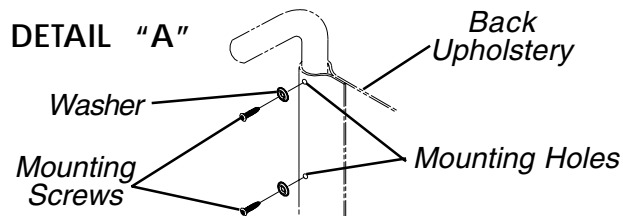


FIGURE 1 - INSTALLING THE BACK EXTENSION BRACKETS

### INSTALLING THE BACK EXTENSION (FIGURE 2)

1. Align the back extension insert tubes with the back extension brackets on the wheelchair as shown in FIGURE 2.
- NOTE: Fastening strips should face the rear of the wheelchair.*
2. Seat the insert tubes firmly into position in the back extension brackets.

*NOTE: Alignment notches should sit firmly on the alignment screws. Insert tubes should NOT rotate in back extension brackets.*

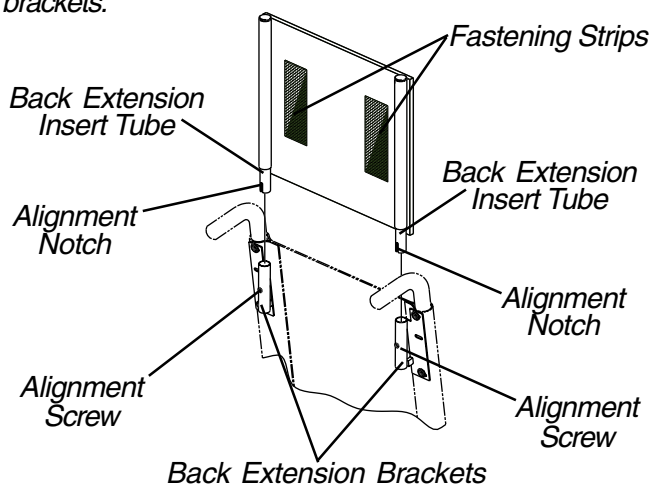


FIGURE 2 - INSTALLING THE BACK EXTENSION

Invacare Corporation [www.invacare.com](http://www.invacare.com)



Yes, you can.™

**USA**  
One Invacare Way  
Elyria, Ohio USA  
44036-2125  
800-333-6900

**Canada**  
5970 Chedworth Way  
Mississauga, Ontario  
L5R 3T9, Canada  
905-890-8838

Invacare and "Yes, you can" are trademarks of Invacare Corporation.  
© 2000 Invacare Corporation  
Form No. 00-107 Part No. 1098351 Rev A (1) 6/00

## Free Manuals Download Website

<http://myh66.com>

<http://usermanuals.us>

<http://www.somanuals.com>

<http://www.4manuals.cc>

<http://www.manual-lib.com>

<http://www.404manual.com>

<http://www.luxmanual.com>

<http://aubethermostatmanual.com>

Golf course search by state

<http://golfingnear.com>

Email search by domain

<http://emailbydomain.com>

Auto manuals search

<http://auto.somanuals.com>

TV manuals search

<http://tv.somanuals.com>