Invacare® Rio™ H605
Bath Lift

User Manual
Manuel d'utilisation

Yes, you can.®

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© 2011 Invacare Corporation. All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by [™] and [®]. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.

User Manual

Invacare® Rio™ H605 Bathlift

DEALER: This manual MUST be given to the user of the product.

USER: BEFORE using this product, read this manual and save for future reference.

1	GENERAL	4
	Symbols	4
	Specifications and Weight Limitation	4
	Technical Data	5
2	SAFETY	6
	Scope of Delivery.....	6
	Intended Use.....	6
	General Warnings.....	7
	Instructions on electromagnetic compatibility.....	7
	Battery Warnings.....	8
3	SETUP	9
	Recharging the Battery	9
	Assembly	10
	Setting Up the Bathlift in the Bathtub	11
	Removing the Bathlift from the Bathtub.....	12
4	USAGE	13
5	MAINTENANCE	15
6	WARRANTY	17
	Limited Warranty - North America.....	17
	Customer Service and Warranty Information	18

I General

I.1 Symbols

Signal words are used in this manual and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.

WARNING

Warning indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

CAUTION

Caution indicates a potentially hazardous situation which, if not avoided, may result in property damage or minor injury or both.

Gives useful tips, recommendations and information for efficient, trouble-free use.

I.2 Service Life

The expected service life of this product is five years when used in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

Maintenance should be done at least every six months.

I.3 Specifications and Weight Limitation

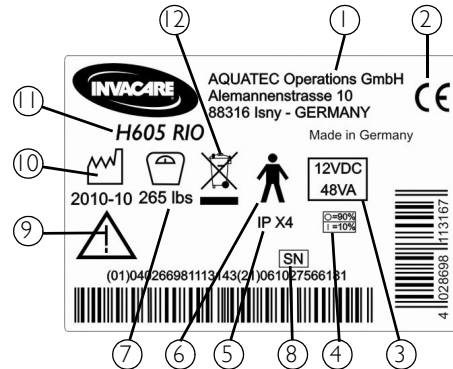
WARNING


ALWAYS observe the weight limit on the labeling of your product. Check that all labels are present and legible. Replace if necessary.

The weight limitation of the bath lift is 265 lb (120 kg).

I.4 Product Labeling

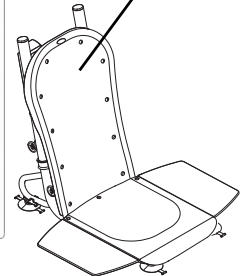The product label is located on the rear of the backrest

1	Manufacturer's Data	7	Load Capacity
2	Conformity Marking	8	Serial Number
3	Nominal voltage/nominal consumption	9	Follow the operating instructions
4	duty cycle (max. 10%, max. 2 min/18 min)	10	Date of manufacture
5	Protection class	11	Type designation
6	Application part of type B	12	Disposal instructions

1.5 Technical Data


Overall Depth (lower end position)	26.8 in	680 mm
Total Height (upper end position)	40.6 in	1030 mm
Height of Seat	2.4 - 17.3 in	60 mm - 440 mm
Width of Seat (without/with flaps)	14.6 in / 28.3 in	370 mm / 720 mm
Width of Backrest	13.8-15.7 in	350-400 mm
Height of Backrest	24.2 in	615 mm
Depth of Seat	17.3 in	440 mm
Angle of Seat	105°	
Total Weight (without battery)	20.9 lb	9.5 kg
Weight of battery	3.5 lb	1.6 kg
Battery type	Lead-acid gel 12 V 3 Ah	
Charger input / output	120 V AC; 50 Hz / 12 V DC; 0.4 A	
Charging time	max 5 h	
Drive protection class	IP66W	
Pendant control protection class	IP67	
Sound power level	52dB(A)	
Actuating force for pendant control buttons	< 5 N	

2 Safety

2.1 Scope of Delivery


- i** Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact carrier/dealer for further instruction.

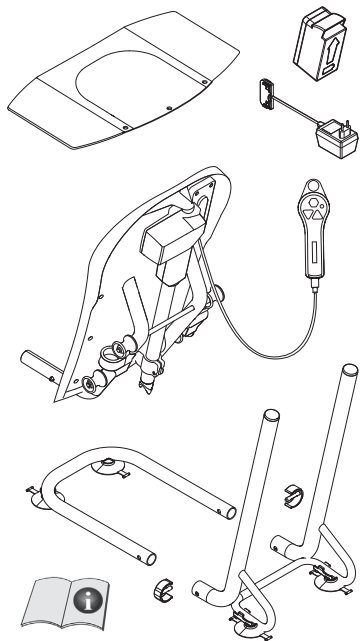If possible, keep the original packaging for transportation at a later date.

2.2 Intended Use

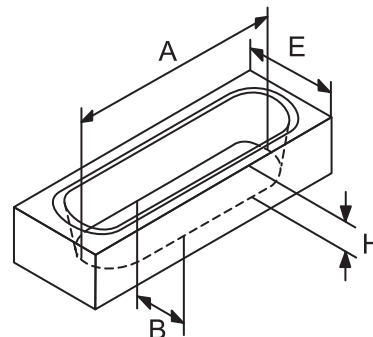The bathlift is intended for lowering a person into a bathtub and lifting a person out of a bathtub only. All other use can cause risk of injury. Only use the bathlift in bathtubs of the specified size:

A	Bathtub length at bottom	min. 31.5 in	min. 80 cm
B	Bathtub width at bottom (inside)	min. 14.2 in	min. 36 cm
H	Bathtub height inside (without optional height adapters)	max. 17.3 in	max. 44 cm
	Bathtub height inside (with optional height adapters)	max. 18.9 in	max. 48 cm
E	Bathtub width outside	max. 27.6 in	max. 70 cm

2.3 General Warnings

WARNING

DO NOT USE THIS PRODUCT OR ANY AVAILABLE OPTIONAL EQUIPMENT WITHOUT FIRST COMPLETELY READING AND UNDERSTANDING THESE INSTRUCTIONS AND ANY ADDITIONAL INSTRUCTIONAL MATERIAL SUCH AS OWNER'S MANUALS, SERVICE MANUALS OR INSTRUCTION SHEETS SUPPLIED WITH THIS PRODUCT OR OPTIONAL EQUIPMENT. IF YOU ARE UNABLE TO UNDERSTAND THE WARNINGS, CAUTIONS OR INSTRUCTIONS, CONTACT A HEALTHCARE PROFESSIONAL, DEALER OR TECHNICAL PERSONNEL BEFORE ATTEMPTING TO USE THIS EQUIPMENT - OTHERWISE, INJURY OR DAMAGE MAY OCCUR.

ACCESSORIES WARNING

Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

WARNING

Risk of Injury

If your state of health does not allow you to bathe on your own, ALWAYS take a bath under supervision. DO NOT make any unauthorized alterations or conversions to the device.

CAUTION

Risk of Damage

DO NOT expose the product to direct sunlight. Only transport or store the product in dry spaces at temperatures between 0°C and 40°C, an air pressure between 860 and 1060 hPa and an air humidity between 30% and 75%.

Note the information on the product label.

Only load the bathlift up to the specified maximum user weight. If you overload the bathlift, it could result in damage to the device.

2.4 Instructions on electromagnetic compatibility

Even if all applicable EMC guidelines and standards are complied with, it is possible that the bathlift could be impaired by other electrical devices (e.g. mobile telephones, CB radio devices, portable radios) or that the lift could impair these devices. If you note any such interference, switch off the other device and/or the bathlift. If you carry other electrically operated medical products on your person and these products cause interference or are impaired, switch off the bathlift and contact your specialist dealer. For more information about the electromagnetic characteristics, see the Technical data section.

2 SAFETY

2.5 Battery Warnings

WARNING

Risk of Injury

Only use original batteries in the device (Invacare H605 Rio Battery).

Only charge the battery with the original charger (Invacare H605 Rio plug-in charger).

Replace batteries with a greatly reduced output (range) in good time.

Do not open or alter the power adapter or the battery.

Before cleaning the charger, always disconnect the power adapter from the power socket.

Never throw the battery into fire and do not store the battery near fire.

If the battery becomes damaged and diluted sulphuric acid comes into contact with eyes, wash eyes immediately with copious amounts of water and see a doctor without delay.

If the battery becomes damaged and diluted sulphuric acid comes into contact with skin or clothing, thoroughly wash the affected area immediately with water.

CAUTION

Risk of Damage

Do not touch the battery's contacts with electrically conductive objects as this could damage the battery (short circuit).

3 Setup

3.1 Recharging the Battery

WARNING

Risk of Injury

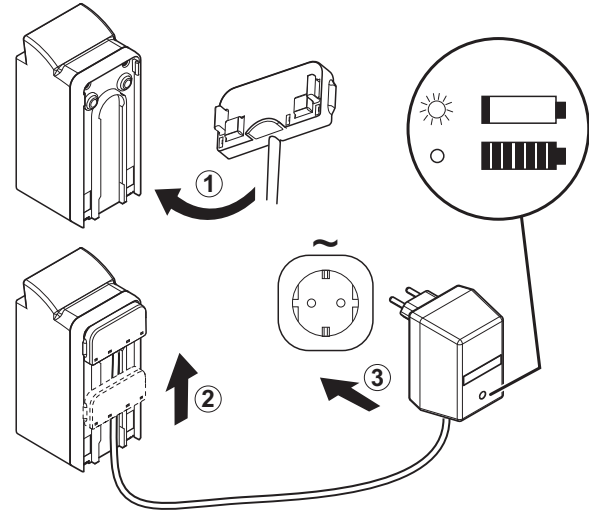
Never touch the battery's charge contacts.

CAUTION

Risk of Damage



When charging the battery, pay careful attention to the sequence of steps.

- Recharge the battery after each use of the device.
 - Only charge the battery in a dry room, never in the bathroom.
 - When charging, ensure that there is sufficient ventilation in the room.
 - The charging time is a maximum of 5 hours.
 - You cannot overcharge the battery; the charger switches off automatically when the battery is fully charged.
 - If not using the device, charge the battery at least once a month.
 - After charging the battery, disconnect the charger and the battery in reverse order.
1. Remove the battery from the bathlift. Turn the battery over to expose the slots for the battery charger adapter.
 2. Slide the battery charger adapter up into the battery.
 3. Plug the battery charger adapter into a wall outlet.

3 SETUP

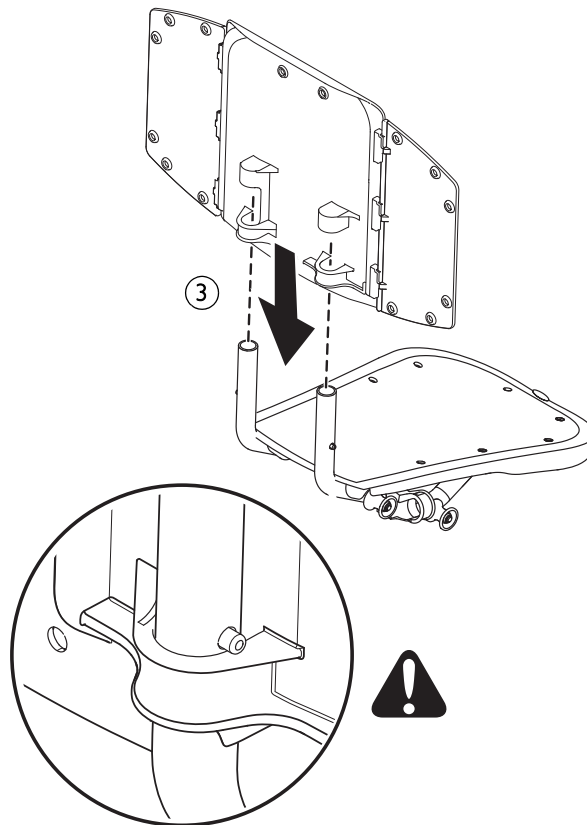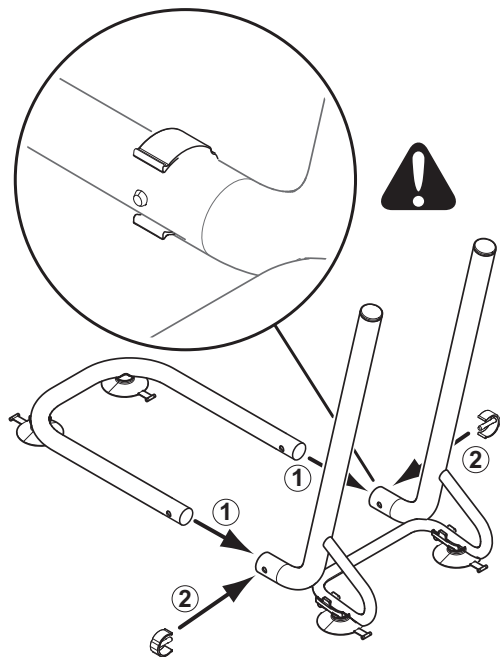3.2 Assembly

3.3 Setting Up the Bathlift in the Bathtub

CAUTION

Risk of Tipping

Check that the device is seated correctly. When the device is loaded, the suction cups should not come loose.

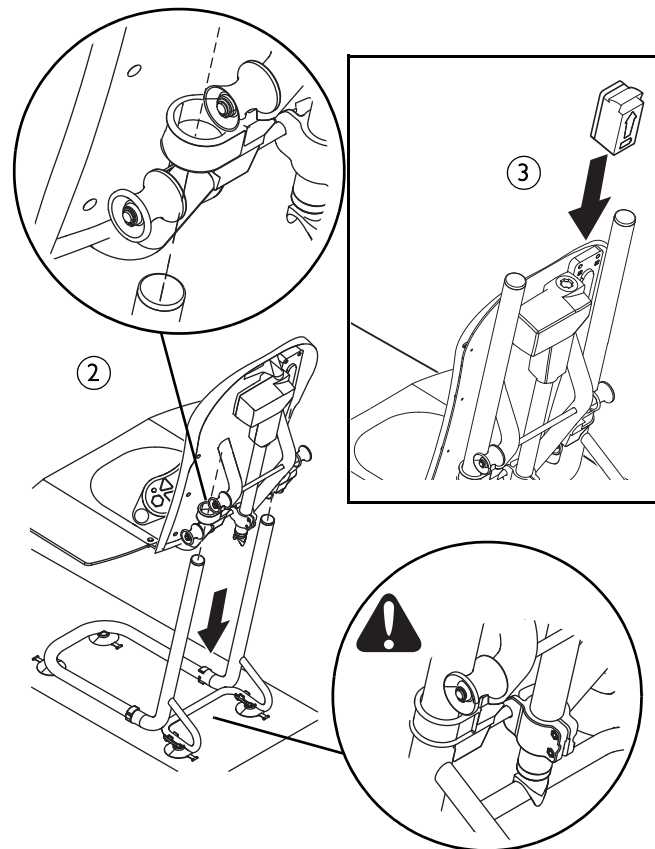
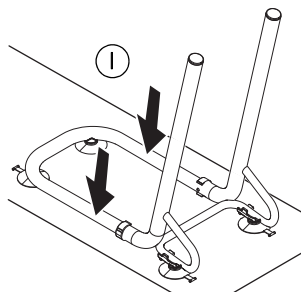
Risk of Pinching

Handle the backrest from the sides only.

Risk of Damage

Do not move the basic frame again after you have set it up in the bathtub.

Before setting up the device, clean the bathtub and the suction cups.

3 SETUP

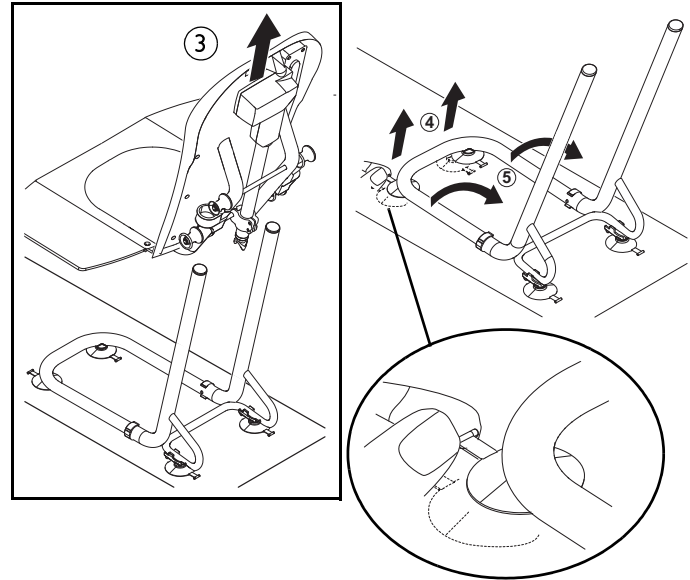
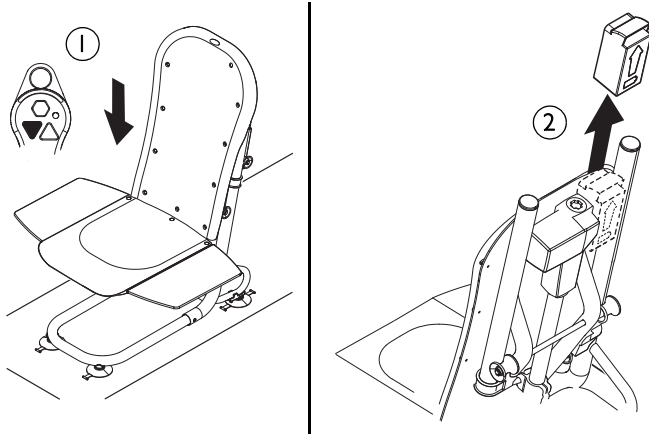
3.4 Removing the Bathlift from the Bathtub

CAUTION

Risk of Damage

Remove the battery before lifting the bathlift parts out of the bathtub.

- Drain the water from the bathtub.
- Rinse away any residue with fresh water.

4 Usage

4.1 Operation

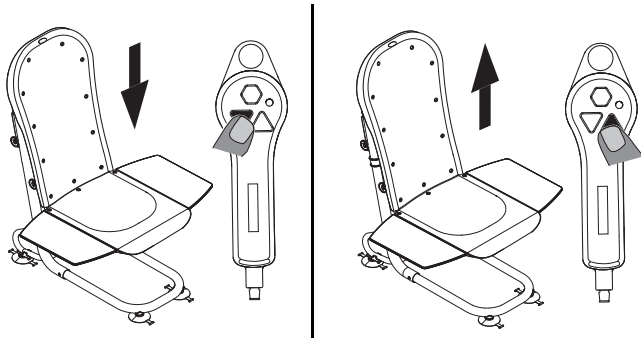
WARNING

Risk of Injury


Check for any external damage each time you use the device. If you identify any damage, contact a qualified technician.

Lowering and Lifting

- If the LED on the hand control unit lights up RED, the battery is not sufficiently charged. The seat does not lower, but can be lifted.
- The bathlift stops as soon as the button is released.

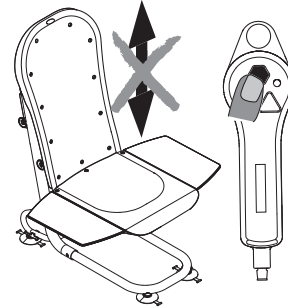Emergency Stop Button

- All seat movement is interrupted immediately when the emergency stop button is pressed.

4 USAGE

4.2 Using the Bathlift

CAUTION

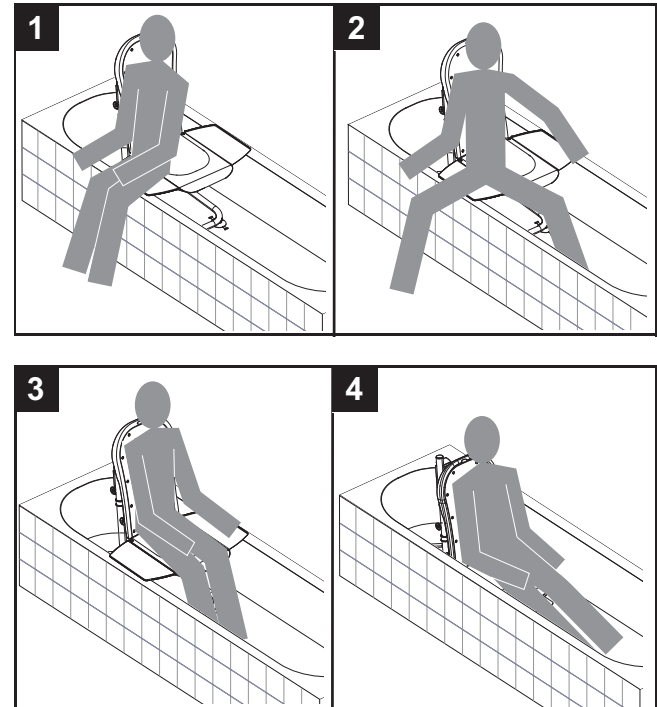
Risk of Injury

Keep long hair well clear of the guide tubes when lowering the seat. If necessary, use a hair net.

DO NOT hold the guide tubes when lowering or raising the seat. Otherwise, pinching could occur.

When you sit down on the seat, the two side flaps must be lying flat on the edge of the bathtub. Lift or lower the bathlift or alter the installation position to achieve this.

To get out of the bath, proceed in the reverse order.

5 Maintenance

5.1 Care instructions

Basic frame and seat frame

- Keep the bathlift free of dirt and clean it at regular intervals.
- Do not use any aggressive (pH value under 4.5 or over 8.5) or abrasive cleaning agents.
- Rinse the bathlift thoroughly after each bath.

Charger

- Disconnect the power adapter from the power socket before cleaning.
- Clean the charger with a dry cloth.

5.2 Maintenance/periodic checks

- If operated and cared for properly, the bathlift is maintenance-free.
- To maintain your bathlift's functional safety, you should check it for technical safety defects before moving it to a new bath or at least every 24 months.
- Note the service instructions. These instructions can be provided on request.
- Any repairs necessary should be carried out by a qualified technician with original replacement parts only. If third-party products are used, responsibility for the replacement is transferred to the implementing party.

5.3 Disposal

When the product can no longer be used, the product, accessories and the packaging materials should be sorted correctly.

This product contains a rechargeable lead battery. Old batteries may contain hazardous materials that could damage the environment or health.

Please be environmentally responsible and recycle this product through your recycling facility at its end of life.

5.4 Reuse

WARNING: RISK OF INJURY

Always have a dealer inspect the product for damage before transferring the product to a different user. If any damage is found, **DO NOT** use the product. Otherwise injury may occur.

This product is suitable for reuse after it has been reconditioned. Follow the instructions in [Maintenance](#) on page 15 when reconditioning the product.

5.5 Transport

- Before transporting, disassemble the device into its component parts, as described in the “Scope of delivery” section.
- Do not transport the device with the battery inserted.
- If possible, transport the device in its original box.

5.6 Storage

- Allow the device to dry fully in the air.
- Store the battery separately from the seat frame.
- If not using the device, recharge the battery at least once a month.

5 MAINTENANCE

5.7 Troubleshooting

FAULT	POSSIBLE CAUSE	REMEDY
No motor sound audible/the lift does not move	Battery is empty or not sufficiently charged	Recharge battery fully
	Battery contacts are dirty	Clean battery contacts
	Battery is not inserted correctly	Insert battery correctly
	Cable for hand control unit is damaged.	Contact a qualified technician.
Lift can only be raised	Battery not sufficiently charged	Recharge battery fully
Lift stops when rising	Maximum load exceeded	Observe maximum load
LED on power socket does not light up during charging	Battery defective	Replace battery
	Charger not inserted correctly	Insert charger correctly
	Charger defective	Replace charger
	Battery is already fully charged	-----
Device slips	Suction cups damaged	Replace suction cups
LED on hand control unit lights up during operation	Battery not sufficiently charged	Recharge battery fully
	Electronics defective	Contact a qualified technician.

6 Warranty

6.1 Limited Warranty - North America

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants this product to be free from defects in materials and workmanship for three years for the original purchaser. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back page. Provide dealer's name, address, model number, date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

Invacare Corporation will issue a return authorization. The defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty (30) days of return authorization date. DO NOT return products to our factory without our prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO: MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN INVACARE DEALER, SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE.

THE FOREGOING EXPRESS WARRANTY IS EXCLUSIVE AND IN LIEU OF ANY OTHER EXPRESS WARRANTIES WHATSOEVER, WHETHER EXPRESS OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

6 WARRANTY

6.2 Customer Service and Warranty Information

Canada	Invacare Canada, Inc., 570 Matheson Blvd. E. Unit 8, Mississauga, Ontario, L4Z 4G4 Tel: (905) 890-8300, Fax: (905) 890-5244, www.invacare.ca
United States	Invacare Corporation, One Invacare Way, Elyria, Ohio 44036-2125 Tel: (800) 333-6900, www.invacare.com

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Aquatec Operations GmbH,