SAFETY SUMMARY

The following recommendations are made for the safe installation of the Commodes:

WARNING

DO NOT install this equipment without first reading and understanding this instruction sheet. If you are unable to understand these instructions, contact a healthcare professional, dealer or technical personnel if applicable before attempting to install this equipment - otherwise, injury or damage may occur.

Check all parts for shipping damage. In case of shipping damage, DO NOT use. Contact Dealer/Carrier for further instruction.

Snap buttons MUST be fully engaged and aligned in the proper height adjustment hole of each leg before use. This ensures that the legs are securely locked and an even height adjustment is achieved.

Make certain ALL screws, nuts and/or bolts are tight at all times.

Inspect rubber tips on the leg extensions for rips, wear or if they are missing. Replace them immediately if any of these conditions exist.

Leg extensions with rubber tips MUST be in contact with the floor at all times.

Users with limited physical capabilities should be supervised or assisted when using the commode.

The Commode Model No. 9630E has a weight limitation of 135 kg.

LEG HEIGHT ADJUSTMENT (FIGURE 1)

1. Select the desired height for the commode by aligning the snap button of each leg extension with the appropriate adjustment hole in the leg frame.
2. Make certain that the snap buttons are fully engaged into the adjustment hole of each leg frame and are at the same height before using.

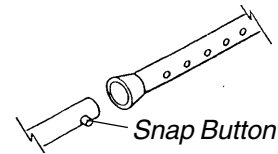
FIGURE 1 - LEG HEIGHT ADJUSTMENT

INSTALLING AND REMOVING THE COMMODE SEAT AND LID (FIGURE 2)

CAUTION

Before removing/installing seat and lid, allow seat/ring to reach room temperature if exposed to cold. This will help prevent the seat clamps from breaking when being removed/installed onto commode back frame rail.

1. Align the seat clamps of the toilet seat onto the back frame rail. **(Each clamp should sit outside the pail holders)**. Push down on seat until it clamps firmly into place.
2. Repeat the same process for the lid.
3. When removing the seat or lid, remove the lid first and then the seat. To remove, grasp the back corner of the lid and/or seat and lift up firmly.

CARE: Use a damp cloth and soap to clean.

NOTE: To make raising or lowering the seat/lid easier, you may lubricate the commode back frame rail with a very small amount of petroleum jelly or furniture polish.

WARNING

Toilet seat MUST be DOWN before sitting on the commode. Pail holders are NOT constructed to support the weight of an individual, ONLY the pail and its contents.

CAUTION

Back Tube Models -9630E; wing nuts must be inspected regularly for proper tightness. (Refer to Item No. 6 in Parts List).

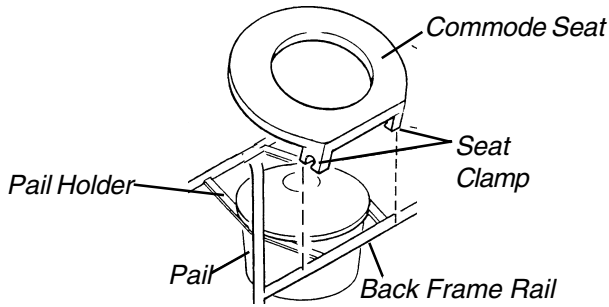
FIGURE 2 - INSTALLING AND REMOVING THE COMMODE SEAT AND LID

REPLACING THE COMMODE ARMRESTS (FIGURE 3)

1. Remove the Phillips Tap Screws, two (2) each from the underside of the arm tube(s).
2. Lift armrest(s) from the arm tube(s).
3. Install/reinstall the armrest(s).


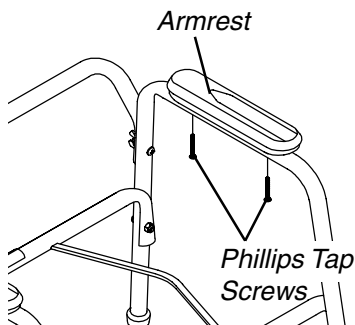
FIGURE 3 - REPLACING THE COMMODE ARMRESTS

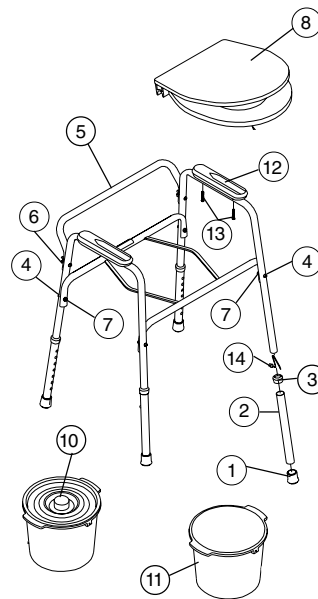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© 2000 Invacare Corporation
Invacare and "Yes, you can" are trademarks of Invacare Corporation.
Invacare Corporation www.invacare.com