

LUMEX®

For The Quality Of Life®

5180A, 5181A, 5182A, 5183A, 5184A, 5185W, 5185BL, 5186B: WOOD CANE ASSEMBLY & OPERATION INSTRUCTIONS

PLEASE SAVE THESE INSTRUCTIONS FOR FUTURE USE

Caution: This Product Contains Natural Rubber Latex Which May Cause Allergic Reactions

- △ **Warning:** Please read all instructions before use. Do not use this product without proper instruction from a healthcare professional.
- △ **Warning:** Wood Canes 5180A, 5181A, 5182A, 5183A, 5184A, 5185W and 5185BL have a maximum user weight limitation of 250 lbs.
- △ **Warning:** Wood Cane 5186B has a maximum user weight limitation of 500 lbs.
- △ **Warning:** Lumex assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.
- △ **Warning:** If components are damaged or missing, contact your dealer immediately. DO NOT use substitute parts.
- △ **Warning:** Before use, ensure that cane tip is properly seated and straight.
- △ **Warning:** Always use caution when using this product on wet or on slippery surfaces.
- △ **Warning:** Inspect Wood Cane weekly for proper function and wear. If any component is not in proper working order or appears worn, contact your Lumex dealer for repair and/or appropriate replacement parts.



Graham-Field Health Products

2935 Northeast Parkway • Atlanta, Georgia 30360

Cane Height

See picture at right:

- Standard-height wooden canes (5180A, 5181A, 5182A, 5183A, 5185W and 5185BL) are designed for users up to 6'2" tall and up to 250 lbs. in weight.
- The extra long wooden cane (5184A) is designed for users over 6'2" tall and up to 250 lbs. in weight.
- The bariatric cane (5186B) is designed for users up to 500 lbs. in weight.
- Please consult a healthcare professional for the appropriate cane height.

Adjust Cane Height

Before using, your cane may be cut to the appropriate height by an authorized Lumex dealer. Please consult a healthcare professional for the appropriate cane height. To cut cane to proper height:

1. Remove the tip which covers the end.
2. Cut the end of the cane at the appropriate height for the user.
3. Ensure that cut is flat and smooth, so that cane end will rest in a stable position when in contact with the floor.
4. Install cane tip onto end of cane.


⚠ **Warning:** Before use, ensure that cane tip is properly seated and straight.

Maintenance

- Wipe cane weekly with a damp cloth.
- Never use abrasive cleansers or scouring pads on any part of cane.
- Replace cane tips as needed. To guarantee the best service from your Wood Cane, always specify Lumex replacement parts. They are available from your Lumex dealer.

Warranty

Lifetime limited warranty. For additional warranty information, please contact your Lumex dealer.

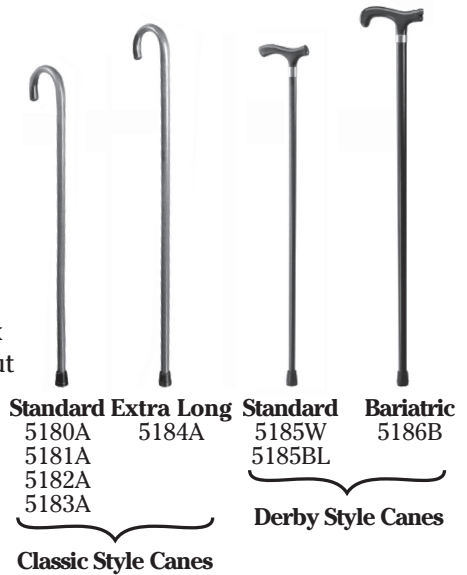

**Graham-Field
Health Products**
2935 Northeast Parkway
Atlanta, Georgia 3360

Free Manuals Download Website

<http://myh66.com>

<http://usermanuals.us>

<http://www.somanuals.com>

<http://www.4manuals.cc>

<http://www.manual-lib.com>

<http://www.404manual.com>

<http://www.luxmanual.com>

<http://aubethermostatmanual.com>

Golf course search by state

<http://golfingnear.com>

Email search by domain

<http://emailbydomain.com>

Auto manuals search

<http://auto.somanuals.com>

TV manuals search

<http://tv.somanuals.com>