#1 CHOICE OF HOSPITALS AND MOTHERS

Symphony[®]

BREASTPUMP

BOMBA DE LACTANCIA

TIRE-LAIT

Instructions for use

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Mode d'emploi

Table of Contents - English

Important Safeguards	1
Uses	2
Cleaning & Sterilization	2
Hygiene	2
Preparation	2
Hospital Sterilization	3
Home Sanitizing	3
After Each Use	3
Tubing Care	4
Battery Care	4
Symphony Parts List	5
Symphony Assembly & Operation	6-11
Symphony 2-Phase Pumping Protocol	12-14
Storing Breastmilk	15
Thawing Breastmilk	16
Troubleshooting	16
Symphony Technical Description	17

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IMPORTANT SAFEGUARDS

When using electrical devices, basic safety precautions should always be followed, including the following:

READ ALL INSTRUCTIONS BEFORE USE

DANGER: To reduce the risk of electrocution:

1. Always unplug electrical devices immediately after using. Separation from the mains is only assured through the disconnection of the plug and socket connection.
2. Do not use while bathing or showering.
3. Do not place in or drop motor unit into water or other liquid.
4. Do not reach for any electrical device if it has fallen into water. Unplug immediately.
5. Do not store or use this product where it can fall or be pulled into a tub or sink.

WARNING: To reduce the risk of burns, electrocution, fire or injury to persons:

1. The pump should never be left unattended when plugged into an electric outlet.
2. Use the pump only for its intended use as described in this manual.
3. Do not use accessory attachments not recommended by the manufacturer, as this may cause hazards.
4. Never operate an electrical device if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or if it has been dropped into water.
5. Keep the cord and all components away from heated surfaces.
6. Do not use outdoors.
7. Only use the electric cord that comes with the Lactina or Symphony Breastpump.
8. The kits are personal care items and are not to be shared between mothers without proper sterilization to prevent the risk of cross contamination.
9. The pump should be cleaned as needed with soapy water or Cavicide® solution. Do not spray or pour liquid directly onto the pump.
10. Supervision is necessary when this product is used in the vicinity of children.
11. Make sure the voltage of the device is compatible with the power source.
12. Pump only with the motor in an upright position.
13. Do not use while sleeping or overly drowsy.
14. There are no user-serviceable parts inside the Lactina® or Symphony® Breastpump. DO NOT ATTEMPT TO REPAIR THE LACTINA OR SYMPHONY BREASTPUMP YOURSELF.

SAVE THESE INSTRUCTIONS

Uses

- To initiate and maintain a milk supply as well as collect breastmilk at work or during other absences from the baby.
- To initiate and maintain a milk supply if direct breastfeeding is not possible.
- To relieve engorgement. Single pumping with simultaneous massage is helpful to relieve engorgement.

Cleaning and Sterilization

Unless instructed otherwise by your health care provider.

IMPORTANT

This product's kit is a single-user item unless properly sterilized between users. Use by more than one user without adequate sterilization may pose a health risk and voids the warranty.

Hygiene

Wash hands before touching breasts or containers, and avoid touching the inside of containers or lids.

Preparation

Follow these cleaning instructions unless you are told otherwise by your healthcare professional. Prior to cleaning, take apart all parts of your breastpump accessory kit. Be especially sure to remove the yellow valve from the breastshield and separate the white membrane from the valve prior to cleaning.

Prior To First Use

The Symphony sanitized version does not require washing before first use.

To Sterilize in the Hospital:

All disassembled parts can be autoclaved to a maximum of 272° F for 3 minutes at 29 PSIG or 250° F for 15-20 minutes at 15 PSIG. *If autoclaved membrane caps may require more force to attach to pump.*

To avoid damage during and immediately after sterilization:

- Sterilize parts without applying external pressure to them.
- Allow parts to cool before applying external pressure.

If gas sterilization is used, residual testing is recommended to determine the proper length of aeration.

To Sanitize at Home:

Disassemble, refer to diagram on page 6. Sanitize by boiling for 10 minutes.

After each use:

- Disassemble and wash all parts that come in contact with the breast and milk in warm soapy water.
- Rinse in clear water.
- Air-dry on clean towel and cover parts when not in use.

All parts may also be washed in top rack of dishwasher.

For Quicker Cleaning:

Medela offers Quick Clean™ Micro-Steam™ Bags for daily kit disinfection.

Tubing Care:

Inspect tubing after each pumping session for condensation and or milk.

If condensation appears in tubing:

To eliminate condensation after you have completed pumping, continue running the pump with the tubing(s) attached for another 1-2 minutes or until dry, while you attend to storing your pumped breastmilk.

If milk appears in tubing:

1. Turn off the pump and unplug from power source.
2. Remove and disassemble the tubing, protective membrane and membrane cap. Wash in soapy water then rinse in cold clear water.
3. Shake water droplets out and hang to air-dry. For faster drying, attach tubings to pump and run pump 1-2 minutes or until dry, or pour a small amount of isopropyl alcohol through the tubings to dry.
4. Make sure that valve membranes are undamaged and clean. Replace if necessary.
5. Use a damp cloth to wipe the pump and the diaphragm. Do not immerse the pump in water.
6. When components are completely dry, reassemble breastpump and kit.

Important Battery Care

Charge Symphony® pump batteries for 12 hours upon receipt.

Pump may be used while charging.

Optimal battery life is achieved when:

- Batteries are not fully drained on a regular basis
- Pump is allowed to charge when not in use

If unable to charge regularly, charge for 12 hours every 2 months at minimum

Pumping time with a completely charged battery is approximately 60 minutes.

Symphony® Breastpump Parts

Item

Order Number

- | | |
|--|---------|
| 1. PersonalFit™ Breastshield, Standard (24mm)..... | 87073 |
| 2. PersonalFit Breastshield Connector | 87071 |
| 3. Valve & Membrane | 8000001 |
| 4. Collection Connector | 8117009 |
| 5. Solid Container Lid | 8107057 |
| 6. Tubing..... | 8107168 |
| 7. Membrane Cap | 8100769 |
| 8. Protective Membrane..... | 8200094 |
| 9. Breastfeeding Information Guide..... | 1907200 |
| 10. Symphony Instruction Manual | 1907724 |
| 11. Symphony Container Stand | 8100552 |
| 12. Symphony Vehicle Lighter Adapter | 67173 |

Call for pricing

Symphony® Assembly & Operation

Kit Assembly

- 1 Follow cleaning instructions on pages 2-3.
- 2 Insert the small end of the Breastshields into the small end of the connectors. (Prior to cleaning, separate these two pieces.)
- 3 Snap white membranes onto yellow valves until membrane lies completely flat. (Prior to cleaning, separate these two pieces.)

8 To unlock case, press the oval release button on top of the Symphony® Breastpump. Keeping button depressed, lift up on the handle to open case and access pumping mechanisms.

9 **Push membrane/cap into place on pumping mechanism. A “click” will be heard once properly seated. To ensure proper fit, be sure to center the membrane caps’ tube ports directly over the crevice on the pump housing.**

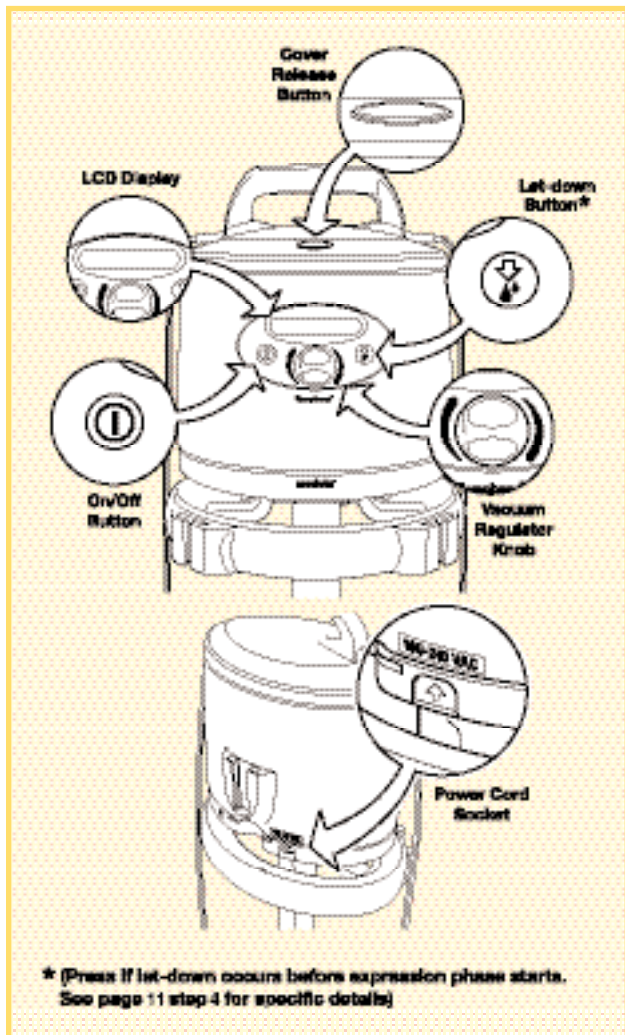
73ETCS2 CS1-A1 T885 005F25-8 GS1 gsqBT/F0 1 Tt8.5 0 0 8.5 10 For a complete list of kits that are compatible with the Symphony Breastpump (or charge its battery if included) by connecting the power cord to a wall outlet and the socket in the back of the pump.

- 11 Center breastshields over nipples. Switch the breastpump on to begin the Stimulation phase by pressing the power button. Vacuum may be adjusted to your comfort level by turning the vacuum regulator knob. After two minutes, the Stimulation phase will automatically change to the Expression phase and you may wish to adjust the vacuum level for comfort. To single pump, simply remove one of the breastshields from your breast.

Important Note: Kits sold separately. For a complete list of kits that are compatible with the Symphony pump, please refer to the Medela Product Catalog or contact Customer Service at 1-888-633-3528.

Important Note: If you experience discomfort, contact your lactation professional as you may need a larger breastshield or other advice. To locate Medela products or a breastfeeding specialist in your area call 1-800-TELL YOU.

Symphony® Controls

Symphony® Program Cards

There are two versions of the Symphony Program Card: the Original Symphony Program Card and the Symphony 2.0 Program Card. The following information is instructions for either card.

Original Symphony Program Card

The original Symphony Program Card is programmed to operate the 2-Phase Expression.

1 **LCD Display.**

When powered off and connected to A/C power supply, the LCD Display will indicate "Mains". When batteries are used to power the Symphony, the LCD Display will remain blank until powered on. The vacuum setting is indicated on the LCD Display with Drops (Stimulation) and Bars (Expression).

2 **Begin in Stimulation Phase.**

Start the pump by pressing the on/off button. The pump will immediately begin pumping in the Stimulation Phase. The initial vacuum level will be indicated by 1 drop on the LCD Display. Stimulation Phase is always limited to two minutes.

3 **Switch to Expression Phase.**

The pump will switch to Expression Phase by one of two ways: 1) automatically, after two minute Stimulation Phase is complete, or 2) by pressing the let-down button. You should press the let-down button immediately after milk ejection and continue pumping in the Expression Phase.

4 **Vacuum Setting.**

When switching from Stimulation to Expression, the pump automatically increases the vacuum setting by adding 3 Bars to the last Stimulation setting. *This vacuum level can be increased by using the Vacuum Regulator Knob.

Symphony® 2.0 Program Card

The Symphony 2.0 Program Card is the second generation program card. It is programmed to operate the 2-Phase Expression similar to the Original Symphony Program Card. Additionally, there are some new features available with the Symphony 2.0 Program Card:

1 **Program Card Name Displayed on LCD Display.**

The Symphony 2.0 Program Card displays the name of the card (Symphony 2.0) on the LCD display when the pump is powered off and connected to A/C power supply. When batteries are used to power the Symphony, the LCD Display will remain blank until powered on.

2 **Minimum Start Vacuum Setting.**

The Symphony 2.0 Program Card will start the pump in the Stimulation Phase. The initial vacuum setting when powering on the pump is visually indicated with 3 drops on the LCD screen. The vacuum setting can be increased or decreased by using the Vacuum Regulator Knob.

3 **Ramp-Up Transition from Stimulation to Expression Phase.**

When switching from Stimulation to Expression Phase, the Symphony 2.0 Program Card increases the vacuum setting by adding 3 bars to the last Stimulation setting in a stair-stepped fashion. The 3 additional bars and increased vacuum setting occur gradually over a 4-second period. The ramp-up transition eliminates potential discomfort of an immediate vacuum setting increase.

4 **Use Let-Down Button to Switch Back to Stimulation Phase.**

The Symphony pump can switch from Stimulation to Expression Phase in one of two ways: 1) automatically after 2 minutes, or 2) manually through pressing the let-down button. With the Symphony 2.0 Program Card, the let-down button can also be used to switch back to Stimulation Phase. For more information on this feature, please review the Symphony 2-Phase Pumping Protocol information in this booklet.

Note: Stimulation Phase is always limited to two minutes. If milk ejection occurs before the 2 minute Stimulation Phase is over, press the let-down button and continue pumping in Expression Phase

5 **Program Remembers Last Stimulation Vacuum Setting.**

When switching back to Stimulation Phase, the Symphony 2.0 Program Card remembers the last Stimulation Phase vacuum setting prior to switching to Expression Phase and returns to this setting in Stimulation Phase.

Symphony 2-Phase Pumping Protocol

Maximum Comfort Vacuum™

Research has shown that you will pump more efficiently—meaning get more milk in less time—when pumping at Maximum Comfort Vacuum in the Expression Phase.

1 **Begin in Stimulation Phase**

First, start the pump by pressing the on/off button. The pump will immediately begin pumping in Stimulation Phase of the 2-Phase Expression.

2 **Switch to Expression Phase**

Once your milk starts to flow, or you sense milk ejection, press the let-down button to switch to the Expression Phase for the most effective pumping and removal of milk. If you do not press the let-down button, the pump will automatically switch to the Expression Phase after two minutes.

3 **Setting your Maximum Comfort Vacuum**

Maximum Comfort Vacuum is the highest vacuum you can use and still feel comfortable; it is different for every mom. To find your maximum comfort vacuum, increase the vacuum by turning the Vacuum Regulator Knob clockwise until pumping feels slightly uncomfortable—never painful. Then, decrease the vacuum one bar by turning the knob counterclockwise.

Symphony 2.0 Program Card – Addressing Special Needs

The Symphony 2.0 Program Card includes a feature that allows moms to switch back to the Stimulation Phase by pressing the let-down button. Some moms find this feature beneficial in addressing the following special needs. Please note that the Stimulation Phase is always limited to two minutes.

1 **No milk ejection occurs in the first two minutes of the Stimulation Phase**

If your milk ejection has not occurred in the first two minutes of the Stimulation Phase and the pump automatically switches to the Expression Phase, press the let-down button once to return to the Stimulation Phase. If you have not achieved a milk ejection after two consecutive Stimulation Phases, take a break, relax and try again in 15-30 minutes.

2 **Engorgement**

If, due to engorgement, you are experiencing difficulties breastfeeding or expressing milk, careful use of the Stimulation Phase may help to provide some relief. If your milk ejection does not occur in the first two minutes of the Stimulation Phase and the pump automatically switches to the Expression Phase, press the let-down button to return to the Stimulation Phase. Repeat this process until your milk ejection occurs. After milk ejection you may find it beneficial to remain in the Stimulation Phase until enough milk has been expressed so that the nipple and areola area are not as hard and engorged. After initially draining the breast to relieve the discomfort, follow up with breastfeeding or pumping and expressing in the Expression Phase to adequately drain the breast. Although Medela strongly encourages pumping in the Expression Phase after milk ejection at Maximum Comfort Vacuum™ for the most effective removal of milk, it may be necessary to address temporary discomfort issues by pumping in the Stimulation Phase. If your engorgement condition persists or worsens, consult your physician or lactation consultant.

③ Sore Nipples

If, due to sore nipples, you are experiencing discomfort while pumping in the Expression Phase, careful use of the Stimulation Phase may help to provide some relief. You can return to the Stimulation Phase at any time by pressing the let-down button. Although Medela strongly encourages pumping in the Expression Phase after milk ejection at Maximum Comfort Vacuum™ for the most effective removal of milk, it may be beneficial to address temporary discomfort issues by pumping in the Stimulation Phase. If your sore nipple condition persists or worsens, consult your physician or lactation consultant.

Important Note:

For the most effective pumping and removal of milk, Medela strongly encourages pumping in the Expression Phase after milk ejection at Maximum Comfort Vacuum™.

Storing Breastmilk

Check with your hospital for specific storage instructions.

When freezing, do not fill containers more than 3/4 full to allow space for expansion. Label containers with the date of pumping.

Breastmilk Storage (For Healthy Term Babies)					
	Room Temperature	Cooler with 3 Frozen Ice Packs	Refrigerator	Self-contained Refrigerator Freezer Unit	Deep Freezer
Freshly expressed breastmilk	4 hours at 66-72°F (19-22°C) ¹	24 hours at 59°F (15°C) ¹	5-7 days at 32-39°F (0°-4°C) ²	3-4 months ³	6-12 months at 0°F (-19°C) ³
Thawed breastmilk (previously frozen)	Do not store	Do not store	24 hours ⁴	Never refreeze thawed milk	Never refreeze thawed milk

¹ Hamosh M, Ellis L, Pollock D, Henderson T, and Hamosh P: Pediatrics, Vol. 97, No. 4, April 1996. pp 492-497. (4 hours at 77° F/25° C).

² Sosa, Roberto; Barness, Lewis: AJDC, Vol. 141, Jan. 1987.

³ Lawrence R, and Lawrence R: Breastfeeding: A Guide For the Medical Profession, 1999, p.894.

⁴ Arnold L; Recommendations for Collection, Storage and Handling of a Mother's Milk for Her Own Infant in the Hospital Setting, 3rd Edition. The Human Milk Banking Association of North America, Inc. 1999, p. 18.

Important Note: *If the temperature in the room, car or outdoors exceeds 77° F / 25° C, chill milk immediately to preserve freshness.*

Defrosting

- Thaw milk overnight in the refrigerator, or hold the bottle under warm running water to quickly thaw. You can also place the sealed container in a bowl of warm water for 20 minutes to bring it to body temperature.
- Thawed milk is safe in the refrigerator for 24 hours.
Do not refreeze.

Caution:

- Do not thaw frozen breastmilk in a microwave or in a pan of boiling water.
- Never microwave breastmilk. Microwaving can cause severe burns to baby's mouth from hot spots that develop in the milk during microwaving. Microwaving can also change the composition of breastmilk.
- If adding expressed breastmilk to container of already frozen breastmilk, make sure to add a lesser amount than the already frozen amount.

Troubleshooting

If There Is Low or No Suction

- Make sure all connections of kit and breastpump are secure.
- If using a unit with a battery, make sure the battery is charged.
- Make sure membrane is lying flat against valve head. Refer to the assembly instructions.
- Make sure the hard yellow ends of the tubing fit securely into breastshield connectors and the other ends into the ports on the pump connector or membrane caps.
- In the US, call Medela at 800-435-8316 or 815-363-1166. In Canada, call Medela at 800-435-8316 or 905-608-7272. In other countries, contact your local distributor or the location where you purchased your pump. Visit us on the internet at www.medela.com.

Symphony® Technical Description

SAFETY INSTRUCTIONS

Medical Electrical Equipment, Breastpump Model Symphony® 600.0837 and 600.0838
CLASSIFIED WITH RESPECT TO ELECTRIC SHOCK, FIRE AND MECHANICAL HAZARDS ONLY IN ACCORDANCE WITH UL60601-1/CAN/CSA C22.2 No. 601.1 22PL.

PROTECTION CLASS II

(DoubleInsulation)

Type B

VACUUM:

-50mmHg – -250mmHg
-7kPa – -33kPa

POWER SUPPLY:

Conventional Outlet:
100-240 V
50/60 Hz
0.5 A

DC Connection:

12 VDC +
25 W
T2.5A

Battery:

2 x 6 V, 1.2 Ah Type: Yuasa NP 1.2-6
Battery(optional): Battery,
DC powered
Charging time=12 hours
Full battery pumping
time=approx. 60 minutes

DISPOSAL:

This breastpump is made of technical plastics and metals that cannot be separated. It must be disposed of in accordance with local regulations.

WARRANTY:

3-years for the unit from the date of delivery, excluding pump set and batteries. The manufacturer is not liable for any damage or consequential damage caused by incorrect operation, inappropriate usage or use by unauthorized persons. Routine checks, servicing and battery exchange may only be carried out by Medela authorized personnel.

DIMENSIONS:

	<i>With Stand</i>	<i>Without Stand</i>
Length	16.1"	8.5"
Width	16.1"	6.1"
Height	35.4"	9.6"

WEIGHT:

With Battery: 6 lbs. 7 oz.
Without Battery: 5 lbs. 1 oz.

ENVIRONMENTAL CONDITIONS FOR OPERATION:

Temperature: +41°F to +104°F
Relative Humidity: 20% to 80%
Atmospheric Pressure: 70 kPa to 106 kPa

ENVIRONMENTAL CONDITIONS TRANSPORT AND STORAGE:

Temperature: -4°F to +122°F
Relative Humidity: 20% to 95%
Atmospheric Pressure: 70 kPa to 106 kPa

STANDARDS:

FDA (GMP)
ISO 9001
ISO 13485
CE (93/42/EEC)

ELECTROMAGNETIC OR OTHER INTERFERENCE:

Complies with IEC 60601-1-2. Unlikely to interfere with other electrical equipment.

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Medela and Symphony are registered trademarks of Medela in the USA. PersonalFit is a trademark of Medela, Inc. in the USA and in other countries. Applications for registration of Medela is pending in other countries.
Medela, Inc.,
Medela Canada, Inc.,