
OMRON®

INSTRUCTION MANUAL

Gentle Temp™

EAR THERMOMETER

Model MC-505

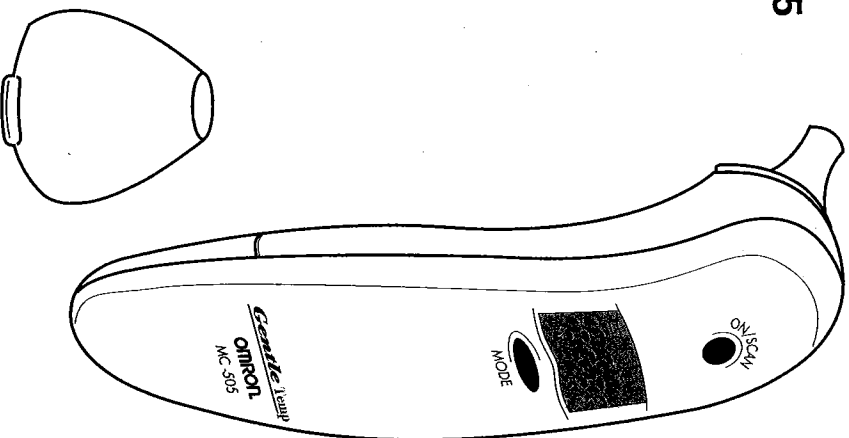ENGLISH

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1601401-4D

TABLE OF CONTENTS

Cautions	4
Introduction	5-6
Components	6
Know Your Unit	7
Display Messages	8
How To Take A Measurement	9-12
Care and Cleaning	13
Battery Replacement	14
Limited Three Year Warranty	15

CAUTIONS

This thermometer is designed to take a temperature through the ear canal. DO NOT use it at any other site or for any other purpose.

DO NOT force Safety Probe into the ear canal.

Keep Lens Covers out of reach of children.

DO NOT use the Gentle Temp™ Ear Thermometer in an infected ear.

DO NOT use a cellular phone in the same room when you are taking a temperature with the thermometer.

For accurate measurement:

DO NOT use the Gentle Temp™ Ear Thermometer without a Lens Cover. Use this thermometer only with a new or clean, intact Lens Cover attached.

Use only Omron Lens Covers (model MC-505LC) made for the Gentle Temp™ Ear Thermometer.

Always attach a clean Lens Cover and the Protective Storage Cap to the Gentle Temp™ Ear Thermometer when not in use.

After each use:

Replace or clean the Lens Cover after every use. To clean leave the Lens Cover on the thermometer Safety Probe and wipe the Lens Cover with a cloth or cotton swab moistened with rubbing alcohol (Isopropyl 70%) or mild soap and water. Dry the Lens Cover with a soft cloth. Allow forty five minutes drying time before taking another temperature.

DO NOT subject the thermometer to shock or drop it on the floor.

DO NOT disassemble the thermometer.

4

INTRODUCTION

Thank you for purchasing the Gentle Temp™ Ear Thermometer. This remarkable instrument offers safe, accurate, and quick temperature measurement from the ear canal.

How Does it Work?

The Gentle Temp™ Ear Thermometer uses a two-step process to take a temperature. First, the Gentle Temp™ measures the temperature of the infrared heat given off by the eardrum and surrounding tissues. Second, it converts this temperature into an oral or rectal temperature. The Gentle Temp™ offers you the convenience of taking a temperature in the ear, and gives you an oral or rectal temperature, such as 98.6°F or 99.6°F, that you understand.

The Gentle Temp™ is less threatening to a child than a rectal thermometer. It's faster, safer and easier to use than an oral thermometer. There's no worry of broken glass or mercury ingestion. Measurements can even be taken while a child is sleeping. For adults, the Gentle Temp™ Ear Thermometer offers fast convenient, accurate readings without the wait of a conventional thermometer.

Temperature From The Ear?

Clinical research has shown that the ear is an ideal site for taking body temperature. The eardrum shares blood vessels with the hypothalamus, the part of the brain that controls body temperature. Therefore, the ear is an accurate indicator of internal body (core) temperature. An ear temperature, unlike an oral temperature, is unaffected by factors such as talking, eating, drinking, and smoking.

5

INTRODUCTION

Gentle Temp™ Ear Thermometer is hospital tested and meets the American Society for Testing and Materials (ASTM) accuracy standards under normal operating conditions.

Practice Makes Perfect

We recommend that you practice with the Gentle Temp™ on yourself and family members when you are healthy. This way you can improve your technique and feel more confident of the measurements you take when a family member is ill.


Please read this instruction manual completely before using your Gentle Temp™ Ear Thermometer. This product is intended for household use only. If you have questions about the meaning of a specific temperature, please talk to your physician.

COMPONENTS



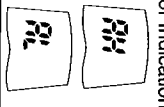
Gentle Temp™ includes the following components:

- Gentle Temp™ Ear Thermometer Model MC-505
- 21 Lens Covers
- Soft Storage Case
- Battery (Type: CR2032 installed)
- Instruction Manual
- Warranty Card

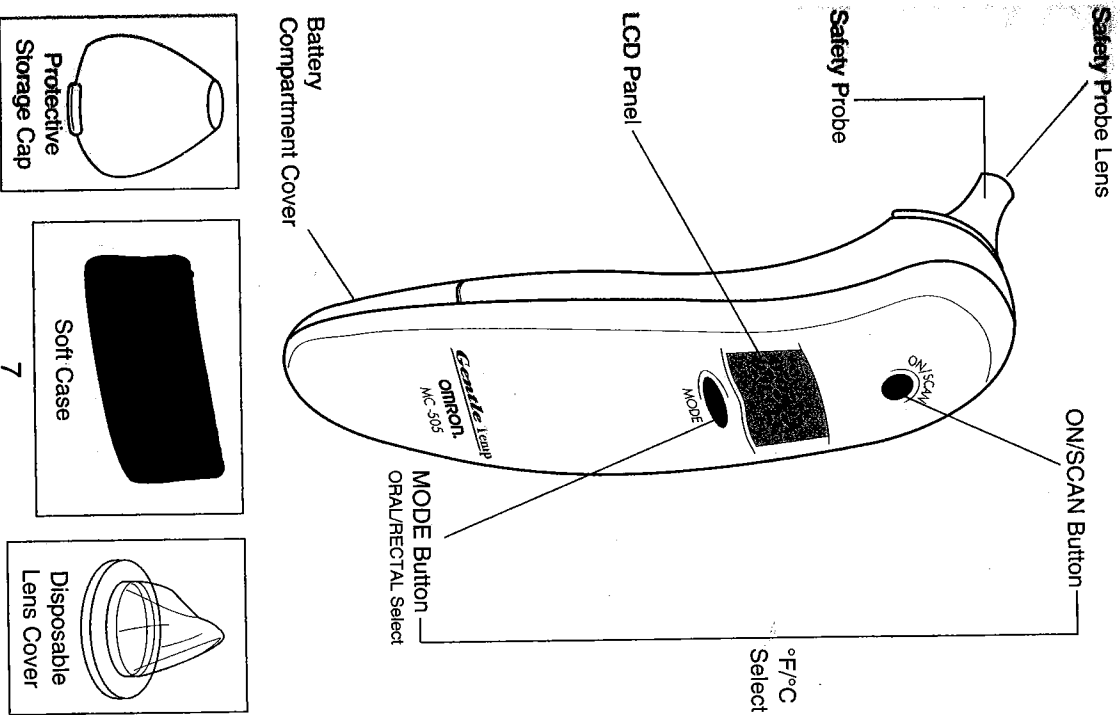KNOW YOUR UNIT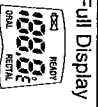

DISPLAY MESSAGES

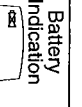
DISPLAY	ACTION
 <p>Full Display</p>	<p>Displays for 2 seconds when the thermometer is first turned on.</p> <p>Wait for "Ready" symbol to appear.</p>
 <p>Ready Indication</p>	<p>Displays when thermometer is ready for measurement.</p> <p>Insert the thermometer probe into the ear canal and begin measurement.</p>
 <p>Error Indications</p>	<p>Displays after "ON/SCAN" button is pushed if the thermometer's own temperature is not within its operating range of 50°F (10°C) to 104°F (40°C).</p> <p>Put the thermometer in a room within the range of 50°F (10°C) to 104°F (40°C). Wait 30 minutes before retaking a temperature.</p>
	<p>Displays after "ON/SCAN" button is pushed if the temperature measurement is not within the human temperature range of 95°F (35°C) to 107.6°F (42°C).</p> <p>Make sure a Lens Cover is properly attached. Fully insert the thermometer into the ear canal. Retake the temperature.</p>
	<p>Displays if the thermometer senses an error in self-check procedure.</p> <p>Return to Omron Healthcare, Inc. for repair.</p>
 <p>Battery Indication</p>	<p>Battery voltage is excessively low.</p> <p>Make sure battery is installed properly. Retest. If display shows battery low symbol or nothing at all, immediately replace with new battery.</p>
 <p>No Display</p>	

If you continue to have difficulty obtaining a reading, call **OMRON HEALTHCARE CUSTOMER SERVICE TOLL FREE 1-800-634-4350**. Please tell the representative you have an Omron Model MC-505 Gentle Temp™ Ear Thermometer.



8

HOW TO TAKE A MEASUREMENT

NOTE: If there is any difference in the room temperature where you store the thermometer and the room temperature where you take the measurement, please let the thermometer sit in the measurement room at least 30 minutes.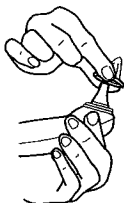

1. Remove the protective storage cap.
2. Make sure that a clean Lens Cover is placed on the Safety Probe. If the Lens Cover is dirty or missing, attach a clean Lens Cover to the Safety Probe Tip by placing your thumb and forefinger on the collar of the Lens Cover and gently pushing it down. Do not touch the plastic film of the Lens Cover. A dirty or damaged Lens Cover may cause an inaccurate reading. Push the Lens Cover down around the Safety Probe until it snaps into place.
3. Press and release the ON/SCAN button. The LCD panel shows the Full Display for two seconds. After a few seconds the thermometer beeps and "READY" appears on the LCD panel.
4. To change the MODE from oral to rectal, or rectal to oral, press the MODE button and hold for about one second. The thermometer beeps when the MODE changes.

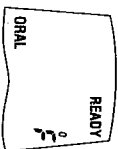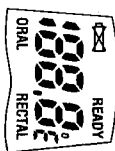
9

HOW TO TAKE A MEASUREMENT

5. To change from °F to °C or from °C to °F press and hold the MODE button then, while still holding the MODE button, press and hold the ON/SCAN button. You will hear a beep signaling that degree MODE has been changed.

6. It is best to approach the ear from behind the person. Carefully insert the Safety Probe into the ear canal until the body of the probe tip fully seals the ear canal.

7. Do an "ear tug." Gently grasp the outer edge of the upper portion of the ear with your thumb and forefinger. Gently but firmly pull the ear up and back as illustrated. Continue pulling the ear until you have finished taking a temperature.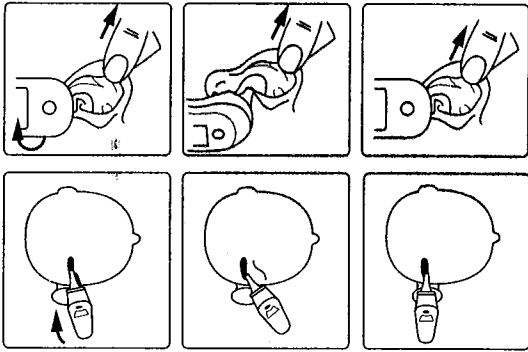


10

HOW TO TAKE A MEASUREMENT

8. While holding the thermometer firmly in place, press and release the ON/SCAN button. You will hear one short beep alerting you that the thermometer is working.

9. When the measurement is complete you will hear four beeps. You may then gently remove the Safety Probe from the ear. Read the measurement on the LCD panel.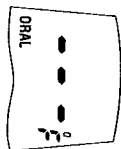

10. If you wish to take another measurement, wait about ten seconds. The thermometer will beep twice and then displays READY. The measurement is taken. See TAKING REPEATED MEASUREMENTS on page 12 for more information.

11. To protect the Safety Probe Lens, leave the Lens Cover on the thermometer when the measurement is complete. Place the Protective Storage Cap over the Safety Probe Tip. Store the thermometer in a dry place at room temperature. You may reuse the Lens Cover provided you clean it and it is intact. To clean, simply wipe with a cotton swab moistened with rubbing alcohol or mild soapy water. Allow the Lens Cover to completely dry before using it again.

12. The thermometer automatically powers off after about one minute.

11

HOW TO TAKE A MEASUREMENT

TAKING REPEATED MEASUREMENTS

The Gentle Temp takes ten seconds between measurements to run an internal accuracy check. You can take another measurement when the thermometer beeps twice and displays "READY".

To ensure an accurate measurement, change or clean the Lens Cover after each use. To change the Lens Cover, push up with your thumb on the plastic collar. The Lens Cover will pop off. Attach a new Lens Cover.

The left and right ear temperature may differ. You can take a temperature from either the left or right ear, but use the same ear each time or an average of both.

REPLACEMENT LENS COVERS


Replacement Lens Covers (MC-505LC) are available from your Omron retailer. The replacement box contains forty Lens Covers (Use only OMRON MC-505LC Lens Covers with the MC-505). Using other Lens Covers may cause an inaccurate reading.

CARE AND CLEANING

Safety Probe Lens:

Keep Clean and Protected

To ensure accurate measurements, keep the Safety Probe Lens clean and free of scratches. Fingerprints, ear wax or dirt will impede the accuracy of the thermometer. The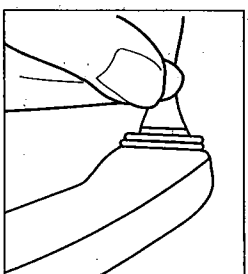

Safety Probe Lens may be gently cleaned with a soft cotton cloth moistened with alcohol. Do not use a paper towel. The fibers in the paper towel will scratch the Safety Probe Lens. Allow forty five minutes for the Safety Probe Lens to dry completely before attaching the Protective Storage Cap. Always attach a clean Lens Cover and the Protective Storage Cap to the thermometer when not in use. If you see tears, holes, or scratches in the the Probe Lens, you should return the thermometer to Omron Healthcare for repair.

Lens Cover: To Change or Clean

Change or clean the Lens Cover after each use. Clean the Lens Cover when it is attached to the thermometer by using a soft cloth moistened with alcohol. Do not use thinner or benzene. Dry completely with a soft cloth before reusing. If you see tears, holes, scratches or discoloration in the Lens Cover, replace it. Always use a Lens Cover when taking a measurement. Always change or clean Lens Covers after each use.

Thermometer: Keep Clean and Protected

Use a soft, dry cloth to clean the body of the thermometer. Never clean the thermometer with an abrasive cleanser, thinner, benzene or submerge the thermometer in water or other liquids. Do not expose the thermometer to extreme temperatures, humidity, direct sunlight or shock. Store in a clean, dry place at room temperature.

BATTERY REPLACEMENT

The Gentle Temp™ Ear Thermometer includes one 3V lithium battery already installed. Approximate battery life is three years. If the Low Battery Indication symbol displays on the thermometer's LCD panel or the thermometer does not function at all, immediately replace the battery as follows:

Unscrew the Battery Cover and lift it off the body. Remove the battery and replace it with a fresh one (CR2032 type). Place new battery with positive side up, replace cover and tighten screw.

After the battery is installed, the MODE is in Oral and °F.

CAUTION: ONLY REPLACE BATTERY WITH TYPE CR2032

USE OF OTHER TYPES OF BATTERIES MAY PRESENT A RISK OF FIRE OR EXPLOSION. SEE BELOW FOR SAFETY INSTRUCTIONS.

CAUTION: BATTERY MAY EXPLODE IF MISTREATED. DO NOT RECHARGE, DISASSEMBLE, HEAT ABOVE 100°C (212°F), OR INCINERATE

KEEP BATTERY OUT OF REACH OF CHILDREN

LIMITED THREE YEAR WARRANTY

Your MC-505 Gentle Temp™ Ear Thermometer is warranted to be free from manufacturing defects for a period of three years under normal use. This warranty extends only to the original retail purchaser.

Should repair be needed within the warranty period, ship the unit prepaid to **Omron Healthcare, Inc., 300 Lakeview Parkway, Vernon Hills, IL 60061, Attn: Service Dept.**, together with \$5.00 for return shipping and insurance. Be sure to include the model number of your unit and your phone number on any correspondence.

We will either repair or replace (at our option) free of charge any parts necessary to correct defects in the materials or workmanship.

The above warranty is complete and exclusive. The warrantor expressly disclaims liability for incidental, special, or consequential damages of any nature. (Some states do not allow the exclusion or limitation of incidental or consequential damages, so the above warranty may not apply to you.)

Any implied warranties arising by the operation of law shall be limited in duration to the term of this warranty. (Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.)

This warranty gives you specific legal rights and you may have other rights which vary from state to state.

**FOR CUSTOMER SERVICE CALL TOLL FREE:
1-800-634-4350**


OMRON®

MANUAL DE INSTRUCCIONES

Gentle Temp™

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Modelo MC-505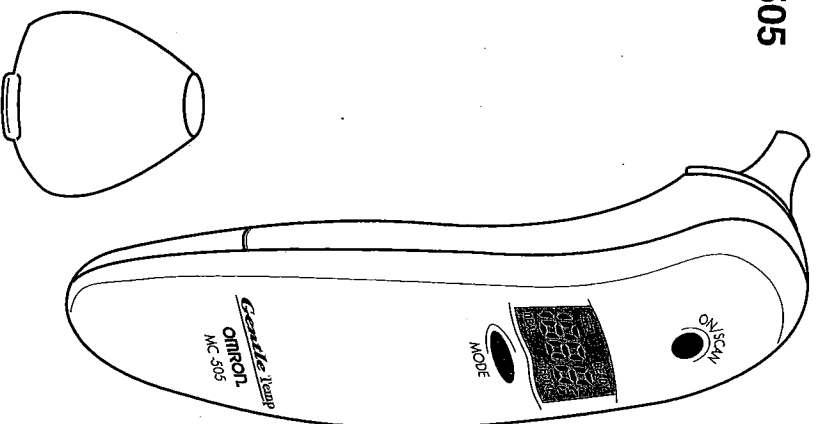

ESPAÑOL

Omron uses recycled paper to help preserve our natural resources.

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Copyright © 1996 OMRON HEALTHCARE, INC.