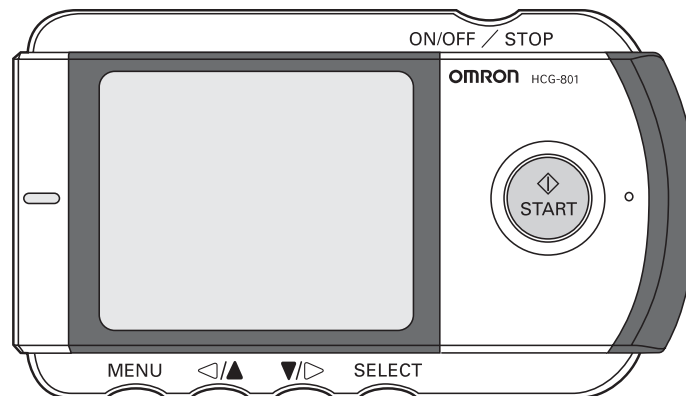
OMRON®

INSTRUCTION MANUAL

Portable ECG Monitor

Model HCG-801

ESPAÑOL

ENGLISH

Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.

TABLE OF CONTENTS

Before Using the Unit

Introduction	4
General Information About the Heart and ECG Measurements	5
Important Safety Notes	6
Know Your Unit	9

Operating Instructions

How to Insert and Replace Batteries	13
Battery Life and Replacement	14
How to Attach the Battery Cover	15
How to Set the Date and Time	16
How to Use the SD Memory Card	18
Inserting and Ejecting SD Memory Card	18
Formatting an SD Memory Card	22
Important Notes on Taking a Measurement	24
How to Take a Measurement	25
Common Causes of Inaccurate Measurement Results	29
How to Display ECG Data	30
How to Adjust the Settings	32
Basic Operation	32
Date and Time	34
Volume	34
Brightness	35
Contrast	36
Language	37
Date Format	38
Message	39
Overwrite	40
About the Overwrite Message	40

TABLE OF CONTENTS

How to Erase, Save and Organize ECG Data	41
Erasing ECG Data	41
Copying Data to an SD Memory Card	44
Copying Data from the SD Memory Card to a PC	46
Copying Data from a PC to the SD Memory Card	47
Messages	48
Data Description Messages	48
Criteria of Data Description Messages	49
Heart Rate Message	49
Action Messages	49
Lists and Explanations of Messages	50
Notice Messages	50
Warning Messages	50
Error Messages	52
Care and Maintenance	
Trouble Shooting	54
Maintenance and Storage	56
FCC Statement	58
Warranty Information	59
Specifications	60
Sample Symptom Log	61
Notes	63

INTRODUCTION

Thank you for purchasing the OMRON® Portable ECG Monitor.

Fill in for future reference.

DATE PURCHASED: _____

SERIAL NUMBER: _____

Staple your purchase receipt here

Your OMRON HCG-801 is an easy-to-use electrocardiograph unit that can record and store electrocardiogram (ECG) measurements of your heart rhythm. Each ECG reading records an approximately 30-second measurement and these ECG readings can help your doctor monitor your condition. It also includes guidance screens to help you obtain good data when taking measurements. This unit is intended for use on the adult population.

The monitor is supplied with an SD memory card that can store up to 300 measurements, including the ECG data along with the date and time of measurement. Storing ECG data on the SD memory card, or in unit memory, allows your doctor to view them later.

IMPORTANT: Please note that while the measurements from an ECG monitor are a useful guide for your doctor, they cannot detect all changes in heart conditions. Always consult your doctor if you notice any change in your condition, regardless of the measurement results. Consult a doctor even if “Regular heart rate, Regular heart rhythm” is displayed as the data message, if you have been diagnosed with a heart condition.

The HCG-801 comes with the following components:

- Main Unit
- SD Memory Card
- Storage Pouch
- 2 × LR03(AAA) Batteries
- Instruction Manual*
- Registration Card
- Quick Reference Card

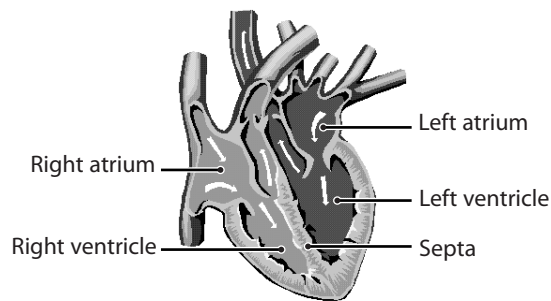
* The illustrations used in this manual may differ slightly from the appearance of the actual unit.

SAVE THESE INSTRUCTIONS

GENERAL INFORMATION ABOUT THE HEART AND ECG MEASUREMENTS

The heart is a muscular pump controlled by electrical impulses generated by the body. It is divided into two sides by the septa and each side has two chambers – an atrium and a ventricle – linked by a one-way valve. The left atrium and ventricle control oxygenated blood, and the right atrium and ventricle de-oxygenated (“used”) blood.

The electrical impulse that causes the heart to beat spreads across the atria, causing the left and right atria to contract and pump blood into the left and right ventricles respectively. The two ventricles then contract and pump blood out of the heart. The heart muscle then relaxes, or re-polarizes, allowing blood to fill up the heart again.

An ECG monitor is able to measure the electrical impulse as it passes across and through the heart, causing the heart to beat. An ECG does not measure the movement of your heartbeat, but rather the electrical activity that causes the heart to beat. The measurements recorded by an ECG monitor, when combined with a medical examination, can help your doctor monitor your heart condition. The ECG measurements recorded by the unit are NOT designed or intended for medical diagnosis. Conditions such as arrhythmia can only be diagnosed by a doctor through a special examination.

IMPORTANT SAFETY NOTES

To assure the correct use of the product basic safety measures should always be followed including the warnings and cautions listed in this instruction manual.

SAFETY SYMBOLS USED IN THIS INSTRUCTION MANUAL	
 WARNING	Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.
 CAUTION	Indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury to the user or patient or damage to the equipment or other property.

THIS UNIT IS DESIGNED FOR RECORDING THE ECG WAVEFORM IN THE ADULT POPULATION. IT IS NOT DESIGNED OR INTENDED FOR MEDICAL DIAGNOSIS. NEVER USE THE UNIT IN CRITICAL CARE SITUATIONS.

- Messages after measurement report heart rate and heart rhythm only.
- This device neither detects nor measures all heart rate, heart rhythm and heart waveform changes, especially those related to ischemic heart conditions.
- Always consult your doctor if you notice change in your condition, regardless of measurement results and messages.

 **WARNING:**

-  Do not use this device if you have a cardiac pacemaker or other implanted devices. 
-  Do not use this device with a defibrillator.
-  Do not use in the presence of flammable anesthetics, drugs or pressurized oxygen (such as in a hyperbaric chamber, ultraviolet sterilizer or oxygen tent).
-  Do not attempt self-diagnosis or self-treatment based on the measurement results and analysis. Always consult your doctor. Self-diagnosis or self-treatment may lead to deterioration in your condition.
-  Do not use this device during an MRI scan.

IMPORTANT SAFETY NOTES

WARNING:

-  Do not use this device on people with sensitive skin or allergies.
-  Do not attempt to use the device over or through clothing.
-  Do not attempt to use the device with your left hand.
-  When you experience symptoms potentially related to heart disease always consult your physician, even if “Regular heart rate, Regular heart rhythm” is displayed.
-  If battery fluid should get in your eyes, immediately rinse with plenty of clean water. Consult a doctor immediately.

CAUTION:

-  Do not measure if dust is on the electrodes. Clean them first.
-  If the portion of the body where the electrode is applied has too much body fat, a successful reading may not be possible.
-  Do not measure with the electrode upside down (Refer to the illustration on page 26).
-  Do not expose the unit to static electricity. Always disperse any static electricity from your body before handling the unit.
-  Do not take measurements in a moving vehicle.
-  Do not use a cellular phone near the unit.
-  Do not disassemble, repair, or modify the unit.
-  Do not insert batteries with their polarities reversed.
-  The chest electrode will be impeded by excessive body hair. Regularly remove hair at the electrode contact point. Omron does not recommend the use of contact gel.
-  Keep out of reach of infants, small children, and compromised people who cannot express their consent.

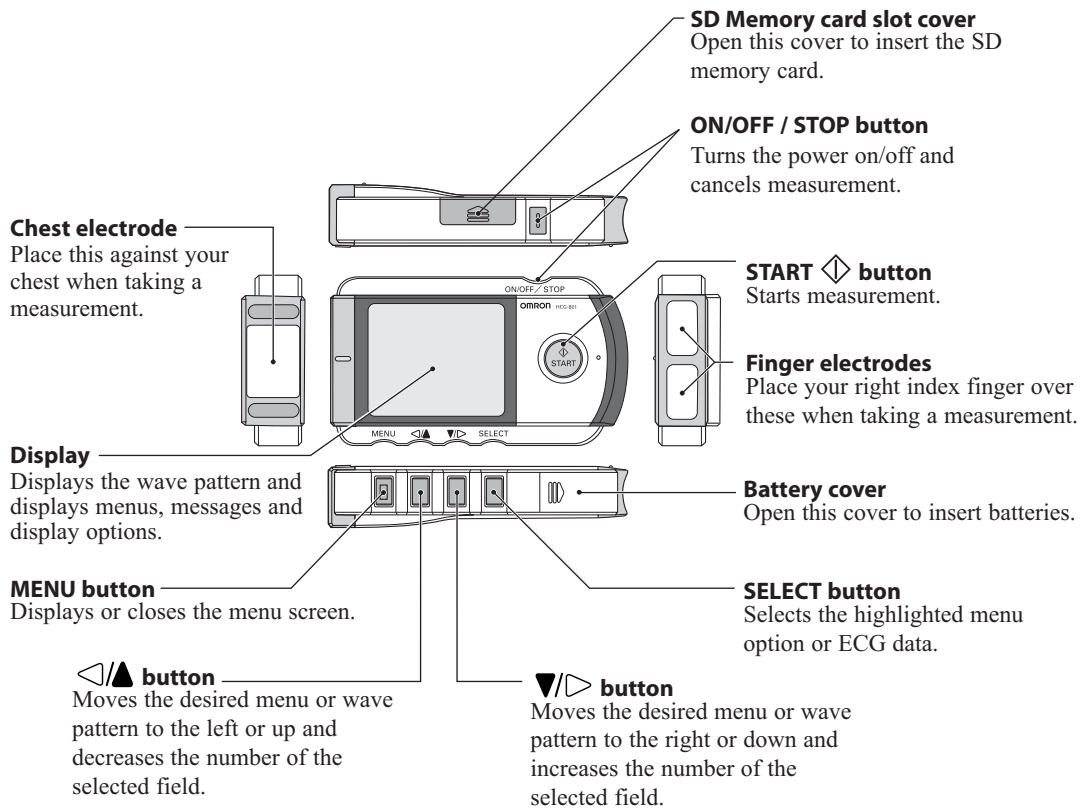
IMPORTANT SAFETY NOTES

 **CAUTION:**

-  Do not use for any purpose other than obtaining an electrocardiograph.
-  Do not expose the unit to strong shocks or vibrations. Do not drop or step on the unit.
-  Do not use batteries of a type other than that specified in this manual.
-  Do not take measurements when there are drops of water on your skin, such as from sweat or after bathing.
-  Do not take measurements where the unit will be exposed to strong electromagnetic forces.
-  If the batteries are removed for more than one hour, the settings and any ECG data stored in the unit may be erased. If the date and time setting is erased, be sure to reset it before using the unit again. Any ECG data erased from the unit cannot be recovered.
-  Avoid extremes in temperature and humidity. Do not use this unit in locations subject to high or low temperatures or humidity. Use at a temperature within +50 to +104°F (+10 to +40°C) and between 30 and 85%RH.
-  Do not sterilize this unit in an autoclave or gas sterilizer (EOG, formaldehyde, high density ozone, etc.)
-  Do not wash this unit with water.
-  Do not store the unit in the following ambient conditions:
 - Locations exposed to direct sunlight.
 - Locations subject to high temperatures and high humidity.
 - Wet or damp locations where water may get on the unit.
 - Dusty locations.
 - Near fires or open flames.
 - Locations exposed to strong vibration.
 - Locations exposed to strong electromagnetic fields.

KNOW YOUR UNIT

MAIN UNIT:

KNOW YOUR UNIT

MAIN UNIT

Display:

The screen is used to display basic instructions and results of ECG readings, to adjust settings, and to review ECG data stored on the SD memory card or in unit memory.

Menus

Menu Screen

The main menu screen is displayed when you press the MENU button.

SET menu

The SET menu is displayed when you select the SET menu and press the SELECT button.

GRAPH menu

This allows you to view ECG readings stored on the SD memory card or in unit memory. (page 30)

SET menu

This allows you to set the date and time, and adjust other settings. (page 32)

GUIDE

This displays the guidance screens that show you how to take a measurement.

Back

This option takes you back to the previous screen, from the SET menu.

Close

This option closes the menu screen.

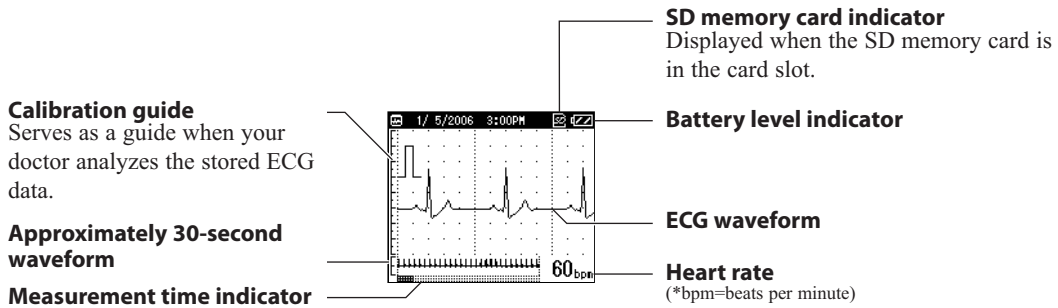
KNOW YOUR UNIT

MAIN UNIT

Display:

ECG Graph

This screen is displayed when you use the GRAPH menu to select ECG data stored on the unit.


NOTE: During measurement, the screen displays the ECG waveform. The progress bar at the bottom of the screen displays measurement progress, and the number on the right shows the heart rate. Depending on the waveform, situations can occur in which the unit cannot assess and display the heart rate during the measurement.

KNOW YOUR UNIT

COMPONENTS:

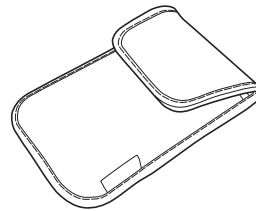- SD memory card
(801SDMEMORY)

- Storage pouch
(801STORAGEPOUCH)

- Instruction Manual
(HCG801INST)

- 2 LR03(AAA)
Batteries

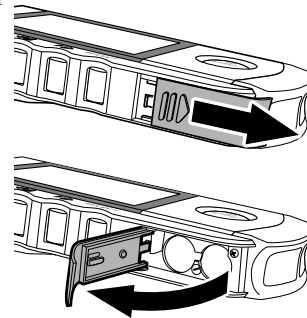
- Quick Reference Card

- Registration Card

HOW TO INSERT AND REPLACE BATTERIES


1. Slide the battery cover in the direction of the arrow, and lift the end of the cover upwards.

NOTE: Do not pull hard on the cover.


2. Insert two LR03(AAA) batteries so the + (positive) and the - (negative) polarities match the polarities on the battery compartment as illustrated.

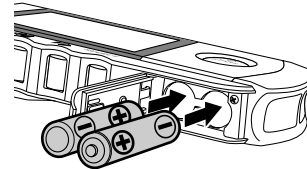NOTE: Always use LR03(AAA) batteries.

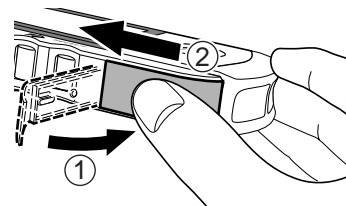3. Close the battery cover.
Lower the battery cover (①), then while gently pressing the cover down, slide it in the direction of the arrow (②).

HOW TO INSERT AND REPLACE BATTERIES

BATTERY LIFE AND REPLACEMENT

When the “Please insert new batteries.” message appears, replace both batteries with new ones.

- Always turn the unit off before replacing the batteries.
- Dispose of the used batteries according to applicable local regulations.

Battery life

- Two new LR03(AAA) batteries will last for approximately 400 measurements, if measurements are taken once a day at room temperature (72°F (22°C)).
- The batteries enclosed in the package are intended for the purpose of demonstration. It is possible these batteries will not last for 400 measurements.
- The battery life can be confirmed in the upper right of the display.

 Full power.	 Half remaining.	 Remaining power is low. Replace the batteries with new ones.
--	--	---

 **WARNING:** If battery fluid should get in your eyes, immediately rinse with plenty of clean water. Consult a doctor immediately.

 **CAUTION:**

- **If the batteries are removed for more than one hour, the settings and any ECG data stored in the unit may be erased. If the date and time setting is erased, be sure to reset it before using the unit again. Any ECG data erased from the unit cannot be recovered.**
- **Do not use batteries not specified for this unit.**
- **Do not insert the batteries with the polarities in the wrong direction.**
- **Do not dispose of batteries in fire.**
- **If battery fluid should get on your skin or clothing, immediately rinse with plenty of clean water.**
- **Remove the batteries from this unit when you are not going to use it for a long period of time (approximately three months or more).**
- **Do not use batteries of different types together.**
- **Do not use new and used batteries together.**

HOW TO ATTACH THE BATTERY COVER

If the battery cover is accidentally detached, attach the battery cover as described in the procedure below.

1. Align the pin of the battery cover with the groove at the top of battery compartment on the START  button side (front) of the unit (①).

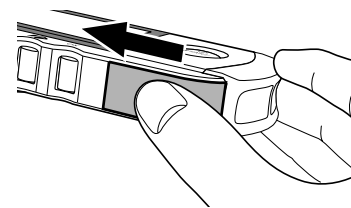
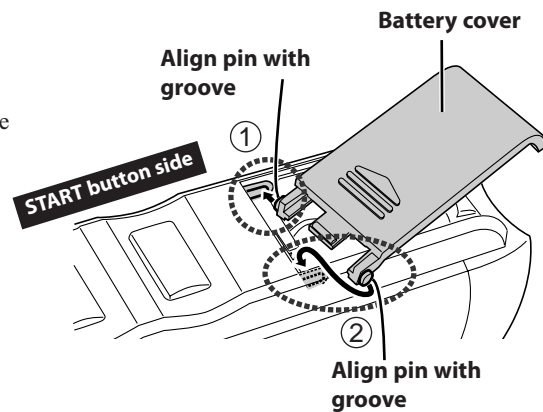
2. Align the other pin of the battery cover with the groove on the other side of the battery compartment (②).

While gently pushing the pin already inserted in the START  button side (front), insert the pin in the groove on the opposite side of the battery compartment.

NOTE: Do not push the pin in the START  button side of the battery compartment hard. This could result in damage to the battery cover.

3. Close the battery cover.

While pressing the battery cover, slide it in the direction of the arrow.


HOW TO SET THE DATE AND TIME

Always set the date and time before using the unit for the first time.


Check that date and time are correct before using the unit, reset them if necessary. The date and time are important indicators of when a measurement was taken.

NOTE: The date and time format can be changed.
(Refer to “Date Format” on page 38)

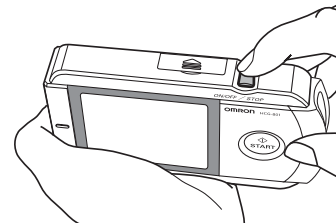
1. Press the ON/OFF / STOP button to turn the unit on.

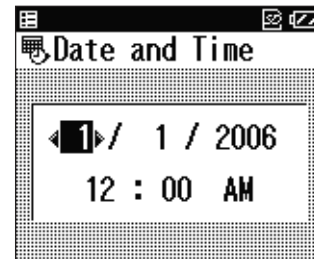When the unit is used for the first time, or if the Date and Time setting has been reset, the Date and Time setting screen is automatically displayed, and the month digits are highlighted.

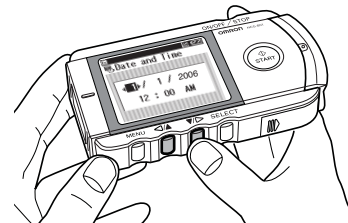2. Press the ▼/▶ button or ◀/▲ button to set the month.
Press the ▼/▶ button to increase the number.
Press the ◀/▲ button to decrease the number.

HOW TO SET THE DATE AND TIME

3. Press the SELECT button to confirm the month setting.

The month setting is set and the day setting is highlighted.

The display cycles through the settings,

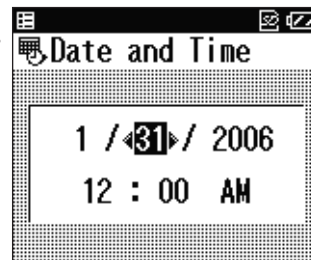
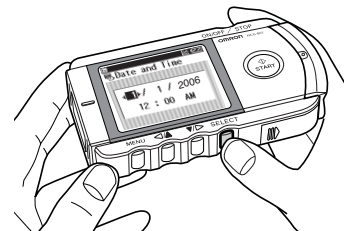
month -> **day** -> **year** -> **hour** -> **minute** -> **AM/PM**, with each press of the SELECT button.

NOTE: If the setting for the date format has been changed (refer to “Date Format” on page 38), the order in which items are highlighted may differ from the example above.

4. In the same way, set the day, year, hour, minute, and AM/PM.

After you have confirmed the last item on the date and time setting, press the SELECT button to complete the date and time setting.

NOTE: If you need to set the date and time again, refer to page 34.


5. Press the ON/OFF / STOP button for two seconds to turn off the power.

The unit beeps, then the power is turned off.

NOTE: If you forget to turn off the power, the unit will turn itself off automatically after five minutes.

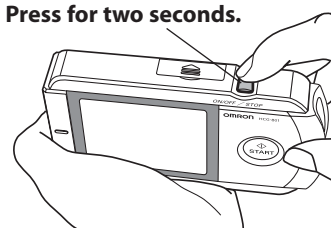Press for two seconds.

HOW TO USE THE SD MEMORY CARD

The unit includes an SD memory card for storing ECG data, which can then be viewed by your doctor later. This section describes how to insert and eject an SD memory card, and how to format a card so it can be used to store ECG data.

NOTES:

- If you need to purchase a new SD memory card, contact Omron Healthcare Customer Service 1-800-829-6427.
- Do not store any data on the card from other devices rather than this unit.
- ECG data stored on an SD memory card can be damaged by static electricity and other causes. Copy data to PC or other device for additional safety.

INSERTING AND EJECTING SD MEMORY CARD

The SD (SD) mark is displayed in the top right corner of the screen while the SD memory card is inserted, and the unit is turned on.


Inserting the SD memory card

1. Check that the unit is turned off.

 **CAUTION: Never insert an SD memory card while the unit is turned on.**

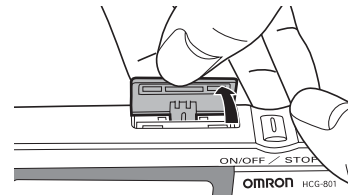HOW TO USE THE SD MEMORY CARD



2. Open the SD memory card slot cover and insert the SD memory card into the slot.


NOTES:

- Check that the card is correctly aligned and insert it into the slot as shown.

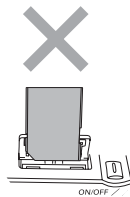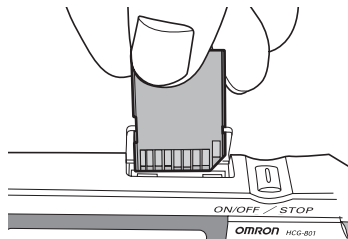
CORRECT ALIGNMENT

Back to front

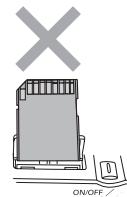Upside down

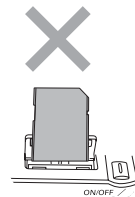
Back to front and upside down

- Insert the card until it clicks into position.

Write Protection (Locking the SD Memory Card)

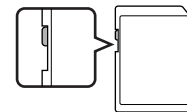
- Make sure that the write-protect switch is not locked, before inserting the SD memory card. If the write-protect switch is locked (down), you will not be able to store new ECG data on the SD memory card.

⚠ CAUTION: When inserting the SD memory card, do not try to force the card into the slot if it is upside down, back to front, etc. This could damage the card or the unit.

HOW TO USE THE SD MEMORY CARD

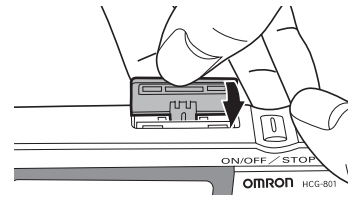3. Close the SD memory card slot cover.

HOW TO USE THE SD MEMORY CARD

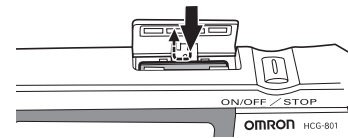
To eject the SD memory card

1. Check that the unit is turned off.


⚠ CAUTION: Eject an SD memory card only when the unit is turned off.

2. Open the SD memory card slot cover and press the card.

The lock is released and the card is ejected slightly.

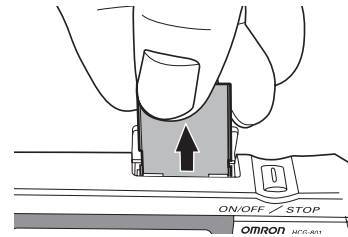3. Remove the SD memory card.

4. Close the SD memory card slot cover.

HOW TO USE THE SD MEMORY CARD

FORMATTING AN SD MEMORY CARD

You will need to format the SD memory card before you use it for the first time, and in the following circumstances.

- If you use an SD memory card that has been used by other devices.
- If the SD memory error occurs.

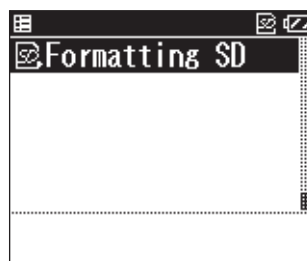
NOTES:

- When you format an SD memory card, any data stored on the card is erased. Always copy any data you wish to keep to a computer before formatting an SD memory card.
- When you format an SD memory card, format it with the unit. Do not format an SD memory card with a computer.

1. Press the MENU button, press the  button to select **SET menu**, then press the SELECT button.

2. Press the  button to scroll through the settings and select **Formatting SD**.


HOW TO USE THE SD MEMORY CARD

3. Press the SELECT button.

The message “All data in SD will be deleted. Format SD?” is displayed.

4. Press the  button to select **YES**, then press the SELECT button.

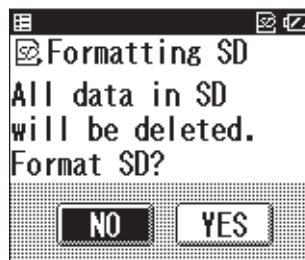If you press the SELECT button while **NO** is selected, the SD memory card will not be formatted.

* By default, **NO** is Highlighted.

5. Press the SELECT button.

A message warning you not to remove the SD memory card while it is being formatted is displayed and the SD memory card is formatted.

Formatting can take up to eight seconds to complete.

 **CAUTION: Never remove an SD memory card while it is being formatted.**

IMPORTANT NOTES ON TAKING A MEASUREMENT

In order to obtain a good ECG reading, it is important that measurements are taken correctly.

Before taking a measurement for the first time, read these instructions carefully and make sure you observe the following points each time you take a measurement.

- Make sure that the electrodes are directly touching your skin.
- If your hands or skin is dry, wipe them with a damp towel so that they are slightly moist.
- If the electrodes are dirty, wipe any dirt off with a soft cloth or a cotton swab moistened with disinfectant alcohol.
 - Do not use too much disinfectant alcohol when cleaning the electrodes.
 - Do not use disinfectant alcohol to clean the unit except the electrodes.
- Do not move during measurement.
 - Movement including talking, coughing, or sneezing during measurement can affect the measurement results.
- If the position of your finger or arm is inappropriate during measurement, the results may be inaccurate, so be sure to check their positions before taking a measurement. (Refer to page 25.)
- After taking a measurement, disinfect the electrodes by wiping them with a soft cloth or cotton swab moistened with disinfectant alcohol. If you do not disinfect the electrodes there is a risk of skin disease.

NOTE: If you have difficulty applying the unit so that the chest electrode is flat against your chest, please ask your doctor for advice.

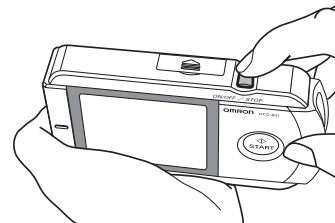
HOW TO TAKE A MEASUREMENT

1. Press the ON/OFF / STOP button to turn the unit on.

The unit displays the following reminder. “Self-measurement is not the same as medical diagnosis. Please consult your doctor for proper interpretation.” Next, the guidance screens are displayed.

NOTES:

- You can press the START  button to begin a measurement even while the guidance screens are being displayed.
- The word “electrode” in this Instruction Manual is described as “sensor” in the guidance screen.

2. Make sure that you are in a relaxed position.

The ideal posture is one where you are seated comfortably with your back straight.

NOTE: You can also take a measurement while you are standing or while lying down.

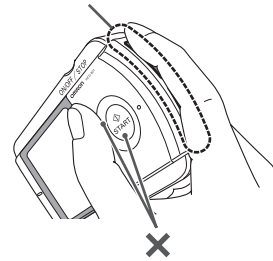
3. Place your index finger so that it fits closely over the two finger electrodes.

NOTE:

- Take care not to press the START  button until you are ready to take the measurement.

⚠ WARNING: Even if you are left-handed, use your right hand to take a measurement.

Fit finger closely on the electrodes.

Do not press START  button until ready to take measurement.

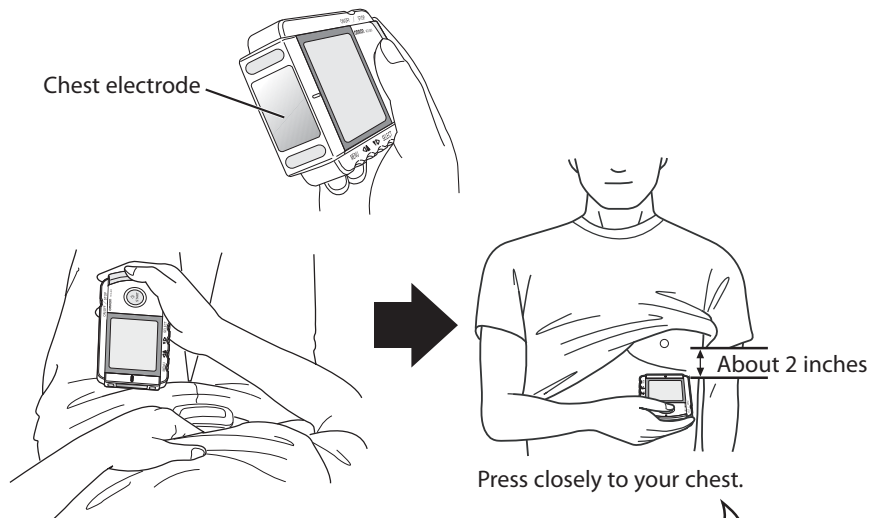
HOW TO TAKE A MEASUREMENT

4. Place the chest electrode on bare skin about 2 inches below your left nipple (IMPORTANT - see below for female application).

NOTE:


- Make sure that you are relaxed and that your chest is free of tension.

⚠ WARNING: Press the chest electrode close against your bare chest. Do not place the electrode on clothing as this will result in an inaccurate measurement.

Female:

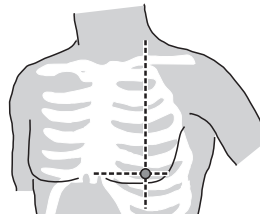
Locate the ECG monitor at the lower end of the breastbone and move horizontally to the left to the middle of the left part of the chest. If necessary, gently lift the breast and place the Chest Electrode under the left breast. Unless the chest electrode comes in contact with the brassiere, it is not necessary to remove it; if you are unsure of exact placement, **please consult your doctor.**


HOW TO TAKE A MEASUREMENT

5. While maintaining the same posture, press the START  button.

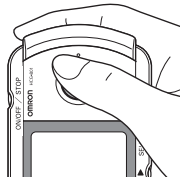
Measurement starts

During measurement

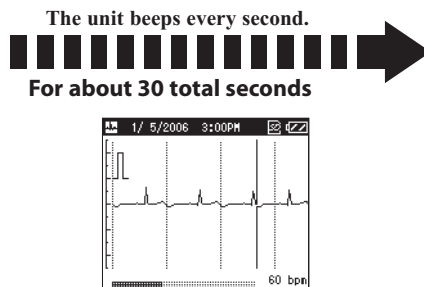Measurement ends

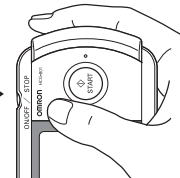The measurement ends automatically, indicated by 4 rapid beeps.

The measurement takes about 30 seconds to complete. The unit beeps every second during measurement.

During measurement, the screen displays the ECG waveform. The progress bar at the bottom of the screen displays measurement progress, and the number on the right shows the heart rate.

Depending on the waveform, situations can occur in which the unit cannot assess and display the heart rate during the measurement.

NOTES:

- During measurement the unit will only respond to the ON/OFF / STOP button to discontinue the measurement.
- If the contact of the electrodes becomes loose, or the conditions change during measurement, the measurement may not be correct.

IMPORTANT: Keep still and do not move until the measurement is complete.

HOW TO TAKE A MEASUREMENT

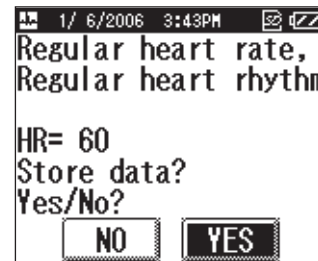
A data description message is displayed when the measurement has been completed. (Refer to page 48 - 49 for details.)

When the SD memory card is inserted, measurement results will be saved preferentially on the card.

Otherwise, they will be saved in unit memory.

NOTES:

- “Store date? Yes/No?” might not be displayed. (Refer to “ACTION MESSAGES” on page 49 for details.)
- A data description message can be selected not to display. (Refer to page 39 for details.)

Example of data description message

If you press the SELECT button while the data description message is displayed, the list of ECG data stored on the SD memory card or in unit memory is displayed.

NOTE: If the “Please insert new batteries.” error message is displayed immediately after measurement, the ECG data may not be stored. If this happens, repeat the measurement after replacing the batteries.

6. Press the ON/OFF / STOP button for two seconds to turn the unit off. The unit beeps, then the power is turned off.

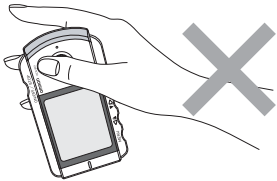
NOTE: If you forget to turn the unit off, it will automatically turn off after five minutes.

HOW TO TAKE A MEASUREMENT


COMMON CAUSES OF INACCURATE MEASUREMENT RESULTS

The following situations can result in inaccurate measurement results or errors.


- The index finger is not placed firmly on the electrodes.


- The unit is held incorrectly.

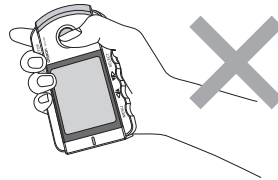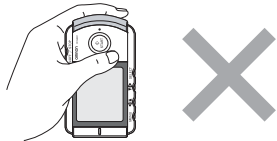
- The unit is being held in the left hand. Always use your right hand to hold the unit when taking a measurement.


- Unit placed on clothing.

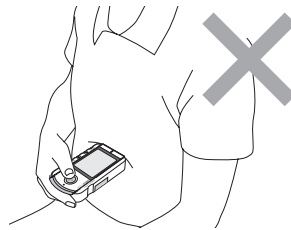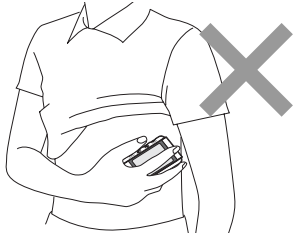- The rear of the unit is placed against the chest.

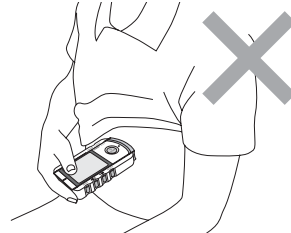- The chest electrode is against the finger and the finger electrode is against the chest.



HOW TO DISPLAY ECG DATA

The **GRAPH menu** is used to display the list of ECG data stored on the SD memory card or in unit memory.

1. Press the MENU button while the unit is turned on, the **GRAPH menu** is highlighted. If you then press the SELECT button, the list of ECG data is displayed.

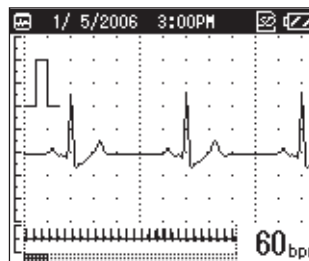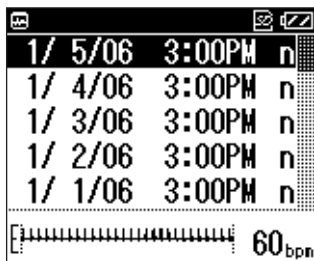2. Select the ECG data you wish to view from the **GRAPH menu** list, then press the SELECT button to display the detailed view of the waveform.

Each stored ECG data is displayed by date, time and a character representing the measurement data description. Refer to page 48 - 49 for an overview of possible characters and messages.

HOW TO DISPLAY ECG DATA

Press the $\blacktriangledown/\blacktriangleright$ button to move down the list, and the $\blacktriangleleft/\blacktriangle$ button to move up.

The most recent measurement result is displayed at the top of the list and is automatically highlighted, with any previous measurement results listed below it in chronological order.

When the SD memory card is in the SD memory card slot, the list of ECG data stored on the card is displayed.

If there is no SD memory card in the card slot, the ECG data stored in unit memory is displayed.

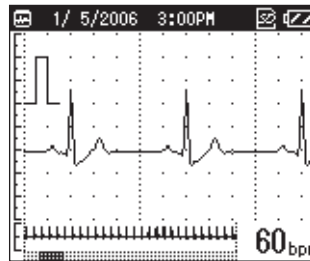
3. If you press the $\blacktriangledown/\blacktriangleright$ button, you can scroll through the ECG waveform measurement.

Press the $\blacktriangleleft/\blacktriangle$ button to return to the previous waveform display.

Two waveforms are displayed (the lower, smaller, waveform being a view of the whole 30-second measurement.)

At the bottom of the screen is a bar with shaded area which indicates the section of the ECG waveform displayed.


The heart rate is shown in the lower right corner of the screen.

4. Press the SELECT button again to view the data description messages and heart rate.

NOTE: You can specify whether a data description message is displayed or not. Refer to page 39 for details.

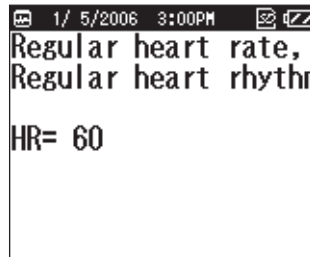5. Press the SELECT button once more to return to the **GRAPH** menu list.

HOW TO ADJUST THE SETTINGS

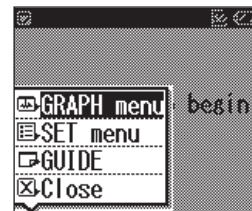
You can use the **SET menu** to adjust various settings.

You can press the **START**  button to begin measurement even while the **SET menu** is displayed.

BASIC OPERATION

The following procedure describes how to select and adjust a setting. Refer to the following pages for details of the options available for each setting.

1. Press the **MENU** button, while the unit is on, to display the menu screen.

* Initially, **GRAPH menu** is highlighted.

2. Press the  button to select (highlight) **SET menu**.

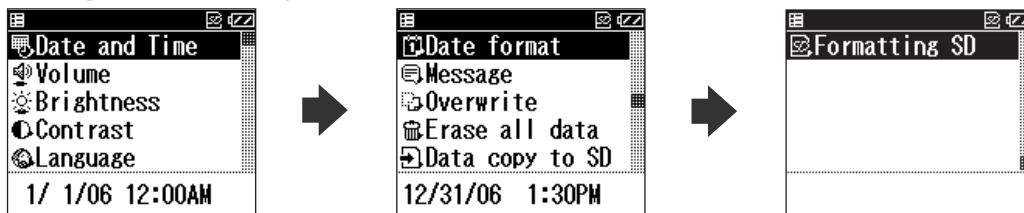
3. Press the **SELECT** button.

Initially, the Date and Time setting is highlighted.

HOW TO ADJUST THE SETTINGS

4. Press the ◀/▲ button or ▼/▷ button to select the desired setting, then press the SELECT button to see the options for that setting.

5. Adjust the settings then press the SELECT button to confirm the changes.

The **SET menu** can be used to adjust the following settings or to organize ECG data stored on the SD memory card or in unit memory:

Date and Time, Volume, Brightness, Contrast, Language, Date format, Message, Overwrite, Erase all data, Data copy to SD, Formatting SD.

Please see the following pages for details on how to change each of these settings.

NOTES:

- Refer to “How to Erase, Save and Organize ECG Data” (pages 41-47) for details on **Erase all data** and **Data copy to SD**.
- Refer to “How to Use the SD Memory Card” (pages 22-23) for details on **Formatting SD**.
- To return to the menu screen from the **SET menu** screens, press the MENU button. To return to the SET menu screen from a setting screen, press MENU button, and highlight Back, then press the SELECT button.
- If you press the MENU button without first pressing the SELECT button, after changes a setting, the changes to the setting will not be implemented.

HOW TO ADJUST THE SETTINGS

DATE AND TIME

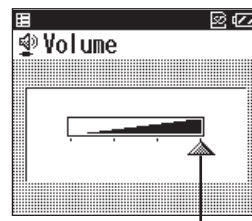
Select this item if you need to reset the date and time. For details on setting the date and time, refer to page 16.

NOTE: You will be prompted to set the date and time when you first insert the batteries.

VOLUME

The current volume level is displayed at the bottom of the screen.

The  mark indicates the volume level.
* The default setting is the maximum volume level.

Press the / button or / button, to adjust the volume.

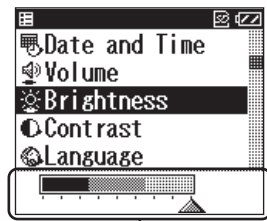
The volume can be set to one of four levels.

The volume increases as the  mark moves to the right.

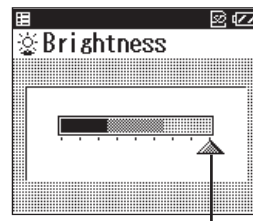
The sound is muted (off) when the  mark is to the far left.

HOW TO ADJUST THE SETTINGS

BRIGHTNESS

The current brightness level is displayed at the bottom of the screen.

The  mark indicates the brightness level.

* The default setting is the maximum brightness level.

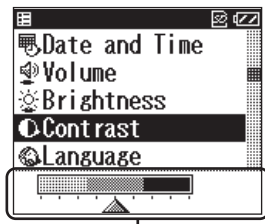
Press the   button or   button, to adjust the brightness.

The brightness can be set to one of nine levels.


Brightness increases as the  marker moves to the right.

HOW TO ADJUST THE SETTINGS

CONTRAST

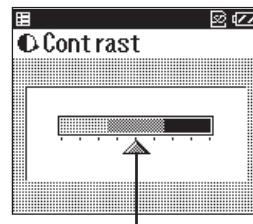The current contrast level is displayed at the bottom of the screen.

The  mark indicates the contrast level.
* The default setting is the medium contrast level.

Press the / button or / button to adjust the contrast.

Contrast can be set to one of nine levels.

Contrast increases as the  marker moves to the right.

HOW TO ADJUST THE SETTINGS

LANGUAGE

The currently selected language is displayed at the bottom of the screen.

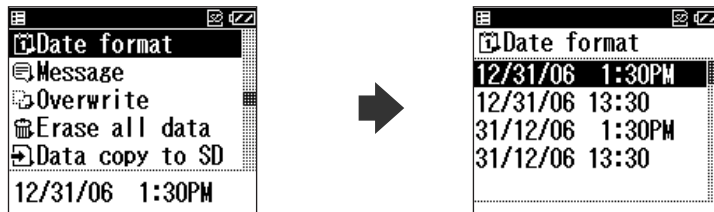
* The default language is English.

Press the /▲ button or ▼/▶ button to select the desired language, then press the SELECT button to confirm your selection.

HOW TO ADJUST THE SETTINGS

DATE FORMAT

You can select one of four formats, MM/DD/YY HH:MM (AM/PM), MM/DD/YY HH:MM (24 hour clock), DD/MM/YY HH:MM (AM/PM), or DD/MM/YY HH:MM (24 hour clock).

The date is displayed at the bottom of the screen using the currently selected format.

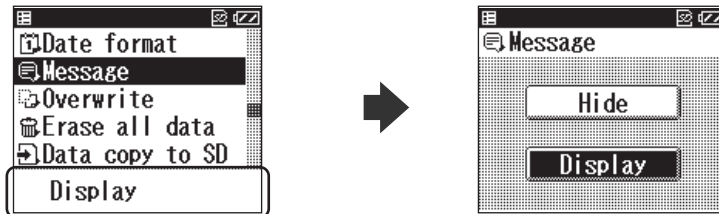
* The default date format is MM/DD/YY HH:MM (AM/PM).

Press the $\triangleleft/\blacktriangle$ button or $\blacktriangledown/\triangleright$ button to select the desired format, then press the SELECT button to confirm your selection.

HOW TO ADJUST THE SETTINGS

MESSAGE

This setting specifies whether to display or hide the data description message.

The current setting is displayed at the bottom of the screen.

* The default setting is Display.

Press the ◀/▲ button or ▼/▶ button to highlight the **Display** or **Hide**, then press the SELECT button to confirm your selection.

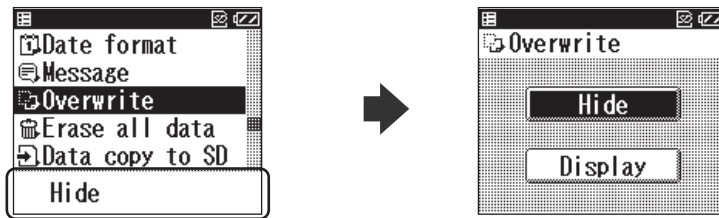
NOTES:

- Even if Hide is selected, data description messages are saved in unit memory or on the SD memory card. (Refer to page 28 for details.)
- When Display is selected again, all the saved data description messages will be displayed.

HOW TO ADJUST THE SETTINGS

OVERWRITE

The unit will overwrite the oldest stored reading when the SD memory card or unit memory is full. A message will warn that this will happen, you can choose to hide or display this message. (The default setting is to hide the message.)

The current setting is displayed at the bottom of the screen.

* The default setting is Hide.

Press the $\triangleleft/\blacktriangle$ button or $\blacktriangledown/\triangleright$ button to highlight **Display** or **Hide**, then press the SELECT button to confirm your selection.

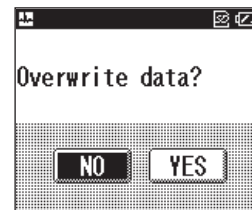
⚠ CAUTION: While **Hide** is selected, the unit will overwrite (erase and replace) the oldest measurement data with the results of the new measurement without displaying a message (when the SD memory card or unit memory is full).

ABOUT THE OVERWRITE MESSAGE

If you select **Display**, the unit will display a message before you take a measurement, asking if you want to overwrite existing data with the new measurement data.

Select **Yes** to overwrite the oldest ECG data with the results of the new measurement.

If you select **No**, measurement is cancelled.

HOW TO ERASE, SAVE AND ORGANIZE ECG DATA

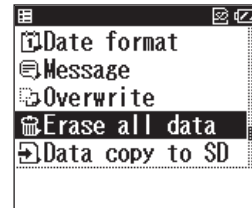
ERASING ECG DATA

Erasing All ECG Data

You can use the **SET menu** to erase all the ECG data stored on the SD memory card or in unit memory.

NOTE: If an SD memory card is inserted in the unit, all data on the card will be erased.


1. Press the **▼/▶** button to scroll through the settings and select **Erase all data**.

2. Press the **SELECT** button.

The message “**Erase all data in SD memory?**” is displayed.

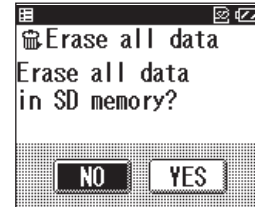
NOTE: If the SD memory card is not inserted and measurements are being stored in unit memory, the message “**Erase all data in the device?**” is displayed.

* By default, NO is highlighted.

HOW TO ERASE, SAVE AND ORGANIZE ECG DATA

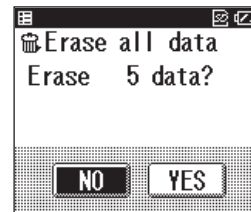3. Press the **▼/▷** button and select **YES**, then press the SELECT button.
The number of sets of data to be deleted is displayed.

4. Press the **▼/▷** button and select **YES**, then press the SELECT button.
All the ECG data is erased and the screen from step 1 appears.

NOTE: If you press the SELECT button after selecting **NO**, the screen from step 1 appears.

Erasing Selected ECG Data

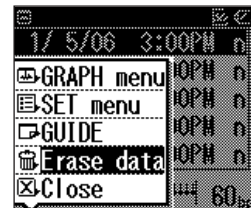
Use the **GRAPH menu** if you only want to delete a selected set of ECG data.

For details on displaying the **GRAPH menu**, follow step 1 in “How to Display ECG Data” on page 30.

1. Press the **▼/▷** button to highlight the data you want to erase.

2. Press the MENU button.

The menu screen is displayed.

3. Press the **▼/▷** button and select **Erase data**.

HOW TO ERASE, SAVE AND ORGANIZE ECG DATA

4. Press the SELECT button.

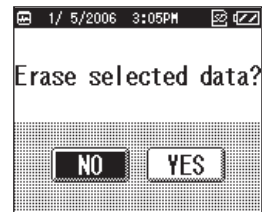
The confirmation message “**Erase selected data?**” is displayed.

5. Press the ▼/▷ button to select **YES**, then press the SELECT button.

The ECG data is erased and the screen from step 1 appears.

NOTE: If you press the SELECT button after selecting **NO**, the ECG data is not erased and the screen from step 1 appears.

⚠ CAUTION: Never remove an SD memory card while data is being erased.

* By default, NO is highlighted.

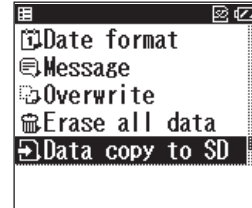
HOW TO ERASE, SAVE AND ORGANIZE ECG DATA

COPYING DATA TO AN SD MEMORY CARD

You can copy ECG data from the unit's internal memory to an SD memory card. This allows you to copy data to the SD memory card if measurements have been taken while the SD memory card was not inserted.


NOTE: The unit's internal memory can only store a maximum of five measurement results.

1. Press the ▼/▶ button and select **Data copy to SD**.

2. Press the SELECT button.

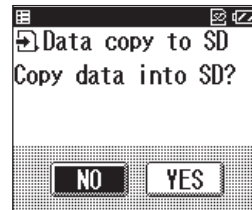The confirmation message "Copy data into SD?" is displayed.

3. Press the ▼/▶ button to select **YES**.

NOTES:

- If the SD memory card is full, the "Overwrite data" message may appear. Refer to page 40 for details.
- ECG data stored on an SD memory card can be damaged by static electricity and other causes. Copy data to PC or other device for additional safety.

*** By default, NO is highlighted.**

HOW TO ERASE, SAVE AND ORGANIZE ECG DATA

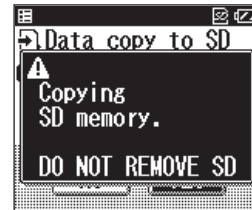
4. Press the SELECT button.

The ECG data is copied to the SD memory card.

⚠ CAUTION: Never remove the SD memory card while data is being copied to it. If the SD memory card is removed while data is being copied, the data may be lost or damaged.

NOTES:

- The copied data is displayed and the screen from step 1 appears.
- If the same ECG data already exists on the SD memory card, the data is not copied.

HOW TO ERASE, SAVE AND ORGANIZE ECG DATA

COPYING DATA FROM THE SD MEMORY CARD TO A PC

Please follow these instructions to copy data from the SD memory card to a PC.

1. Make sure the PC is equipped with an SD memory card port or connect a commercially available SD memory card reader/writer to your PC.
2. Create a new folder on your PC where you want to save a copy of the ECG data.
3. Insert the SD memory card and browse to the SD memory card data as explained in the instruction manual of your PC and /or software. The ECG data is located in the *:\hcg801\data\ (where * stands for the character of a removable disk).
4. Select all files and copy them in the folder you have created earlier (step 2).
5. When copying is finished, remove the SD memory card.
6. The SD memory card can now be formatted as described in “Formatting an SD Memory Card” on page 22, if you wish to delete data on the SD card.

HOW TO ERASE, SAVE AND ORGANIZE ECG DATA

COPYING DATA FROM A PC TO THE SD MEMORY CARD

Please follow these instructions to copy data from a PC to the SD memory card.

1. Make sure the PC is equipped with an SD memory card port or connect a commercially available SD memory card reader/writer to your PC.
2. Insert the SD memory card in the SD port of your PC and confirm that there is an “hcg801” folder in the SD memory card. If there is not, format the SD memory card in the HCG-801 as described in “Formatting an SD Memory Card” on page 22.
3. Browse to the folder where you saved the ECG data in the PC. Select all files and copy them to the SD memory card in the *:\hcg801\data\ (where * stands for the character of a removable disk)
4. When copying is finished, insert the SD memory card in the HCG-801 as described in “Inserting and Ejecting SD Memory Card” on page 18.
5. Do not store more than 300 measurement files in this folder on the SD memory card. When more than 300 files are copied in this folder, the HCG-801 will display an error message.

MESSAGES

This section includes a list of all messages that may appear on the display. Refer to these if you are not sure what a displayed message means.

DATA DESCRIPTION MESSAGES

After finishing a measurement, the unit will display a message describing the data from the measurement.

A letter of the alphabet indicates each of the possible messages (see list below). This letter is displayed together with the time and date of the measurement when the **GRAPH** menu is displayed.

 **WARNING:**

- **Never interpret measurement results yourself. Always consult your doctor.**
- **These messages support heart rate and heart rhythm only. Waveform changes especially those related to ischemic heart conditions are not supported.**

n	Regular heart rate, Regular heart rhythm
v	Slow heart rate.
w	Fast heart rate.
x	Irregular heart rhythm.
y	Slow heart rate, Irregular heart rhythm.
z	Fast heart rate, Irregular heart rhythm.
m	Analysis impossible. Please measure again

MESSAGES

CRITERIA OF DATA DESCRIPTION MESSAGES

MESSAGES	CONDITIONS
Fast heart rate	Fast (more than 100 BPM)
Slow heart rate	Slow (less than 60 BPM)
Irregular heart rhythm	no R-waves detected for 2 seconds and more
	R-R variation rate is 10% and above
	R-R interval is less than 80% of the normal intervals
	when VPC detected
Analysis impossible. Please measure again	Noise
	R-R regulation cannot be analyzed
Regular heart rate, Regular heart rhythm	None of the above

HEART RATE MESSAGE

HR - In addition to the above messages the unit will display the heart rate in beats per minute.

ACTION MESSAGES

Finally, the unit will display one of the following messages:

Store data? Yes/No?

This message will be displayed when the unit can find no irregularities (see Data Description Message “n” on page 48) or when the unit cannot measure properly (see Data Description Message “m” on page 48).

This data can be saved by selecting Yes and pressing the SELECT button.

Data is stored. Please consult your doctor.

This message will be displayed when the unit finds an irregularity (see Data Description Messages “v through z” on page 48). Always write down your symptoms when you get this message. You will find a form for tracking your symptoms in the back of this manual. The unit will automatically store the data. Contact your doctor to determine the data’s significance.

MESSAGES

LISTS AND EXPLANATIONS OF MESSAGES

Notice Messages

MESSAGE	CAUSE	CORRECTION
Analyzing.	The ECG data is being analyzed.	No action required. Wait until analysis ends.
Cannot measure. Please follow the instructions for use.	An ECG wave-pattern could not be detected.	Check the instructions and repeat measurement.
Measurement was stopped. Push START to begin measurement again.	The measurement was interrupted or cancelled.	Press START  to resume the measurement.
Copied  *1 data into SD.	The indicated ECG data has been copied to an SD memory card.	No action required. Wait until the message disappears.
Erased  *2 data.	The indicated ECG data has been erased.	

*1: Number of readings from 0 to 5. The number displayed represents the amount of copied ECG data.

*2: Number of readings from 1 to 300. The number displayed represents the amount of erased ECG data.

Warning Messages

MESSAGE	CAUSE	CORRECTION
Formatting SD memory. DO NOT REMOVE SD	The SD memory card is being formatted.	Wait until the message disappears. Do not remove the SD memory card until formatting has been completed.

MESSAGES

MESSAGE	CAUSE	CORRECTION
Storing on SD memory. DO NOT REMOVE SD	ECG data is being written (saved) to the SD memory card.	Wait until the message disappears. Do not remove the SD memory card until data storage has been completed.
Copying SD memory. DO NOT REMOVE SD	ECG data is being copied to the SD memory card.	Wait until the message disappears. Do not remove the SD memory card until data copying has been completed.
Data in device memory is being erased.	ECG data stored on the unit is being erased.	Wait until the message disappears.
Data in SD is being erased. DO NOT REMOVE SD until erasing finishes.	ECG data stored on the SD memory card is being erased.	Wait until the message disappears. Do not remove the SD memory card until data erasing has been completed.
Reading data... DO NOT REMOVE SD.	The unit is loading ECG data stored on the SD memory card.	Wait until the message disappears. Do not remove the SD memory card until the data has been loaded by the unit.
No SD is inserted.	There is no SD memory card in the card slot.	Turn off the power and insert the SD memory card. Never insert the SD memory card while the power is turned on.
SD is write-protected.	The unit is unable to access data because the SD memory card inserted is write-protected.	Refer to the instruction supplied with the memory card and insert the SD memory card with write-protection turned off.
No saved data.	The battery power has run low and the ECG data stored on the unit has disappeared.	Insert new batteries. Reset the Date and Time and other settings. Any ECG data that was stored on the unit cannot be recovered.

MESSAGES

MESSAGE	CAUSE	CORRECTION
Set clock.	The battery power has run low and the Date and Time setting on the unit has disappeared.	Insert new batteries and reset the Date and Time from the SET menu .
No saved data. Set clock.	The battery power has run low and the ECG data stored on the unit has disappeared or the clock has been reset.	Insert new batteries and reset the Date and Time and other settings.

Error Messages

ERROR #	MESSAGE	CAUSE	CORRECTION
01	Please insert new batteries.	The batteries are worn out.	Replace both batteries with new ones.
11	Device memory error. Cannot read data.	An error occurred while trying to read ECG data stored in unit memory.	Contact Omron Healthcare Customer Service at 1-800-829-6427.
12	Device memory error. Cannot erase data.	An error occurred while trying to erase ECG data stored in unit memory.	
13	Device memory error. Cannot write data.	An error occurred while trying to save ECG data to unit memory.	
21	SD memory error. Cannot read data.	An error occurred while trying to read ECG data stored on the SD memory card.	Format the SD memory card. Refer to page 22. * Copy any important data to a PC before formatting the SD memory card as all data on the card will be erased. Use a new/different SD memory card. (Always turn off the unit before removing or inserting SD memory cards.)
22	SD memory error. Cannot erase data.	An error occurred while trying to erase ECG data stored on the SD memory card.	
23	SD memory error. Cannot write data.	An error occurred while trying to save ECG data on the SD memory card.	

MESSAGES

ERROR #	MESSAGE	CAUSE	CORRECTION
24	Error formatting SD memory.	An error occurred when formatting the SD memory card.	Use a new/different SD memory card. (Always turn off the unit before removing or inserting SD memory cards.)
25	SD memory error. SD memory unknown.	The SD memory card may be damaged.	Format the SD memory card. Refer to page 22. * Copy any important data to a PC before formatting the SD memory card as all data on the card will be erased. Use a new/different SD memory card. (Always turn off the unit before removing or inserting SD memory cards.)
26	SD memory full. Please delete unnecessary data.	The SD memory card is full and there is not enough space to save new ECG data.	Either delete unnecessary data or use a new SD memory card. (Always turn off the unit before removing or inserting SD memory cards.)
27	The number of data in SD is over the limit. Please delete unnecessary data.	The number of ECG data in the SD memory card is over the limit.	
91	Device error.	The unit may have been damaged or is otherwise unable to operate normally.	Turn the unit off, then on again. If this error still occurs, contact: Omron Healthcare Customer Service at 1-800-829-6427.

TROUBLE SHOOTING

PROBLEM	CAUSE	SOLUTION
Nothing happens when you press the ON/OFF / STOP button.	The batteries are worn out. The batteries are inserted incorrectly.	Replace both batteries with new ones. Insert the batteries with their polarities correctly aligned. Refer to page 13
Measurement starts but the message "Cannot measure. Please follow the instructions for use." appears.	An ECG wave-pattern could not be detected.	Check the instructions and repeat measurement.
Measurement finishes but the message "Analysis impossible. Please measure again." appears.	The electrodes are not making good contact with your finger or chest.	Check the instructions and repeat measurement.
	Your body is too tense.	Relax and repeat the measurement.
	Electrode contact was broken before 30 seconds of data has been recorded.	Make sure that your finger and chest maintain contact with the electrodes for the full 30 seconds of the measurement.
	The chest electrode has been applied over clothing.	Apply the chest electrode to bare skin.
	Your skin is dry.	Moisten your skin with a damp towel.
	The ECG signal is too weak.	Make sure that the electrodes are applied correctly. However, for some individuals measurement may not be possible.

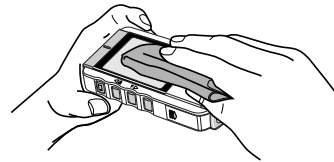
TROUBLE SHOOTING

PROBLEM	CAUSE	SOLUTION
ECG data has disappeared.	The batteries are worn out or have been removed for over one hour.	The unit will not be able to store data if left with worn batteries for an extended period. Replace the batteries with new ones as soon as possible after the warning message appears.
ECG data has disappeared from an SD memory card.	The SD card was exposed to strong electromagnetic noise or static that may damage the stored data.	Erased data cannot be recovered. Format the SD memory card again.
Other problems		Turn off the unit. Turn on the unit and try again. If the date and time or other settings have been reset to their default values, set them again.

If the problem still occurs, contact Omron Healthcare Customer Service at 1-800-829-6427.

MAINTENANCE AND STORAGE

- Clean the unit with a cloth dampened with water or mild detergent. Then wipe it dry with a dry cloth. Do not wipe the SD memory card slot cover.

- Wipe the electrodes using a cloth dampened with disinfectant alcohol.
- Do not wipe the unit with benzene, gasoline, paint thinner, concentrated alcohol, or other volatile detergents.

- Do not place objects on top of the unit. This could damage the unit.

- Do not disassemble the unit. There are no user serviceable parts. Repairs should only be carried out by Omron authorized personnel.
- This unit does not require calibration during its expected life cycle.

⚠ CAUTION: Do not sterilize the unit in an autoclave, ultraviolet sterilizer or gas sterilizer (EOG, formaldehyde, high density ozone, etc.)

MAINTENANCE AND STORAGE

CAUTION:

- Do not wash the unit with water.
- Do not store the unit in the following ambient conditions:
 - Locations exposed to direct sunlight.
 - Locations subject to high temperatures and high humidity.
 - Wet or damp locations where water may get on the unit.
 - Dusty locations.
 - Near fires or open flames.
 - Locations exposed to strong vibration.
 - Locations exposed to strong electromagnetic fields.
- Remove the batteries from this unit when you are not going to use it for a long period of time (approximately three months or more).

FCC STATEMENT

NOTE:

POTENTIAL FOR RADIO/TELEVISION INTERFERENCE (for U.S.A. only)

This product has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC rules.

These limits are designed to provide reasonable protection against harmful interference in a residential installation. The product generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If the product does cause harmful interference to radio or television reception, which can be determined by turning the product on and off, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna
- Increase the separation between the product and the receiver.
- Connect the product into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

POTENTIAL FOR RADIO/TELEVISION INTERFERENCE (for Canada only)

This digital apparatus does not exceed the Class B limits for radio noise emissions from digital apparatus as set out in the interference-causing equipment standard entitled “ Digital Apparatus”, ICES-003 of the Canadian Department of Communications.

Changes or modifications not expressly approved by the party responsible for compliance could void the user’s authority to operate the equipment.

WARRANTY INFORMATION

Limited Warranty

Your Portable ECG Monitor HCG-801, excluding the SD memory card and storage pouch, is warranted to be free from defects in materials and workmanship appearing within 2 years from the date of purchase, when used in accordance with the instructions provided with the monitor. The above warranties extend only to the original retail purchaser.

We will, at our option, repair or replace without charge any monitor covered by the above warranties. Repair or replacement is our only responsibility and your only remedy under the above warranties. To obtain warranty service contact Omron Healthcare Customer Service at 1-800-829-6427 for the address of the repair location and the return shipping and handling fee. Information for warranty service is available on our website at www.omronhealthcare.com.

Enclose the Proof of Purchase. Include a letter, with your name, address, phone number, and description of the specific problem. Pack the product carefully to prevent damage in transit. Because of possible loss in transit, we recommend insuring the product with return receipt requested.

ALL IMPLIED WARRANTIES, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR PARTICULAR PURPOSE, ARE LIMITED TO THE DURATION OF THE APPLICABLE WRITTEN WARRANTY ABOVE.

Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.

OMRON SHALL NOT BE LIABLE FOR LOSS OF USE OR ANY OTHER INCIDENTAL, CONSEQUENTIAL OR INDIRECT COSTS, EXPENSES OR DAMAGES IN ANY WAY CONNECTED WITH THE GOODS OR USE OF THE GOODS. Some states do not allow the exclusion or limitation of incidental or consequential damages, so the above exclusions may not apply to you.

This warranty gives you specific legal rights, and you may also have other rights which may vary from state to state.

Omron Healthcare Customer Service (US and Canada)

Visit our web site at:

www.omronhealthcare.com

Call toll free:

1-800-829-6427

SPECIFICATIONS

Model:	HCG-801
Induction system:	Bipolar single channel
Heart rate range:	2 to 200 beats/min.
Bandwidth:	0.05 to 40Hz
Sampling Rate:	125Hz
Enclosure ingress protection:	IP20
Display:	Graphic LCD
Memory:	5 measurements (using the unit's internal memory) 300 measurements (using the supplied SD memory card)
Electrical Safety:	Internally powered equipment, B-type device
Power Supply:	2 × LR03 (AAA) batteries
Battery life:	Approximately 400 measurements with alkaline batteries (When measurements are taken once a day at room temperature (72 °F (22 °C)))
Operating Environment:	+50 to +104°F (+10 to +40 °C), 30 to 85%RH, 700 to 1060 hPa
Storage Environment:	-4 to +140 °F (-20 to +60 °C), 10 to 95%RH, 700 to 1060 hPa
Weight:	Approximately 4 5/8 oz. (130 g) (including batteries)
Dimensions:	4 3/4" (W) × 2 5/8" (H) × 1" (D) (121 mm × 67 mm × 24 mm)
Contents:	Main Unit, SD memory card, Storage pouch, 2 × LR03(AAA) batteries, Instruction manual, Registration card, Quick reference card.

NOTE: Specifications may be changed without prior notice.

- Disposal of this product, the optional accessories and used batteries should be carried out in accordance with the national regulations for the disposal of electronic products. Unlawful disposal may cause environmental pollution.
- Do not use in the presence of flammable anesthetics, drugs or pressurized oxygen (such as in a hyperbaric chamber, ultraviolet sterilizer or oxygen tent).

= Type B

ECG TYPE3

SD Logo is a trademark.

PORTABLE ECG MONITOR

with respect to electric shock, fire and mechanical hazards only in accordance with UL60601-1 and CAN/CSA C22.2 No. 601.1

- This device supports SDA standard SD memory cards.
- OMRON Healthcare, Co., Ltd. is a member of the SD Card Association.
- This device uses bitmap font from Ricoh Co., Ltd.
- The SD logo is a registered trademark.
- This product meets EMC standards IEC60601-1-2:2001. Using this device with or near other medical/electrical equipment could produce mutual interference. Be sure to read the Instruction Manual for correct installation and use. Also, be sure to read the Instruction Manuals for all other electronic equipment nearby.

SAMPLE SYMPTOM LOG

You may wish to make photocopies of this log and use them to fill in any information. Always record your symptoms in the right hand column every time you take a measurement. Be sure to always include the date and time of your measurement so your doctor can effectively review your data. If you are unsure of what constitutes a symptom, contact your physician for further information.

MEASUREMENT DATE	MEASUREMENT TIME	WRITE DOWN YOUR SYMPTOMS HERE

MEASUREMENT DATE	MEASUREMENT TIME	WRITE DOWN YOUR SYMPTOMS HERE

NOTES

Manufactured by Omron Healthcare
Made in Japan

Distributed by:
OMRON HEALTHCARE, INC.
1200 Lakeside Drive
Bannockburn, Illinois 60015
www.omronhealthcare.com

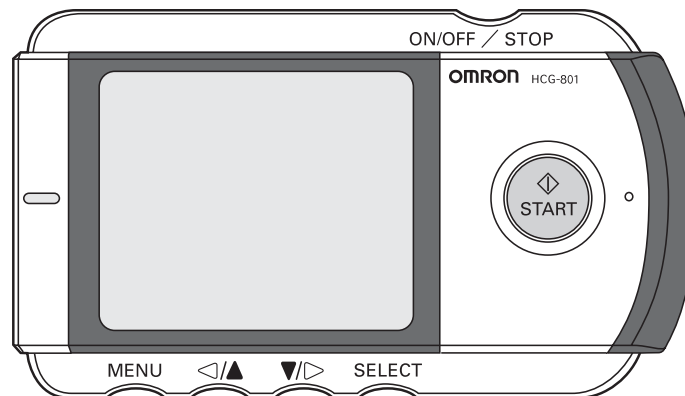
World Rights Reserved
© 2006 Omron Healthcare, Inc.

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- OMRON Healthcare Co., Ltd. es miembro de la SD Card Association.
- Este dispositivo usa fuentes de mapa de bits de Ricoh Co., Ltd.