Panasonic

Upper Arm Blood Pressure Monitor

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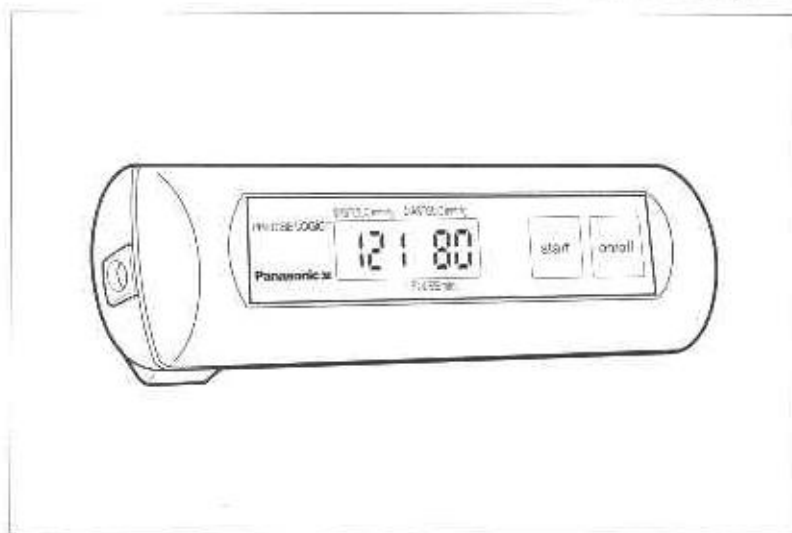
Model No. EW254

Modelo No. EW254

Before operating this device, please read these instructions completely.

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Introduction

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Thank you for purchasing the Panasonic Blood Pressure Monitor EW254. The monitor incorporates Precise Logic™ technology to provide accurate, reliable measurements.

Measuring your own blood pressure is an important way of monitoring your health. High blood pressure (hypertension) is a major health problem which can be treated effectively once detected. Measuring your blood pressure between doctor visits on a regular basis in the comfort of your home, and keeping a record of the measurements using the monitor, will help you monitor any significant changes in your blood pressure. Keeping an accurate record of your blood pressure will help your doctor diagnose and possibly prevent any health problems in the future.

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Your heart acts like a pump, sending blood surging through your blood vessels each time it contracts. Blood pressure is the pressure exerted by blood pumped from the heart on the walls of blood vessels. Systolic pressure is the pressure exerted when the heart contracts and pumps blood into the arteries. Diastolic pressure is the pressure exerted when the heart expands, or relaxes. When you or your doctor take your blood pressure, both your systolic and diastolic pressures are measured. If your blood pressure measurement is 120 over 80 (120/80), for example, your systolic pressure is 120 while your diastolic pressure is 80.

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Important Instructions Before Use

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1. Do not confuse self-monitoring with self-diagnosis. Blood pressure measurements should only be interpreted by a health professional who is familiar with your medical history.

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2. If you are taking medication, consult with your physician to determine the most appropriate time to measure your blood pressure. NEVER change a prescribed medication without first consulting with your physician.

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3. Blood pressure can vary based on many factors, including age, gender, weight and physical condition. In general, a person's blood pressure is lower during sleep and higher when he or she is active. Blood pressure can change easily in response to physiological changes. The setting in which a person's blood pressure is measured can also affect the results. Having one's blood pressure measured by a healthcare professional in a hospital or clinic can cause nervousness and may result in a temporarily elevated reading. Because blood pressure measurements taken in a clinical setting can vary considerably from those taken at home, a person's blood pressure should be measured not only occasionally in the doctor's office, but also on a regular basis at home. Also, if you find that your blood pressure is lower at home, this is not unusual. To accurately compare with your physician's reading, take your Panasonic blood pressure monitor to your doctor's office and compare readings in this setting.

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4. **People suffering from cardiac arrhythmia, vascular constriction, liver disorders or diabetes, people with cardiac pacemakers or a weak or irregular pulse, and women who are pregnant should consult their physician before measuring their pressure themselves. Different values may be obtained due to their condition.**

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5. **Try to take your blood pressure measurements at the same time and under the same conditions every day.**

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- **The ideal time to measure your blood pressure (to obtain your so-called “base blood pressure”) is in the morning just after waking up, before having breakfast and before any major activity or exercise. If this is not possible, however, try to take measurements at a specified time prior to breakfast, and before you have become active. You should relax for about 5 minutes before taking the measurement.**

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- **The following situations may cause substantial variations in blood pressure readings and should therefore be avoided at least 30 minutes prior to taking your blood pressure.**

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6. Measurements may be impaired if this device is used near a television, microwave oven, X-ray equipment or other devices with strong electrical fields. To prevent such interference, use the monitor at a sufficient distance from such devices or turn the devices, off.

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7. This device is designed for use by adults. Consult with your physician before using this device on a child. Do not use on infants or toddlers.

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8. This device is not suitable for continuous monitoring during medical emergencies or operations.

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9. Do not use the unit for any purpose other than measuring blood pressure.

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10. Improper handling of batteries may result in battery explosion or in corrosion from battery leakage. Please observe the followings to ensure proper use of batteries.

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a. Be sure to turn off the power of the device after use.

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b. Do not mix different types or sizes of batteries.

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c. Replace all batteries at the same time. Do not mix old and new batteries.

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d. Be sure to insert batteries with correct polarity, as instructed.

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e. Remove batteries when they are worn out, and dispose of them properly according to all applicable environmental regulations.

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f. Do not disassemble or throw batteries into a fire.

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g. Do not short-circuit batteries.

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h. Do not charge the batteries included with this unit.

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
i. If the device requires repressurization to get a proper measurement, there will be a longer measurement time. This may temporarily affect the blood flow in your upper arm, which may feel slightly numb until normal blood flow resumes.

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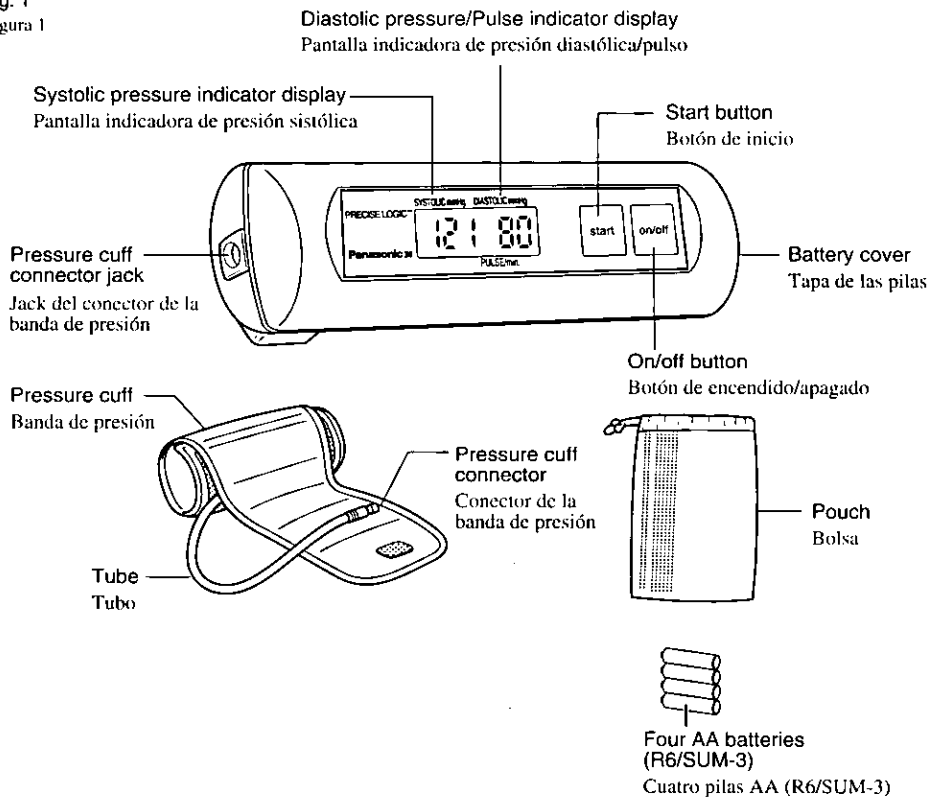Fig. 1
Figura 1


Loading Batteries

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1. Lightly press the battery cover on the rear side of the main unit, and push it in the direction of the arrow.

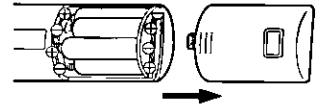Fig. 2
Figura 2

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2. Insert four AA (R6, SUM-3) batteries in the battery compartment, observing the (+) and (-) signs to ensure correct polarity.

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- Approximately 200 measurements can be taken with a new set of Panasonic batteries (at an ambient temperature of 22 °C or 71.6 °F, pressurization of 150 mmHg).

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NOTE :

NOTA :

When the batteries have been almost fully drained, the low battery symbol (Fig. 3) may not disappear immediately, even after the unit is turned off.

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2. When nothing is displayed on the display even when the unit is turned on.

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Either of the above conditions indicates that the batteries are exhausted. You should replace them with four new batteries of the same type.

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NOTE :

NOTA :

Remove the batteries from the unit if it will not be used for an extended period of time (30 days or more).

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CAUTION :

PRECAUCIÓN:

Do not throw batteries into a fire. Doing so may result in an accident or explosion.

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Positioning the Pressure Cuff

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CAUTIONS :

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- Do not use the unit for any purpose other than measuring blood pressure. Also, do not use the unit in conjunction with other devices. Improper use may cause a malfunction or personal injury.

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- Keep the unit out of the reach of children.

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1. Blood pressure is measured on the upper arm. Blood pressure is best measured on the arm opposite from your writing arm (non-dominant arm).

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2. As shown in the Fig.4-1, wrap the cuff around the arm 2 to 3 cm (1 to 1-1/2 inch) above the elbow, with the tube on the palm side. The tube should be set slightly towards the inside of your arm.

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- Correct measurements cannot be taken if the edge of the pressure cuff extends below the elbow.

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3. Wrap the cuff firmly and then fasten it with the Velcro fastener. (Fig. 4-2)

(Make sure that your sleeve does not get in the cuff.)

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
4. Keep the cuff level with your heart (in line with your breast). (Fig. 4-3)

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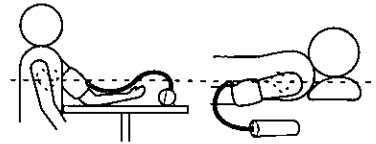Fig. 4-3
Figura 4-3

Measuring Your Blood Pressure Medida de su presión arterial

Make sure the pressure cuff is correctly attached before measuring. The pressure cuff connector at the end of the cuff tube must be firmly inserted into the pressure cuff connector jack on the main unit.

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Relax during measurement.

Relájese durante la medida.

1. Press the on/off button to turn on the unit.

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- The entire display will flash for about 2 seconds, and then a zero will be displayed, indicating that the unit is ready. (Fig. 5-1)

(When the power is turned on and off, you will hear a noise. This is caused by the opening of an air valve and does not indicate a malfunction.)

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2. Press the start button and inflation will begin automatically.

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- Inflation will continue until about 150 mmHg. (Fig. 5-2)

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- The symbol "●" flashes on the display to indicate that measurement is in progress.

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4. The blood pressure reading is displayed when measurement is completed. (Fig. 5-4)

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- The blood pressure reading is displayed as soon as measurement is completed.

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5. Your pulse rate is automatically displayed alternately with your blood pressure readings. (Fig. 5-5)

Fig. 5-1
Figura 5-1

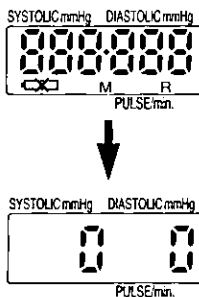
Fig. 5-2
Figura 5-2

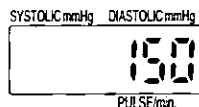
Fig. 5-3
Figura 5-3



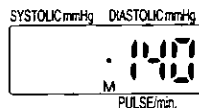
Fig. 5-4
Figura 5-4

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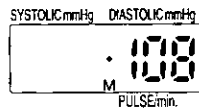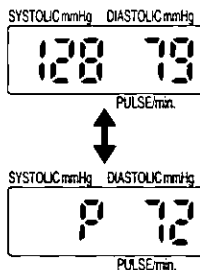Fig. 5-5
Figura 5-5

- Blood pressure reading is displayed for about 3 seconds.
La indicación de la presión arterial se visualiza durante unos 3 segundos.
- Pulse rate is displayed for about 2 seconds. (The pulse rate is calculated from the pulse count over a minute during measurement of the blood pressure.)

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Error:

Error:

- If "E" (meaning error) appears on the display during measurement, an error has occurred, e.g., you have moved your hand during measurement. Check to make sure the pressure cuff is positioned and wrapped securely. Check the pressure cuff connection to the main unit, and repeat measurement making sure you remain quiet and still.

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- When the correct pulse is not measured, "E" also appears on the area of diastolic pressure indication. Noise or an uneven pulse is the probable cause. Please take another measurement.

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6. To take another readings, repeat the procedure beginning with step 2.

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- Wait at least 5 minutes before taking another measurement.

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7. Press the on/off button to turn off the unit.

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- The unit turns off automatically after approximately 5 minutes.

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CAUTION :

RECAUCIÓN :

- Do not confuse self-monitoring with self-diagnosis.

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- NEVER change a prescribed medication without first consulting with your physician.

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Easily Check Your Blood Pressure Readings Against the World Health Organization (WHO) Guidelines

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IMPORTANT :

IMPORTANTE :

- WHO does not define a low blood pressure area. For reference purposes, systolic pressure readings of less than 90 mmHg are considered to indicate low blood pressure regardless of the diastolic pressure.

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- Do not be alarmed by temporarily high or low readings because fluctuations in a person's blood pressure are not uncommon. If possible, measure and record your blood pressure at the same time every day, and consult your physician if you have questions or concerns.

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- If abnormal variations in blood pressure are observed in measurement, please consult your physician.

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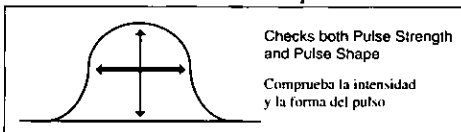
About the Precise Logic™ Measurement System

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
The blood flow (pulse wave) which influences blood pressure monitors varies according to the conditions of the blood vessels and the structure of the arm, making measurements by conventional methods difficult in some cases. For more consistent, accurate results, this device employs a Precise Logic™ measurement system that verifies the strength and shape of the pulse wave and then measures the blood pressure.

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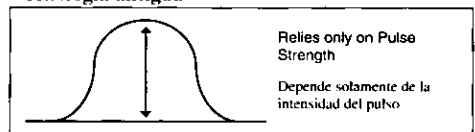By using the Precise Logic™ measurement system, measurement reliability and accuracy have been improved.

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Precautions to Ensure Safe, Reliable Operation

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Care and Maintenance

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1. Use a dry, soft cloth to clean the device. If desired, use a cloth lightly dampened with tap water and a mild detergent to clean the device. **DO NOT** use alcohol, benzene, thinner or other harsh chemicals to clean the device or cuff.

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2. When the unit will not be used for an extended period of time (30 days or more), be sure to remove the batteries. Otherwise, the batteries may leak and damage the unit.

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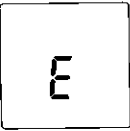
3. A replacement pressure cuff for this device can be purchased from an authorized service center.

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**If the device still appears to provide unusual or erroneous readings, consult your physician.
If the device still does not function properly, contact Panasonic at 1-800-338-0552.**

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Specifications are subject to change without notice.

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Model EW254 is intended to be used for the oscillometric PRECISE LOGIC™ measurement of systolic and diastolic blood pressure and pulse rate using a pressurized cuff.

Blood pressure measurements determined with this device are equivalent to those obtained by a trained observer using the cuff/stethoscope auscultation method, within the limits prescribed by the American National Standard, Electronic or automated sphygmomanometers.

For questions or assistance with your blood pressure monitor, call us at 1-800-338-0552.

**Panasonic Personal and Professional
Products Company
A Unit of Matsushita Electric Corporation
of America**

One Panasonic Way 4A-3
Secaucus, NJ 07094

© 2000 Matsushita Electric Works, Ltd. All Rights Reserved.

Free Manuals Download Website

<http://myh66.com>

<http://usermanuals.us>

<http://www.somanuals.com>

<http://www.4manuals.cc>

<http://www.manual-lib.com>

<http://www.404manual.com>

<http://www.luxmanual.com>

<http://aubethermostatmanual.com>

Golf course search by state

<http://golfingnear.com>

Email search by domain

<http://emailbydomain.com>

Auto manuals search

<http://auto.somanuals.com>

TV manuals search

<http://tv.somanuals.com>