INSTRUCTION MANUAL

Automatic Wrist Blood Pressure Monitor Model 6021REL

SPANISH

ENGLISH

TABLE OF CONTENTS

Before Using the Monitor

| | |
|-----------------------------------|---|
| Introduction | 3 |
| Safety Information | 4 |
| Operating the Device | 4 |
| Care and Maintenance | 5 |
| Before Taking a Measurement | 6 |

Operating Instructions

| | |
|-------------------------------|----|
| Know Your Unit | 8 |
| Unit Display | 9 |
| Battery Installation | 10 |
| Applying the Wrist Cuff | 12 |
| Taking a Measurement | 13 |

Care and Maintenance

| | |
|---|----|
| Care and Maintenance | 15 |
| Error Indicators and Troubleshooting Tips | 16 |
| FCC Statement | 18 |
| Limited Warranty | 19 |
| Specifications | 20 |
| Notes | 21 |

INTRODUCTION

Thank you for purchasing the ReliOn® 6021REL Automatic Wrist Blood Pressure Monitor.

Fill in for future reference.

DATE PURCHASED: _____

SERIAL NUMBER: _____

Staple your purchase receipt here

The monitor is compact and easy-to-use at home, at work, and during travel. Perfect for people who frequently monitor their own blood pressure.

Your new digital blood pressure monitors use the oscillometric method of blood pressure measurement. This means the monitor detects your blood's movement through your brachial artery and converts the movements into a digital reading. An oscillometric monitor does not need a stethoscope so the monitor is simple to use.

Clinical research has proven a direct relationship between blood pressure in the wrist and blood pressure in the arm. Changes in wrist blood pressure reflect changes in arm blood pressure because the arteries in the wrist and the arm are close to each other. Frequently measuring the blood pressure in your wrist will provide your physician with an accurate indication of changes in your blood pressure.

The ReliOn® 6021REL Automatic Wrist Blood Pressure Monitor is intended for home use.

SAVE THESE INSTRUCTIONS

SAFETY INFORMATION

To assure the correct use of the product, basic safety measures should always be followed including the warnings and cautions listed in this instruction manual.

| SAFETY SYMBOLS USED IN THIS INSTRUCTION MANUAL | |
|--|--|
|  WARNING | Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury. |
|  CAUTION | Indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury to the user or patient or damage to the equipment or other property. |

OPERATING THE DEVICE

- ⚠ Contact your physician for specific information about your blood pressure. Self-diagnosis and treatment using measured results may be dangerous. Follow the instructions of your physician or licensed healthcare provider.
- ⚠ The monitor is not intended to be a diagnostic device.
- ⚠ If battery fluid should get in your eyes, immediately rinse with plenty of clean water. Contact a physician immediately.
- ⚠ If you have a condition that may compromise circulation, you may get an inaccurate reading with this device. Consult your physician before using this device.
- ⚠ Read all of the information in the instruction manual and any other literature in the box before operating the unit.
- ⚠ This device is intended for use in measuring blood pressure and pulse rate in the adult population. Do not use this device on infants or persons who cannot express their intentions.

SAFETY INFORMATION

OPERATING THE DEVICE (continued)

- ⚠ Operate the device only as intended. Do not use the device for any other purpose.
- ⚠ Dispose of the device, components and optional accessories according to applicable local regulations. Unlawful disposal may cause environmental pollution.
- ⚠ Do not use a cellular phone near the device. It may result in an operational failure.
- ⚠ Use only 1.5V “AAA” alkaline batteries with this device. Do not use any other type of battery. This may damage the unit.

CARE AND MAINTENANCE

- ⚠ Do not subject the monitor to strong shocks, such as dropping the unit on the floor.
- ⚠ Do not submerge the device or any of the components in water.
- ⚠ Store the device and the components in a clean, safe location.
- ⚠ Changes or modification not approved by the manufacturer will void the user warranty. Do not disassemble or attempt to repair the unit or components.

BEFORE TAKING A MEASUREMENT

To ensure a reliable reading follow these recommendations:

1. Avoid eating, drinking alcohol, smoking, exercising, and bathing for 30 minutes before taking a measurement. Rest for at least 15 minutes before taking the measurement.
2. Stress raises blood pressure. Avoid taking measurements during stressful times.
3. The cuff can be applied to your left or right wrist.
4. Measurements should be taken in a quiet place.
5. Position the unit at heart level throughout the measurement.
6. Remain still and do not talk during the measurement.
7. Wait 2-3 minutes between measurements. The wait time allows the arteries to return to the condition prior to taking the blood pressure measurement. You may need to increase the wait time depending on your individual physiological characteristics.
8. Keep a record of your blood pressure and pulse readings for your physician. A single measurement may not provide an accurate indication of your true blood pressure. You need to take and record several readings over a period of time. Try to measure your blood pressure at the same time each day for consistency.

BEFORE TAKING A MEASUREMENT

The American Heart Association (AHA) recommends the following guideline for upper limit of normal for home blood pressure readings:

| AHA Home Guideline for Upper Limit of Normal BP | |
|--|----------|
| Systolic Blood Pressure | 135 mmHg |
| Diastolic Blood Pressure | 85 mmHg |

This is a general guideline because blood pressure depends on age and morbidity. The AHA recommends a lower home blood pressure goal for certain patients, including diabetic patients, pregnant women, and patients with renal failure¹.

The monitor is not intended to be a diagnostic device.


⚠ WARNING

Contact your physician for specific information about your blood pressure. Self-diagnosis and treatment using measured results may be dangerous. Follow the instructions of your physician or licensed healthcare provider.


¹American Heart Association Scientific Statement 2004

KNOW YOUR UNIT


Main Unit: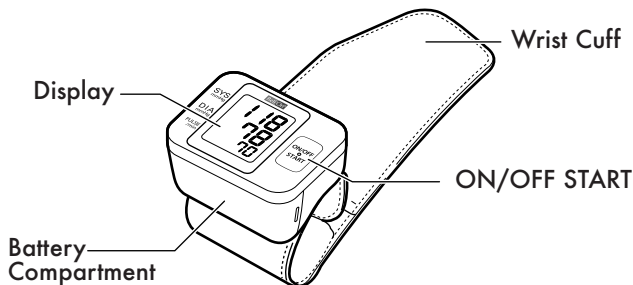


Components:

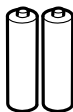Two "AAA" Alkaline Batteries

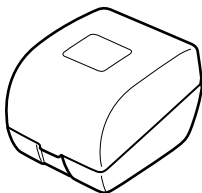
Storage Case

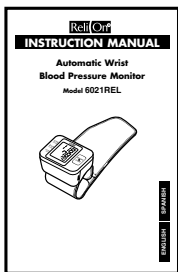Instruction Manual

UNIT DISPLAY

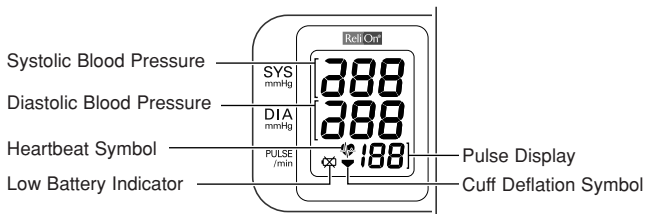
BATTERY INSTALLATION


1. Press both sides of the battery cover and lift the cover off in the direction of the arrow.

2. Install 2 “AAA” size batteries so the + (positive) and - (negative) polarities match the polarities of the battery compartment as indicated.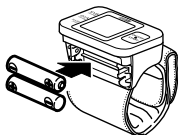

3. Replace the Battery Cover.
Make sure the battery cover is securely in place.

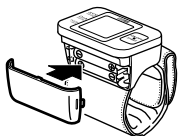BATTERY INSTALLATION

BATTERY REPLACEMENT

Low Battery Indicator

When the Low Battery Indicator appears on the display screen, turn the monitor off and remove all the batteries. Replace with two new batteries at the same time. Long-life alkaline batteries are recommended.

⚠ WARNING

If battery fluid should get in your eyes, immediately rinse with plenty of clean water. Contact a physician immediately.

NOTE: The batteries supplied with the device may have a shorter life.

⚠ CAUTION


Dispose of the device, components and optional accessories according to applicable local regulations. Unlawful disposal may cause environmental pollution.

APPLYING THE WRIST CUFF

Wrapping the wrist cuff is one of the most important points for taking a correct measurement.


APPLYING THE CUFF ON THE LEFT WRIST

1. Roll up sleeve. Make sure your sleeve is not rolled up too tightly on your arm. This may constrict the flow of blood in your arm.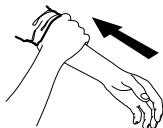

2. Wrap the cuff directly against your skin. Do not apply over clothing. Place the cuff over your left wrist with your left thumb facing upward.
3. Position the cuff leaving a clearance of approximately 1/2 to 1 inch between the cuff and the bottom of your palm.

4. Hold the bottom part of the cuff and wrap it around the wrist so it fits comfortably and securely around your wrist.

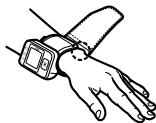
NOTE: Make sure the wrist cuff does not cover the protruding part of the wrist bone on the outside of the wrist.

5. Fold the remaining part of the wrist cuff back out of the way.

APPLYING THE CUFF ON THE RIGHT WRIST

When taking a measurement using the right wrist position the cuff as shown in the illustration.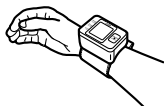

TAKING A MEASUREMENT

1. Sit comfortably in a chair with your feet flat on the floor.


2. Hold your arm across your chest at heart level. Hold your elbow firmly to avoid body movement. Sit still and do not talk or move during the measurement.

3. Press the ON/OFF START button.

All symbols appear on the display.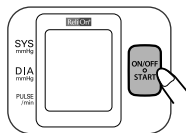

The cuff starts to inflate automatically.

As the cuff inflates, the monitor automatically determines your ideal inflation level. This monitor detects your blood pressure and pulse rate during inflation.

The Heartbeat Symbol () flashes at every heartbeat.

Remain still and do not move until the entire measurement process is completed.

NOTE: To stop the inflation or measurement, push the ON/OFF START button. The monitor will stop inflating, start deflating, and will turn off.

TAKING A MEASUREMENT

4. After the monitor has detected your blood pressure and pulse rate, the cuff automatically deflates. Your blood pressure and pulse rate are displayed.

5. Press the ON/OFF START button to turn the monitor off.

NOTE: The monitor will automatically turn off after two minutes.

CARE AND MAINTENANCE

To keep your digital blood pressure monitor in the best condition and protect the unit from damage, follow the directions listed below:

Do not forcefully bend the cuff. Do not fold tightly.

Clean the monitor with a soft dry cloth. Do not use any abrasive or volatile cleaners. Do not attempt to clean the cuff.

⚠ CAUTION

Do not submerge the device or any of the components in water. Do not subject the monitor to extreme hot or cold temperatures, humidity or direct sunlight.

⚠ CAUTION

Store the device and the components in a clean, safe location.


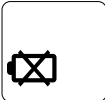
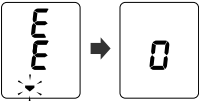
⚠ CAUTION

Do not subject the monitor to strong shocks, such as dropping the unit on the floor.

Remove the batteries if the unit will not be used for three months or longer. Always replace all the batteries with new ones at the same time.

ERROR INDICATORS AND TROUBLESHOOTING TIPS

ERROR INDICATORS
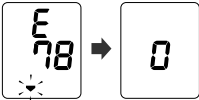
| SYMBOL | CAUSE | CORRECTION |
|---|-----------------------------------|--|
|  | Movement during measurement. | Press the ON/OFF START button to turn the unit off. Remove the monitor from your wrist. Refer to "Taking a Measurement". Wait 2-3 minutes. Take another measurement. |
|  | The cuff is not wrapped securely. | Remove the monitor from your wrist. Read, "Applying the Wrist Cuff". Wait 2-3 minutes. Take another measurement. |
|  Blinks or lights | Batteries are worn. | Replace the two "AAA" batteries. Refer to "Battery Installation". |
|  | Cuff over-inflated. | Remove the monitor from your wrist. Read, "Applying the Wrist Cuff". Wait 2-3 minutes. Take another measurement. |
|  | Abnormal unit function. | Repair Service Required. Refer to "Warranty". |

ERROR INDICATORS AND TROUBLESHOOTING TIPS

TROUBLESHOOTING TIPS

| PROBLEM | CAUSES AND SOLUTIONS |
|---|--|
| No power. No display appears on the unit. | Replace worn batteries. Check the battery installation for proper placement of the battery polarities. |
| Measurements values appear too high or too low. | Blood pressure varies constantly. Many factors including stress, time of day, how you wrap the cuff, may affect your blood pressure. Review the sections “Before Taking a Measurement” and “Taking a Measurement”. |

FCC STATEMENT

NOTE:

POTENTIAL FOR RADIO/TELEVISION INTERFERENCE (for U.S.A. only) This product has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC rules.

These limits are designed to provide reasonable protection against harmful interference in a residential installation. The product generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If the product does cause harmful interference to radio or television reception, which can be determined by turning the product on and off, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the product and the receiver.
- Connect the product into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

POTENTIAL FOR RADIO/TELEVISION INTERFERENCE
(for Canada only)

This digital apparatus does not exceed the Class B limits for radio noise emissions from digital apparatus as set out in the interference-causing equipment standard entitled "Digital Apparatus", ICES-003 of the Canadian Department of Communications.

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

LIMITED WARRANTY

Your ReliOn® 6021REL Automatic Wrist Blood Pressure Monitor is warranted to be free from defects in materials and workmanship appearing within one year from the date of purchase, when used in accordance with the instructions provided with the monitor. The above warranty extends only to the original retail purchaser.

We will, at our option, repair or replace without charge any monitor covered by the above warranty. Repair or replacement is our only responsibility and your only remedy under the above warranty.

To obtain warranty service contact Omron Healthcare for the address of the repair location. Shipping instructions and repair information is available by calling **1-800-634-4350** or visit our website at www.omronhealthcare.com.

Enclose the Proof of Purchase. Include \$5.00 for the return shipping and handling.

Include a letter, with your name, address, phone number, and description of the specific problem. Pack the product carefully to prevent damage in transit. Because of possible loss in transit, we recommend insuring the product with return receipt requested.

ALL IMPLIED WARRANTIES, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR PARTICULAR PURPOSE, ARE LIMITED TO THE DURATION OF THE APPLICABLE WRITTEN WARRANTY ABOVE. Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.

OMRON SHALL NOT BE LIABLE FOR LOSS OF USE OR ANY OTHER INCIDENTAL, CONSEQUENTIAL OR INDIRECT COSTS, EXPENSES OR DAMAGES. Some states do not allow the exclusion or limitation of incidental or consequential damages, so the above exclusions may not apply to you.

This warranty gives you specific legal rights, and you may also have other rights which may vary from state to state.

FOR CUSTOMER SERVICE

Visit our web site at:

www.omronhealthcare.com

Call toll free:

1-800-634-4350

SPECIFICATIONS

| | |
|---|--|
| Model: | 6021REL (HEM-6021-WM) |
| Display: | LCD Digital Display |
| Measurement Range: | Pressure: 0 to 299 mmHg, Pulse: 40 to 180/min. |
| Accuracy/Calibration: | Pressure: ± 3 mmHg Pulse: $\pm 5\%$ of reading |
| Inflation: | Automatic by electric pump |
| Deflation: | Automatic rapid deflation |
| Pressure Detection: | Capacitive pressure sensor |
| Measurement Method: | Oscillometric method |
| Power Source: | Two "AAA" batteries |
| Battery Life: | Approximately 300 measurements when using alkaline batteries at the room temperature of 73°F (23°C) and by using three times a day and inflating to 170 mmHg |
| Operating Temperature /Humidity: | 50°F to 104°F (10°C to 40°C) / 15 to 90% RH |
| Storage Temperature /Humidity/Air Pressure: | -4°F to 140°F (-20°C to 60°C) / 10 to 95% RH / 700 - 1060 hPa |
| Main Unit Weight: | Approximately 4 1/4 oz. (120g) not including batteries |
| Main Unit Dimensions: | 2 7/8" (w) \times 2 1/4" (h) \times 1 3/4" (d) (72 mm \times 56 mm \times 43 mm) (not including the wrist cuff) |
| Measurable Circumference of Wrist: | Approximately 5 1/4" to 8 1/2" (13.5 to 21.5 cm) |
| Contents: | Main Unit, Instruction Manual, two "AAA" alkaline batteries and Storage Case |

NOTE: These specifications are subject to change without notice.

NOTES

Manufactured by OMRON HEALTHCARE

Made in China

Distributed by:

OMRON HEALTHCARE, INC.

1200 Lakeside Drive
Bannockburn, Illinois 60015
www.omronhealthcare.com

Copyright © 2009 Omron Healthcare, Inc.

ReliOn® is a registered trademark used under license
by Omron Healthcare, Inc.

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ReliOn® es una marca comercial registrada utilizada bajo licencia por Omron Healthcare, Inc.
Wal-Mart® Stores, Inc.