
ReliOn®

# INSTRUCTION MANUAL

## Automatic Blood Pressure Monitor Model 7100REL

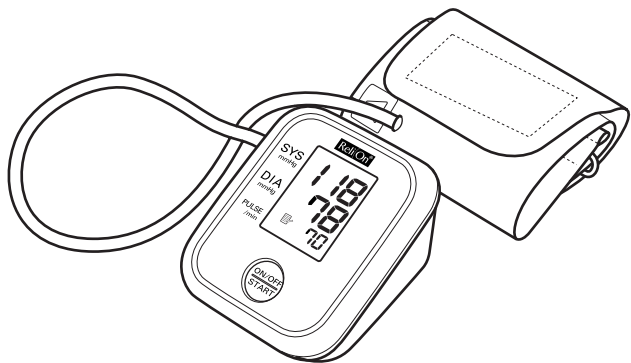ESPAÑOL

ENGLISH

# TABLE OF CONTENTS

## Before Using the Unit

Introduction	3
Safety Information	4
Operating the Device	4
Care and Maintenance	6
Before Taking a Measurement	7

## Operating Instructions

Know Your Unit	8
Unit Display	9
Display Symbols	10
Heartbeat Symbol (  )	10
Battery Installation	11
Applying the Arm Cuff	13
Taking a Measurement	16
Using the Memory function	18

## Care and Maintenance

Care and Maintenance	19
Error Indicators and Troubleshooting Tips	20
FCC Statement	22
Warranty	23
Specifications	24
Notes	25

# INTRODUCTION

Thank you for purchasing the ReliOn® 7100REL Automatic Blood Pressure Monitor.

*Fill in for future reference.*

DATE PURCHASED: \_\_\_\_\_

SERIAL NUMBER: \_\_\_\_\_

*Staple your purchase receipt here*

Your new digital blood pressure monitor uses the oscillometric method of blood pressure measurement. This means the monitor detects your blood's movement through your brachial artery and converts the movements into a digital reading.

The 7100REL comes with the following components:

- Monitor
- Standard Adult Arm Cuff
- 4 “AAA” Batteries
- Instruction Manual printed in English and Spanish

The ReliOn® 7100REL Automatic Blood Pressure Monitor is intended for home use.

## SAVE THESE INSTRUCTIONS

# SAFETY INFORMATION

*To assure the correct use of the product basic safety measures should always be followed including the warnings and cautions listed in this instruction manual.*

SAFETY SYMBOLS USED IN THIS INSTRUCTION MANUAL	
 <b>WARNING</b>	Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.
 <b>CAUTION</b>	Indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury to the user or patient or damage to the equipment or other property.

## OPERATING THE DEVICE

- ⚠ Contact your physician for specific information about your blood pressure. Self-diagnosis and treatment using measured results may be dangerous. Follow the instructions of your physician or licensed healthcare provider.
- ⚠ The monitor is not intended to be a diagnostic device.
- ⚠ If battery fluid should get in your eyes, immediately rinse with plenty of clean water. Contact a physician immediately.
- ⚠ Read all of the information in the instruction manual and any other literature in the box before operating the unit.
- ⚠ This device is intended for use in measuring blood pressure and pulse rate in the adult population. Do not use this device on infants or persons who cannot express their intentions.

# SAFETY INFORMATION

## OPERATING THE DEVICE (continued)

- ⚠ Read carefully “Taking a Measurement” No.3 of the instruction manual if your systolic pressure is known to be higher.
- ⚠ Inflating to a higher pressure than necessary may result in bruising where the cuff is applied.
- ⚠ Operate the device only as intended. Do not use the device for any other purpose.
- ⚠ Dispose of the device, components and optional accessories according to applicable local regulations. Unlawful disposal may cause environmental pollution.
- ⚠ Do not use a cellular phone near the device. It may result in an operational failure.
- ⚠ Do not inflate the arm cuff when it is not wrapped around your arm.
- ⚠ Use only authorized parts and accessories. Parts and accessories not approved for use with the device may damage the unit.
- ⚠ Use only 1.5V “AAA” alkaline batteries with this device. Do not use any other type of battery. This may damage the unit.

# SAFETY INFORMATION

## CARE AND MAINTENANCE

- ⚠ Do not subject the monitor to strong shocks, such as dropping the unit on the floor.
- ⚠ Do not submerge the device or any of the components in water.
- ⚠ Store the device and the components in a clean, safe location.
- ⚠ Changes or modification not approved by the manufacturer will void the user warranty. Do not disassemble or attempt to repair the unit or components.


# BEFORE TAKING A MEASUREMENT

To ensure a reliable reading follow these recommendations:


1. Avoid eating, drinking alcohol, smoking, exercising, and bathing for 30 minutes before taking a measurement. Rest for at least 15 minutes before taking the measurement.
2. Stress raises blood pressure. Avoid taking measurements during stressful times.
3. Measurements should be taken in a quiet place.
4. Remove tight-fitting clothing from your arm.
5. Sit on a chair with your feet flat on the floor. Rest your arm on a table so that the cuff is at the same level as your heart.
6. Remain still and do not talk during the measurement.
7. Wait 2-3 minutes between measurements. The wait time allows the arteries to return to the condition prior to taking the blood pressure measurement. You may need to increase the wait time depending on your individual physiological characteristics.
8. Keep a record of your blood pressure and pulse readings for your physician. A single measurement does not provide an accurate indication of your true blood pressure. You need to take and record several readings over a period of time. Try to measure your blood pressure at about the same time each day for consistency.

# KNOW YOUR UNIT

## Main Unit: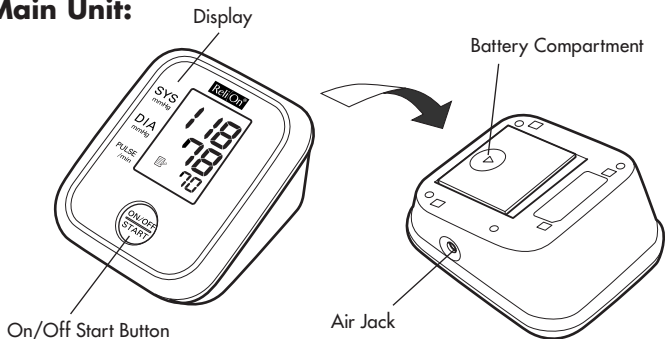


## Components:

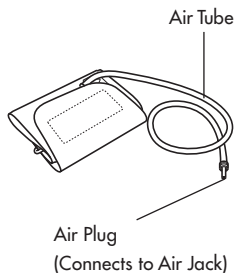Arm Cuff


Four "AAA" Alkaline  
(LR03) Batteries

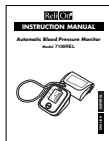Instruction Manual

# UNIT DISPLAY
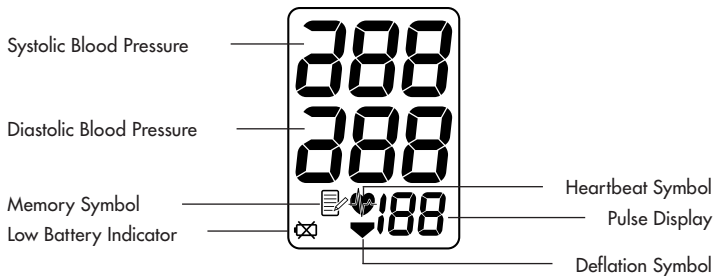
# DISPLAY SYMBOLS

The American Heart Association (AHA) recommends the following guideline for upper limit of normal for home blood pressure readings:

<b>AHA Home Guideline for Upper Limit of Normal BP</b>	
Systolic Blood Pressure	135 mmHg
Diastolic Blood Pressure	85 mmHg

This is a general guideline because blood pressure depends on age and morbidity. The AHA recommends a lower home blood pressure goal for certain patients, including diabetic patients, pregnant women, and patients with renal failure<sup>1</sup>.

## **⚠ WARNING**

- The monitor is not intended to be a diagnostic device.
- Contact your physician for specific information about your blood pressure. Self-diagnosis and treatment using measured results may be dangerous. Follow the instructions of your physician or licensed healthcare provider.

## **Heartbeat Symbol ( )**



### **TAKING A MEASUREMENT**

The Heartbeat Symbol flashes on the display during the measurement. The Heartbeat Symbol flashes at every heartbeat.

<sup>1</sup>American Heart Association Scientific Statement 2004

# BATTERY INSTALLATION

1. Turn the main unit upside down.
2. Press the ▲ indicator on the battery cover and slide the cover off in the direction of the arrow.
3. Install 4 “AAA” size batteries so the + (positive) and - (negative) polarities match the polarities of the battery compartment as indicated.
4. Replace the battery cover. Slide the battery cover as indicated until it clicks into place.

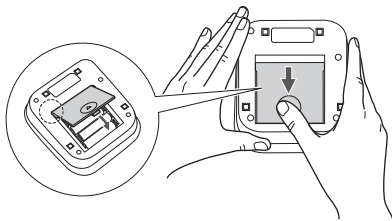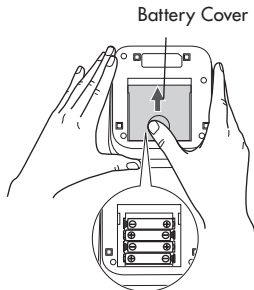# BATTERY INSTALLATION

## BATTERY REPLACEMENT

Low Battery Indicator

When the Low Battery Indicator appears on the display screen turn the monitor off and remove all the batteries. Replace with four new batteries at the same time.

### **⚠ WARNING**

If battery fluid should get in your eyes, immediately rinse with plenty of clean water. Contact a physician immediately.

**NOTE:** The batteries supplied with the device may have a shorter life.

### **⚠ CAUTION**

Dispose of the device, components and optional accessories according to applicable local regulations. Unlawful disposal may cause environmental pollution.

# APPLYING THE ARM CUFF

## ⚠ CAUTION


Read all of the information in the instruction manual and any other literature in the box before operating the unit.

## ⚠ CAUTION

This device is intended for use in measuring blood pressure and pulse rate in the adult population. Do not use this device on infants or persons who cannot express their intentions.


## APPLYING THE CUFF ON THE LEFT ARM

1. Make sure the air plug is securely inserted in the main unit.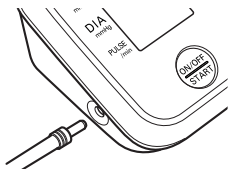


**NOTE:** If the cuff is not assembled, lay the cuff flat with the metal D ring facing down and on the right side. Pass the left edge through the metal D ring to form a loop. The smooth cloth should be inside of the loop.

2. Remove tight-fitting clothing from your upper left arm.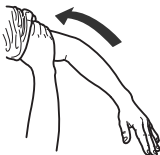


# APPLYING THE ARM CUFF

3. Apply the cuff to your left upper arm so the blue arrow is centered on the inside of your arm and aligned with your middle finger. The air tube runs down the inside of your arm. The bottom of the cuff should be approximately 1/2" above your elbow.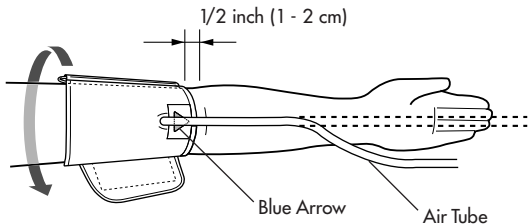

4. Wrap the cuff firmly in place around your arm using the closure strip.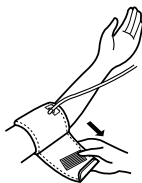


# APPLYING THE ARM CUFF

## APPLYING THE CUFF ON THE RIGHT ARM

When taking a measurement using the right arm use this instruction for Step 4 on page 14.

Apply the cuff to your right upper arm so the blue arrow and the air tube runs along the side of your arm. The bottom of the cuff should be approximately 1/2" above your elbow.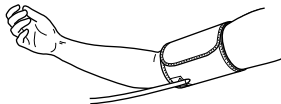


**NOTE:** Be careful not to rest your arm on the air tube. This will restrict the flow of air to the cuff.

# TAKING A MEASUREMENT

## ⚠ CAUTION


Operate the device only as intended. Do not use the device for any other purpose.

1. Sit on a chair with your feet flat on the floor. Place your left arm on a table so the cuff is level with your heart.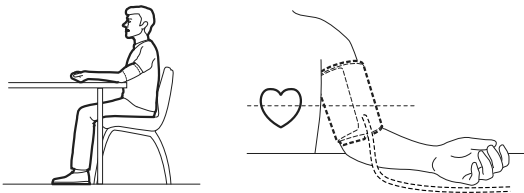

Remain still and do not move your arm until the entire measurement is complete.

2. Press the ON/OFF START button.

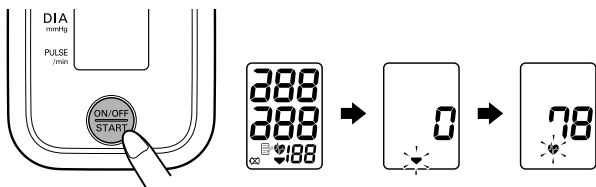All symbols appear on the display.

## ⚠ CAUTION

Inflating to a higher pressure than necessary may result in bruising where the cuff is applied.

# TAKING A MEASUREMENT

3. The cuff starts to inflate. The unit automatically inflates to 170mmHg. The cuff needs to inflate 30 to 40 mmHg higher than your expected systolic pressure.

**NOTES:**

- Do not apply more pressure than necessary.
- The monitor will not inflate above 299 mmHg.
- To stop the inflation or measurement, press the ON/OFF START button. The monitor will stop inflating, start deflating and will turn off.

If your systolic pressure is known to be higher, after the unit starts to inflate the cuff, press and hold the ON/OFF START button until the desired inflation value is reached.

For example: If your systolic pressure is known to be 180 mmHg, press and hold the ON/OFF START button until the inflation level is 210 to 220 mmHg.

4. Inflation stops automatically and the measurement is started.

As the cuff deflates, decreasing numbers appear on the display.


The Heartbeat Symbol (  ) flashes at every heartbeat.

5. When the measurement is complete, the arm cuff completely deflates. Your blood pressure and pulse rate are displayed.

6. Press the ON/OFF START button to turn the monitor off.

**NOTE:** The monitor will automatically turn off after 5 minutes.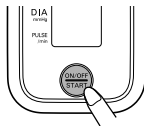


# USING THE MEMORY FUNCTION

The monitor automatically stores the last measurement values in the memory.

## TO DISPLAY THE MEASUREMENT VALUES

1. Press and hold the ON/OFF START button until the last measurement values are displayed along with the Memory Symbol (☰).

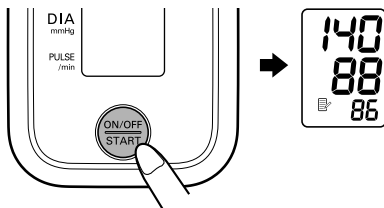2. Press the ON/OFF START button to turn the power off.

# CARE AND MAINTENANCE

*To keep your digital blood pressure monitor in the best condition and protect the unit from damage, follow the directions listed below:*

**Do not forcefully bend** the arm cuff or air tube. Do not fold tightly.

**Clean the monitor with a soft dry cloth.** Do not use any abrasive or volatile cleaners. Do not attempt to wash the cuff.

## **⚠ CAUTION**

Do not submerge the device or any of the components in water. Do not subject the monitor to extreme hot or cold temperatures, humidity or direct sunlight.

## **⚠ CAUTION**

Store the device and the components in a clean, safe location.

## **⚠ CAUTION**

Do not subject the monitor to strong shocks, such as dropping the unit on the floor.

**Remove the batteries** if the unit will not be used for three months or longer. Always replace all the batteries with new ones at the same time.

**Use the unit consistent with the instruction provided in this manual.** Use only authorized parts and accessories.

## **⚠ CAUTION**

Changes or modification not approved by the manufacturer will void the user warranty. Do not disassemble or attempt to repair the unit or components.

## **⚠ CAUTION**

Use only authorized parts and accessories. Parts and accessories not approved for use with the device may damage the unit.

# ERROR INDICATORS AND TROUBLESHOOTING TIPS

## ERROR INDICATORS

<b>SYMBOL</b>	<b>CAUSE</b>	<b>CORRECTION</b>
	Monitor could not detect pulse wave.	Take another measurement and remain still until the measurement is complete.
	Air plug is not connected.	
	Cuff under-inflated. Cuff not applied correctly.	Remove the arm cuff. Read “Taking a Measurement”. Take another measurement.
	Cuff over-inflated	Remove the arm cuff. Read “Taking a Measurement”. Take another measurement without inflating above 299 mmHg.
	Batteries are worn	Replace the four batteries. Refer to “Battery Installation.”

# ERROR INDICATORS AND TROUBLESHOOTING TIPS

## TROUBLESHOOTING TIPS

<b>PROBLEM</b>	<b>CAUSES AND SOLUTIONS</b>
No power. No display appears on the unit.	Replace all four batteries with new ones. Check the battery installation for proper placement of the battery polarities.
Measurement values appear too high or too low.	Blood pressure varies constantly. Many factors including stress, time of day, and how you wrap the cuff, may affect your blood pressure. Review the sections “Before Taking a Measurement” and “Taking a Measurement.”

# FCC STATEMENT

**Note:****POTENTIAL FOR RADIO/TELEVISION INTERFERENCE (for U.S.A. only)**

This product has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC rules.

These limits are designed to provide reasonable protection against harmful interference in a residential installation. The product generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If the product does cause harmful interference to radio or television reception, which can be determined by turning the product on and off, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the product and the receiver.
- Connect the product into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

**POTENTIAL FOR RADIO/TELEVISION INTERFERENCE (for Canada only)**

This digital apparatus does not exceed the Class B limits for radio noise emissions from digital apparatus as set out in the interference-causing equipment standard entitled "Digital Apparatus", ICES-003 of the Canadian Department of Communications.

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Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

# WARRANTY

Your ReliOn® 7100REL Automatic Blood Pressure is warranted to be free from defects in materials and workmanship appearing within one year from the date of purchase, when used in accordance with the instructions provided with the monitor. The above warranty extend only to the original retail purchaser.

We will, at our option, repair or replace without charge any monitor or arm cuff covered by the above warranty. Repair or replacement is our only responsibility and your only remedy under the above warranty.

To obtain warranty service contact Omron Healthcare for the address of the repair location. Shipping instructions and repair information is available by calling **1-800-634-4350** or visit our website at [www.omronhealthcare.com](http://www.omronhealthcare.com).

Enclose the Proof of Purchase. Include \$5.00 for the return shipping and handling. Include a letter, with your name, address, phone number, and description of the specific problem. Pack the product carefully to prevent damage in transit. Because of possible loss in transit, we recommend insuring the product with return receipt requested.

**ALL IMPLIED WARRANTIES, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR PARTICULAR PURPOSE, ARE LIMITED TO THE DURATION OF THE APPLICABLE WRITTEN WARRANTY ABOVE.** Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.

**OMRON SHALL NOT BE LIABLE FOR LOSS OF USE OR ANY OTHER INCIDENTAL, CONSEQUENTIAL OR INDIRECT COSTS, EXPENSES OR DAMAGES.** Some states do not allow the exclusion or limitation of incidental or consequential damages, so the above exclusions may not apply to you.

This warranty gives you specific legal rights, and you may also have other rights which may vary from state to state.

## FOR CUSTOMER SERVICE

Visit our web site at:

[www.omronhealthcare.com](http://www.omronhealthcare.com)

Call toll free:

1-800-634-4350

# SPECIFICATIONS

Model:	<b>7100REL (HEM-7102-WM)</b>
Display:	LCD Digital Display
Measurement Range:	Pressure: 0 to 299 mmHg, Pulse: 40 to 180/min.
Accuracy/Calibration:	Pressure: $\pm 3$ mmHg or 2% of reading Pulse: $\pm 5$ % of reading
Inflation:	Automatic by electric pump
Deflation:	Automatic pressure release valve
Rapid Air Release:	Automatic exhaust valve
Pressure Detection:	Capacitive pressure sensor
Measurement Method:	Oscillometric method
Pulse Wave Detection:	Capacitance type pressure sensor
Power Source:	1.5 V 4 "AAA" batteries
Battery Life:	Approx. 300 uses when used once a day for 2 minutes with 4 new alkaline batteries
Operating Temperature /Humidity:	50°F to 104°F (10°C to 40°C) /15 to 90% RH
Storage Temperature /Humidity/Air Pressure:	-4°F to 140°F (-20°C to 60°C) /10 to 95% RH /700 - 1060 hPa
Main Unit Weight:	Approximately 12 oz. (340 g) not including batteries
Main Unit Dimensions:	Approximately 4 1/8" (l) $\times$ 4 7/8" (w) $\times$ 3 1/8" (h) (103 mm $\times$ 125 mm $\times$ 80 mm)
Cuff Size:	Approximately 5 3/4" (w) $\times$ 17 1/2" (l) (146 mm $\times$ 446 mm) Cuff tube 23 5/8" (600 mm)
Cuff Circumference:	Fits arm circumferences 9" to 13" (220 mm to 320 mm)
Contents:	Main Unit, Cuff, Instruction Manual and 4 "AAA" batteries

**NOTE: These specifications are subject to change without notice.**

# NOTES

Manufactured by OMRON HEALTHCARE

Made in China

Distributed by:

**OMRON HEALTHCARE, INC.**

1200 Lakeside Drive

Bannockburn, Illinois 60015

[www.omronhealthcare.com](http://www.omronhealthcare.com)

Copyright © 2008 Omron Healthcare, Inc.

ReliOn® is a registered trademark used under license by Omron Healthcare, Inc.

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por Omron Healthcare, Inc.