
INSTRUCTION MANUAL

Automatic Blood Pressure Monitor with Two Cuffs Model 7300REL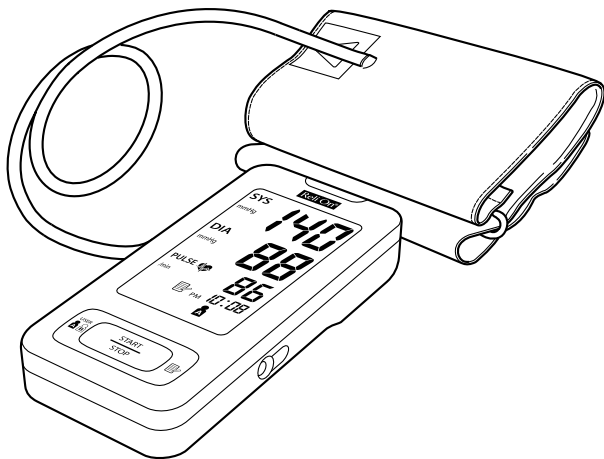

ESPAÑOL

ENGLISH

TABLE OF CONTENTS

Before Using the Monitor

Introduction	3
Safety Information	4
Operating the Device	4
Risk of Electrical Shock	6
Care and Maintenance	6
Before Taking a Measurement	7

Operating Instructions

Know Your Unit	8
Unit Display	10
Display Symbols	11
Irregular Heartbeat Symbol (☹)	11
Movement Error Symbol (🚫)	11
User ID Symbol (👤 / 🏠)	11
Heartbeat Symbol/Hypertension Indicator (💓)	12
Average Value Symbol (📊)	12
Using the AC Adapter	15
Battery Installation	16
Setting the Date and Time	18
Setting the Medication Reminder Alarm	22
Applying the Arm Cuff	26
Taking a Measurement	30
Using the Memory Function	33
Averaging Function	33
To Display the Measurement Values	33
To Delete All Values Stored in the Memory	35

Care and Maintenance

Care and Maintenance	36
Error Indicators and Troubleshooting Tips	38
FCC Statement	40
Limited Warranty	41
Specifications	42

INTRODUCTION

Thank you for purchasing the ReliOn® 7300REL IntelliSense® Automatic Blood Pressure Monitor.

Fill in for future reference.

DATE PURCHASED: _____

SERIAL NUMBER: _____

- *Staple your purchase receipt here*
- *Register your product on-line at www.register-omron.com*

Your new digital blood pressure monitor uses the oscillometric method of blood pressure measurement. This means the monitor detects your blood's movement through your brachial artery and converts the movements into a digital reading. An oscillometric monitor does not need a stethoscope so the monitor is simple to use.

The 7300REL comes with the following components:

- Monitor
- Standard Arm Cuff
- Large Arm Cuff
- AC Adapter
- 4 "AAA" Batteries
- Storage Case
- Instruction Manual printed in English and Spanish
- Quick Start Guide
- Log Book
- Cuff Guide

The 7300REL Automatic Blood Pressure Monitor is intended for home use.

 Please read this instruction manual thoroughly before using the unit. Please keep for future reference.

For specific information about your own blood pressure, **CONSULT YOUR DOCTOR.**

SAVE THESE INSTRUCTIONS

SAFETY INFORMATION

INTENDED USE

This product is designed to measure the blood pressure and pulse rate in adult patient population, following the instructions in this manual. It is mainly designed for general home use.

To assure the correct use of the product, basic safety measures should always be followed including the warnings and cautions listed in this instruction manual.

SAFETY SYMBOLS USED IN THIS INSTRUCTION MANUAL

Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

Indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury to the user or patient or damage to the equipment or other property.

OPERATING THE DEVICE

- ▲ Contact your physician for specific information about your blood pressure. Self-diagnosis and treatment using measured results may be dangerous. Follow the instructions of your physician or licensed healthcare provider.
- ▲ **DO NOT** adjust medication based on measurement results from this blood pressure monitor. Take medication as prescribed by your physician.
Only a physician is qualified to diagnose and treat High Blood Pressure.

SAFETY INFORMATION

OPERATING THE DEVICE (continued)

- ⚠ The monitor is not intended to be a diagnostic device.
- ⚠ If battery fluid should get in your eyes, immediately rinse with plenty of clean water. Contact a physician immediately.
- ⚠ Read all of the information in the instruction manual and any other literature in the box before operating the unit.
- ⚠ This device is intended for use in measuring blood pressure and pulse rate in the adult population. Do not use this device on infants or persons who cannot express their intentions.
- ⚠ Operate the device only as intended. Do not use the device for any other purpose.
- ⚠ Dispose of the device, components and optional accessories according to applicable local regulations. Unlawful disposal may cause environmental pollution.
- ⚠ Do not use a cellular phone near the device. It may result in an operational failure.
- ⚠ Use only authorized parts and accessories. Parts and accessories not approved for use with the device may damage the unit.
- ⚠ Use only 1.5V alkaline batteries with this device.
Do not use other types of batteries. This may damage the unit.

SAFETY INFORMATION

RISK OF ELECTRICAL SHOCK

- ⚠ Do not plug or unplug the power cord into the electrical outlet with wet hands.
- ⚠ Do not overload power outlets. Plug the device into the appropriate voltage outlet.

CARE AND MAINTENANCE

- ⚠ Do not subject the monitor to strong shocks, such as dropping the unit on the floor.
- ⚠ Do not submerge the device or any of the components in water.
- ⚠ Store the device and the components in a clean, safe location.
- ⚠ Changes or modification not approved by the manufacturer will void the user warranty. Do not disassemble or attempt to repair the unit or components.


BEFORE TAKING A MEASUREMENT

To ensure a reliable reading follow these recommendations:

1. Avoid eating, drinking alcohol, smoking, exercising, and bathing for 30 minutes before taking a measurement. Rest for at least 15 minutes before taking the measurement.
2. Stress raises blood pressure. Avoid taking measurements during stressful times.
3. Measurements should be taken in a quiet place.
4. Remove tight-fitting clothing from your arm.
5. Sit on a chair with your feet flat on the floor. Rest your arm on a table so that the cuff is at the same level as your heart.
6. Remain still and do not talk during the measurement.
7. Wait 2-3 minutes between measurements. The wait time allows the arteries to return to the condition prior to taking the blood pressure measurement. You may need to increase the wait time depending on your individual physiological characteristics.
8. Keep a record of your blood pressure and pulse readings for your physician. A single measurement may not provide an accurate indication of your true blood pressure. You need to take and record several readings over a period of time. Try to measure your blood pressure at about the same time each day for consistency.

KNOW YOUR UNIT


Main Unit: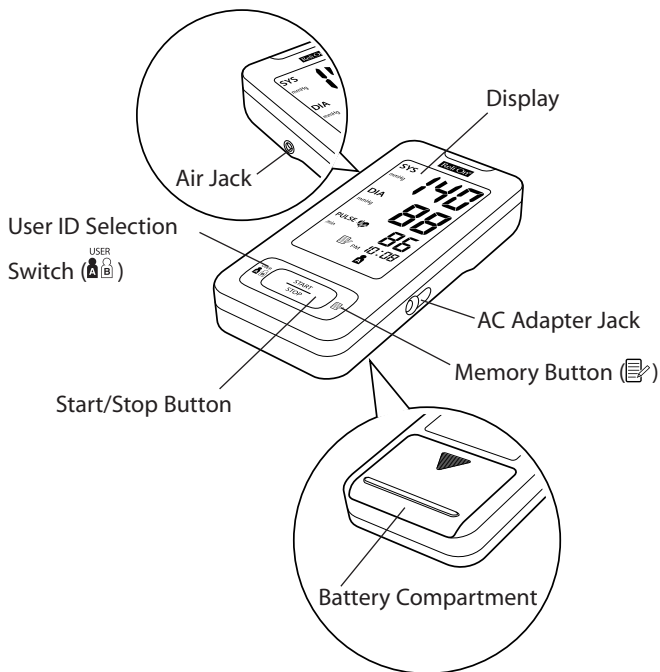


KNOW YOUR UNIT

Components:


Arm Cuffs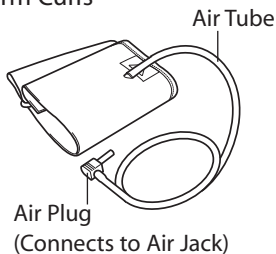


Standard Arm Cuff

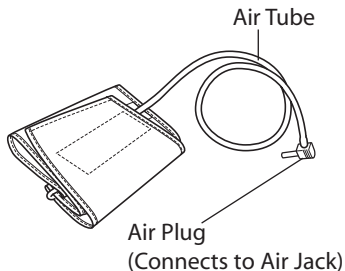Large Arm Cuff

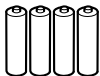
Four "AAA" Batteries

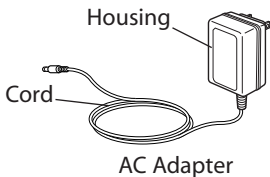
AC Adapter


Storage Case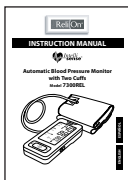

Instruction Manual

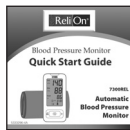Quick Start Guide

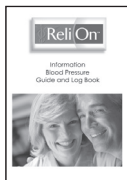
Log Book


Cuff Guide

UNIT DISPLAY

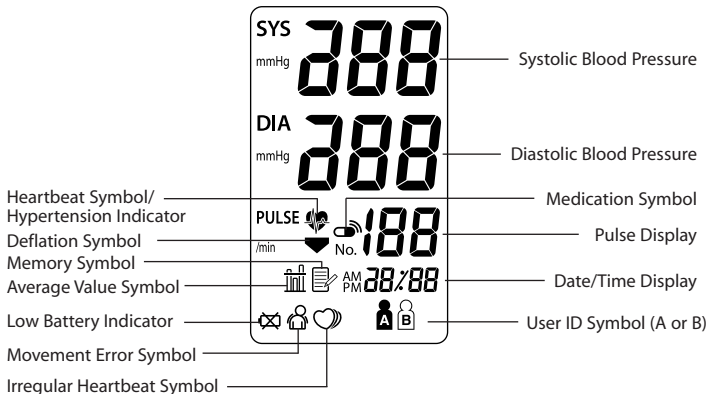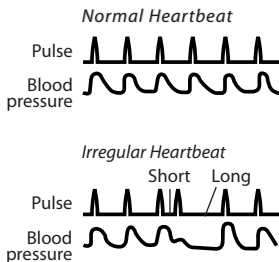
DISPLAY SYMBOLS

IRREGULAR HEARTBEAT SYMBOL ()

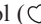When the monitor detects an irregular rhythm two or more times during the measurement, the Irregular Heartbeat Symbol () will appear on the display with the measurement values.

An irregular heartbeat rhythm is defined as a rhythm that varies by less than 25% from the average rhythm or 25% more than the average rhythm detected while the monitor is measuring the systolic and diastolic blood pressure.

If the Irregular Heartbeat Symbol () displays with your measurement results, we recommend you consult your physician. Follow the directions of your doctor.

MOVEMENT ERROR SYMBOL ()

The Movement Error Symbol () is displayed if you move your body during the measurement. Please remove the arm cuff, and wait 2-3 minutes. Take another measurement.

USER ID SYMBOL (/)

The User ID Symbol ( / ) indicates User A or User B as selected by the user to take a measurement or when using the memory function.

To change the User ID, press the User ID button on the monitor (Refer to "Taking a Measurement" section for more information).

DISPLAY SYMBOLS

HEARTBEAT SYMBOL/HYPERTENSION INDICATOR ()

TAKING A MEASUREMENT

The Heartbeat Symbol () flashes on the display during the measurement. The Heartbeat Symbol () flashes at every heartbeat.

When the measurement is complete, the Heartbeat Symbol () flashes on the display with your blood pressure and pulse rate if the reading is above 135 for the Systolic Blood Pressure value and/or 85 for the Diastolic Blood Pressure value.

USING THE MEMORY FUNCTION

The Heartbeat Symbol () flashes when the blood pressure reading stored in the memory is above 135 for the Systolic Blood Pressure value and/or 85 for the Diastolic Blood Pressure value.

Omron Healthcare recommends you contact your physician if the Heartbeat Symbol () flashes on the display with your measurement values.

AVERAGE VALUE SYMBOL ()

The Average Value Symbol () is displayed when you press the Memory button (). The most recent average reading appears on the display screen.

DISPLAY SYMBOLS

Internationally recognized guidelines for upper limit of normal home blood pressure readings are as follows:

Home Guideline for Upper Limit of Normal BP*	
Systolic Blood Pressure	135 mmHg
Diastolic Blood Pressure	85 mmHg

This is a general guideline because blood pressure depends on age and morbidity. A lower home blood pressure goal may be recommended for certain patients, including diabetic patients, pregnant women, and patients with renal failure.

⚠ WARNING

The monitor is not intended to be a diagnostic device.

⚠ WARNING

Contact your physician for specific information about your blood pressure. Self-diagnosis and treatment using measured results may be dangerous. Follow the instructions of your physician or licensed healthcare provider.

* Source: Hypertension. 2008; 52: 10-29

USING THE AC ADAPTER

IMPORTANT SAFETY INSTRUCTIONS SAVE THESE INSTRUCTIONS

**- DANGER -
TO REDUCE THE RISK OF
FIRE OR ELECTRIC SHOCK,
CAREFULLY FOLLOW THESE INSTRUCTIONS.**

- ▲ For connection to a power supply not in the U.S.A., use an attachment plug adapter of the proper configuration for the power outlet.
- ▲ The power unit (AC Adapter) is intended to be correctly oriented in a vertical or floor mount position.

- NOTES:**
- Use only the authorized AC Adapter that came with this monitor.
 - We recommend you install the batteries even if the AC Adapter is used. If no batteries are installed, you may need to reset the date and time if the AC Adapter is disconnected. The measurement results will not be deleted.


▲ CAUTION

Use only authorized parts and accessories. Parts and accessories not approved for use with the device may damage the unit.

USING THE AC ADAPTER

TO CONNECT THE AC ADAPTER

- ① Insert the AC Adapter Plug into the AC Adapter Jack on the right side of the monitor as illustrated.
- ② Plug the AC Adapter into a 120V AC outlet (60 cycles).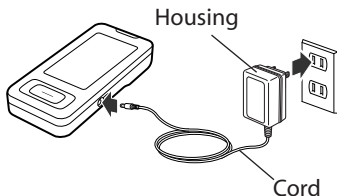

TO DISCONNECT THE AC ADAPTER

- ① Hold and pull the Housing to remove the AC Adapter from the electrical outlet. Do not remove by pulling the cord.
- ② Remove the AC Adapter plug from the monitor.

⚠ WARNING


Do not plug or unplug the power cord into the electrical outlet with wet hands.

⚠ WARNING


Do not overload power outlets. Plug the device into the appropriate voltage outlet.

BATTERY INSTALLATION

1. Press the ▼ indicator on the battery cover and slide the cover off in the direction of the arrow.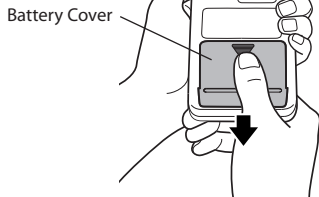

2. Install 4 “AAA” size batteries so the + (positive) and – (negative) polarities match the polarities of the battery compartment as indicated.
3. Replace the battery cover.

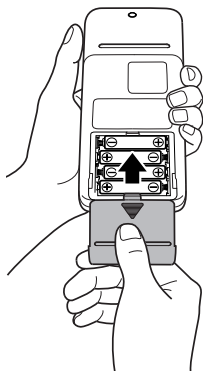BATTERY INSTALLATION

BATTERY REPLACEMENT

Low Battery Indicator

When the Low Battery Indicator appears on the display screen, turn the monitor off and remove all the batteries. Replace with four new batteries at the same time.

⚠ WARNING

If battery fluid should get in your eyes, immediately rinse with plenty of clean water. Contact a physician immediately.

- NOTES:**
- Turn the unit off before replacing the batteries. If the batteries are removed while the unit is still on, the date and time will be reset to that of the previous use. However, the measurement results are not deleted.
 - When the batteries are replaced, you may need to reset the date and time. If “-:--” appears on the display screen, refer to “Setting the Date and Time”.
 - The supplied batteries may have a shorter life.

⚠ CAUTION

Dispose of the device, components and optional accessories according to applicable local regulations. Unlawful disposal may cause environmental pollution.


SETTING THE DATE AND TIME

Set the monitor to the current date and time before taking a measurement for the first time. If the date and time are not set correctly, the measurement values stored in the memory and the average values may not be correct.

If the date and time are not set, “-:--” appears during or after measurement.

1. TO START THE SETTING

Press and hold the USER ID Selection Switch () to adjust the date and time.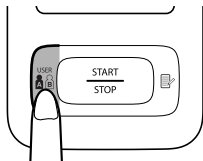

The year flashes on the display.


SETTING THE DATE AND TIME

2. SETTING THE YEAR


The year can be set between 2011 and 2040.
When the display reaches 2040, it will return to 2011.

Press the Memory button (📄) to advance by one year increments.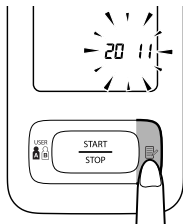

Press the User ID Selection Switch (USER A/B) to set the current year.

The month flashes on the display.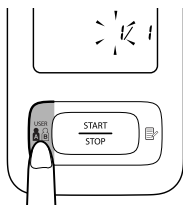


NOTE: Press and hold the Memory button (📄) to advance the date and time values faster.

3. SETTING THE MONTH

Press the Memory button (📄) to advance by one month increments.

Press the User ID Selection Switch (USER A/B) to set the current month.

The day flashes on the display.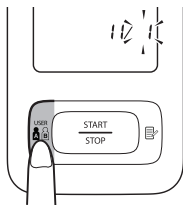


SETTING THE DATE AND TIME

4. SETTING THE DAY

Press the Memory button (📄) to advance by one day increments.

Press the USER ID Selection Switch (USER A/B) to set the current day.

The hour flashes on the display.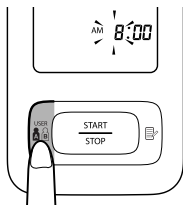


5. SETTING THE HOUR

The time is set using AM or PM.

Press the Memory button (📄) to advance by one hour increments.

Press the USER ID Selection Switch (USER A/B) to set the current hour.

The minute flashes on the display.

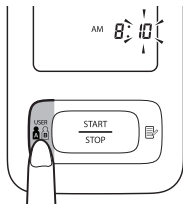6. SETTING THE MINUTE

Press the Memory button (📄) to advance by one minute increments.

Press the USER ID Selection Switch (USER A/B) to set the minute setting.

7. TO TURN THE MONITOR OFF

Press the START/STOP button.

SETTING THE DATE AND TIME

RESETTING THE DATE AND TIME

To adjust the date or time after it has been setup when you first use the unit.

Start with the monitor in the off position.

1. Press and hold the USER ID Selection Switch ()

The User ID Symbol ( or ) and the Medication Symbol () will appear on the display.

Press the USER ID Selection Switch () repeatedly until the year flashes on the display.

2. Refer to “Setting the Date and Time”.

The User ID Symbol ( or ) , the Medication Symbol () No.1, and the current setting (**on** or **off**) will appear on the display, after the minute has been set.

3. Press the START/STOP button to turn the monitor off.

NOTE: If you need to change the Date and Time, always adjust the Date and Time setting before taking the next measurement.

SETTING THE MEDICATION REMINDER ALARM


The Medication Reminder Alarm can be set to sound twice a day to alert the user to take a medication. The Medication Reminder Alarm can be set for either AM or PM alerts. Only set the Medication Reminder Alarm after the date and time has been set. The Medication Reminder Alarm is set to off at the time of purchase.

- NOTES:**
- Press the START/STOP button to stop the Medication Reminder Alarm.
 - The Medication Reminder Alarm will automatically stop sounding after one minute.
 - The Medication Reminder Alarm will not sound while setting the date and time, setting the Medication Reminder Alarm, and taking a measurement.
 - The Medication Reminder Alarm will not sound without setting the date and time.

1. TO START THE SETTING

Press and hold the USER ID Selection Switch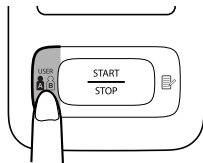


SETTING THE MEDICATION REMINDER ALARM

2. SELECTING THE USER ID

Press the Memory button (📄) to select your ID (A or B).

Press the USER ID Selection Switch (USER A B) to set the User ID.

The User ID Symbol (A or B), the Medication Symbol (📄) No.1, and the **OFF** setting appear on the display.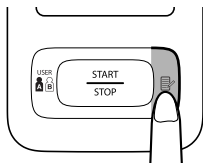


3. SETTING ON OR OFF

Press the Memory button (📄) to select on.

Press the USER ID Selection Switch (USER A B).

The hour flashes on the display.

- NOTES:**
- Press the Memory button (📄) to alternate between **on** or **off**.
 - If the Medication Reminder Alarm 1 is set to off, the Medication Reminder Alarm 2 setting display appears.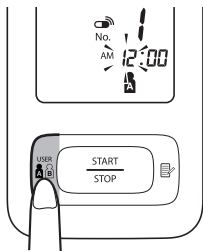

SETTING THE MEDICATION REMINDER ALARM


4. SETTING THE HOUR

The time is set using AM or PM.

Press the Memory button (📄) to advance by increments of one hour.

Press the USER ID Selection Switch (USER A/B) to set the hour.

The minute flashes on the display.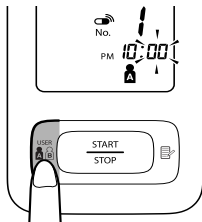


5. SETTING THE MINUTE

Press the Memory button (📄) to advance by increments of one minute.

Press the USER ID Selection Switch (USER A/B) to set the minute.

The Medication Symbol (📄) No.2 and the **OFF** setting appears on the display.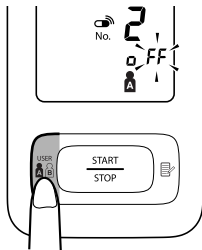


SETTING THE MEDICATION REMINDER ALARM


6. SETTING THE MEDICATION REMINDER ALARM 2

Repeat steps from 2 to 4 to set the Medication Reminder Alarm 2.

NOTE: If the Medication Reminder Alarm 2 is set to off, the year flashes on the display.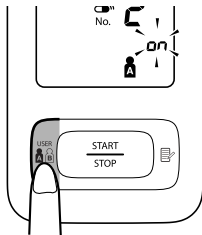

The year flashes on the display, after the Medication Reminder Alarm 2 has been set.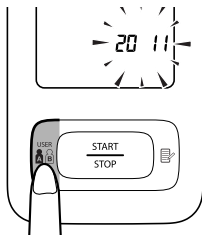

7. TO TURN THE MONITOR OFF

If the date and time are correct, press the START/STOP button to turn the monitor off.

To change the date and time refer to “Resetting the Date and Time” on page 21.

APPLYING THE ARM CUFF

⚠ CAUTION

Read all of the information in the instruction manual and any other literature in the box before operating the unit.

⚠ CAUTION

This device is intended for use in measuring blood pressure and pulse rate in the adult population. Do not use this device on infants or persons who cannot express their intentions.


USE CUFF GUIDE TO CHOOSE THE RIGHT CUFF

There are two cuffs included in this unit, standard size and large size. Use the Cuff Guide to determine which size cuff is right for you, based on the circumference of your arm.



1. Make a loop with your Cuff Guide.

Insert and pull the end with the arrow through the slot.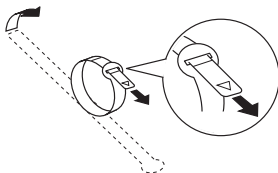


APPLYING THE ARM CUFF

- Fit the looped Cuff Guide on your upper arm.
- The Cuff Guide will indicate which arm cuff should be used. Use the cuff with the same colored sticker.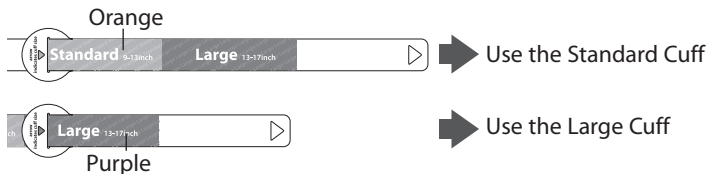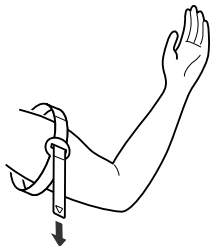


APPLYING THE CUFF ON THE LEFT ARM

- Make sure the air plug is securely inserted in the main unit.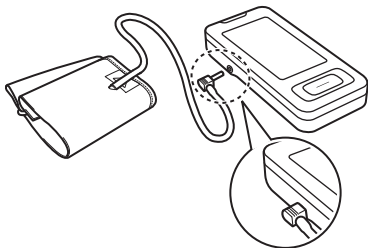


APPLYING THE ARM CUFF

2. Remove tight-fitting clothing from your upper left arm.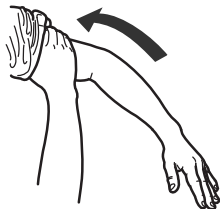

3. Apply the cuff to your left upper arm so the blue arrow is centered on the inside of your arm and aligned with your middle finger. The air tube runs down the inside of your arm. The bottom of the cuff should be approximately 1/2" above your elbow.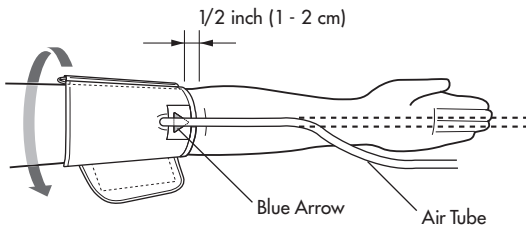

4. Wrap the cuff firmly in place around your arm using the closure strip.


APPLYING THE ARM CUFF

APPLYING THE CUFF ON THE RIGHT ARM

When taking a measurement using the right arm use this instruction for Step 3 on page 28.

Apply the cuff to your right upper arm so the blue arrow and the air tube runs along the inside of your elbow. The bottom of the cuff should be approximately 1/2" above your elbow.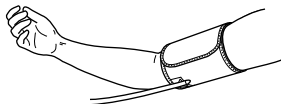

NOTE: Be careful not to rest your arm on the air tube. This will restrict the flow of air to the cuff.

TAKING A MEASUREMENT

The monitor is designed to take measurements and store the measurement values in the memory for two people using User ID A and User ID B.

⚠ WARNING


Contact your physician for specific information about your blood pressure. Self-diagnosis and treatment using measured results may be dangerous. Follow the instructions of your physician or licensed healthcare provider.

⚠ CAUTION

Operate the device only as intended. Do not use the device for any other purpose.


TAKING A MEASUREMENT

1. Sit on a chair with your feet flat on the floor. Place your left arm on a table so the cuff is level with your heart.

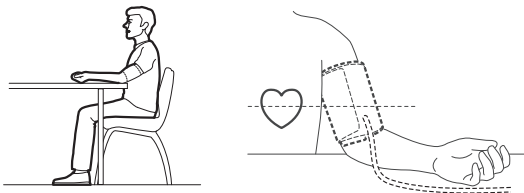2. Press the START/STOP button.

All symbols appear on the display. The User ID Symbol (A or B) with the current time will appear.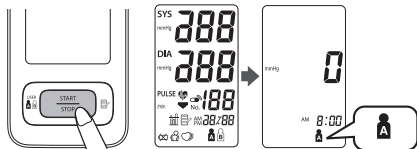


- NOTES:**
- The User ID last used will appear first.
 - The default setting is A.

TAKING A MEASUREMENT

3. Press the USER ID Selection Switch () to select your ID (A or B).

A and B are alternately displayed.

You do not need to press this button if no change is needed.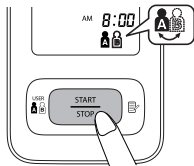


NOTE: Always use the same User ID when taking a measurement. The monitor stores the measurement values in the memory by the User ID selected.

4. Press the START/STOP button again.

The cuff starts to inflate automatically. As the cuff inflates, the monitor automatically determines your ideal inflation level.

This monitor detects your blood pressure and pulse rate during inflation. The Heartbeat Symbol () flashes at every heartbeat. Do not move your arm and remain still until the entire measurement process is completed.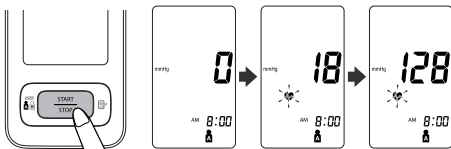

NOTE: To stop the inflation or measurement, press the START/STOP button. The monitor will stop inflating, start deflating and will turn off.


TAKING A MEASUREMENT

5. When the measurement is complete, the arm cuff completely deflates. Your blood pressure and pulse rate are displayed.

6. Press the START/STOP button to turn the monitor off.

NOTE: The monitor will automatically turn off after two minutes.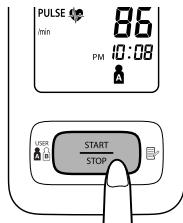

USING THE MEMORY FUNCTION

The monitor is designed to store the blood pressure and the pulse rate in the memory for two people (USER A and USER B) every time a measurement is completed.

The monitor automatically stores up to 60 sets of measurement values (blood pressure and pulse rate) for each user (A and B). When 60 sets of measurement values are stored, the oldest record is deleted to save the most recent values.

AVERAGING FUNCTION

The monitor calculates the average reading based on the three most recent sets of measurement values taken within 10 minutes of the most recent reading.

- NOTES:**
- The average is based on the reading taken with setting the date and time.
 - If two sets of measurement values are stored in the memory for the 10 minutes period, the average is based on the two sets of measurement values. If one set of measurement values is stored, this is displayed as the average.
 - When viewing the reading taken without setting the date and time, “-:--” is displayed instead of the date and time.


TO DISPLAY THE MEASUREMENT VALUES

1. Press the START/STOP button to turn the monitor on.
2. Select your User ID (A or B).

USING THE MEMORY FUNCTION

3. Press the Memory button (📄).

The Average Symbol (📊) appears on the display when the average reading is shown.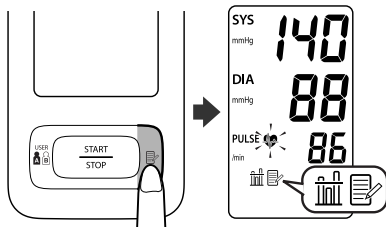


4. Press the Memory button (📄) to display the most recent set of measurement values on the screen. The values are displayed from the most recent to the oldest.

NOTE: The date and time are alternately displayed with the measurement values.

Press the Memory button (📄) to display the next set of values.

Press and hold the Memory button (📄) to display the values faster.

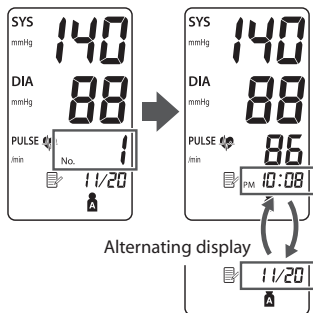3. Press the START/STOP button to turn the power off.

USING THE MEMORY FUNCTION

TO DELETE ALL VALUES STORED IN THE MEMORY

The values stored in the memory are deleted by User ID.


You cannot partially delete values stored in the memory. All values for the user you select will be deleted.

1. Press the START/STOP button to turn the monitor on.
2. Press the USER ID Selection Switch () to select your ID (A or B).

3. Press the Memory button ().

While holding it down, press the START/STOP button together for more than two seconds.

4. Press the START/STOP button to turn the monitor off.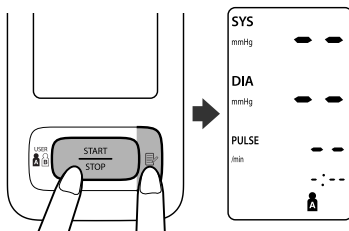


NOTE: The monitor will automatically turn off after two minutes.

CARE AND MAINTENANCE

To keep your digital blood pressure monitor in the best condition and protect the unit from damage, follow the directions listed below:

Keep the monitor in the Storage Case when not in use.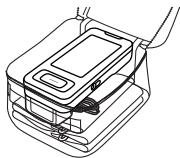

Do not forcefully bend the arm cuff or air tube. Do not fold tightly.

Clean the monitor with a soft dry cloth. Do not use any abrasive or volatile cleaners. Do not attempt to clean the cuff.

⚠ CAUTION

Do not submerge the device or any of the components in water. Do not subject the monitor to extreme hot or cold temperatures, humidity or direct sunlight.

⚠ CAUTION

Store the device and the components in a clean, safe location.

⚠ CAUTION

Do not subject the monitor to strong shocks, such as dropping the unit on the floor.

Remove the batteries if the unit will not be used for three months or longer. Always replace all the batteries with new ones at the same time.

CARE AND MAINTENANCE

Use the unit consistent with the instruction provided in this manual.

⚠ CAUTION


Changes or modification not approved by the manufacturer will void the user warranty. Do not disassemble or attempt to repair the unit or components.

⚠ CAUTION

Use only authorized parts and accessories. Parts and accessories not approved for use with the device may damage the unit.

ERROR INDICATORS AND TROUBLESHOOTING TIPS

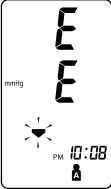ERROR INDICATORS

SYMBOL	CAUSE	CORRECTION
	Cuff over-inflated	Remove the arm cuff. Read, "Taking a Measurement". Take another measurement.
	Monitor could not detect pulse wave.	Take another measurement and remain still until the measurement is complete.
	Air plug is not connected.	
	Device error.	Contact Customer Services.
	Batteries are worn	Replace the four batteries. Refer to "Battery Installation."

ERROR INDICATORS AND TROUBLESHOOTING TIPS

TROUBLESHOOTING TIPS

PROBLEM	CAUSES AND SOLUTIONS
No power. No display appears on the unit.	Replace all four batteries with new ones. Check the battery installation for proper placement of the battery polarities.
Measurement values appear too high or too low.	Blood pressure varies constantly. Many factors including stress, time of day, and how you wrap the cuff, may affect your blood pressure. Review the sections “Before Taking a Measurement” and “Taking a Measurement”.

FCC STATEMENT

NOTE:

POTENTIAL FOR RADIO/TELEVISION INTERFERENCE (for U.S.A. only)

This product has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC rules.

These limits are designed to provide reasonable protection against harmful interference in a residential installation. The product generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If the product does cause harmful interference to radio or television reception, which can be determined by turning the product on and off, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the product and the receiver.
- Connect the product into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

POTENTIAL FOR RADIO/TELEVISION INTERFERENCE (for Canada only)

This digital apparatus does not exceed the Class B limits for radio noise emissions from digital apparatus as set out in the interference-causing equipment standard entitled “Digital Apparatus”, ICES-003 of the Canadian Department of Communications.

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Changes or modifications not expressly approved by the party responsible for compliance could void the user’s authority to operate the equipment.

LIMITED WARRANTY

Your ReliOn® 7300REL Automatic Blood Pressure Monitor, excluding the arm cuffs and batteries, is warranted to be free from defects in materials and workmanship appearing within 5 years from the date of purchase, when used in accordance with the instructions provided with the monitor. The arm cuffs are warranted to be free from defects in materials and workmanship appearing within one year from the date of purchase when the monitor is used in accordance with the instructions provided with the monitor. The above warranties extend only to the original retail purchaser.

We will, at our option, repair or replace without charge any monitor or arm cuffs covered by the above warranties. Repair or replacement is our only responsibility and your only remedy under the above warranties.

To obtain warranty service contact Customer Service by calling **1-800- 634-4350** for the address of the repair location and the return shipping and handling fee.

Enclose the Proof of Purchase. Include a letter, with your name, address, phone number, and description of the specific problem. Pack the product carefully to prevent damage in transit. Because of possible loss in transit, we recommend insuring the product with return receipt requested.

THE FOREGOING IS THE SOLE WARRANTY PROVIDED BY OMRON IN CONNECTION WITH THIS PRODUCT, AND OMRON HEREBY DISCLAIMS ANY OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. IMPLIED WARRANTIES AND OTHER TERMS THAT MAY BE IMPOSED BY LAW, IF ANY, ARE LIMITED IN DURATION TO THE PERIOD OF THE ABOVE EXPRESS WARRANTY.

OMRON SHALL NOT BE LIABLE FOR LOSS OF USE OR ANY OTHER SPECIAL, INCIDENTAL, CONSEQUENTIAL OR INDIRECT COSTS, EXPENSES OR DAMAGES.

This warranty provides you with specific legal rights, and you may have other rights that vary by jurisdiction. Because of special local requirements, some of the above limitations and exclusions may not apply to you.

FOR CUSTOMER SERVICE

Visit our web site at:

www.omronhealthcare.com

Call toll free:


1-800-634-4350

SPECIFICATIONS

Model:	7300REL (HEM-7300-WM)
Display:	LCD Digital Display
Measurement Range:	Pressure: 0 to 299 mmHg, Pulse: 40 to 180/min.
Accuracy/Calibration:	Pressure: ± 3 mmHg or 2% of reading Pulse: $\pm 5\%$ of reading
Inflation:	Automatic by electric pump
Deflation:	Automatic rapid deflation pressure
Rapid Air Release:	Automatic exhaust valve
Measurement Method:	Oscillometric method
Power Source:	AC/DC adapter (Included; Input 100-240V 50Hz/60Hz, Output 6V/9.6W) or 4 "AAA" batteries (1.5V Alkaline)
Battery Life:	Approximately 300 uses when used 3 times a day with new alkaline batteries when inflated to 170mmHg at 73°F (23°C).
Operating Temperature /Humidity:	50°F to 104°F (10°C to 40°C) /15 to 90% RH
Storage Temperature /Humidity/Air Pressure:	-4°F to 140°F (-20°C to 60°C) /10 to 95% RH /700 - 1060 hPa
Main Unit Weight:	Approximately 8 1/2 oz. (240 g) not including batteries
Main Unit Dimensions:	Approximately 6 3/16" (l) \times 2 15/16" (w) \times 1 7/32" (h) (157 mm \times 74 mm \times 30.6 mm)
Cuff Size:	Standard Cuff Approximately 5 3/4" (w) \times 17 1/2" (l) (146 mm \times 446 mm) Cuff tube 23 5/8" (l) (600 mm) Large Cuff Approximately 6 7/8" (w) \times 25 11/16" (l) (174 mm \times 652 mm) Cuff tube 35 7/16" (l) (900 mm)
Cuff Circumference:	Standard Cuff fits arm circumferences 9" to 13" (220 mm to 320 mm) Large Cuff fits arm circumferences 13" to 17" (320 mm to 420 mm)

SPECIFICATIONS

Memory:	Up to 60 per user
Contents:	Main Unit, Standard and Large Arm Cuffs, Instruction Manual, AC Adapter, 4 “AAA” batteries, Storage Case, Quick Start Guide, Log Book, and Cuff Guide

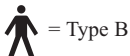NOTE: These specifications are subject to change without notice.

Manufactured by OMRON HEALTHCARE Co., Ltd.

Made in China

Distributed by:

OMRON HEALTHCARE, INC.

1925 West Field Court

Lake Forest, IL 60045 U.S.A.

www.omronhealthcare.com

Copyright © 2011 OMRON HEALTHCARE, INC.

ReliOn® is a registered trademark used under license by OMRON HEALTHCARE, INC.

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Fabricado por OMRON HEALTHCARE Co., Ltd.
Wal-Mart® Stores, Inc.