
ReliOn®

INSTRUCTION MANUAL

MANUAL BLOOD PRESSURE MONITOR

Model HEM-18SREL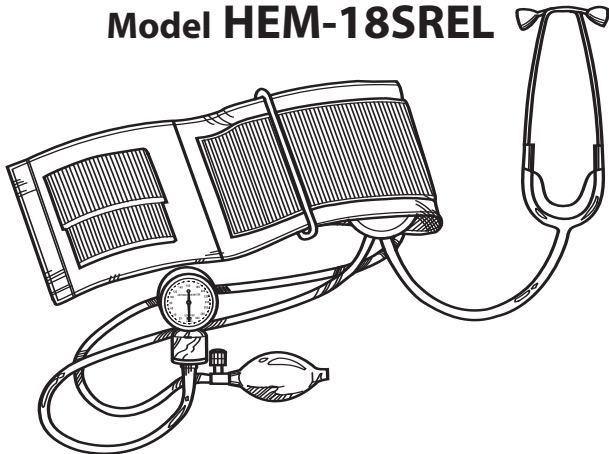

ESPAÑOL

ENGLISH

CAUTION:

This product contains natural rubber latex that may cause allergic reactions.

TABLE OF CONTENTS

Introduction	2
Important Safety Notes	3
Unit Assembly	4
Taking a Measurement	6
Care and Maintenance	9
Warranty Information	10
Quick Reference Guide	11

INTRODUCTION

Thank you for purchasing the ReliOn® HEM-18SREL Manual Blood Pressure Monitor.

The HEM-18SREL comes with the following components:

- Gauge
- Adult Arm Cuff with attached Stethoscope
- Stethoscope Binaural
- Inflation Bulb
- Instruction Manual printed in English and Spanish
- BP Guide and Log Book printed in English and Spanish
- Storage Case

The ReliOn® HEM-18SREL Manual Blood Pressure Monitor is intended for home use.

SAVE THESE INSTRUCTIONS

IMPORTANT SAFETY NOTES

To assure the correct use of the product basic safety measures should always be followed including the precautions listed below:

- ☑ Read all information in the instruction book and any other literature included in the box before using the unit.
- ☑ Contact your physician for specific information about your blood pressure. Follow the instructions of your healthcare provider.
- ☑ Operate the unit only as intended. Do not use for any other purpose.
- ☑ The unit is intended for use in measuring blood pressure in adult patient population.
- ☑ Assemble the unit correctly. Make sure parts are attached securely. Failure to do so may result in injury.
- ☑ Before you put the stethoscope binaural in your ears, make sure the eartips are securely fastened to the binaural.
- ☑ This product contains natural rubber latex that may cause allergic reactions.
- ☑ Changes or modifications not approved by the manufacturer will void the user warranty. Do not disassemble or attempt to repair the unit or components.

UNIT ASSEMBLY

HOW TO FOLD THE CUFF

The cuff should already be folded through the metal D-ring when you remove the cuff from the box. If the cuff is not prefolded, lay the cuff flat in front of you. Place the metal D-ring on your left. The stethoscope faces up. Pull the end of the cuff through the metal D-ring. When the cuff is folded correctly, the hook and pile material faces the outside of the cuff loop.


HOW TO ASSEMBLE THE MONITOR

Lay the parts out as illustrated on the next page. Refer to the illustration to help you put the unit together.

Follow these four easy steps:

- 1.) Attach the gauge to tubing on the left.
- 2.) Attach the inflation bulb to the tubing on the right.
- 3.) Attach the single end of the Y-tubing to the stethoscope chestpiece.
- 4.) Attach the double end of the Y-tubing to the binaural.

UNIT ASSEMBLY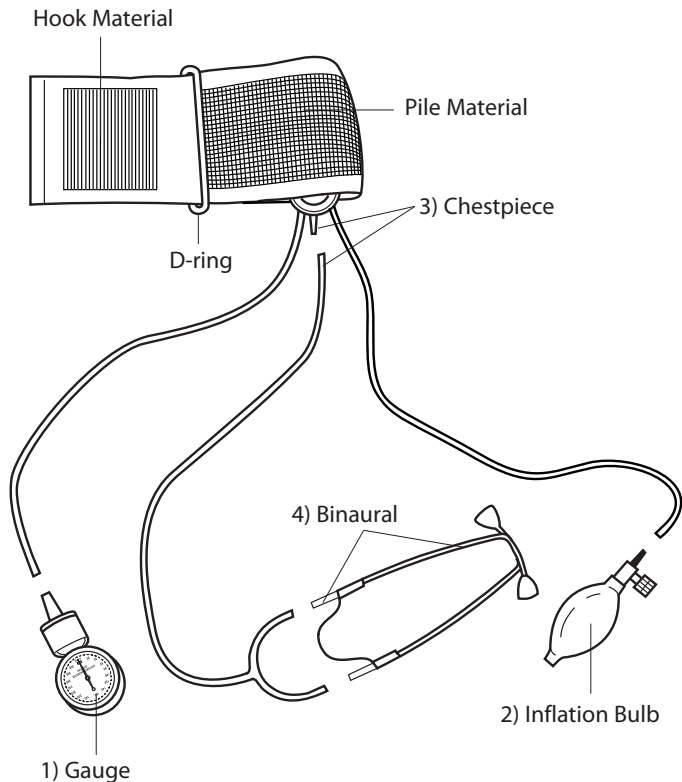

TAKING A MEASUREMENT




1. Sit upright but relaxed in a chair with a back. Keep your feet flat on the floor.
2. Remove tight fitting clothing from your left arm.
3. Make sure the parts are securely and correctly assembled.

CAUTION: Failure to assemble the parts correctly may result in injury or an incorrect reading.

4. Press two fingers one inch above the bend of your elbow on the inside of your arm. You may feel the pulse of the brachial artery. Even if you do not feel your pulse, the stethoscope will allow you to hear the pulse sounds.
5. Slip the cuff up your arm. Place the bottom edge of the cuff one-half inch above the elbow. Adjust the cuff so the Chestpiece sits on the inside of the arm over your brachial artery.
6. Pull the end of the cuff downward until the cuff fits snugly.


NOTE: Make sure the metal D-ring does not pinch you skin.

Press the hook material firmly against the pile material. Make sure the chestpiece sits over the brachial artery.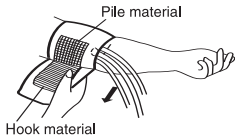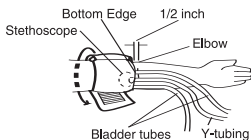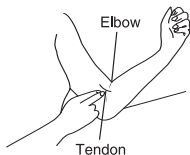

TAKING A MEASUREMENT

7. Place your arm on a table so the cuff is at the same level as your heart.

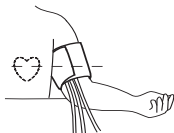8. **CAUTION:** Before you put the stethoscope binaural in your ears make sure the eartips are securely fastened to the binaural.

Insert the eartips into your ears. Adjust the metal binaural for comfort.

9. Clip the gauge to a hard cover book on the table in front of you. Hold the inflation bulb in your right hand. Close the air release valve attached to the inflation bulb by turning the valve clockwise.

10. To inflate the cuff, slowly and steadily pump the inflation bulb.
NOTE: If you know your systolic reading, pump the inflation bulb until the needle on the gauge reaches approximately 50 mmHg above your average systolic reading. If you do not know your systolic reading, pump the inflation bulb until the needle on the gauge reaches 180 mmHg.


CAUTION: DO NOT inflate the cuff above 280 mmHg. You may damage the unit or cause personal injury. If you are unsure, consult your healthcare provider for the correct inflation level.

TAKING A MEASUREMENT

11. Slowly open the air release valve by turning it counterclockwise. Release the air at a rate of 2 mmHg per second.

12. Listen carefully for your pulse beat. Watch the gauge needle. When you hear the first thumping sound note the reading on the gauge. This is your **SYSTOLIC** blood pressure value.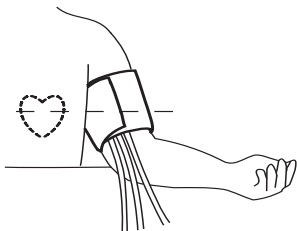

13. When you no longer hear any sounds note the reading on the gauge. This is your **DIASTOLIC** blood pressure value.

14. Turn the air release valve counterclockwise to release the remainder of the air. Remove the stethoscope binaural from your ears. Remove the cuff from your arm. Record your reading in the Blood Pressure Log.

NOTE: If you make a mistake, do not immediately reinflate the cuff. Release all the air in the cuff. Wait 5 minutes. The wait time allows the arteries to return to the condition prior to taking the blood pressure measurement.

CARE AND MAINTENANCE

Your new blood pressure unit has been carefully checked to ensure reliability and accuracy prior to shipment and use. As with any sensitive instrument subjected to repeated use, the manufacturer recommends your blood pressure gauge be checked annually by a medical professional for proper calibration. The unit is properly calibrated when the needle rests at zero.

To keep your blood pressure monitor in the best condition and protect the unit from damage follow the directions listed below:

Avoid washing or moistening the cuff. Never immerse any of the parts in water.

Store the monitor in a safe and dry location. Do not fold the cuff and tubing tightly.

Avoid subjecting the instrument to strong shocks, such as dropping the gauge on the floor or hitting the surface of the stethoscope. When the cuff is fully deflated, the needle stays within the zero indicator on the gauge. If the needle points outside of the zero indicator the gauge gives inaccurate readings. Return the unit to the manufacturer for calibration.

Use the unit consistent with the instruction provided in this manual. Use only authorized parts and accessories.

WARRANTY INFORMATION

LIMITED WARRANTIES

Your HEM-18SREL, ReliOn Manual Blood Pressure Monitor, is warranted to be free from defects in materials and workmanship appearing within 5 years from the date of purchase, when used in accordance with the instructions provided with the monitor. The above warranties extend only to the original retail purchaser.

We will, at our option, repair or replace without charge any monitor or monitor cuff covered by the above warranties. Repair or replacement is our only responsibility and your only remedy under the above warranties.

To obtain warranty service contact Omron Healthcare for the address of the repair location. Shipping instructions and repair information is available by calling **1-800-634-4350** or visiting our website at www.omronhealthcare.com.

Enclose the Proof of Purchase. Include \$5.00 for the return shipping and handling. Include a letter, with your name, address, phone number, and description of the specific problem. Pack the product carefully to prevent damage in transit. Because of possible loss in transit, we recommend insuring the product with return receipt requested.

ALL IMPLIED WARRANTIES, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR PARTICULAR PURPOSE, ARE LIMITED TO THE DURATION OF THE APPLICABLE WRITTEN WARRANTY ABOVE. Some provinces do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.

OMRON SHALL NOT BE LIABLE FOR LOSS OF USE OR ANY OTHER INCIDENTAL, CONSEQUENTIAL OR INDIRECT COSTS, EXPENSES OR DAMAGES. Some states do not allow the exclusion or limitation of incidental or consequential damages, so the above exclusions may not apply to you.

This warranty gives you specific legal rights, and you may also have other rights which may vary from state to state.

For Customer Service call toll free:

1-800-634-4350

QUICK REFERENCE GUIDE

BEFORE TAKING A MEASUREMENT

1. Relax for at least 5 minutes before taking the measurement.
2. Remove tight fitting clothing from you upper left arm.
3. Do not eat, smoke or exercise for at least 30 minutes before taking the measurement.
4. Do not take a measurement when you feel stressed.

TAKING A MEASUREMENT

1. Sit in a chair with your feet flat on the floor. Rest your left arm on a table so cuff is at the same level as your heart.
2. Apply the cuff at heart level.
3. Chestpiece sits over the brachial artery.
4. Eartips are securely fastened. Put the binaural in ears.
5. Inflate the cuff to the proper level.
6. Deflate the cuff at 2mmHg per second.
7. Listen for the first thumping sound, SYSTOLIC value.
8. No longer hear any sounds, DIASTOLIC value.
9. Record your reading in your BP Log.

AFTER TAKING A MEASUREMENT

1. Wait 5 to 10 minutes before taking another measurement.
2. Consult your healthcare provider for questions regarding your blood pressure.

Made in China

Manufactured for:

OMRON HEALTHCARE, INC.

1200 Lakeside Drive

Bannockburn, Illinois 60015

www.omronhealthcare.com

Made in China

© 2005 Omron Healthcare, Inc.

18SRELINST Rev A

Download from Www.Somanuals.com. All Manuals Search And Download.

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