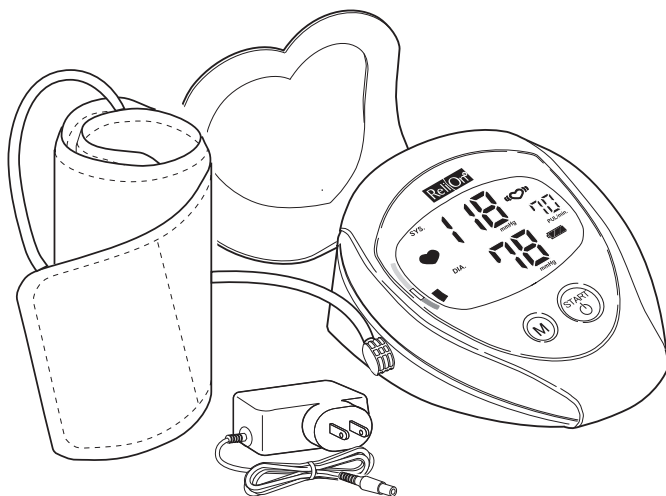
INSTRUCTION MANUAL

BLOOD PRESSURE MONITOR DESIGNED FOR WOMEN

Model UA-787REL

Bilingual Instruction Guide - English • Español
Includes Logbook

Congratulations on purchasing a state-of-the-art ReliOn® blood pressure monitor - one of the most technologically advanced monitors available today. Designed for ease of use and accuracy, this monitor will facilitate your daily blood pressure regimen.

Physicians agree that daily self-monitoring of blood pressure is an important step individuals can take to maintain cardiovascular health and prevent the serious consequences associated with undetected and untreated hypertension.

We are committed to providing you and your family with monitoring devices that provide the utmost in accuracy and convenience.

THIS MONITOR IS CLINICALLY VALIDATED

This ReliOn® blood pressure monitor has undergone and passed a Clinical Validation study to determine the performance and accuracy. This monitor is clinically validated according to the European Society of Hypertension (ESH) standard. The ESH is one of the most prestigious international blood pressure organizations. The published study was performed by a reviewing committee consisting of physicians and/or nurses.

IMPORTANT INFORMATION

Please be sure to read this instruction manual thoroughly to fully understand your monitor's operation, cautions, performance and limitations. After fully acquainting yourself with the manual, please keep it with your monitor.

- ☞ Please remember that only a medical practitioner is qualified to interpret your blood pressure measurements. Use of this device should not replace regular medical examinations.
- ☞ Consult your physician if you have any doubt about your readings. Should a mechanical problem occur, contact ReliOn® at 1-800-726-9855.
- ☞ Do not attempt to service, calibrate, or repair this monitor.
- ☞ Because your UA-787REL monitor contains delicate, high-precision parts, avoid exposing it to extremes in temperature or humidity or to direct sunlight, shock and dust. ReliOn® guarantees the accuracy of this monitor only when it is stored and used within the temperature and humidity ranges noted on page E-18.
- ☞ Clean the monitor and cuff with a dry, soft cloth or a cloth dampened with water and a mild detergent. Never use alcohol, benzene, thinner or other harsh chemicals to clean monitor or cuff.
- ☞ Remove and replace batteries if monitor is not used for more than six months. Alkaline batteries recommended.

PRECAUTIONS

The UA-787REL is designed to be used at home, by those who are eighteen (18) years and older, to monitor blood pressure (systolic and diastolic) and pulse rate.

IMPORTANT SAFETY INSTRUCTIONS

DANGER - TO REDUCE THE RISK OF FIRE OR ELECTRIC SHOCK, CAREFULLY FOLLOW THESE INSTRUCTIONS.

- ☞ For connection to a supply not in the U.S.A., use an attachment plug adapter of the proper configuration for the power outlet.
- ☞ The power unit (AC Adapter) is intended to be correctly oriented in a vertical or floor mount position.

TABLE OF CONTENTS


WHAT DISPLAY SYMBOLS MEAN	E-1
MONITOR COMPONENTS	E-2
HOW UA-787REL WORKS	E-3

BEFORE YOU START.	E-3
INSTALLING THE CUFF HOLDER.	E-4
SELECTING THE CORRECT CUFF SIZE	E-4

TAKING YOUR BLOOD PRESSURE	E-5
WHAT IS AN IRREGULAR HEARTBEAT	E-8
ABOUT THE CUFF INFLATION METER	E-8
ABOUT AVERAGE READING.	E-9
ABOUT THE PRESSURE RATING INDICATOR™	E-9
ABOUT MEMORY	E-10

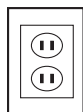USING THE MONITOR WITH AC ADAPTER	E-11
---	------

ABOUT BLOOD PRESSURE	E-12
--------------------------------	------

TROUBLESHOOTING	E-17
SPECIFICATIONS.	E-18

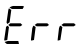
WARRANTY INFORMATION.	E-19
CONTACT INFORMATION	E-20



BLOOD PRESSURE LOG SHEETS ¹	E-21
--	------

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
WHAT DISPLAY SYMBOLS MEAN

Display Symbol	Condition/Cause	Recommended Action
	The symbol shown while measurement is in progress. It blinks while detecting the pulse.	Measurement is in progress, remain as still as possible.
	An irregular heartbeat or body movement may have occurred. Refer to page E-8 for more information on irregular heartbeats.	Take measurement again and consult with your physician.
	Previous measurement stored in MEMORY.	
 Battery Full	The battery power indicator during measurement.	Replace all batteries with new ones when the symbol blinks. Alkaline batteries recommended.
 Battery Low	The batteries are low when it blinks.	
	Unstable blood pressure due to excessive body movement.	Try the measurement again. Remain very still during the measurement.
	The systolic and diastolic values are within 10 mmHg of each other.	Fasten the cuff correctly, and try the measurement again.
	The pressure value did not increase during inflation.	Check for air leaks along the tube and around the air socket.
	The cuff is not fastened correctly.	Refasten the cuff and retake measurement.
	There is an air leak in the cuff or monitor.	Make sure tube is properly connected to cuff and monitor.
	The pulse is not detected correctly.	Try the measurement again. Remain very still during the measurement.
	Cuff Inflation Meter	Measurement is in progress, remain as still as possible.
	Pressure Rating Indicator™	Refer to the section “About Pressure Rating Indicator” for further explanation. Page E-9.

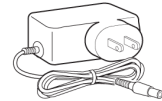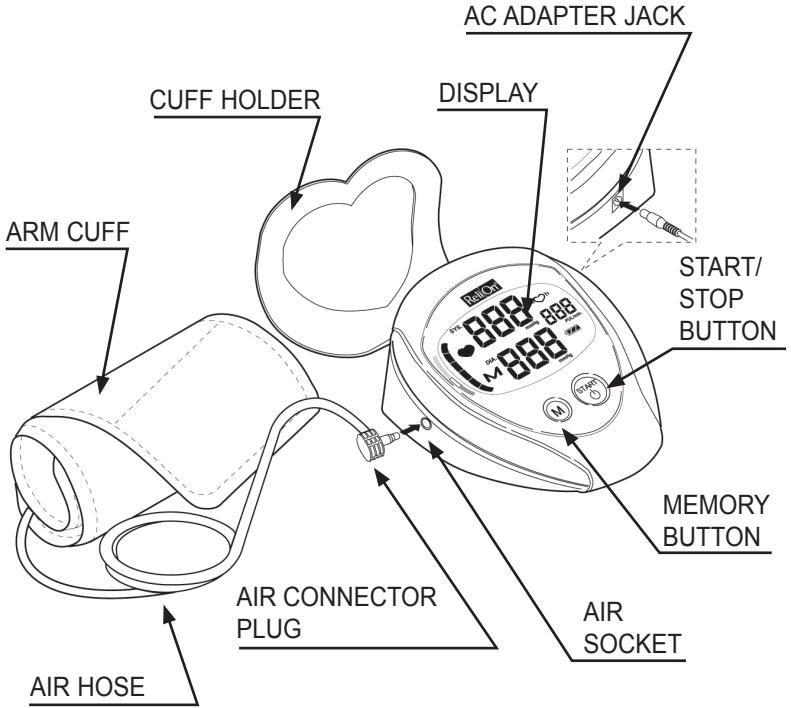MONITOR COMPONENTS

AC ADAPTER

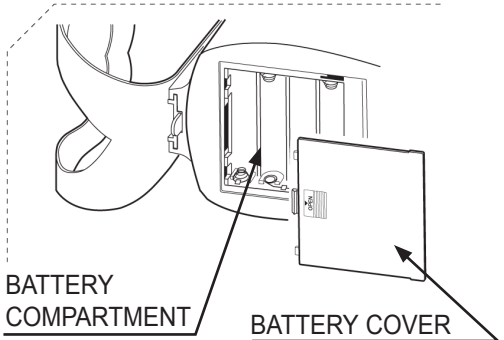HOW UA-787REL WORKS

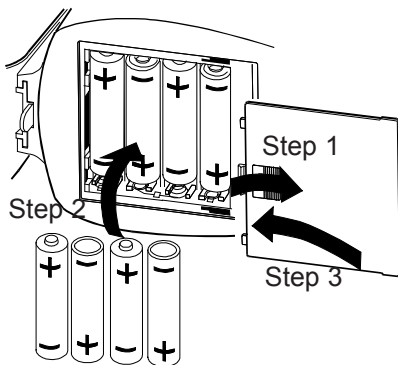
ReliOn® blood pressure monitors are easy to use, accurate and digitally display full measurement readouts. Our technology is based on the “oscillometric method” – a noninvasive blood pressure determination. The term “oscillation” refers to any measure of vibrations caused by the arterial pulse. Our monitor examines the pulsatile pressure generated by the arterial wall as it expands and contracts against the cuff with each heartbeat.

The cuff is inflated until the artery is fully blocked. The inflation speed is maximized and the pressure level is optimized by the device. The monitor takes measurements while the cuff is both inflating and deflating. This results in a faster measurement providing greater comfort to the user.

BEFORE YOU START

You must install 4 type AA (1.5 volt) batteries (alkaline batteries recommended), or use the AC Adapter (see page E-11 for using the Monitor with AC Adapter) and attach the cuff to the monitor before using it. To install batteries (or replace them if the “Low Battery” symbol appears on display), proceed as follows:

1. Remove battery compartment cover by gently pushing down on arrow and sliding cover forward.
2. Put in bottom row of batteries first. Place the batteries in compartment with positive (+) and negative (-) terminals matching those indicated in the compartment. Be sure batteries make contact with compartment terminals.
3. Replace cover by sliding it into the compartment and gently pressing into place.


NOTE: Rechargeable batteries will not work with this monitor.

INSTALLING THE CUFF HOLDER

This monitor comes equipped with a removable cuff holder for your convenience. When not in use, simply slip your Easy Wrap cuff into the holder for storage.


To install the cuff holder:

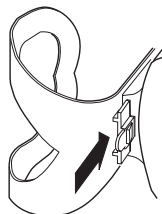1. Align the cuff holder with the rear of the blood pressure monitor.
2. Slide the cuff holder into the rear of the blood pressure monitor until it snaps securely into place.

To remove the cuff holder:

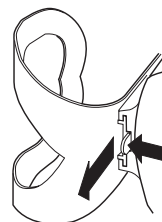1. Push the tab on the cuff holder away from the blood pressure monitor.
2. Slide the cuff holder away from the blood pressure monitor as shown to the left.

SELECTING THE CORRECT CUFF SIZE

Using the right sized cuff is essential for an accurate measurement. Our unique Easy Wrap cuff is designed to accommodate most arm sizes, from 9-17" (23-43 cm). The Easy Wrap cuff is contoured to provide you with a quick and comfortable fit. Just slide the cuff onto your upper arm and secure – it's that simple! To place the cuff on properly, please read the section Taking Your Blood Pressure (see pg. E-5).

If your arm circumference is greater than 17" (43 cm) or smaller than 9" (23 cm), you will need to purchase a difference size cuff than the one that is included in this box. Please call ReliOn® at 1-800-726-9855 for more information on our alternate cuff sizes.

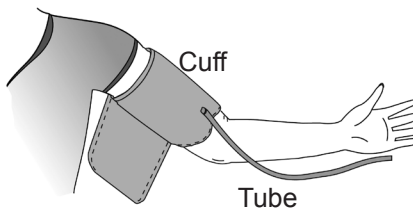
TAKING YOUR BLOOD PRESSURE

Tips for Blood Pressure Monitoring:


- ☞ Relax for about 5 minutes before measurement.
- ☞ Do not smoke or ingest caffeine at least 30 minutes prior to measurement.
- ☞ Remove constricting clothing and place cuff on bare arm.
- ☞ Unless your physician recommends otherwise, use left arm to measure pressure.
- ☞ Do not talk during measurement.
- ☞ Do not cross legs and keep your feet flat on floor during measurement.

Now you are ready. Follow these simple steps:


1. Sit comfortably with your left arm resting on a flat surface so that the center of your upper arm is at the same height as your heart.

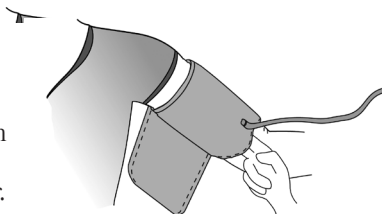2. Lay left arm on the table, and place Easy Wrap cuff on arm with tube pointing away from you. Then position the tube off-center toward the inner side of arm in line with the little finger.

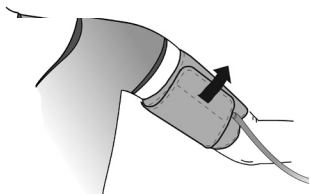3. Secure Easy Wrap cuff around arm. The cuff should be snug but not too tight. You should be able to insert two fingers between the cuff and your arm.


4. Press the **START** button. The average reading (see Pg. E-9) is displayed, the cuff pressurizes, measurement will begin and the Cuff Inflation Meter will show on the display screen. It is normal for the cuff to feel very tight.

Note: If an appropriate pressure is not obtained, the device automatically starts to inflate again.

Caution: If you wish to stop inflation at any time, press the **START/STOP** button again.

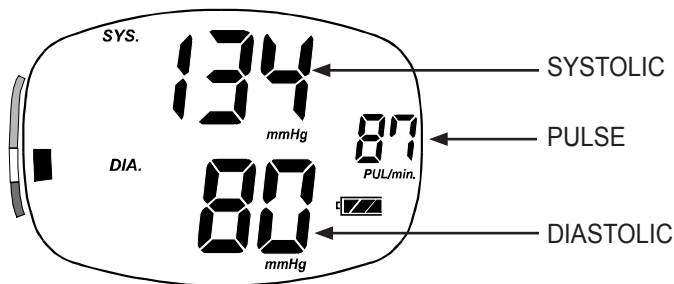5. When the measurement is complete, the systolic and diastolic pressure readings and pulse rate are displayed. The cuff deflates and the monitor automatically shuts off after 60 seconds, or you can turn it off by pressing the **START** button. Your measurement is automatically stored in memory (the monitor can store up to 90 readings in memory).

6. Remove cuff and make note of your blood pressure and pulse rate on the Blood Pressure Record chart in the back of the manual, indicating the date and time of measurement.

IMPORTANT: Measure blood pressure at the same time each day.

TAKING YOUR BLOOD PRESSURE

Measurement with the desired systolic pressure:

If your systolic pressure is expected to exceed 230 mmHg follow these steps:

1. Place the cuff on the left arm, unless your physician tells you otherwise.
2. Press and hold the START button until a number 30 to 40 mmHg higher than your expected systolic pressure appears.
3. Release the START button when the desired number is reached. 280 mmHg is the highest pressure level the monitor can reach. Measurement will then begin. Continue to measure your blood pressure as described on Page E-5.

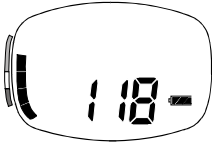
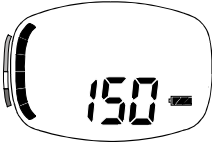
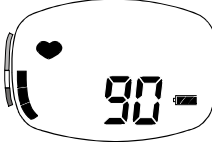
WHAT IS AN IRREGULAR HEARTBEAT

The UA-787REL Blood Pressure Monitor provides a blood pressure and pulse rate measurement even when an irregular heartbeat is detected. The Irregular Heartbeat symbol “♥” will appear in the display window in the event an Irregular Heartbeat has occurred during measurement. An irregular heartbeat is defined as a heartbeat that varies by 25% from the average of all heartbeat intervals during the blood pressure measurement. It is important that you relax, remain still and refrain from talking during measurements.

NOTE: We recommend contacting your physician if you see this “♥” symbol frequently.


ABOUT THE CUFF INFLATION METER

The Cuff Inflation Meter is located on the left side of the display screen to tell you when the blood pressure monitor is inflating and deflating the cuff. The Cuff Inflation Meter moves up during inflation and moves down during deflation.

Inflation in Progress	Inflation Complete	Deflation/ Measurement in Progress
 The display shows a cuff inflation meter on the left with a vertical bar rising to approximately 118. The digital display shows "118-".	 The display shows a cuff inflation meter on the left with a vertical bar rising to approximately 150. The digital display shows "150-".	 The display shows a cuff inflation meter on the left with a vertical bar falling from approximately 150 to 90. A heart symbol "♥" is positioned above the meter. The digital display shows "90-".


ABOUT AVERAGE READING

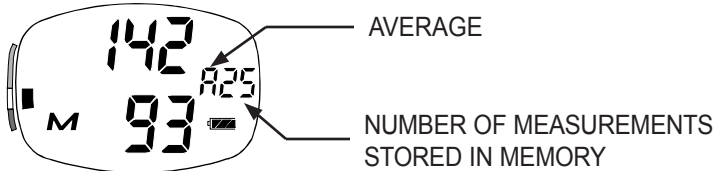The UA-787REL provides an average blood pressure reading based on the total measurements stored in memory. The average blood pressure reading appears briefly when you push the start button.

ABOUT THE PRESSURE RATING INDICATOR™

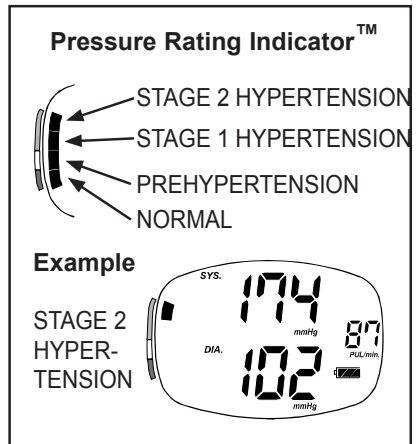The Pressure Rating Indicator™ is a feature which provides a snapshot of your blood pressure classification based on your measurements. This will let you quickly know what your blood pressure readings mean. Each segment of the bar indicator corresponds to the Seventh Report of the Joint National Committee (JNC7) on Prevention, Evaluation and Treatment of High Blood Pressure from the National Heart, Lung and Blood Institute - May 2003. For a more detailed look at this blood pressure classification, please refer to “Assessing High Blood Pressure” on page E-13.


The indicator displays a segment, based on the current measurement, corresponding to the JNC7 Classification.

NOTE: Due to other risk factors (e.g. diabetes, obesity, smoking, etc.) in addition to your blood pressure measurement, the Pressure Rating Indicator is approximate. Please consult with your physician for interpretation and diagnosis of your blood pressure measurements.

ABOUT MEMORY

NOTE: This device stores the last 90 measurements in memory. The data is retained as long as the batteries are in the device. When you remove the batteries or the batteries are drained, the data will be erased.

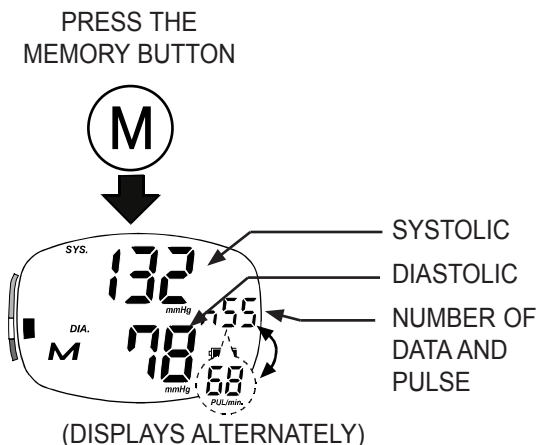1. Press the MEMORY button.
2. The newest data are displayed. Each time the MEMORY button is pressed, the memorized data is displayed. After the last reading is displayed, the display is turned off.

Clearing the memory data

Press and hold the MEMORY button until the “M”(memory) mark blinks to clear the stored data.

USING THE MONITOR WITH AC ADAPTER

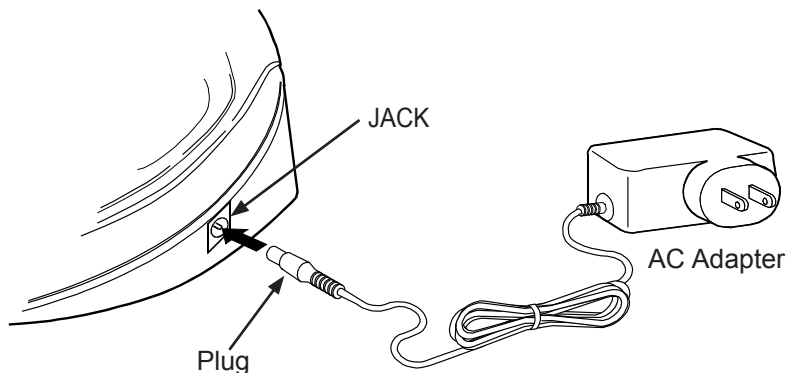
The UA-787REL has an AC adapter jack to allow you to supply power from an outlet in your home. We recommend you use only the supplied AC adapter to avoid potential damage to the monitor.

Connecting the AC adapter to the monitor:

1. Gently insert the AC adapter plug into a 120V AC outlet.
2. Connect the AC adapter plug into the jack on the right side of the monitor.

Disconnecting the AC adapter from the monitor:

1. Turn the unit off by pressing the START button.
2. Disconnect the plug from the jack of the monitor quickly.
3. Gently remove the AC adapter from the outlet.

ABOUT BLOOD PRESSURE

■ What Is Blood Pressure?

Blood pressure is the force exerted by blood against the walls of the arteries. Systolic pressure occurs when the heart contracts; diastolic pressure occurs when the heart expands. Blood pressure is measured in millimeters of mercury (mmHg).

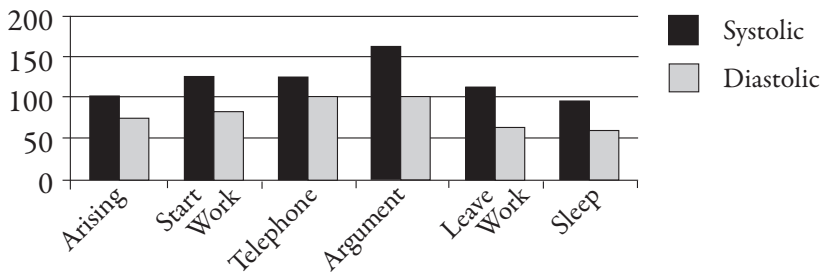
■ What Affects Blood Pressure?

Blood pressure is affected by many factors: age, weight, time of day, activity level, climate, altitude and season. Certain activities can significantly alter one's blood pressure. Walking can raise systolic pressure by 12 mmHg and diastolic pressure by 5.5 mmHg. Sleeping can decrease systolic blood pressure by as much as 10 mmHg. Taking your blood pressure repeatedly without waiting an interval of 2 - 3 minutes between readings, or without raising your arm to allow blood to flow back to the heart, can also affect it. In addition to these factors, beverages containing caffeine or alcohol, certain medications, emotional stress and even tight-fitting clothes can make a difference in the readings.

■ What Causes Variations In Blood Pressure?

An individual's blood pressure varies greatly from day to day and season to season. For hypersensitive individuals, these variations are even more pronounced. Normally, blood pressure rises during work or play and falls to its lowest levels during sleep.

FLUCTUATION WITHIN A DAY

ABOUT BLOOD PRESSURE

■ Assessing High Blood Pressure for Adults

The following standards for assessing high blood pressure (without regard to age or gender) have been established as a guideline. Please note that other risk factors (e.g. diabetes, obesity, smoking, etc.) need to be taken into consideration and may affect these figures. Consult with your physician for an accurate assessment.

JNC7 Classification Table – for adults within the U.S.

BP Classification	Systolic (mmHg)		Diastolic (mmHg)
Normal	< 120	and	< 80
Prehypertension	120-139	or	80-89
Stage 1 Hypertension	140-159	or	90-99
Stage 2 Hypertension	≥ 160	or	≥ 100

SOURCE: The Seventh Report of the Joint National Committee on Prevention, Evaluation and Treatment of High Blood Pressure for Adults. National Heart, Lung and Blood Institute - May 2003.

■ What Is Hypertension?

Hypertension (high blood pressure) is the diagnosis given when readings consistently rise above normal. It is well known that hypertension can lead to stroke, heart attack or other illness if left untreated. Referred to as a “silent killer” because it does not always produce symptoms that alert you to the problem, hypertension is treatable when diagnosed early.

ABOUT BLOOD PRESSURE

■ Can Hypertension Be Controlled?

In many individuals, hypertension can be controlled by altering lifestyle and minimizing stress, and by appropriate medication prescribed and monitored by your doctor. The American Heart Association recommends the following lifestyle suggestions to prevent or control hypertension:

- Don't smoke.
- Have regular physical checkups.
- Reduce salt and fat intake.
- Maintain proper weight.
- Monitor your blood pressure at periodic intervals.
- Exercise routinely.

■ Why Measure Blood Pressure at Home?

It is now well known that, for many individuals, blood pressure readings taken in a doctor's office or hospital setting might be elevated as a result of apprehension and anxiety. This response is commonly called "white coat hypertension." In any case, self-measurement at home supplements your doctor's readings and provides a more accurate, complete blood pressure history. In addition, clinical studies have shown that the detection and treatment of hypertension is improved when patients both consult their physicians and monitor their own blood pressure at home.

■ **Answers to Why Your Readings Are Different Between Home and at the Doctor's Office**

Your blood pressure readings taken in a doctor's office or hospital setting may be elevated as a result of apprehension and anxiety. This response is known as white coat hypertension.

When I bring my monitor to a doctor's office, why do I get a different measurement from my monitor to that taken by a doctor or nurse?

The healthcare professional may be using a different sized cuff. The size of the bladder inside the cuff is critical for the accuracy of the measurement. This may give you a different reading. A cuff too large will produce a reading that is lower than the correct blood pressure; a cuff that is too small will produce a reading that is higher than the correct blood pressure. There may also be other factors that can cause the difference in measurements.

■ **Keys to Successful Monitoring:**

Blood pressure fluctuates throughout the day. We recommend that you are consistent in your daily measurement routine:

- Measure at the same time each day.
- Sit in the same chair/position.
- Do not cross legs and keep your feet flat on the floor.
- Relax for 5 minutes before measurement.
- Use the correct cuff size to get an accurate reading.
- Sit still during measurement – no talking, eating or sudden movements.
- Record your measurement in the logbook.

ABOUT BLOOD PRESSURE

■ Establishing Baseline Measurements

The most important method to get an accurate blood pressure measurement is consistency. To get the most benefit out of your monitor, it is important to establish a “baseline measurement.” This helps build a foundation of measurements that you can use to compare against future readings. To build this baseline measurement, devote two weeks for consistent blood pressure monitoring. This involves doing everything the same way when you measure (e.g. measuring during the same time of day, in the same location, sitting in the same chair, using the same cuff, etc.). Once you establish your baseline measurement, you can start evaluating if your measurement has been affected based on things lifestyle changes or medication treatment.

■ How Do I Record My Blood Pressure?

Blood pressure readings are typically recorded with the systolic pressure written first, followed by a slash mark and the diastolic pressure. For example, 120 mmHg systolic and 80 mmHg diastolic measurements are written as 120/80. Pulse is simply written with the letter “P” followed by the pulse rate—P 72, for example. Please see the back of the manual for the blood pressure tracking log.

■ Customer Support Tools Online

Additional tools are available on www.LifeSourceOnline.com to help you get the most out of your blood pressure monitoring. These include:

- Animated Operating Instructions
- Additional Logbook Sheets

TROUBLESHOOTING

Problem	Probable cause	Corrective action
Nothing appears in the display, after I press the START button.	Batteries are drained.	Replace all batteries with new ones. Alkaline batteries recommended.
	Battery terminals are not in the correct position.	Reinstall the batteries with negative and positive terminals matching those indicated in the battery compartment.
The cuff does not inflate.	Battery voltage is too low. Low battery symbol blinks. [If the batteries are drained completely, the mark does not appear.]	Replace all batteries with new ones. Alkaline batteries recommended.
The unit does not measure. Readings are too high or too low.	The cuff is not fastened properly.	Fasten the cuff correctly.
	You moved your arm or body during the measurement.	Make sure you remain very still and quiet during the measurement.
	The cuff position is not correct.	Sit comfortably and still. Make sure the cuff is at the same level as your heart.
	You are using the wrong size cuff.	See Pg. E-4 "Selecting the Correct Cuff."
The value is different from that measured at a clinic or doctor's office.	The healthcare professional may be using a different sized cuff.	See Pg. E-15 "Answers to Why Your Readings Are Different Between Home and the Doctor's Office."
	Your measurements may be elevated by white coat hypertension	See Pg. E-14 "Why Measure Blood Pressure at Home."

NOTE: If the actions described above do not solve the problem, call 1-800-726-9855. Do not attempt to repair the device yourself.

SPECIFICATIONS

MODEL	UA-787REL
TYPE	OSCILLOMETRIC
DISPLAY	DIGITAL CHARACTER HEIGHT PRESSURE (17.3MM) PULSE (8.6MM) DISPLAYED SIMULTANEOUSLY
MEMORY	90 READINGS
MEASUREMENT RANGE	PRESSURE: 20 MMHG TO 280 MMHG PULSE: 40 PULSES TO 180 PULSES
ACCURACY	PRESSURE: +3 MMHG OR +2%, WHICHEVER IS GREATER, PULSE: +5%
PRESSURIZATION	AUTOMATIC, USING MICROPUMP
DEPRESSURIZATION	ACTIVE EXHAUST VALVE
POWER SOURCE	4 TYPE "AA" (1.5 VOLT) ALKALINE BATTERIES (NOT INCLUDED) OR 120 V AC ADAPTER
BATTERY LIFE	APPROXIMATELY 4 MONTHS WITH ONE DAILY MEASUREMENT
OPERATING ENVIRONMENT	50°F TO 104°F (10°C TO 40°C) LESS THAN 85% RELATIVE HUMIDITY
STORAGE ENVIRONMENT	14°F TO 140°F (-10°C TO 60°C) LESS THAN 95% RELATIVE HUMIDITY
DIMENSIONS	LENGTH: 5.8" (146.5 MM) WIDTH: 4.6" (116 MM) HEIGHT: 3.5" (88.5 MM)
WEIGHT	11.3 OZ. (320 G) WITHOUT BATTERIES

Blood pressure measurements determined by the UA-787REL are equivalent to those obtained by a trained observer using the cuff/stethoscope auscultation method within the limits prescribed by the American National Standards Institute for electronic or automated sphygmomanometers.

WARRANTY INFORMATION

UA-787REL DIGITAL BLOOD PRESSURE MONITOR LIFETIME WARRANTY

A&D Medical (the “Company”) warrants to the first retail purchaser that this Digital Blood Pressure Monitor (the “Product”) will be free from defects in material and workmanship for the life of the original owner. To obtain warranty service for such defects, return the Product, freight and insurance prepaid, together with satisfactory proof of date of purchase such as, a sales receipt or warranty card and shipping costs to:

A&D Medical
Attn: LW
1756 Automation Parkway
San Jose, CA 95131
1-800-726-9855

Include a check for return shipping and insurance. Contact A&D Medical at 1-800-726-9855 for appropriate shipping costs.

At A&D Medical’s option, the Company will repair or replace any Product found to be defective and return it to you. (Any replacement product may be new or refurbished.) This warranty will not apply if the defect or malfunction results from: subjecting the Product to any but the specified voltage; modification, alteration or repairs of the Product by persons not authorized by A&D Medical; misuse, abuse, damage during transit or lack of reasonable care with respect to the Product; or failure to follow the written instructions enclosed with the Product. **THE WARRANTY AND REMEDY PROVIDED ABOVE ARE EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES. IN NO EVENT SHALL A&D MEDICAL BE LIABLE FOR ANY LOSS, INCIDENTAL, OR CONSEQUENTIAL DAMAGES, RESULTING FROM THE USE OR INABILITY TO USE THE PRODUCT, WHETHER RESULTING FROM BREACH OF WARRANTY OR ANY OTHER LEGAL THEORY, EVEN IF A&D MEDICAL HAS BEEN ADVISED OF THE POSSIBILITY OF SUCH INCONVENIENCE, DAMAGE OR LOSS.**

Some states do not allow the exclusion of incidental or consequential damage, so that the above exclusions may not apply to you. This warranty gives you specific legal rights, and you may also have other rights that may vary from state to state.

CONTACT INFORMATION

This ReliOn® blood pressure product is covered by a Lifetime Warranty. See warranty card for details.

For more information regarding use, care or servicing of your blood pressure monitor, contact:

**ReliOn® Customer Service
Toll-Free: 1-800-726-9855**

Visit our web site at www.LifeSourceOnline.com for warranty information and registration.

IMPORTANT!

If You Need Assistance with the Set-Up or Operation

We Can Help!

Please call us FIRST before contacting your retailer at

ReliOn® Customer Service Line

1-800-726-9855 - Toll Free

A specially trained representative will assist you

Distributed by:

A&D Engineering, Inc.
1756 Automation Parkway
San Jose, CA 95131
1-888-726-9855
www.LifeSourceOnline.com

Pressure Rating Indicator is a trademark of A&D Medical and is patent pending.

©2007 A&D Medical. All rights reserved.

ReliOn® is a registered trademark used under license by A&D Medical.

Specifications subject to change without notice.

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