DOC NO	268106
REV	1
PAGE	1 of 195

SPECIFICATION - PRINTED MATERIALS

Rev	Change Note	Date	Document Prepared by	Document Checked by (Name)
1	C16389	24 Nov 06	Roberto Fermin	Andrew Cheung or delegate (refer to Change Note)

VPAP Malibu User Guide USA Multilingual

1. ORACLE DESCRIPTION

VPAP Malibu User Gd USA ML

2. DETAILS

Cover

Dimensions: Trimmed to A5 Portrait (210 H x 148 W ± 2 mm)
Style: Covers printed one side only. Inside cover is blank.
Stock: 250gsm Chorus Silk
Colour of printing: 4 colour process

TEXT

Dimensions: A5 Portrait (210 H x 148 W)mm ± 2mm
Style: Printed double-sided, bound and trimmed to make A5 portrait sized booklet. 80 gsm offset.
Colour of printing: Black.

Art Work: As shown on following pages. If re-typeset, the same styles and sizes must be maintained.

Manufacturer: Small quantities may be printed in-house and large quantities externally.

3. RESMED QC INSPECTION

Refer to AU10006 - Printed Materials/Labels/CDs/Videos Section 19.1

RESMED

VPAP Malibu™
User Guide

English • Français • Español • Português

268106/1 <small>06 12</small>
VPAP Malibu
User
AMER ML

RESMED

VPAP Malibu™

User Guide

1

English

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45

Français

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93

Español

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141

Português

ResMed Ltd (Manufacturer) 1 Elizabeth Macarthur Drive Bella Vista NSW 2153 Australia

ResMed Corp (US Designated Agent) 14040 Danielson Street Poway CA 92064-6857 USA

ResMed (UK) Ltd (EU Authorized Representative) 65 Milton Park Abingdon Oxfordshire OX14 4RX UK

ResMed Offices Australia, Austria, Brazil, Finland, France, Germany, Hong Kong, Japan, Malaysia, Netherlands, New Zealand, Norway, Singapore, Spain, Sweden, Switzerland, UK, USA (see www.resmed.com for contact details).

Protected by patents: AU 691200, AU 697652, AU 699726, AU 702820, AU 709279, AU 713679, AU 724589, AU 764761, EP 0651971, EP 0661071, EP 0858352, EP 0920845, EP 0927538, EP 0934723, JP 3638613, US 4944310, US 5199424, US 5245995, US 5522382, US 5704345, US 6029665, US 6138675, US 6213119, US 6363933, US 6398739, US 6591834, US 6635021, US 6675797, US 6705315, US 6745768, US 6770037, US 6935337, US 7004908. Other patents pending.

Protected by design registrations: AU 147283, AU 147336, CH 128709, CH 128710, DE 40202007.3, DE 40202020.0, ES 153514, ES 153515, FR 021396, FR 021397, GB 3001791, GB 3001819, JP 1164087, JP 1164265, SE 75599, SE 75600, US D467335, US D468011. Others pending.

Humidaire, Meridian, Mirage, Mirage Swift, Mirage Vista, SmartStart, ResLink, Ultra Mirage, VPAP and VPAP Malibu are trademarks of ResMed Ltd. Humidaire, Mirage, SmartStart, Vista and VPAP are registered in U.S. Patent and Trademark Office. SmartMedia is a registered trademark of Toshiba.

© 2006 ResMed Ltd.

RESMED

**VPAP Malibu™
User Guide**

English

English

Contents

Introduction	3
Definitions	3
User/Owner Responsibility	3
Medical Information	5
What the VPAP™ Malibu Is Intended For	5
Tell Your Doctor If...	5
Warnings	5
Cautions	7
The VPAP Malibu System	9
Mask	9
Supplemental Oxygen	11
Preparing for Use	13
Setting Up the VPAP Malibu	13
Operating Instructions	21
Starting Treatment	21
Stopping Treatment	23
Using the Mask-Fitting Feature	24
Using DC Power to Run the VPAP Malibu	25
Helpful Hints	25
Cleaning and Maintenance	27
Daily	27
Weekly	27
Periodically	27
Replacing the Air Filter	28
Replacing the Antibacterial Filter	28
Servicing	28
Troubleshooting	31
Technical Specifications	35
Limited Warranty	41
Index	43

Introduction

Definitions

This guide contains special terms and icons that appear in the margins to draw your attention to specific and important information.

 **WARNING**
Alerts you to possible injury.

 **CAUTION**
Explains special measures for the safe and effective use of the device.

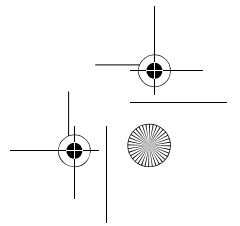
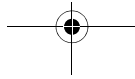
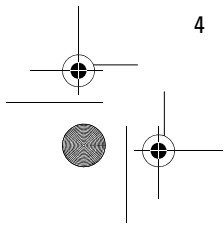
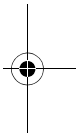
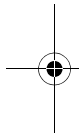
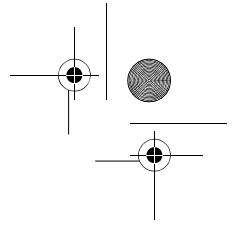
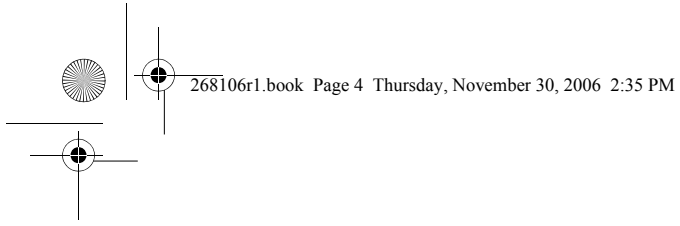
Note: Is an informative or helpful note.

User/Owner Responsibility

The user or owner of this system shall have sole responsibility and liability for any injury to persons or damage to property resulting from:

- operation which is not in accordance with the operating instructions supplied
- maintenance or modifications carried out unless in accordance with authorized instructions and by authorized persons.

Please read this guide carefully before use.

Medical Information

What the VPAP Malibu™ Is Intended For

The VPAP Malibu is indicated for the treatment of adult patients with obstructive sleep apnea (OSA). It is intended for use in the hospital and home.

CAUTION (USA ONLY)

Federal law restricts this device to sale by or on the order of a physician.

Tell Your Doctor If...

You must tell your doctor if you have any of the following conditions, and CPAP (continuous positive airway pressure) or bilevel therapy must not be used:

- pneumothorax or pneumomediastinum
- severe heart failure, low blood pressure, or dehydration
- surgery to the brain, middle or inner ear, pituitary gland, or sinuses
- respiratory distress syndrome
- middle ear infection or perforated ear drum
- severe nosebleed.

Warnings

CPAP or bilevel therapy should be used with caution if you have any of the following conditions:

- respiratory failure
- cavities or cysts in the lung, or previous history of pneumothorax
- previous history of severe nosebleed
- sinus infection.

Tell your doctor if you have any of these conditions. Your doctor will advise you whether the likely benefits of CPAP or bilevel therapy outweigh the expected risks.

Special care should be exercised if you are dehydrated, or may become dehydrated, for example as a result of fluid restriction or diuretic therapy (including changes in therapy).

Discontinue therapy and seek medical advice if, during therapy or when you start therapy each night, you feel faint or light-headed.

The following are general warnings that pertain to your use of a VPAP Malibu unit. Specific warnings appear next to the relevant instructions in this guide.

WARNING

- **This is NOT a life support device.** It may stop operating with power failure or if a fault occurs in the unit.
- The air flow for breathing produced by this device can be as much as 11°F (6°C) higher than the temperature of the room. Caution should be exercised if the room temperature is warmer than 90°F (32°C).
- The VPAP Malibu unit should only be used with masks (and connectors) recommended by ResMed, or by your physician or respiratory therapist. A mask should not be used unless the VPAP Malibu unit is turned on and operating properly. The vent hole or holes associated with the mask should never be blocked.

Explanation: The VPAP Malibu unit is intended to be used with special masks (or connectors) which have vent holes to allow continuous flow of air out of the mask. When the device is turned on and functioning properly, new air from the device flushes the exhaled air out through the mask vent holes. However, when the device is not operating, insufficient fresh air will be provided through the mask, and the exhaled air may be rebreathed. Rebreathing of exhaled air for longer than several minutes can, in some circumstances, lead to suffocation. This applies to most models of CPAP or bilevel devices.

- The VPAP Malibu unit should only be connected to the components, humidifiers, or accessories specified in this guide. Connection of other items may result in injury, or damage to the VPAP Malibu unit.
- At low pressures, the flow through the exhalation ports of your mask may not clear all exhaled gas from the tubing. Some rebreathing may occur.
- Explosion hazard—do not use in the vicinity of flammable anesthetics.
- If oxygen is used with this device, the oxygen flow must be turned off when the device is not operating.

Explanation of the warning: When the device is not in operation and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the device enclosure. Oxygen accumulated in the device enclosure will create a risk of fire. This warning applies to most types of flow generators.

- Oxygen supports combustion. Oxygen should not be used while smoking or in the presence of an open flame.
- Always ensure air flow is being generated by the device before the oxygen supply is turned on.
- Always turn the oxygen supply off before stopping the air flow from the device.

Note: At a fixed rate of supplemental oxygen flow, the inhaled oxygen concentration will vary, depending on where the oxygen is introduced, pressure settings, patient breathing pattern, mask selection, and leak rate. This applies to most types of CPAP machines.

Warnings Related to Treatment

WARNING

- If you stop your CPAP or bilevel treatment, your sleep apnea will return immediately.
- Always consult your clinician if you expect to be in a situation where you cannot use your VPAP Malibu unit.
- If you are admitted to a hospital or prescribed any other form of medical treatment, always inform the medical staff that you are being treated with CPAP or bilevel. It is also important to contact the clinician who is treating you for sleep apnea.
- If you experience an infection of the upper respiratory tract, middle ear, or sinuses, contact your clinician before continuing your CPAP or bilevel treatment. You may be advised to stop treatment until the infection has cleared. If you continue with treatment during an infection, be sure to clean your mask and tubing after every use.

Cautions

The following are general cautions. Specific cautions appear next to the relevant instructions in this guide.

CAUTION

- Do not open the VPAP Malibu case. There are no user serviceable parts inside. Repairs and internal servicing should only be performed by an authorized service agent.

Possible Side Effects

The VPAP Malibu flow generator is designed to help you have a good night's sleep. However, you need to be aware of possible problems that may arise during CPAP or bilevel treatment.

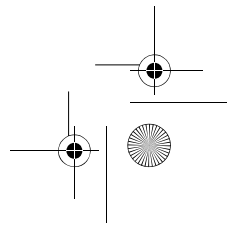
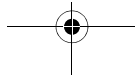
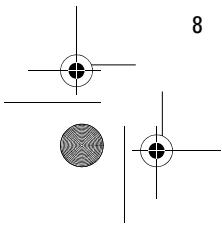
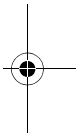
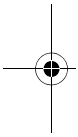
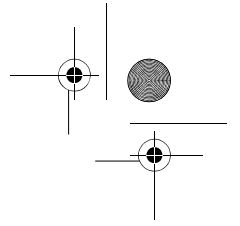
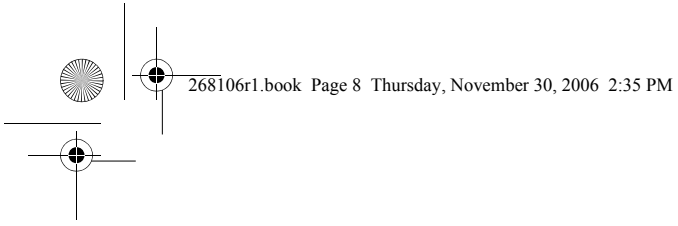
WARNING

Consult your clinician immediately if you experience any of the following symptoms during your CPAP or bilevel treatment:

- headache
- middle ear or sinus discomfort
- dryness of the nose, mouth, or throat
- feeling bloated due to air swallowing
- air continually leaking out of the mouth while sleeping
- recurrence of any sleep apnea symptoms while on CPAP or bilevel.


Skin irritation may occur from sensitivity to the mask materials or from excessively tight headgear straps. A correctly fitted mask and appropriately adjusted straps will often prevent skin irritation. If problems persist, contact your clinician for advice.

The VPAP Malibu System

Please identify and familiarize yourself with the following components of the VPAP Malibu system:

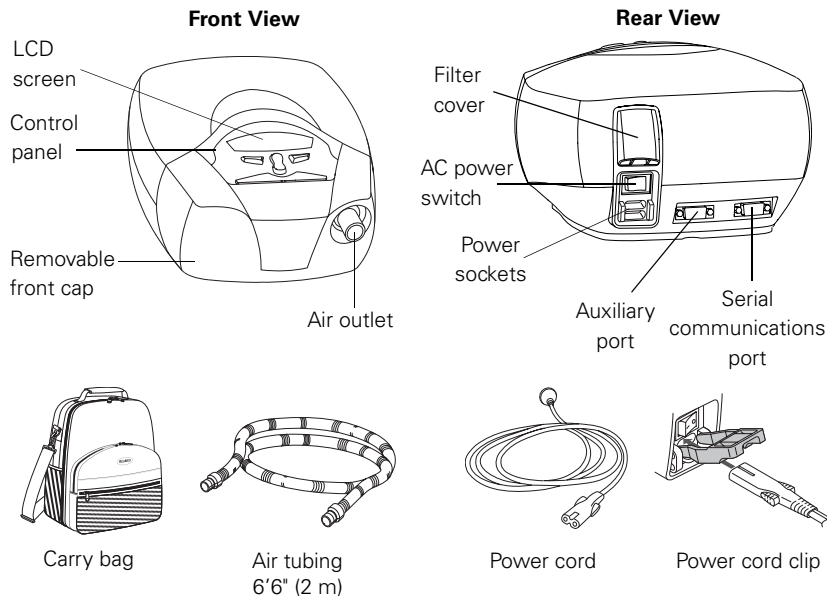VPAP Malibu Unit

Mask

You will also need a **ResMed mask system** (supplied separately). The following ResMed mask systems are recommended for use with the VPAP Malibu:

Nasal Masks

- Meridian™ Nasal Mask
- Mirage Vista™ Nasal Mask
- Ultra Mirage™ Nasal Mask
- Ultra Mirage II Nasal Mask.

Nasal Pillow System

- Mirage Swift™ Nasal Pillows System.

Full Face Masks

- Ultra Mirage Full Face Mask.

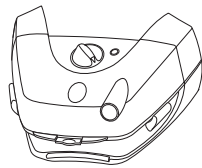
Notes:

- *ResMed devices have been designed and manufactured to provide optimum performance using ResMed vented mask systems. Other mask systems may be used, however performance and data outputs may be affected.*

- In VSet mode, only ResMed masks are compatible for use with the VPAP Malibu.
- Your clinician needs to set up your VPAP Malibu for use with your particular mask.

Humidifier

A humidifier may be required if you are experiencing dryness of the nose, throat, or mouth. The VPAP Malibu is compatible for use with the following humidifier.

HumidAire 2i™ heated humidifier

Please refer to the relevant clinician's guide for details.

WARNING

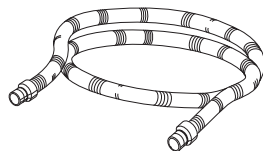
Only the HumidAire 2i is compatible for use with the VPAP Malibu. Please refer to Warnings on page 6.

Accessories


The following accessories are available for use with the VPAP Malibu:

VPAP Malibu Accessories


- Hypoallergenic air filter (not shown)

Air tubing 9'9" (3 m)

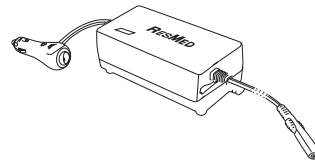DC-24/30 Converter

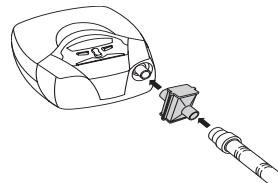Antibacterial air filter

Note: ResMed recommends the following tubing products for use with the VPAP Malibu: 3 m air tubing, gray (PN: 14980); 2 m air tubing, gray (PN: 14948); disposable air tubing, clear (PN: 14912).

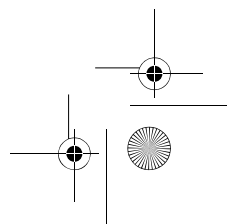
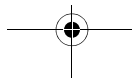
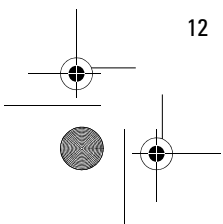
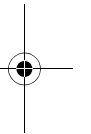
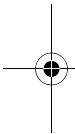
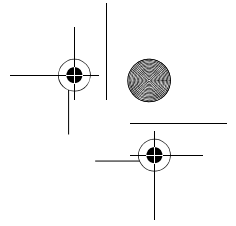
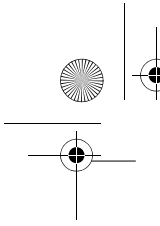
Supplemental Oxygen

The VPAP Malibu is designed to be compatible with the use of low flow oxygen therapy provided from an external source and supplied at a fixed rate of up to 15 L/min (S mode, CPAP mode) or 5 L/min (VSet mode). At a fixed flow rate of supplemental oxygen flow, the inhaled oxygen concentration will vary, depending on the pressure settings, patient breathing pattern, mask selection, and the leak rate. This warning applies to most types of CPAP machines.

WARNING

- If oxygen is used with this device, the oxygen flow must be turned off when the device is not operating.
Explanation: When the device is not in operation, and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the device enclosure and create a risk of fire.
- Always begin VPAP Malibu therapy before the oxygen supply is turned on.
- Always turn the oxygen supply off before stopping VPAP Malibu therapy.
- Oxygen supports combustion. Oxygen should not be used while smoking or in the presence of an open flame.

Preparing for Use

Setting Up the VPAP Malibu

WARNING

- Do not connect any device (other than the ResLink™ module) to the auxiliary port. Although your health care provider may connect specially designed devices to the auxiliary port of the VPAP Malibu unit, connection of other devices could result in injury to persons, or damage to the unit.
- In the clinical environment any PC that is used with the VPAP Malibu system must be at least 5' (1.5 m) away from, or at least 8' (2.5 m) above the patient. It must also comply with IEC 60950 or equivalent.

Note: The air filter cover protects the device in the event of accidental liquid spillage onto the device. Ensure that the air filter and air filter cover are fitted at all times.

- 1 Place the VPAP Malibu unit on a flat surface near the head of your bed. If the unit is placed on the floor, ensure that the area is free from dust and clear of bedding, clothes, or any other objects that could block the air inlet.

CAUTION

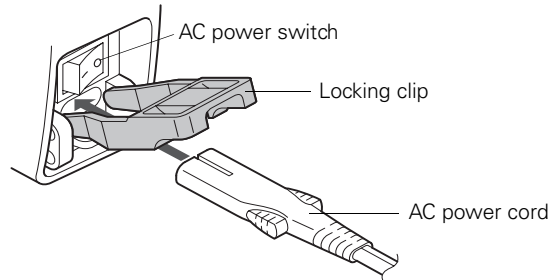
Be careful not to place the device where it can be bumped or where someone is likely to trip over the power cord.

- 2 Connect the power cord:

There are two sockets at the rear of the flow generator—the top one for an AC (standard mains electricity) and the lower one for a DC power cord (see “Using DC Power to Run the VPAP Malibu” on page 25). ResMed recommends using the AC power cord supplied with the unit.

The customized ResMed power cord can be held in place with the locking clips. To insert the locking clip, pinch the free ends together and fit the pins into the holes on both sides of the socket. Insert the power cord into the socket. Push the locking clip down so that the groove holds the power cord in place. Plug one


end of the power cord into the power socket at the rear of the unit, and the other end into a power outlet.

CAUTION
Do not connect both AC and DC power cords to the VPAP Malibu at the same time.

WARNING
Make sure the power cord and plug are in good condition and the equipment is not damaged.

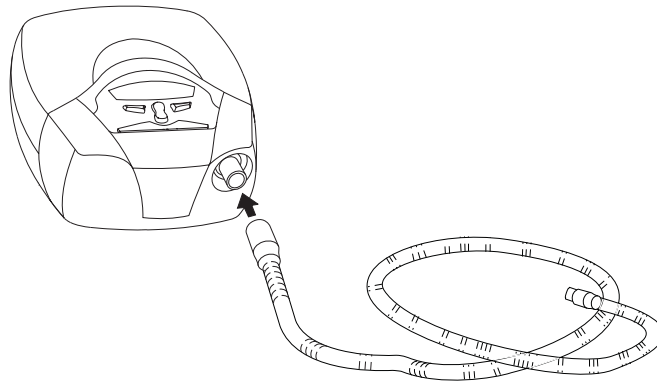
3 Connect one end of the air tubing firmly onto the air outlet of the unit.

WARNING
Only ResMed air tubing should be used with your flow generator. A different type of air tubing may alter the pressure you actually receive reducing the effectiveness of your treatment.

4 Assemble your mask system according to the mask user instructions.

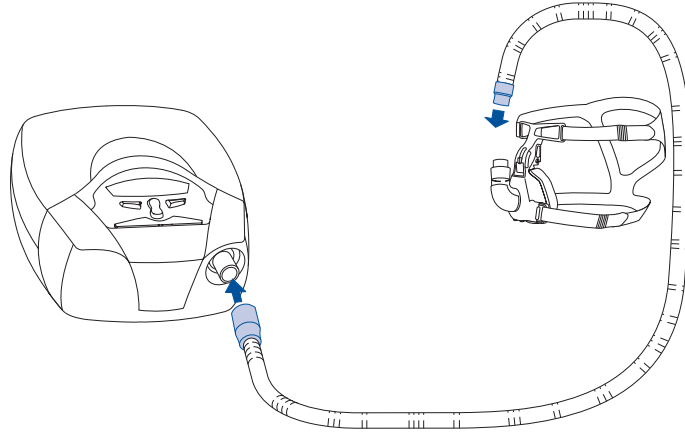- 5 Connect your mask system to the free end of the air tubing.

The VPAP Malibu is now ready for use. To start treatment, see "Operating Instructions" on page 21.

English

Humidifier Use


WARNING

The VPAP Malibu is not compatible with external humidifiers.

HumidAire 2i

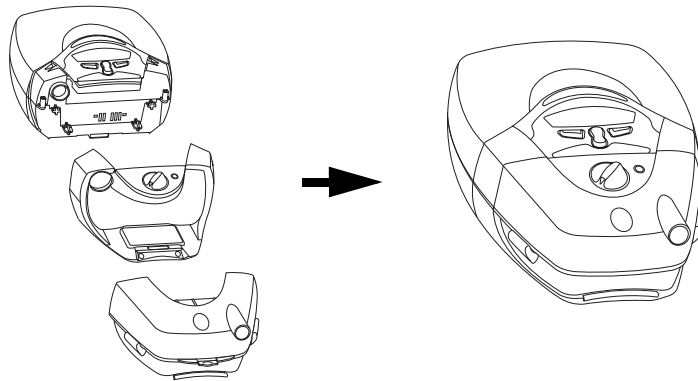The HumidAire 2i attaches to the front of the VPAP Malibu to provide heated humidification. No other accessories are required for its use. The VPAP Malibu automatically detects the presence of the HumidAire 2i. No menu changes are required. Please refer to the relevant user manual for details.

WARNING

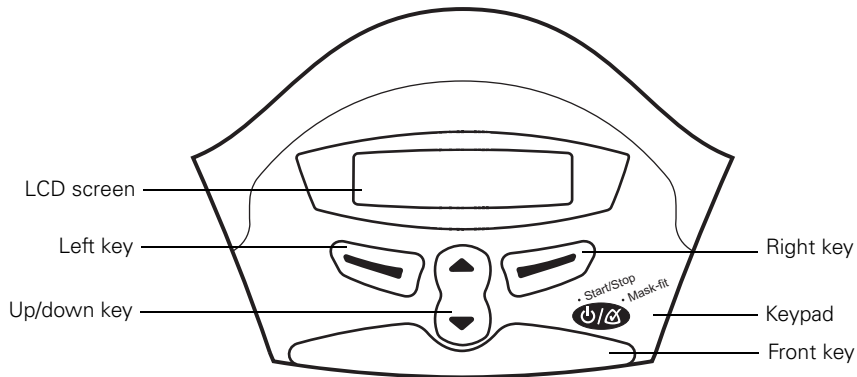
- Make sure that the power cord and plug are in good condition and the equipment is not damaged.
- Make sure that the water chamber is empty and thoroughly dried before transporting the HumidAire 2i.
- When using the carry bag, always separate the VPAP Malibu unit and the HumidAire 2i before placing in their respective compartments.

The VPAP Malibu is now ready for use. To start treatment, see "Operating Instructions" on page 21.

Features of the VPAP Malibu

LCD Screen and Keypad

The control panel of the VPAP Malibu includes an LCD screen and keypad.

The VPAP Malibu keypad has the following keys:

Key	Function
Front 	<ul style="list-style-type: none"> Starts or stops treatment. Extended hold for at least 3 seconds starts the Mask-Fitting feature.
Up/Down 	<ul style="list-style-type: none"> Allows you to scroll through the VPAP Malibu menus, submenus, and setting options.
Left 	<ul style="list-style-type: none"> Performs the function indicated by the guiding text displayed above it on the LCD screen. Guiding text includes menu, enter, change, and apply.
Right 	<ul style="list-style-type: none"> Performs the function indicated by the guiding text displayed above it on the LCD screen. Guiding text includes exit and cancel.

To assist you in adjusting the VPAP Malibu, the keypad and LCD are equipped with a backlight. The LCD backlight comes on when the unit is turned on or when you press a key, and turns off after two minutes. The keypad backlight is on at all times when the VPAP Malibu is powered.

Using the Menus

The VPAP Malibu unit provides a set of functions which are arranged in menus and submenus. Via the LCD screen, the menus and submenus allow you to view and change the settings for a particular function. You can access the menus regardless of whether the VPAP Malibu is in standby mode or delivering therapy. After the Welcome screen appears and device self-checks are complete, you will see either the Ramp screen (S mode and CPAP mode) or the Settling screen (VSet mode).

Ramp Screen

(S mode, CPAP mode) If your clinician has set a maximum ramp time, the Ramp screen is displayed after the Welcome screen. On the Ramp screen, you can immediately set a ramp time. Ramp time is the period during which the pressure increases from a low pressure to the prescribed treatment pressure. See "Ramp Time" on page 23.

Ramp time can be altered in five minute increments (from OFF to a maximum ramp time set by your clinician) by the using the **Up/Down** key.

Settling Screen

(VSet mode) If your clinician has set a maximum settling time, the Settling screen is displayed after the Welcome screen. On the Settling screen, you can immediately set a settling time. Settling time is the period for which the VPAP Malibu remains at Minimum Pressure, before increasing pressure in response to events. See "Settling Time" on page 23.

Settling time can be altered in five minute increments (from OFF to a maximum settling time set by your clinician) by the using the **Up/Down** key.

Menus

The menu allows you to view details about the time used and the current software version of your VPAP Malibu. The Used (time) screen displays the total number of hours for which you have used the device since it was last reset by the clinician. It also displays the number of days the VPAP Malibu was used out of the total number of days available for use. The SW screen displays the current software version that is installed in your VPAP Malibu.

Figure 1 summarizes the VPAP Malibu standard menu series.

- To access the VPAP Malibu menus:
Press the **Left** key (menu) while the VPAP Malibu (or Ramp or Settling) screen is displayed.
- To scroll through items within the menu:
Press the **Up/Down** key
- To exit out of the menu:
Press the **Right** key (exit)

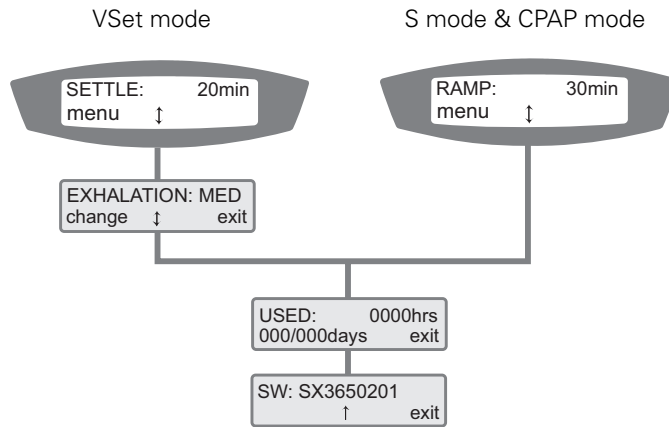
Figure 1: VPAP Malibu Menu Series

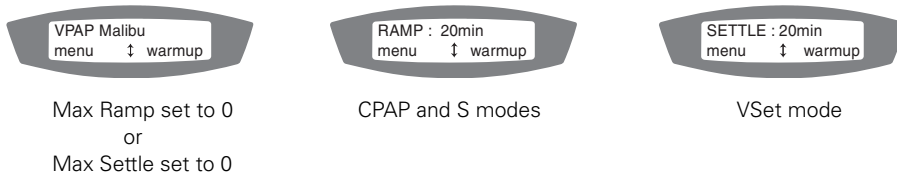
Notes:

- The Settling and Ramp screens are only available if Max Settle or Max Ramp has been set by a clinician.
- The Exhalation screen is only available if enabled by a clinician.

Menus with a Humidifier

If the HumidAire 2i is connected to the VPAP Malibu, and treatment has not commenced, the VPAP Malibu will display one of the following screens:

Prior to starting treatment:

After starting treatment:

Note: The VPAP Malibu screen displays for three seconds, then defaults to the treatment screens (see "Starting Treatment" on page 19).

Figure 2: Menu display with humidifier

Menus with a ResLink and Oximeter

If your clinician has recommended that you use a ResLink module, and the ResLink and an oximeter are connected to the VPAP Malibu, then display of SpO₂ (oxygen saturation) and HR (heart rate) takes priority. If treatment has not commenced, the VPAP Malibu will display one of the following screens, whether a humidifier is connected or not:

Prior to starting treatment:


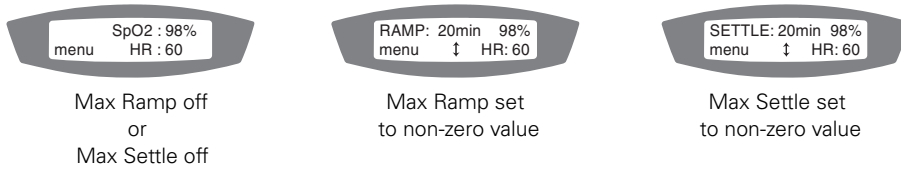
Figure 3: Menu display with ResLink and oximeter

Operating Instructions

Starting Treatment

The VPAP Malibu unit should be assembled beside your bed with the air tubing and mask system connected. See "Setting Up the VPAP Malibu" on page 13.

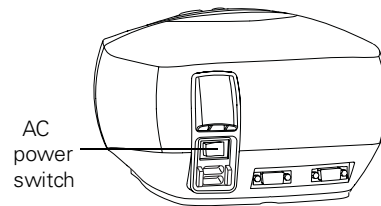- 1 Turn the main power switch at the back of the unit to on (I).

When the VPAP Malibu is turned on, a welcome message is displayed on the LCD screen. The VPAP Malibu (or Ramp or Settling) screen then appears.

Note: If you have the HumidAire 2i attached, please refer to the relevant user manual for operating instructions.

- 2 Fit your mask as described in the mask user instructions.
- 3 Lie down and arrange the air tubing so that it is free to move if you turn in your sleep.

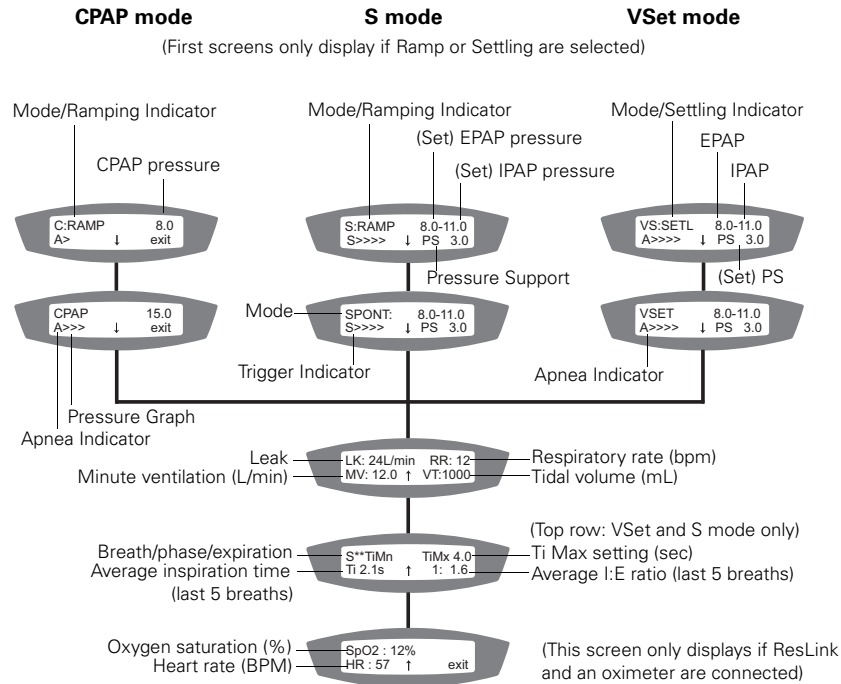
CAUTION

- Do not leave long lengths of air tubing around the top of your bed. It could twist around your head or neck while you are sleeping.
- Make sure the area around the flow generator is dry and clean. It should also be clear of bedding, clothes, and other potential blockages.

- 4 To start treatment, simply breathe into the mask (see "SmartStart™/SmartStop" on page 23).

Air will begin flowing slowly and will build up to full operating pressure in about 10 to 15 seconds.

Note: If the VPAP Malibu does not start automatically, you can also press the **Front** key.

The treatment screens contain the following information:

Treatment mode: Mode of treatment set by your clinician. Options are: CPAP, S and VSet.

Ramping indicator: (CPAP mode, S mode) Appears if the VPAP Malibu is in ramp mode. This disappears once the ramp time has elapsed.

Settling indicator: (VSet mode) Appears if the VPAP Malibu is in settling mode. This disappears once the settling time has elapsed.

Set pressure(s): In CPAP mode, this displays the set treatment pressure (centimeters of water). In S mode, it displays exhalation and inhalation pressures (centimeters of water). In VSet mode, it displays the AutoSet pressure (mean mid-point pressure between inhalation and exhalation).

Trigger indicator: (S mode) How the VPAP Malibu changes the pressure when you are inhaling. "S" (Spontaneous) indicates a patient triggered change.

Apnea indicator: (CPAP mode, VSet mode) Visual indicator that displays an "A" to indicate an apnea is occurring.

Pressure bar graph: Graphical display of the changing pressure.

Leak: Current mask leak (liters per minute).

Respiratory rate: Number of breaths per minute.

Minute ventilation: Volume of air inhaled per minute (liters per minute).

Tidal volume: Volume of air inhaled per breath (milliliters per breath).

Breath/phase/expiration: This screen item displays in the format: breath type/phase display/expiration type as shown below (eg, S**TiMn):

Breath type: S=spontaneous

Phase display: **=inspiration, (blank)=expiration

Expiration type:

TiMn=expiration triggered at completion of minimum inspiration-hold time

TiMx=expiration triggered at completion of maximum inspiration-hold time

C=expiration triggered spontaneously within TiMin and TiMax

Oxygen saturation: The saturation of blood hemoglobin (%).

Heart rate: The number of hearts beats in a 60-second time frame (beats per minute).

Ramp Time

(S mode, CPAP mode) Ramp time is a feature which can be enabled by your clinician. If you have difficulty falling asleep with full pressure, select a ramp time. The airflow will start very gently while you fall asleep. The pressure will slowly increase to full operating pressure over the selected ramp time. The clinician has set a maximum ramp time; you may select any value up to the maximum.

Settling Time

(VSet mode) Settling time is a feature which can be enabled by your clinician. If you have difficulty falling asleep with the VPAP Malibu operating, select a settling time. The airflow will remain at Minimum Pressure for the period of time that you set, giving you time to fall asleep. After this time, the VPAP Malibu will be able to increase pressure in response to events

The clinician will set a maximum settling time; you may select any value up to the maximum.

SmartStart™/SmartStop

The VPAP Malibu has a function SmartStart which starts therapy automatically when you breathe into the mask. If SmartStop is enabled, therapy will automatically stop when you take your mask off. This means you do not have to press the **Front** key to begin or end treatment.

Note: *SmartStop will not work if:*

- you have selected **FULL FACE** for your mask type.
- your clinician has enabled **Mask Alarm**.

Stopping Treatment

To stop treatment at any time, simply remove your mask and treatment will end (if your clinician has enabled SmartStop).

or

remove your mask and press the **Front** key.

Using the Mask-Fitting Feature

The VPAP Malibu Mask-Fitting feature can be used to help you fit your mask properly. The mask-fitting feature delivers air pressure for a three-minute period, prior to starting treatment, for checking and adjusting your mask fit to minimize leaks. If a Ramp time (S mode, CPAP mode) or Settling time (VSet mode) is selected, this allows you to adjust the mask at a pressure closer to the prescribed pressure. To use the mask-fitting feature:

- 1 Fit your mask as described in the user instructions.
- 2 Hold down the **Front** key for at least three seconds until air pressure delivery starts.

The following display will appear on the LCD screen indicating that the Mask-Fitting feature is in operation. The flow generator will ramp to the Mask-Fit pressure and will remain at this pressure for three minutes. A Mask-Fit star rating is also displayed. See "Definitions of Mask-Fit Star Rating" on page 25.

Notes:

- The Mask-Fit star rating display disappears after three minutes.
 - Mask-Fit is accessible when the LCD shows the VPAP Malibu screen. It is also accessible from the Ramp screen or Settling screen (these are only displayed if a maximum ramp or settling time has been set).
 - In S and CPAP modes the Mask-Fit pressure is the set treatment pressure or 10 cm H₂O, whichever is greater.
 - In VSet mode, the mask-fit pressure is 10 cm H₂O.
 - Mask Alarm and SmartStop functions are disabled while using Mask Fit.
- 3 Adjust your mask, mask cushion, and headgear until you have a secure and comfortable fit.

Once you have a secure and comfortable fit, check your Mask-Fit star rating on the LCD screen. Definitions of the Mask-Fit star ratings are presented in Table 1.

Note: If there is another person nearby to check your Mask-Fit star rating, you can adjust your mask, mask cushion, and headgear while lying down.
 - 4 After three minutes, treatment will begin.
 - If you do not wish to wait three minutes, hold down the **Front** key for at least three seconds and treatment will begin immediately.
 - If you press the **Front** key for less than three seconds, the unit will return to standby mode (the VPAP Malibu, Ramp or Settling screen is displayed).

Table 1: Definitions of Mask-Fit Star Rating

Star rating	Definition
*****	Excellent
****_	Very good
***_ _	Good
**_ _ _	Adjust mask
*_ _ _ _	Adjust mask
HIGH LEAK	Adjust mask

Using DC Power to Run the VPAP Malibu

The VPAP Malibu can be run on DC in the case of power failure. ResMed supplies separately a converter that allows a 12V or 24V battery to be connected to the DC input of the VPAP Malibu. When using DC, power is turned on and off via the switch on the connector to the power source, not the power switch at the back of the VPAP Malibu. The DC converter is a separate accessory that can be purchased through your ResMed distributor.

The humidifier heater will be automatically disabled when the VPAP Malibu is operating from the DC supply.

Power consumption of the VPAP Malibu varies with treatment settings. Adequate battery type and size is required before operation on DC, and it is recommended that you contact ResMed's Technical Services division or ResMed's website for information on your specific application.

CAUTION

- The VPAP Malibu uses 24V DC power. Only connect the VPAP Malibu to DC power using the ResMed DC-24/30 Converter specified for this application.
- Do not connect both AC and DC power cords to the VPAP Malibu at the same time.

Helpful Hints

Starting Out

Mouth Leaks

When using a nasal mask, try to keep your mouth closed during treatment. Air leaks from your mouth can decrease the effectiveness of your treatment. If mouth leaks are a problem, a full face mask or chin strap may help. Contact your clinician or equipment supplier for further details.

Mask-Fitting

The flow generator delivers the most effective treatment when the mask is well fitted and comfortable. Treatment can be affected by leaks, so it is important to eliminate any leaks that may arise.

If you have problems trying to get a comfortable mask fit, contact your sleep clinic or equipment supplier. You may benefit from a different mask size.

You can also use the Mask-Fitting feature to help you fit your mask properly. See "Using the Mask-Fitting Feature" on page 24.

Before wearing your mask, wash your face to remove excess facial oils. This will allow a better fit and prolong the life of the mask cushion.

Nasal Irritation

Dryness

You may experience dryness of the nose, mouth, and/or throat during the course of treatment, especially during winter. In many cases, a humidifier may resolve this discomfort. Contact your clinician for advice.

Runny or Blocked Nose

You may experience sneezing and/or a runny or blocked nose during the first few weeks of treatment. In many cases, nasal irritation can be resolved with a humidifier. Consult your clinician for advice.

Traveling with the VPAP Malibu

International Use

Your VPAP Malibu flow generator has an internal power adapter that enables it to operate in other countries. It will operate on power supplies of 100–240V and 50–60Hz. No special adjustment is necessary, but you will require an approved power cord for the region for which you are using the device.

Cleaning and Maintenance

You should regularly carry out the cleaning and maintenance described here.

Daily

1. Disconnect the air tubing and hang it in a clean, dry place until next use. Do not hang the air tubing in direct sunlight as it may harden and crack over time.
2. Clean the mask according to the mask user instructions.
3. If you are using a humidifier, clean it according to the instructions in the manual.

Weekly

1. Remove the air tubing from the VPAP Malibu unit and the mask.
2. Wash the mask system according to the instructions supplied with it.
3. Wash the air tubing in warm water using mild detergent. Rinse thoroughly, hang and allow to dry.
4. Before next use, assemble the mask and headgear according to the mask user instructions.
5. Reconnect the air tubing to the air outlet and mask.

CAUTION

- Do not use bleach, chlorine-, alcohol- or aromatic-based solutions (including all scented oils), moisturizing, or antibacterial soaps to clean the cushion, mask, air tubing, or the VPAP Malibu. These solutions may cause hardening and reduce the life of the product.
- Do not wash or dry the mask frame at a temperature above 176°F (80°C). Exposure to higher temperatures may reduce the life of the product.
- Do not hang the air tubing in direct sunlight as the tubing may harden over time and eventually crack.

Periodically

1. The mask and air tubing are subject to normal wear and tear. Inspect them regularly for damage.
2. Clean the exterior of the flow generator with a damp cloth and mild detergent.
3. Inspect the air filter to check if it is blocked by dirt or contains holes. See "Replacing the Air Filter" on page 28.

WARNING

Beware of electric shock. Do not immerse the flow generator or power cord in water. Always unplug the flow generator before cleaning and be sure that it is dry before reconnecting.

CAUTION

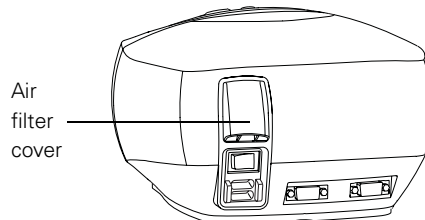
Do not attempt to open the VPAP Malibu. There are no user serviceable parts inside. Repairs and internal servicing should only be performed by an authorized service agent.

Replacing the Air Filter

Inspect the air filter every month to check if it is blocked by dirt or contains holes. With normal use of a VPAP Malibu unit, the air filter needs to be replaced every six months (or more often if your unit is in a dusty environment). To replace the air filter:

Note: For details of how to check and replace the hypoallergenic filter (if used) see the *Hypoallergenic Air Filter User Instructions*.

1. Remove the air filter cover at the back of the VPAP Malibu.

2. Remove and discard the old air filter.
3. Insert a new filter with the blue tinted side facing out.
4. Replace the air filter cover.

WARNING

Do not wash the air filter. The air filter is not washable or reusable.

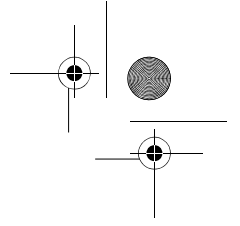
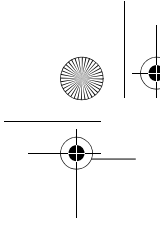
Replacing the Antibacterial Filter

You should regularly inspect and replace the antibacterial filter, as outlined in the Antibacterial Filter User Instructions.

Note: ResMed recommends that you use the Pall BB50TE filter.

Servicing

This product (VPAP Malibu) should be inspected by an authorized ResMed Service Center 5 years from the date of manufacture. Prior to this, the device is intended to provide safe and reliable operation provided that it is operated and maintained in accordance with the instructions provided by ResMed. Applicable ResMed warranty details are provided with the device at the time of original supply. Of course, as with all electrical devices, if any irregularity becomes apparent, you should exercise caution and have the device inspected by an Authorized ResMed Service Center.






If you feel that your unit is not performing properly, see "Troubleshooting" on page 31.








CAUTION

Inspection and repair should only be performed by an authorized agent. Under no circumstances should you attempt to service or repair the flow generator yourself.

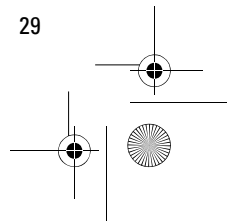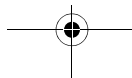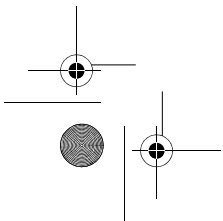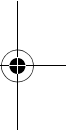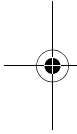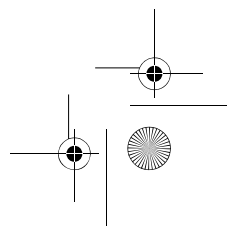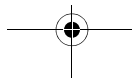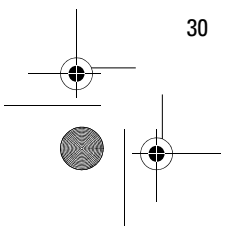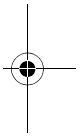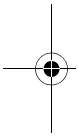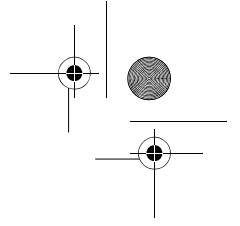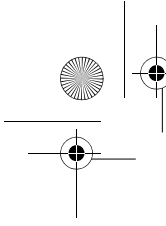English








Troubleshooting

If there is a problem, try the following suggestions. If the problem cannot be solved, contact your equipment supplier or ResMed. Do not attempt to open the unit.

Problem	Possible Cause	Solution
<i>No display.</i>	Power not connected or switch at back is not on.	Ensure the power cable is connected and that the switch at the back of the unit is in the ON position.
<i>Insufficient air delivered from the VPAP Malibu.</i>	Ramp or Settling function is in use.	Wait for air pressure to build up.
	Air filter is dirty.	Replace air filter.
	Air tubing is kinked or punctured.	Straighten or replace tubing.
	Air tubing not connected properly.	Check air tubing.
	Mask and headgear not positioned correctly.	Adjust position of mask and headgear.
	Plug(s) missing from access port(s) on mask.	Replace plug(s).
<i>The VPAP Malibu does not start when you breathe into the mask.</i>	Pressure required for treatment may have changed.	See your clinician to adjust the pressure.
	Power cord not connected properly.	Connect power cord firmly at both ends.
	Power outlet may be faulty.	Try another power outlet.
	The VPAP Malibu unit not switched on.	Switch power switch at rear of the VPAP Malibu to ON.
Breath is not deep enough to trigger SmartStart.	Take a deep breath in and out through the mask.	

Problem	Possible Cause	Solution
	There is excessive leak.	Adjust position of mask and headgear. Plugs may be missing from ports on mask. Replace them. Air tubing not connected properly. Connect firmly at both ends. Air tubing kinked or punctured. Straighten or replace.
<i>VPAP Malibu unit does not stop when you remove your mask.</i>	Mask Alarm has been enabled; SmartStop has automatically been disabled. Use of a full face mask.	Disable Mask Alarm to enable SmartStop. SmartStop does not work with a full face mask.
<i>SmartStop is enabled but the flow generator does not stop automatically when you remove your mask.</i>	Incompatible humidifier or mask system being used.	Use only equipment as recommended and supplied by ResMed.
<i>Display error message:</i> Check tube!! Key if done	The air tubing is loose. Air tubing is blocked. The device motor may not be running at the correct speed..	Check that the air tubing is connected securely to your mask and the air outlet on the front of the VPAP Malibu. Check air tubing for blockage. Return your unit for servicing. To clear the error message, press any key on the VPAP Malibu keypad.

Problem	Possible Cause	Solution
<i>Displays error message:</i> SYSTEM ERROR Call service!	Component failure.	Return your VPAP Malibu for servicing.
<i>Excessive motor noise.</i>	Component failure.	Return your unit for servicing.

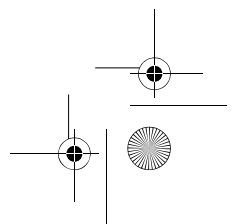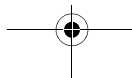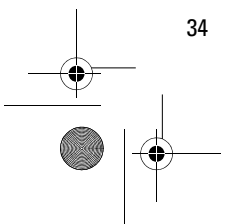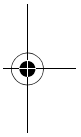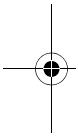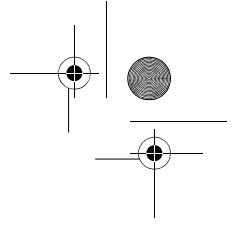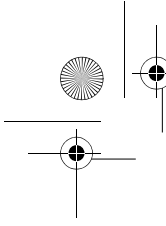Technical Specifications

Performance

Operating pressure range: 3 to 25 cm H₂O
 Maximum single fault pressure: 40 cm H₂O
 Pressure measurement tolerance: ± 0.5 cm H₂O + 4% of the measured reading
 Flow measurement tolerance: ± 0.2 L/s

Dynamic pressure characteristics

VSet mode: 4 to 25 cm H₂O (measured at the end of standard 6'6" air tubing);
 AutoSet Max pressure: 20 cm H₂O; Pressure Support: 0 to 10 cm H₂O

S mode: IPAP: 3 to 25 cm H₂O (measured at the end of standard 6'6" air tubing);
 EPAP: 3 to 25 cm H₂O (measured at the end of standard 6'6" air tubing)

CPAP mode: 4 to 20 cm H₂O (measured at the end of standard 6'6" air tubing)

Dynamic flow characteristics

130 L/min at 3 to 25 cm H₂O

Sound pressure level: <30 dB (tested in accordance with the requirements of ISO 17510-1:2002)

Data storage: 365 days of usage, leak, pressure, AHI and AI data

Dimensions (L x W x H): 10.6" x 9.1" x 5.6"

Weight: 5.1lb

Power supply: Input range 100–240V, 50–60Hz, 40VA (typical power consumption), < 100VA (maximum power consumption)

Housing construction: Flame retardant engineering thermoplastic

Environmental conditions

Operating temperature: 41°F to 104°F (+5°C to +40°C)

Operating humidity: 10–95% non-condensing

Storage and transport temperature: -4°F to 140°F (-20°C to +60°C)

Storage and transport humidity: 10–95% non-condensing

Electromagnetic Compatibility

Product complies with all applicable electromagnetic compatibility requirements (EMC) according to IEC60601-1-2, for residential, commercial, and light industry environments. For further details, see "Guidance and Manufacturer's Declaration - Electromagnetic Emissions and Immunity" on page 37.

Air filter: Two-layered, powder-bonded, polyester non-woven fiber

Air tubing: Flexible plastic, 1 x 6'6" (2 m) or 1 x 9'10" (3 m) length

Air outlet: 22mm taper, compatible with EN 1281-1:1997 Anaesthetic & Respiratory Equipment - Conical Connectors

IEC 60601-1 classification: Class II (double insulated), Type CF

Note: The manufacturer reserves the right to change these specifications without notice.

Table 2: Displayed values

Value	Range	Accuracy	Display Resolution
Pressure sensor at air outlet			
Pressure	-5 to 30 cm H ₂ O	±0.5 cm H ₂ O	0.1 cm H ₂ O
Flow sensor in flow generator*			
Leak	0–120 L/min	**	1 L/min
Tidal volume	100–3000 mL	**	1 mL
Respiratory rate	6–60 BPM	±0.5 BPM	0.1 BPM
Minute ventilation	0.6–60 L/min	**	0.25 L/min

* Results may be inaccurate in the presence of leaks.

** The displayed values are estimates. They are provided for trending purposes only.

Table 3: Maximum low impedance flow at stated pressures

Pressure (cm H ₂ O)	Flow (L/min)
6.6	177
13.2	202
20.0	202

Note: The manufacturer reserves the right to change these specifications without notice.

Symbols which appear on the product

Attention, consult accompanying documents

Class II equipment

Type CF equipment

IPX1

Drip Proof

Start/Stop or Mask-Fit

Guidance and Manufacturer's Declaration - Electromagnetic Emissions and Immunity

Guidance and manufacturer's declaration – electromagnetic emissions

The VPAP Malibu is intended for use in the electromagnetic environment specified below. The customer or the user of the VPAP Malibu should assure that it is used in such an environment.

Emissions test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group 1	The VPAP Malibu uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The VPAP Malibu is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage network that supplies buildings used for domestic purposes.
Harmonic Emissions IEC 61000-3-2	Class A	
Voltage Fluctuations/Flicker Emissions IEC 61000-3-3	Complies	

Medical Electrical Equipment needs special precautions regarding EMC and needs to be installed and put into service according to EMC information provided in this document.

Warnings: The VPAP Malibu should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, the VPAP Malibu should be observed to verify normal operation in the configuration in which it will be used.

The use of accessories (eg, humidifiers) other than those specified in this guide is not recommended. They may result in increased emissions or decreased immunity of the VPAP Malibu.

Guidance and manufacturer's declaration – electromagnetic immunity

The VPAP Malibu is intended for use in the electromagnetic environment specified below. The customer or the user of the VPAP Malibu should assure that it is used in such an environment.

Immunity test	IEC60601-1-2 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV contact ±8 kV air	±6 kV contact ±8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	±2 kV for power supply lines ±1 kV for input/output lines	±2 kV Not Applicable	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	±1 kV differential mode ±2 kV common mode	±1 kV differential mode ±2 kV common mode	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines. IEC 61000-4-11	<5% U_t (>95% dip in U_t) for 0.5 cycle 40% U_t (60% dip in U_t) for 5 cycles 70% U_t (30% dip in U_t) for 25 cycles <5% U_t (>95% dip in U_t) for 5 sec	< 12V (>95% dip in 240V) for 0.5 cycle 96V (60% dip in 240V) for 5 cycles 168V (30% dip in 240V) for 25 cycles <12V (>95% dip in 240V) for 5 sec	Mains power quality should be that of a typical commercial or hospital environment. If the user of the VPAP Malibu requires continued operation during power mains interruptions, it is recommended that the VPAP Malibu be powered from an uninterruptible power source
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment

NOTE: U_t is the AC mains voltage prior to application of the test level.

(Continued next page)

Guidance and manufacturer's declaration – electromagnetic immunity (Continued)

The VPAP Malibu is intended for use in the electromagnetic environment specified below. The customer or the user of the VPAP Malibu should assure that it is used in such an environment.

Immunity test	IEC60601-1-2 test level	Compliance level	Electromagnetic environment – guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	3 Vrms	Portable and mobile RF communications equipment should be used no closer to any part of the VPAP Malibu, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance $d = 1.17 \sqrt{P}$
Radiated RF IEC 61000-4-3	10 V/m 80 MHz to 2.5 GHz	10 V/m	$d = 0.35 \sqrt{P}$ 80 MHz to 800 MHz $d = 0.70 \sqrt{P}$ 800 MHz to 2.5 GHz where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, ^a should be less than the compliance level in each frequency range. ^b Interference may occur in the vicinity of equipment marked with the following symbol: 

NOTE 1: At 80 MHz and 800MHz, the higher frequency range applies.

NOTE 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

^a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the VPAP Malibu is used exceeds the applicable RF compliance level above, the VPAP Malibu should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the VPAP Malibu.

^b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 10 V/m.

Recommended separation distances between portable and mobile RF communications equipment and the VPAP Malibu

The VPAP Malibu is intended for use in an environment in which radiated RF disturbances are controlled. The customer or the user of the VPAP Malibu can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the VPAP Malibu as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz $d = 1.17 \sqrt{P}$	80 MHz to 800 MHz $d = 0.35 \sqrt{P}$	800 MHz to 2.5 GHz $d = 0.70 \sqrt{P}$
0.01	0.17	0.04	0.07
0.1	0.37	0.11	0.22
1	1.17	0.35	0.70
10	3.69	1.11	2.21
100	11.70	3.50	7.00

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be determined using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1: At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

Limited Warranty

ResMed warrants that your ResMed product shall be free from defects in material and workmanship for the period specified below from the date of purchase by the initial consumer. This warranty is not transferable.

Product	Warranty Period
ResMed humidifiers, ResControl™, ResLink™, ResTraxx™	1 Year
ResMed flow generators	2 Years
Accessories, mask systems (including mask frame, cushion, headgear and tubing). Excludes single-use devices.	90 Days

Note: Some models are not available in all regions.

If the product fails under conditions of normal use, ResMed will repair or replace, at its option, the defective product or any of its components. This Limited Warranty does not cover:

- a) any damage caused as a result of improper use, abuse, modification or alteration of the product;
- b) repairs carried out by any service organization that has not been expressly authorized by ResMed to perform such repairs;
- c) any damage or contamination due to cigarette, pipe, cigar or other smoke;
- d) any damage caused by water being spilled on or into a flow generator.

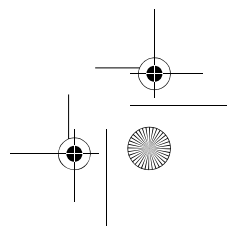
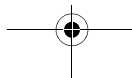
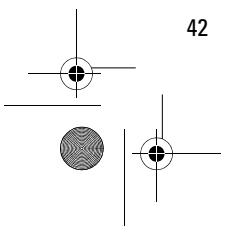
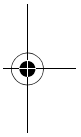
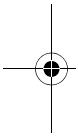
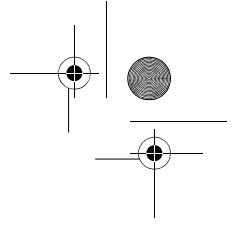
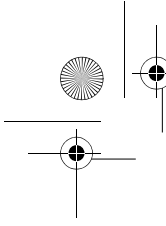
Warranty is void on product sold, or resold, outside the region of original purchase. Warranty claims on defective product must be made by the initial consumer at the point of purchase.

This warranty is in lieu of all other express or implied warranties, including any implied warranty of merchantability or fitness for a particular purpose. Some regions or states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.

ResMed shall not be responsible for any incidental or consequential damages claimed to have occurred as a result of the sale, installation or use of any ResMed product. Some regions or states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitation may not apply to you. This warranty gives you specific legal rights, and you may also have other rights which vary from region to region.

For further information on your warranty rights, contact your local ResMed dealer or ResMed office.

R001-307/2 05 06

Index

A

Accessories 10
Air filter 28
Antibacterial filter 28

B

Backlight, LCD and keypad 17

C

Cleaning 27
Components 9

D

Daily cleaning 27
DC power
 using 25
Definitions 3
Dryness 26
Dynamic pressure characteristics 35

E

Electromagnetic emissions and immunity
 37
Error messages 32

F

Features of the VPAP Malibu 17
Filter 28
Flow characteristics 35
Front key 17

H

Helpful hints 25
HumidAire 2i 16
Humidifier 10, 16
Humidifier use 16

I

International use 26

K

Keys, functions 17

L

LCD screen and keypad 17
Left key 17
Limited Warranty 41

M

Maintenance 27
Mask 9
Mask-Fit star rating, definitions 25
Mask-Fitting 24, 26
Medical information 5
Menus 18
 using 18
 with humidifier 19
 with ResLink and oximeter 20
Mouth leaks 25

N

Nasal irritation 26

O

Operating instructions 21
Oxygen
 supplemental 11

P

Periodic cleaning 27
Preparing for use 13

R

Ramp screen 18
Ramp time 18, 23
Replacing the air filter 28
Replacing the antibacterial filter 28
Responsibility, user/owner 3
Right key 17
Runny or blocked nose 26

S

Servicing 28
Setting up 13
Settling screen 18
Settling time 18, 23
Side effects 7
SmartStart 23
SmartStop 23
Starting treatment 21
Stopping treatment 23
Supplemental oxygen 11
Symbols 36

T

- Technical specifications 35
- Traveling 26
- Troubleshooting 31

U

- Up/Down key 17
- Used (time) screen 18
- User/owner responsibility 3
- Using the Mask-Fitting feature 24
- Using the menus 18

W

- Warnings related to treatment 7
- Warranty 41
- Weekly cleaning 27

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