
 **SENNHEISER**

Gebrauchsanleitung
Instructions for use
Instructions pour l'usage
Istruzioni per l'uso
Modo de empleo

RI 100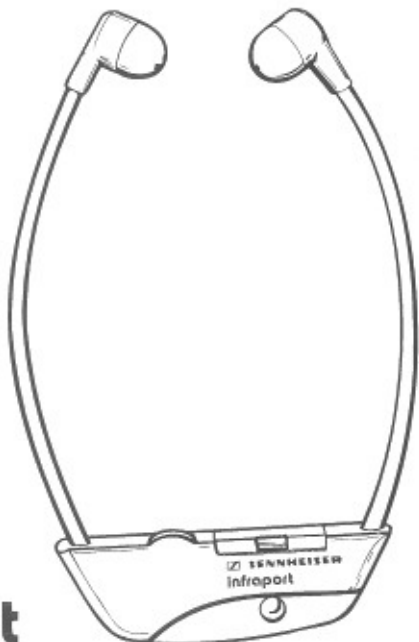

infraport

Download from www.Somanuals.com. All Manuals Search And Download.

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RI 100 infrared mono hearing aid

The wireless infrared mono hearing aid RI 100 receives music and speech radiated by an appropriate infrared transmitter.

Types of application:

- TV sound reproduction
- in theatres or during conferences as a hearing aid

Features:

- excellent sound reproduction
- high wearing comfort
- high volume
- individually adjustable to different degrees of hardness of hearing
- easy to handle
- economic accu operation causing no harm to the environment
- recharging of BA 90 accu in the charging port of the transmitter or using the L 90 charging unit.

Technical data:

Receiving frequency	95 kHz \pm 25 kHz, FM
Frequency range	30-18,000 Hz
S/N ratio	65 dB(A)
Max. sound pressure level at 1 kHz	124 dB
THD at 1 kHz	< 1 %
Controllable bass cut Level pre-adjustment	up to 20 dB/100 Hz approx. attenuation of up to 30 dB, separately adjustable for the left and the right channel
Power supply	BA 90 accu (Item No. 03261)
Operating time	approx. 10 hrs
Accu recharging time	approx. 18 hrs
Weight	approx. 40 g (including accu)

Fault finding:

Hissing noise only	Transmitter is not set into operation or it is defective
Volume turned to max. but no signal available	Signal is missing, check connection to transmitter! Volume can be adjusted separately at headphone socket? See manual of TV/hi-fi components!
Sound is disturbed	Accu is completely discharged, please recharge it immediately!
Sound is slightly noisy	The transmitter is not positioned in an optimum way, reception is obstructed.

Before using it for the first time, the accu has to be charged for 24 hrs!
Please switch off the device after use!

Ear cushions (removable) have to be cleaned with soap-suds from time to time!

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