GUARDIAN®

Trapeze and Floor Stand

Trapeze Model IC-7740,
Floor Stand Model IC-7714

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usuario

⚠ WARNING

- The Fixed Trapeze should **ONLY** be used on beds with metal-framed headboards.
- The Head Trapeze, used in conjunction with a bed, can support an individual weighing up to 250 lbs. (114 kg.).
- The Fixed Trapeze is designed to provide support, increase stability and assist the user when repositioning himself/herself in bed. The Fixed Trapeze is **NOT** designed to support the total weight of an individual.
- Do **NOT** install or adjust Trapeze when bed is occupied.
- Do **NOT** attach swivel Trapeze to the floor stand. This may cause the stand to tip over, causing injury to the user and/or assistant.
- Before lowering bed, ensure trapeze is in the highest position.
- Make sure the snap buttons fully protrude through the adjustment holes of the floor stand. This insures that the floor stand is properly installed and securely locked in position.
- After installation and **BEFORE** use, make sure all attaching hardware is tight and secure.
- The trapeze is not intended to provide a mounting point for either powered or non-powered traction equipment.

ATTACHING TRAPEZE TO BED END (Figure 1)

1. Locate the lower bed bracket and attach to the underneath of the bed headboard. The tubular hook on the bracket should face up.
2. Tighten the wing nuts to secure the bracket in place.
3. Place the Trapeze on a flat surface for easy installation of the upper bed bracket.
4. Loosen the wing nuts on the upper bed bracket and slide the bracket 3/4 of the way up the Trapeze bar. Once in position, tighten the wing nuts to secure.
5. Place the Trapeze bar assembly into the socket of the lower bracket now secured on the bottom of the bed headboard.
6. Position the upper bracket so its tubular hook rests on the top of the bed headboard. Tighten the wing nuts to secure the bracket in place.

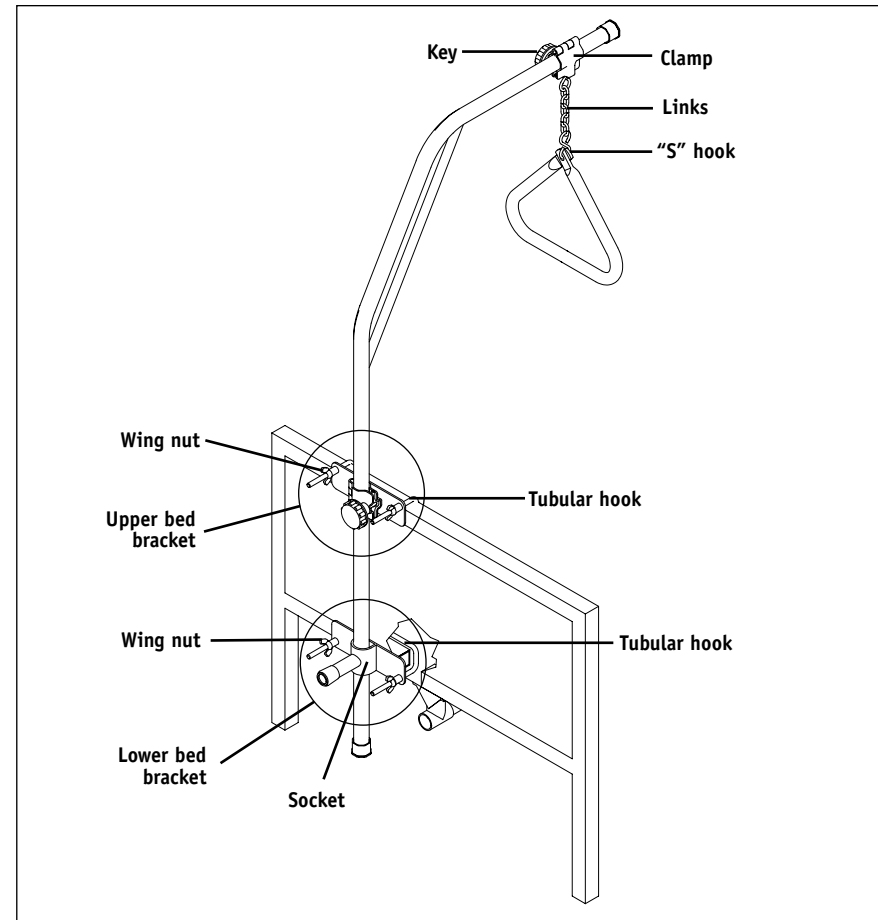
Figure 1

ASSEMBLING FLOOR STAND (Figure 2)

1. Place the "A" frame into the floor stand leg assembly with the snap buttons facing the short length of the floor stand.
2. Press the snap button on one leg of the "A" frame and slide it into the leg assembly until it snaps into place.
3. Make sure the snap buttons fully protrude through the holes in the leg assembly.
4. Attach Trapeze to floor stand.

ATTACHING TRAPEZE TO FLOOR STAND (Figure 2)

1. Locate the Trapeze lower bed bracket and attach to the lower rail of the floor stand. The tubular hook on the bracket should face down.
2. Tighten the wing nuts to secure the bracket in place.
3. Loosen the wing nuts on the upper bed bracket and slide the bracket 3/4 of the way up the Trapeze bar. Once in position, tighten the wing nuts to secure.
4. Place the Trapeze bar assembly into the socket of the lower bracket now secured on the lower floor stand rail.
5. Position the upper bracket, tubular hook facing down, on the top rail of the floor stand. Tighten the wing nuts to secure the bracket in place.

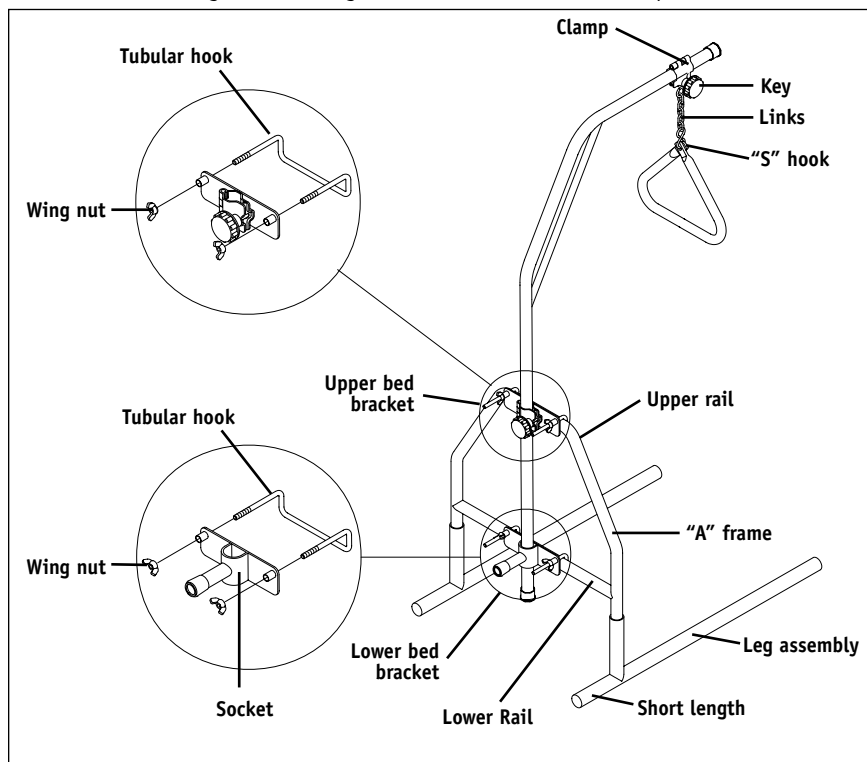
Figure 2

ADJUST THE HEIGHT/POSITION OF THE TRAPEZE

Height:

Place any of the grab-bar chain links onto the "S" hook.

Position:

1. Loosen grab-bar key by turning counter-clockwise.
2. Slide the grab-bar clamp to desired position on the Trapeze bar.
3. Turn the grab-bar key clockwise to secure in place.

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© 2002, Sunrise Medical 11.02
Sunrise Medical Inc.