GUARDIAN[®]

Component Mobility System *User Instructions*

Model IC-5140

IMPORTANT WARNINGS

⚠ WARNING

Inspect all parts for damage to the carton or its contents. If damage is evident, do not use. Contact your local Sunrise Medical dealer immediately.

⚠ WARNING

Read all instructions before using this product. Failure to follow these instructions may result in injury or damage.

⚠ WARNING

Make sure all attaching hardware is tightened securely at all times.

⚠ WARNING

The recommended weight capacity of the component mobility system is 100 lbs.

INSTALLATION

1. Remove head and foot boards from bed. For complete instructions, refer to Bed Instruction Manual (IM-429000).
2. Connect wheels to 1/2" tubing on both ends of connector, making sure that the side of the wheel with six ribs faces the inside axle assembly and the side with three ribs faces the outside (Figure 1).

NOTE– To help the wheels fit onto the tubing more easily, tap the wheel with a rubber or plastic mallet.

3. Fasten both wheels into place with hitch pin.
4. Position mobility system over the bed end hook (Figure 2).
5. Push down so that the pins of the bed end hook depress the spring button. Slide the mobility system to the left to fasten it to the head or foot spring.
6. Turn the head or foot spring around and place the wheels on the floor.
7. To remove, depress the spring button with finger, slide the mobility system to the right and remove from bed end hook.

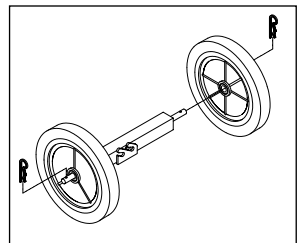
Figure 1

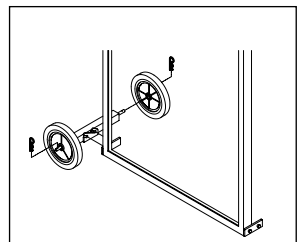
Figure 2

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© 2002, Sunrise Medical 11.02
Sunrise Medical Inc.