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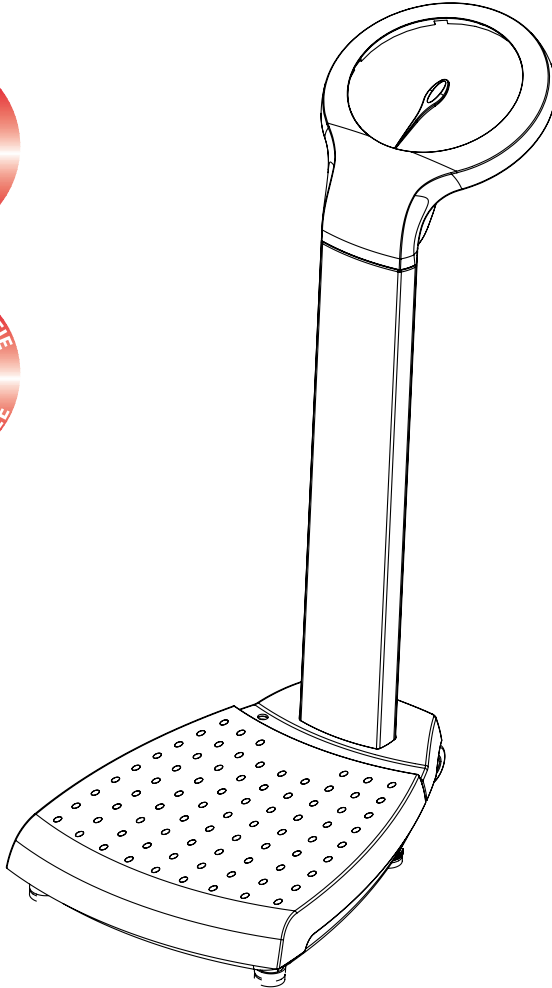
After Sales Service : Terraillon France & Headquarters - B.P.73 -78403 Chatou Cedex-France

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FOOK TIN TECHNOLOGIES LTD

4/F, Eastern Centre, 1065 King Road, HONG KONG

www.fooktin.com.hk

MECHANICAL MEDICAL SCALE

Dear Customer,

Thank you for purchasing this product. We hope you will find it satisfactory. Please read these instructions carefully to take full advantage of your product.

READ BEFORE USE

The Mechanical Medical Scale is designed and manufactured in a facility certified in the following Management Systems : ISO 9001 Quality, ISO14001 Environment, OHSAS18001 Health and Safety.

This scale is manufactured to conform with the United States FDA Class I requirements for certification as a Patient medical weighing device. Regulation 880.770. Establishment Registration N° 3005374248.

Applications

The mechanical medical scale's intended use is for weighing patients for the purposes of monitoring, diagnosis and medical treatment.

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		This symbol indicates general precautions that should be taken when using this product.
	WARNING	This symbol indicates the possibility of serious injury if the product is mishandled or instructions are ignored.
	CAUTION	This symbol indicates the possibility of physical injury or equipment damage if instructions are ignored.

WARNING

- Use the product on a firm and flat surface.
- Clean after use with a lightly dampened cloth without allowing any water to get into the device. Do not use solvents or immerse the product in water.
- Avoid excessive impact or vibration to the unit, such as dropping the product onto the floor.
- Do not jump on the weighing platform.

CAUTION

- Do not disassemble the scale as incorrect handling may cause injury.

MAINTENANCE

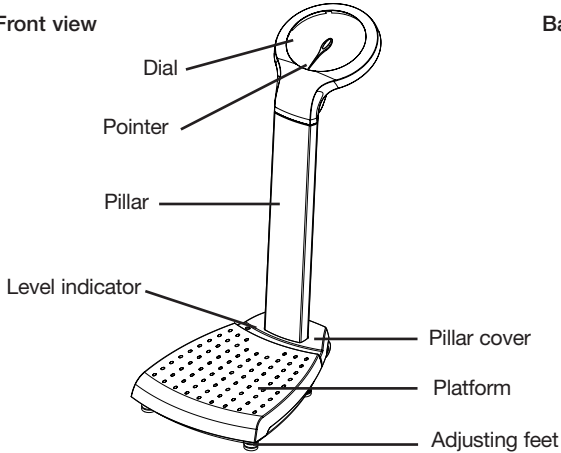
- The product has been manufactured with precision. The user should not disassemble the product on its own.

PRODUCT SPECIFICATION


- Weighing function
- Level indicator and adjusting feet
- Large dial : 8"
- Capacity: 400lb / 180 kg
- Graduation: 1lb / 0.5 kg

MECHANICAL MEDICAL SCALE

Front view

Back view

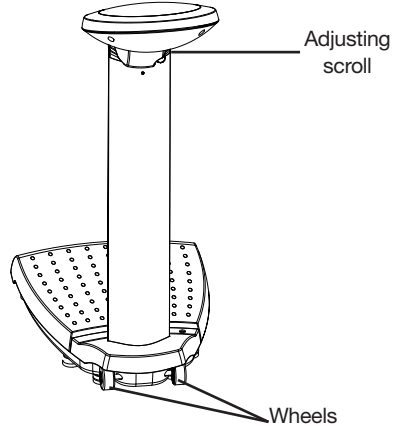
ITEMS INCLUDED IN THE PACKAGE

- Top part with dial and pillar.
- Pillar cover.
- Base part with platform and adjusting feet.
- 3 pcs M5 x 10 screws (1 pc screw as spare part).
- 1 M4 Allen Key.
- 1 screw driver.
- Instruction and assembly manual.

PROCEDURES AND INSTRUCTIONS FOR PRODUCTS ASSEMBLY

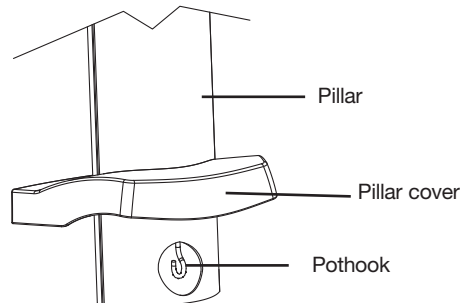
1. Insert the pillar into the hole of the pillar cover and move the cover temporarily up for easier assembly.


MECHANICAL MEDICAL SCALE

2. Insert the pillar into the hole at the base, while holding the pothook with your hand.

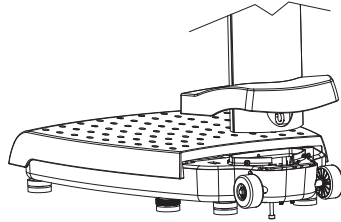Important: Ensure pillar is completely seated onto base otherwise the scale will not operate correctly. When the red indicator line (located on the pillar) meets the edge of the collar on the base, the pillar will be properly seated in the base of the scale.

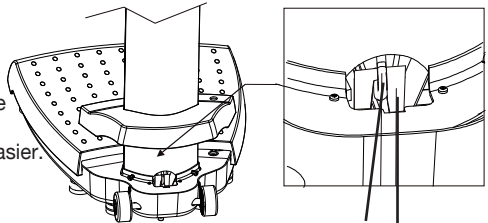
3. Use screw driver as well as your hand to hook the pothook into the hole of the long arm connecting plate. The steel screw on the base (which will be removed in the next step) can be adjusted inward to move the long arm connecting plate up towards the pothook and make the connection between the two points easier.


Pothook
Long arm connecting plate

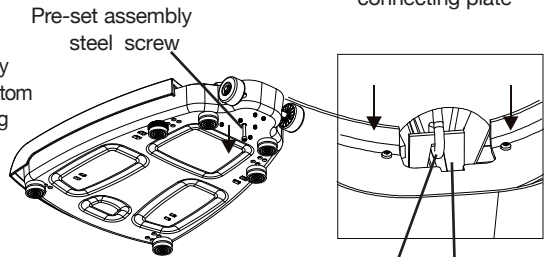4. Using Allen key to remove the pre-set assembly steel screw out from the small hole (located in the bottom part of the base) to release long arm connecting plate down to its place.

Pothook
Long arm connecting plate

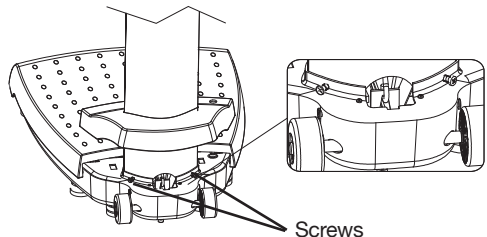
5. Using screw driver to fix the pillar to the base by screwing in tightly the 2 pcs of screws at the left and right side of the pillar.

Screws

MECHANICAL MEDICAL SCALE


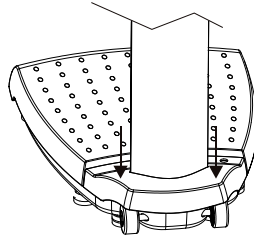
6. Move the pillar cover down and push it onto the base until it clicks on both sides


PRODUCT USAGE

Initialisation

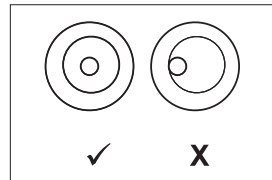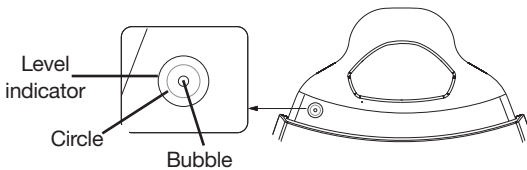- Before you use the scale for the first time or when the scale has been moved to another place, the zero point and balance of the scale must be set.
1. Do not step on the scale or/and remove all weight from the platform.
 2. Place the scale on a hard and flat surface. Use the adjusting feet to adjust the balance until the bubble inside the level indicator is located in the middle of the drawn circle.

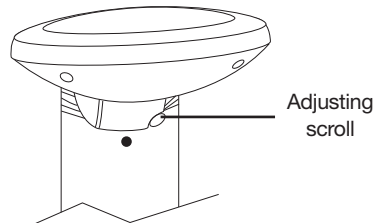3. Adjust the pointer to “Zero” using adjusting scroll.

Operation

1. Step on the platform and stand still during the weighing operation process.
2. The weight will be shown on the dial.

MECHANICAL MEDICAL SCALE

GUARANTEE

This product is guaranteed against defects in materials or manufacturing. During the guarantee period, any such defects will be repaired free of charge (proof of purchase must be shown in the case of a complaint under guarantee). The guarantee does not cover damage resulting from improper use or negligence. In the event of a claim, first contact the shop where you purchased the product.

AFTER SALES SERVICE

For calibration and after sales services, please contact Terrailon consumer service at the following address:

After Sales Service : Terrailon France & Headquarters - B.P.73 -78403 Chatou Cedex-France
Service consommateurs : 0 826 88 1789
serviceconsommateurs@terrailon.fr - www.terrailon.com

FOOK TIN TECHNOLOGIES LTD
4/F, Eastern Centre, 1065 King Road, HONG KONG
www.fooktin.com.hk

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Viale Sarca 45
Milano - Italy
Tel: +39 (0)2 66114199

Terraillon Asia Pacific Ltd

4/F, Eastern Centre
1065 King's Road
Quarry Bay - Hong Kong
Tel: +852 (0)2960 7288

Terraillon Corp

1860 Renaissance Blvd
Sturtevant, WI 53177 - USA
Tel: +1-866-696-6066
customerserviceusa@terraillon.com

Terraillon
PROFESSIONAL
www.terraillon.com

Free Manuals Download Website

<http://myh66.com>

<http://usermanuals.us>

<http://www.somanuals.com>

<http://www.4manuals.cc>

<http://www.manual-lib.com>

<http://www.404manual.com>

<http://www.luxmanual.com>

<http://aubethermostatmanual.com>

Golf course search by state

<http://golfingnear.com>

Email search by domain

<http://emailbydomain.com>

Auto manuals search

<http://auto.somanuals.com>

TV manuals search

<http://tv.somanuals.com>

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Hanson (UK) Ltd
Terraillon Deutschland GmbH
Terraillon SpA