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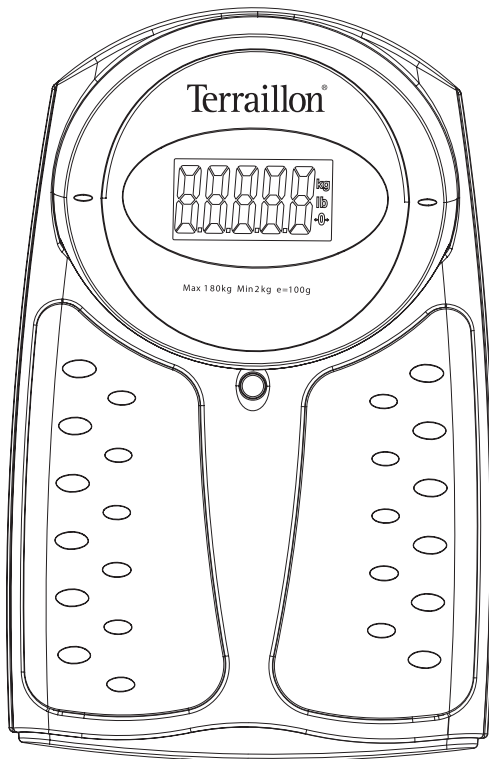
FOOK TIN TECHNOLOGIES LTD
 4/F, Eastern Centre, 1065 King's Road, HONG KONG
www.fooktin.com.hk

INSTRUCTION MANUAL

CLASS III ELECTRONIC MEDICAL SCALE

Dear Customer,

Thank you for purchasing this product. We hope you will find it satisfactory.

Please read these instructions carefully to take full advantage of your product.

READ BEFORE USE

The Electronic Medical Scale is designed and manufactured in a facility certified in the following Management Systems: ISO 9001 Quality, ISO14001 Environment, OHSAS18001 Health and Safety and ISO13485 Medical Devices Quality.

Applications

The electronic medical scale's intended use is for weighing patients for the purposes of monitoring, diagnosis and medical treatment.

Precautions of use

		This symbol indicates general precautions that should be taken when using this product.
	WARNING	This symbol indicates the possibility of serious injury if the product is mishandled or instructions are ignored.
	CAUTION	This symbol indicates the possibility of physical injury or equipment damage if instructions are ignored.

Warning

- Use the product on a firm and flat surface.
- Clean after use with a lightly dampened cloth without allowing any water to get into the device. Do not use solvents or immerse the product in water.
- Avoid excessive impact or vibration to the unit, such as dropping the product onto the floor.
- Do not jump on the weighing platform.
- Only use the adaptor which has the characteristics mentioned in the paragraph: Product Specification.

Caution

- Do not disassemble the scale as incorrect handling may cause injury.

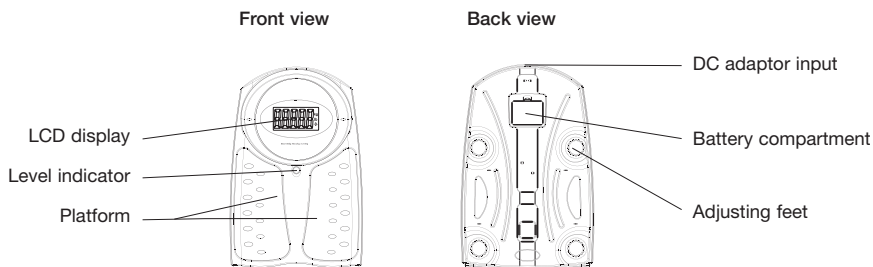
MAINTENANCE

- The product has been manufactured and calibrated with precision. The user should not disassemble or attempt to calibrate the product on its own.
- Remove batteries if the product is not in use for an extended period.

CLASS III ELECTRONIC MEDICAL SCALE

PRODUCT SPECIFICATION



- Class I medical device (EC Directive 93/42 EEC).
- Class III OIML compliant.
- Class III Non-Automatic Weighing Instruments (EC Directive 90/384 EEC) compliant.
- Weighing function.
- Level indicator and adjusting feet.
- Tap-on and auto-off functions.
- Low-battery indicator.
- Capacity: 180 kg.
- Graduation: 100 g.
- Operates through 1 piece 9V battery (6LR61 or equivalent included) or with AC 100-240V adaptor input/DC 9V adaptor output (adaptor not included).

PRODUCT USAGE

Initialisation

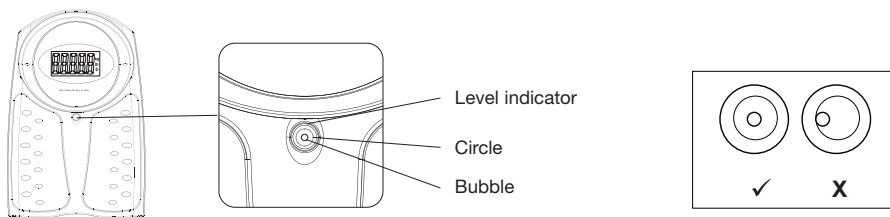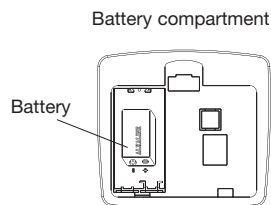- Insert the battery(s) in the battery compartment, making sure that the polarities are correct or power up the product by connecting the DC adaptor to the power socket. The battery type is specified inside the battery compartment.
- If it is necessary to set the balance of the product before you use it for the first time or if the product has been moved to another place:
 1. Do not step on the scale or/and remove all weight from the platform.
 2. Place the scale on a hard and flat surface. Use the adjusting feet to adjust the balance until the bubble inside the level indicator is located in the middle of the drawn circle.

CLASS III ELECTRONIC MEDICAL SCALE

Operation

- Tap lightly on the center of the scale beneath the display to turn it on. The following displays will appear in the following order: software version of the scale, all digits and symbols, "0.0".

- Step on the scale while the display shows "0.0" and stand still while the weight is computing. Weight reading will be shown automatically:

- The product will turn off automatically after approximately 10 seconds.

Warning indicators

Err0	Initial zero error. Step off the scale, wait until the scale shows 0.0 and repeat measurement.	adErr	Hardware problem. Please contact place of purchase.
Err2	Overload: remove weight immediately.	EEP	Hardware problem. Please contact place of purchase.
bAtt	Low battery indication: replace it.	Ustb	Unstable zero. Please wait until the scale shows 0.0 and repeat measurement.

PROTECT THE ENVIRONMENT

Dispose of used batteries in a designed container so that they can be collected and recycled. Do not mix different types of batteries. Do not mix new and used batteries.

After life, dispose of this product at a designated waste recycling point.

GUARANTEE

This product is guaranteed against defects in materials or manufacturing. During the guarantee period, any such defects will be repaired free of charge (proof of purchase must be shown in the case of a complaint under guarantee). The guarantee does not cover damage resulting from improper use or negligence. In the event of a claim, first contact the shop where you purchased the product.

AFTER SALES SERVICE

For calibration and after sales services, please contact Terraillon consumer service at the following address:

After Sales Service: Terraillon France & Headquarters
 B.P.73 -78403 Chatou Cedex-France
 Service consommateurs :0 826 88 1789
 serviceconsommateurs@terraillon.fr
www.terraillon.com

FOOK TIN TECHNOLOGIES LTD
 4/F, Eastern Centre, 1065 King's Road, HONG KONG
www.fooktin.com.hk

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Viale Sarca 45
Milano - Italy
Tel: +39 (0)2 66114199

Terrailon Asia Pacific Ltd

4/F, Eastern Centre
1065 King's Road
Quarry Bay - Hong Kong
Tel: +852 (0)2960 7288

Terrailon Corp

1860 Renaissance Blvd
Sturtevant, WI 53177 - USA
Tel: +1 (0)866 696 6066
customerserviceusa@terraillon.com

Terrailon
PROFESSIONAL

www.terraillon.com

P/N:700351253(Rev.06.11)

Free Manuals Download Website

<http://myh66.com>

<http://usermanuals.us>

<http://www.somanuals.com>

<http://www.4manuals.cc>

<http://www.manual-lib.com>

<http://www.404manual.com>

<http://www.luxmanual.com>

<http://aubethermostatmanual.com>

Golf course search by state

<http://golfingnear.com>

Email search by domain

<http://emailbydomain.com>

Auto manuals search

<http://auto.somanuals.com>

TV manuals search

<http://tv.somanuals.com>

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Terrailon Deutschland GmbH
Terrailon SpA