# **TOPCOM<sup>®</sup>**

## **BPM Wrist 2301b** **Blood Pressure Monitor**

**HANDLEIDING**  
**MODE D'EMPLOI**  
**BRUGSANVISNING**  
**USER GUIDE**  
**v.1.2**

**CE 0434**

- NL** De in deze handleiding beschreven mogelijkheden worden gepubliceerd onder voorbehoud van wijzigingen.
- DK** Vi forbeholder os retten til ændringer af de specifikationer, der er beskrevet i denne brugsanvisning.
- FR** Les possibilités décrites dans ce manuel sont publiées sous réserve de modifications.
- GB** The features described in this manual are published with reservation to modifications.

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<b>1</b>	<b>Introduction</b>	<b>52</b>
<b>2</b>	<b>Helpfull tips</b>	<b>52</b>
<b>3</b>	<b>Safety instructions</b>	<b>53</b>
<b>4</b>	<b>Blood pressure</b>	<b>54</b>
4.1	What is blood pressure?	54
4.2	Why measure your blood pressure?	54
4.3	Blood pressure fluctuation	54
4.4	Blood pressure standard	55
<b>5</b>	<b>Battery installation</b>	<b>56</b>
<b>6</b>	<b>Buttons</b>	<b>57</b>
<b>7</b>	<b>Display</b>	<b>57</b>
<b>8</b>	<b>Applying the cuff</b>	<b>58</b>
<b>9</b>	<b>Measuring your bloodpressure</b>	<b>59</b>
9.1	Posture	59
9.2	Important	59
9.3	Measurement	60
9.4	Memory	61
<b>10</b>	<b>Setting date and time</b>	<b>61</b>
<b>11</b>	<b>Technical specifications</b>	<b>62</b>
<b>12</b>	<b>Error messages</b>	<b>63</b>
<b>13</b>	<b>Topcom warranty</b>	<b>63</b>
13.1	Warranty period	63
13.2	Warranty handling	64
13.3	Warranty exclusions	64
13.4	Returning defect warranty units	64

## 1 Introduction

Congratulations with your purchase of the Topcom BPM wrist 2301b. This fully automatic, wrist mounted blood pressure monitor is both easy to use and ideally suited for daily measurements. The large display includes systolic, diastolic and pulse, all clearly shown at the completion of each reading.

In addition, you can store up to 30 measurements per Memory Zone, ideal for users who want to monitor and track their blood pressure on a regular basis. The BPM wrist 2301b is compact and portable, making it ideal for home use and travel.

## 2 Helpfull tips

Here are a few helpful tips to help you obtain more accurate readings:

- Do not measure your blood pressure immediately after consuming a large meal. Wait one hour before measuring.
- Do not smoke or drink alcohol before measuring your blood pressure.
- You should not be physically tired or exhausted while taking a measurement.
- Try to take 15-minutes rest and relax before a reading.
- Do not take measurements if you are under stress or under tension.
- Take your blood pressure at normal body temperature. If you are feeling cold or hot, wait a while before taking a measurement.
- If the monitor is stored at very low temperature, have it placed at a warm location for at least one hour before using it.

### 3 Safety instructions

- **This product is not intended as a substitute for consultation with your doctor.** It is for household use only!
- For users diagnosed with common arrhythmia (atrial or ventricular premature beats or atrial fibrillation), diabetes, poor circulation of blood, kidney problems, or for users suffered from stroke, or for unconscious users, the device might not be suitable for use. Consult your doctor in case of any doubt.
- The unit should not be operated by children so to avoid hazardous situations.
- The unit contains high-precision assemblies. Therefore, avoid extreme temperatures, humidity, and direct sunlight. Avoid dropping or strongly shocking the main unit, and protect it from dust.
- Leaky batteries can damage the unit. Remove the batteries when the unit is not used for a long time.
- Do not press '**START**' when the cuff is not placed around the wrist.
- Do not disassemble the main unit or cuff.
- If the unit is stored in a cold place, allow it to acclimate at room temperature before use.
- Clean the blood pressure monitor body and the cuff carefully with a slightly damp, soft cloth. Do not press. Do not bend the pre-formed cuff inside out. Do not wash the cuff or use chemical cleaner on it. Never use thinner, alcohol or petrol (gasoline) as cleaner.
- Dispose of the unit and used batteries in an environment friendly manner. Do not dispose of batteries in a normal household garbage.

## 4 Blood pressure

### 4.1 What is blood pressure?

Blood pressure is the pressure exerted on the artery tube while blood flows through the arteries. The pressure measured when the heart contracts and sends blood out of the heart is **systolic** (highest). The pressure measured when the heart dilates with blood flowing back into the heart is called **diastolic** (lowest) blood pressure.

### 4.2 Why measure your blood pressure?

Among the various health problems afflicting modern people, problems associated with high blood pressure are by far the most common. High blood pressure's dangerously strong correlation with cardiovascular diseases and high morbidity has made measuring blood pressures a necessity of identifying those at risk.


### 4.3 Blood pressure fluctuation

Blood pressure fluctuates all the time!

You should not be worried if you encountered two or three measurements at high levels. Blood pressure changes over the month and even throughout the day depending on circumstances (state of mind, temperature, ...)


## 4.4 Blood pressure standard

The World Health Organization (WHO) and National High Blood Pressure Education Program has developed a blood pressure standard, according to which areas of low and high-risk blood pressure are identified. This standard however, is a general guideline as individual's blood pressure varies among different people and different groups...etc.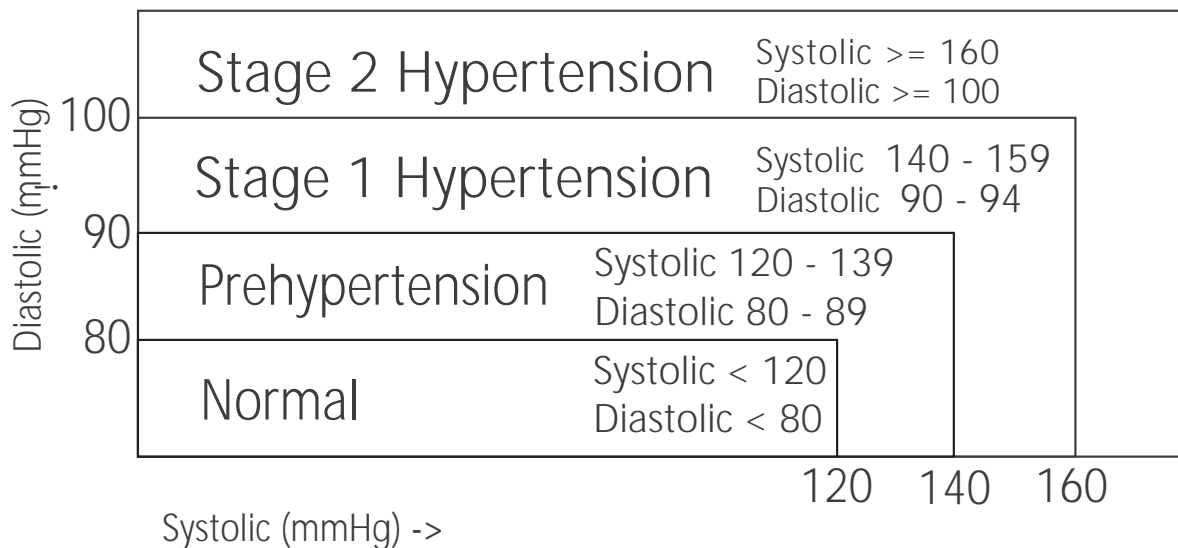

**It is important that you consult with your physician regularly. Your physician will tell you your normal blood pressure range as well as the point at which you will be considered at risk.**

## 5 Battery installation

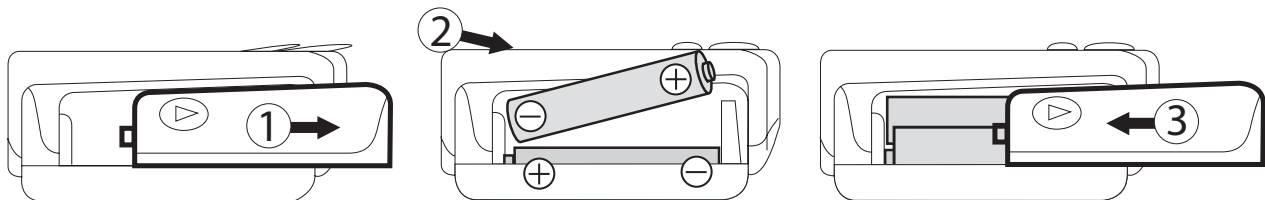
- Open the battery compartment by sliding the cover away
- Insert 2 AAA Alkaline batteries. Follow the polarity as indicated in the battery compartment.

**Dispose of the batteries at a designated battery disposal unit and not in the household waste. Do not shortcircuit & dispose of in fire. Remove the battery if this device is not going to be used for a long period.**


## 6 Buttons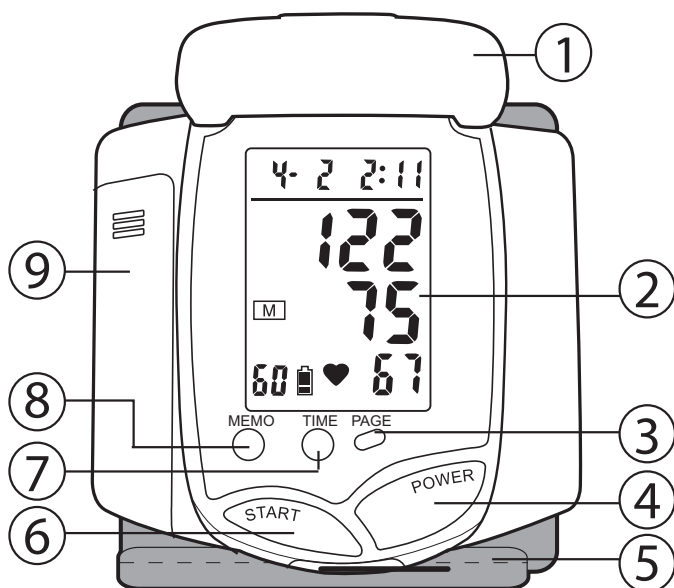

1. Display cover
2. LCD Display
3. Page button
4. Power button
5. Cuff
6. Start button
7. Time button
8. Memo button
9. Battery compartment


## 7 Display

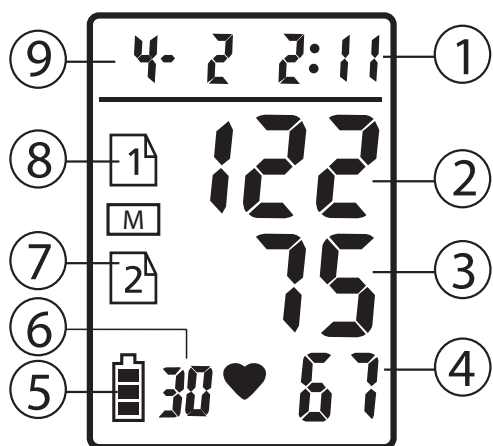
1. Hour/Minutes
2. Systolic pressure
3. Diastolic pressure
4. Pulse rate
5. Battery Low indicator
6. Memory number
7. Stored in Memory Zone 2
8. Stored in Memory Zone 1
9. Month/Day

## 8 Applying the cuff


1. Remove all watches, jewelry, etc. prior to attaching the wrist monitor. Clothing sleeves should be rolled up and the cuff should be wrapped on bare skin for correct measurements. Apply the cuff to your **left** wrist with palm facing up.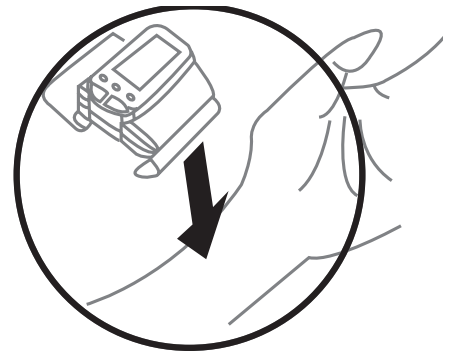

2. Make sure the edge of the cuff is about 1 cm from the palm. In order to insure accurate-measurements.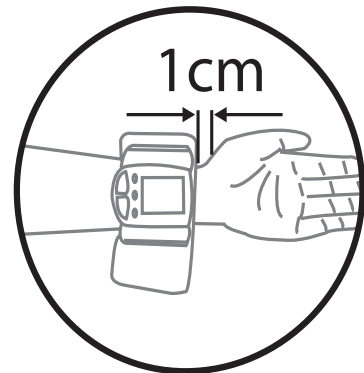

3. Fasten the velcro strap securely around your wrist so there is no extra space between the cuff and the wrist. If the cuff is not wrapped tight enough, the measurement values will be false.

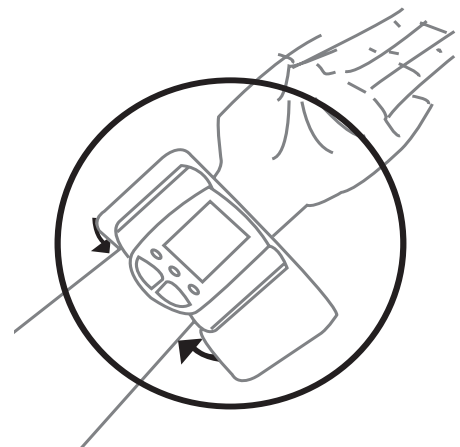**If your physician has diagnosed you with poor circulation on your left arm, place the cuff around your right wrist.**

## 9 Measuring your bloodpressure

### 9.1 Posture

1. Relax your entire body, especially the area between your elbow and fingers. Place your elbow on a table so that the cuff is at the same level as your heart.

**Your heart is located slightly below your arm pit.**

2. If the cuff is not at the same level as your heart or if you can not keep your arm completely still throughout the reading, use a soft object such as folded towel to support your arm.
3. Turn your palm upwards. Sit upright in a chair, and take 5-6 deep breaths. Avoid leaning back during measurement.

### 9.2 Important

- This monitor automatically switches off 3 minutes after the last key operation.
- To interrupt the measurement, simply press '**POWER**'. The cuff will deflate.
- During the measurement, do not talk or move your arm or hand muscles.

## 9.3 Measurement

- Press '**POWER**'. All digits will light up, checking the display functions. The checking procedure will complete after about 2 seconds.
- After all symbols disappear, the display will show "0" with the designated Memory Zone. At this time, the monitor is "Ready to Measure".
- Press '**PAGE**' to select if the result must be stored in memory zone  or . (see §9.4)
- Press '**START**'. Once the monitor automatically inflates the cuff to approximately 180 mmHg, the measurement will begin.
- When the measurement is completed, **systolic, diastolic** and the **Pulse Rate** will be shown simultaneously on the LCD screen (see §7).
- When a mistake was detected during measurement the Error mark '**Err**' appears on the display (see §12).
- After deflation, the measurement of blood pressures is completed. Press '**POWER**' button to switch off electrical supply.
- If you forget to push the '**POWER**' button, the power will turn off automatically within 3 minutes.

## 9.4 Memory

The BPM 2301b has two Memory zones. Each can store up to 30 measurements including the date and time

The main unit automatically stores the previous measured values.

**Remove the batteries for more than 1 minute to erase the entire memory.**

- Press the **'POWER'** button to activate the display.
- Press **'PAGE'** to select memory zone  or .
- Press **'MEMO'**. The latest stored measurement result is displayed.
- Press the **'MEMO'** button successively to read previous measurement results.

## 10 Setting date and time

The Date and time is displayed on top of the LCD panel.

To adjust the date and time:

- Press the **'POWER'** button to activate the unit.
- Press **'TIME'** at least 5 seconds, the 'year' blinks on the display.
- Press **'MEMO'** to increase or **'PAGE'** to decrease the "Year".
- Press **'TIME'** to confirm, the 'month' digits start blinking.
- Follow the same instructions as used for the year to change Month, Day, Hours and Minutes.
- After confirming the Minutes, you will leave the time setting menu.
- Press the **'POWER'** button to deactivate the unit.

## 11 Technical specifications

Measurement Range	Oscillometric Pressure:20-280mmHg Pulse rate:40-195beats/ min
Accuracy	Pressure: within $\pm 3$ mmHg Pulse rate:within $\pm 5\%$ of reading
Inflation	Automatic inflation with pump
Pressure Detection	Semi –Conductor
Power Supply	2X "AAA" alkaline batteries (3V)
Auto Power Off	3 minutes after last key operation
Operating Temperature/ Humidity	+10°C to +40°C, 30% to 85% RH
Storage Temperature Humidity	-20°C to +60°C, 10% to 95% RH
Weight (without batteries)	Approximately 140 g
External Dimensions	80(W)X85(H)X60(D) mm
Wrist Circumference	135mm-220mm
Limited patient Classification	Over 18 years old Classification of type BF

## 12 Error messages

Err P	The cuff is too loosen. Fasten it.
Err 1	The unit deflates to fast. Return the unit to the shop.
Err 2	Keep still during measurement. Remove interfering sources as magnetic materials and mobile phones.
Err 3	The cuff might be placed incorrect. Re-wrap the cuff in the correct position and remeasure.

## 13 Topcom warranty

### 13.1 Warranty period

The Topcom units have a 24-month warranty period. The warranty period starts on the day the new unit is purchased. Consumables or defects causing a negligible effect on operation or value of the equipment are not covered.

The warranty has to be proven by presentation of the original purchase receipt, on which the date of purchase and the unit-model are indicated.

## **13.2 Warranty handling**

If the unit develops a fault during the warranty period, Topcom or its officially appointed service centre will repair any defects caused by material or manufacturing faults free of charge.

Topcom will at its discretion fulfil its warranty obligations by either repairing or exchanging the faulty units or parts of the faulty units. In case of replacement, colour and model can be different from the original purchased unit.

The initial purchase date shall determine the start of the warranty period. The warranty period is not extended if the unit is exchanged or repaired by Topcom or its appointed service centres.

## **13.3 Warranty exclusions**

Damage or defects caused by incorrect treatment or operation and damage resulting from use of non-original parts or accessories not recommended by Topcom are not covered by the warranty. The warranty does not cover damage caused by outside factors, such as lightning, water and fire, nor any damage caused during transportation. No warranty can be claimed if the serial number on the units has been changed, removed or rendered illegible.

## **13.4 Returning defect warranty units**

In case of defect, return the unit to the shop with a valid purchase note.

<p>The CE symbol indicates that the unit complies with the essential requirements of the directive 93/42/EEC and 89/336/EEC.</p>
--

# Topcom BPM Wrist 2301b

visit our website  
**[www.topcom.net](http://www.topcom.net)**

U8006409

## Free Manuals Download Website

<http://myh66.com>

<http://usermanuals.us>

<http://www.somanuals.com>

<http://www.4manuals.cc>

<http://www.manual-lib.com>

<http://www.404manual.com>

<http://www.luxmanual.com>

<http://aubethermostatmanual.com>

Golf course search by state

<http://golfingnear.com>

Email search by domain

<http://emailbydomain.com>

Auto manuals search

<http://auto.somanuals.com>

TV manuals search

<http://tv.somanuals.com>