
# Owner's Manual

## Mounting Clamp for Medical-Grade Power Strips

**Model: PSCLAMP**  
—Patent Pending—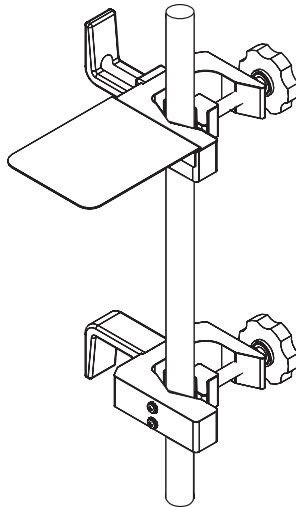

### Table of Contents

1. Important Safety Instructions	2
2. Parts and Features	2
3. Installation	3
4. Lifetime Limited Warranty	4
Français	5

**TRIPP·LITE**

1111 W. 35th Street, Chicago, IL 60609 • [www.tripplite.com/support](http://www.tripplite.com/support)

# 1. Important Safety Instructions

## SAVE THESE INSTRUCTIONS

This manual contains instructions and warnings that must be followed during the installation of the product described in this manual. Read **ALL** instructions before attempting to install this product. Failure to comply may invalidate any warranty and cause serious property damage and/or personal injury.

## 2. Parts and Features



### The PSCLAMP mounting kit includes:

- 2 Clamps
- 2 Cord Wraps
- 1 Drip Shield
- 5 Screws
  - (3) M4 6 mm Pan Head Machine Screws with Lockwashers for Power Strip and Drip Shield
  - (2) M6 12 mm Socket Head Cap Screws for Cord Wraps

The PSCLAMP is designed to be used with Tripp Lite's Medical-Grade Power Strips. To accommodate a wide variety of applications, the clamps include small-tube adapters **A** that allow them to fit tubes from  $\frac{3}{4}$ " up to  $2\frac{1}{4}$ ".

The clamps also include lock nuts **B** that can be tightened depending on mounting location and equipment.

- For units meeting **UL1363a** standards, the lock nuts **MUST** be tightened and secure when mounting on mobile cart, IV pole or other certified usages.  
Tripp Lite Models: PS-410-HGOEMCC, PS-415-HG-OEM and PS-615-HG-OEM.
- For units meeting **UL 60601-1** standards, the lock nuts **MAY** be tightened to user preference.  
Tripp Lite Models: PS-415-HGULTRA and SPS415HGULTRA.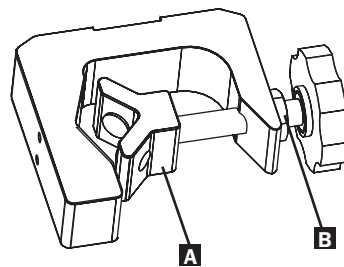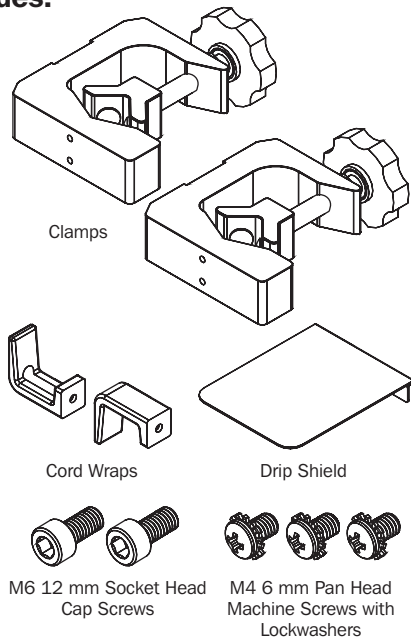


## 3. Installation

Before beginning installation, verify that all necessary package contents are present and that the power strip to be mounted conforms to the standards necessitated by its mounting location.


- 1 Secure clamps **A** loosely to each end of the power strip **B** using two of the M4 screws provided.

*Note: If attaching drip shield in next step, mount power strip with power cord down according to the diagram.*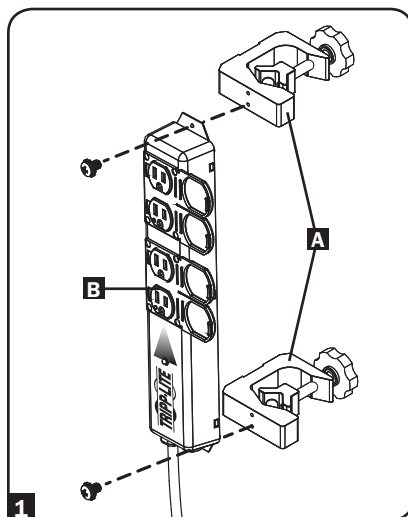


- 2 Depending on your application and mounting location, you may wish to attach the drip shield **C** to the top clamp. Secure the drip shield using one M4 screw. To ensure proper fit, the top edge of the drip shield will rest on top of the clamp.

*Note: Attaching the drip shield is optional.*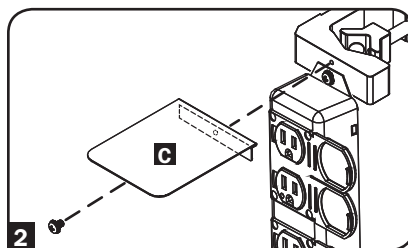


- 3 Attach the cord wraps **D** to the clamps using the provided M6 screws. Make sure that the arms will face away from the power strip.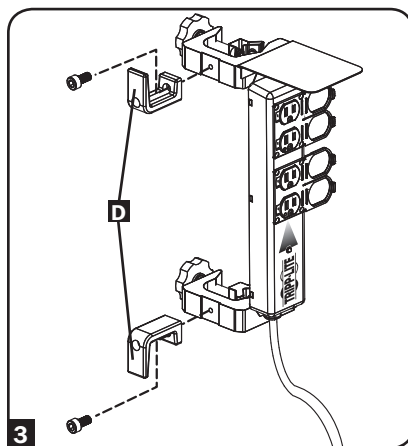

## 3. Installation

**4** Mount the power strip to the mobile cart, IV tube or other application by turning the knobs to tighten the clamps securely in place. Tighten lock nuts.

- For units meeting **UL1363a** standards, the lock nuts **MUST** be tightened and secure when mounting on mobile cart, IV pole or other certified usages.  
Tripp Lite Models:  
PS-410-HGOEMCC,  
PS-415-HG-OEM and  
PS-615-HG-OEM.
- For units meeting **UL 60601-1** standards, the lock nuts **MAY** be tightened to user preference.  
Tripp Lite Models:  
PS-415-HGULTRA and  
SPS415HGULTRA.

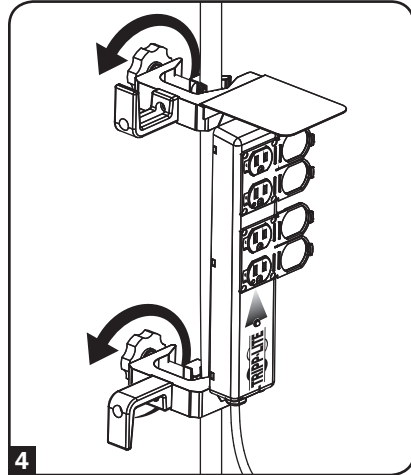**5** Tighten the mounting screws used to attach the power strip to the clamps.

## 4. Lifetime Limited Warranty

Seller warrants this product, if used in accordance with all applicable instructions, to be free from original defects in material and workmanship for its lifetime. If the product should prove defective in material or workmanship within that period, Seller will repair or replace the product, in its sole discretion. Service under this warranty can only be obtained by your delivering or shipping the product (with all shipping or delivery charges prepaid) to: Tripp Lite, 1111 W. 35th Street, Chicago, IL 60609. Seller will pay return shipping charges. Call Tripp Lite at 773.869.1234 before sending any equipment back for repair.

The warranties of all TRIPP LITE surge suppressors are null and void if they have been connected to the output of any UPS system. The warranties of all TRIPP LITE UPS Systems are null and void if a surge suppressor has been connected to its output receptacles.

THIS WARRANTY DOES NOT APPLY TO NORMAL WEAR OR TO DAMAGE RESULTING FROM ACCIDENT, MISUSE, ABUSE OR NEGLIGENCE. SELLER MAKES NO EXPRESS WARRANTIES OTHER THAN THE WARRANTY EXPRESSLY SET FORTH HEREIN. EXCEPT TO THE EXTENT PROHIBITED BY APPLICABLE LAW, ALL IMPLIED WARRANTIES, INCLUDING ALL WARRANTIES OF MERCHANTABILITY OR FITNESS, ARE LIMITED IN DURATION TO THE WARRANTY PERIOD SET FORTH ABOVE; AND THIS WARRANTY EXPRESSLY EXCLUDES ALL INCIDENTAL AND CONSEQUENTIAL DAMAGES. (Some states do not allow limitations on how long an implied warranty lasts, and some states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitations or exclusions may not apply to you. This Warranty gives you specific legal rights, and you may have other rights which vary from jurisdiction to jurisdiction.)

**WARRANTY REGISTRATION:** Visit [www.tripplite.com/warranty](http://www.tripplite.com/warranty) today to register the warranty for your new Tripp Lite product. You'll be automatically entered into a drawing for a chance to win a FREE Tripp Lite product!\*

\* No purchase necessary. Void where prohibited. Some restrictions apply. See website for details.

**WARNING:** The individual user should take care to determine prior to use whether this device is suitable, adequate or safe for the use intended. Since individual applications are subject to great variation, the manufacturer makes no representation or warranty as to the suitability or fitness of these devices for any specific application.

Tripp Lite follows a policy of continuous improvement.  
Product specifications are subject to change without notice.

1111 W. 35th Street, Chicago, IL 60609  
[www.tripplite.com/support](http://www.tripplite.com/support)

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