



TV • EARS®

Dr. Recommended TV Listening Devices

TV Ears Voice Cord Instructions

Power Microphone



Voice Cord



Extender Cord



**Insert Power
Microphone
plug into the
Voice Cord
jack**



**Insert voice cord plug into
the "Accessory" jack on
your headset**



**Velcro the Power
Microphone to the
Voice Cord by
removing the thin
protective covering
on the Power
Microphone**



Add more length to your audio cord using your Extender Cord.
Simply connect the Extender Cord between the Power Microphone and Voice Cord.

Battery Replacement

To replace your battery follow the directions below:



The voice cord is the small black box and requires only ONE 'AA' battery

To place battery into battery pack, remove the screw with a small Phillips head screwdriver



Place the 'AA' battery into the empty slot marked UM 3x2 in the battery pack

TV Ears Inc. 2810 Via Orange Way Suite 'A' Spring Valley CA 91978
Phone: (888) 883-3277 Fax: (888) 958-7899
www.tvears.com

Free Manuals Download Website

<http://myh66.com>

<http://usermanuals.us>

<http://www.somanuals.com>

<http://www.4manuals.cc>

<http://www.manual-lib.com>

<http://www.404manual.com>

<http://www.luxmanual.com>

<http://aubethermostatmanual.com>

Golf course search by state

<http://golfingnear.com>

Email search by domain

<http://emailbydomain.com>

Auto manuals search

<http://auto.somanuals.com>

TV manuals search

<http://tv.somanuals.com>