



















ADA COURTROOM ASSISTIVE LISTENING SYSTEM **Description:**

The Language Interpretation System, model WIR SYS 4, is the ideal package for courtrooms that may be required to translate up to four languages simultaneously. Participants wear the RX22-4 body-pack style receiver to listen to the language of their choice. They can sit anywhere in a 11,000 ft² area and can adjust the volume to meet their comfort level. For the hard of hearing participant, the RX22-4 receiver can be used with a neckloop (included) to amplify the participant's telecoil equipped hearing aid. SoundPlus infrared technology ensures privacy and security: the message of the proceeding will not travel outside the walls of the courtroom.

Each WIR SYS 4 package includes: two (2) TX9 emitters, two (2) MOD 232 modulators, four (4) RX22-4 receivers, four (4) HED 021 headphones, one (1) NKL 001 neckloop, and one (1) RPK 005 rack panel kit. The WIR SYS 4 meets and exceeds government ADA regulations for public hearing assistance, and is backed by a five-year warranty.*

MOD 232 Modulator:

8.5" W x 8.2" D x 1.7" H (21.5 cm x 20.8 cm x 4.4 cm), 3.1 lbs (1.5 kg) Size, Weight:

Color: Black epoxy paint with white legends

Rack Mount: 1/2 rack space wide, 1 rack space high, one or two modulators may be mounted in a single

IEC rack space with RPK 005 (single) or RPK 006 (double) Rack Mount Kits

Power Supply: Wall Transformer, 24VAC, 50-60 Hz, 15VA

North America: TFP 016, UL/CSA

Europe: TFP 027-01, 2-pin Schuko plug, CE UK: TFP 027-02, 3-pin UK plug, CE

Modulation: FM Wideband, +50kHz deviation, 50uS pre-emphasis

Carrier Frequency: Channel A: Selectable, 2.3/2.8/3.3/3.8 MHz, Channel B: Selectable, 2.3/2.8/3.3/3.8 MHz

Signal-to-Noise Ratio: More than 60dB

Frequency Response: 30 to 16,000 Hz, +1 dB, -3 dB, electrical response


Total Harmonic Distortion: Less than 2%, electrical response

Audio Processina: Compression (slope) adjustable from 1:1 to 4:1

Switchable compression gain: Moderate: 16 dB. Max: 33 dB

Auto Carrier Shut-Off: 15-minute timer shuts off carrier when no audio is present (can be disabled)

Fig. 1: MOD 232 Front Panel

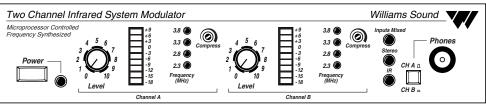

Power Switch: Two-position push button, ON/OFF

Power Indicator: Green LED

Audio Level Controls: CHA and CHB Input Level, rotary knobs Audio Indicators: CHA and CHB Audio Level, 10-segment LED's

Carrier LEDs: 4 green LED carrier "on" indicators per channel (indicates frequency, malfunctions)

Compress Control: 1:1 to 4:1

Indicates inputs A and B audio are mixed and transmitted by CHA, CHB off Input Mix LED:

Stereo LED: Indicates stereo mode

Selects CH1 or CH2 for phones when not in stereo mode Phones Switch:

Phones Output: 1/4" TRS headphone jack. Accepts stereo, mono, and any impedance phones.

Infrared Test LED: IR LED for receiver testing, monitoring, and audio signal testing.


NOTE: SPECIFICATIONS SUBJECT TO CHANGE WITHOUT NOTICE!

*90 days on accessories





Fig. 2: MOD 232 Rear Panel

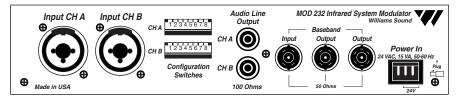

Power Input: 3-Pin Molex, 24 VAC, 50-60 Hz, 15 VA

Audio Input Jack: CHA and CHB combination XLR/TRS jack

Mic Level: Balanced, Lo-Z, 100 μ V min. to 90 mV max., 1mV nominal, 3 $k\Omega$ input impedance, sup-

plies switchable simplex power per DIN 45596 for condenser mics

Line Level: Balanced or unbalanced, 21 mV min. to 10 V max., 212 mV nominal, 100 $k\Omega$

Audio Line Output Jacks: RCA Jack, CHA and CHB, 500 mV, unbalanced, 100 Ω source impedance, load imped-

ance must be greater than 1kΩConfiguration Switches: CHA and CHB 8-position DIP switch, selects Mic/Line input, compressor gain, simplex

power, discrete or mixed inputs, carrier frequency, channel disable, auto shut-off

timer

Baseband Input Jack: BNC, allows mixing with additional MOD 232 Modulator (4CH operation), 100mV, 50\Omega

input impedance, use with MOD 232, BNC, RG-58 Cable

Baseband Output Jack: Two BNC jacks carry baseband signal, 100 mV/channel, 50 Ω source impedance, for use

with WIR TX9 or MOD 232 only

Approvals: CE, FCC, RoHS, WEEE

Operating Requirements: 0-50° C ambient temperature, non-condensing, non-corrosive atmosphere

Warranty: 5 years on modulator, 90 days on accessories

WIR TX9 Emitter:

Dimensions, Weight: 11.25" W x 6.25" H x 2.125" D (28.6 cm x 15.9 cm x 5.4 cm), 1.9 lbs (0.9 kg)

Color: Black with white legends, red acrylic lens

Power Supply: Wall Transformer, 24 VAC, 50-60 Hz, 35 VA, 3-pin MOLEX Connector

North America: TFP 010, UL/CSA

Europe: TFP 027-01, 2-pin Schuko plug, CE UK: TFP 027-02, 3-pin UK plug, CE Note: Each WIR TX9 requires its own power supply

Power Cable: NEC Class 2 wiring, two-conductor, 18 ga, 200' (61 m) max. length

Indicators: Green LED power indicator, red LED baseband indicator

Carrier Frequency: 50 kHz to 8 MHz

Emitter IR Power: 3.5 watts

Coverage Area: 28,000 ft² (2,600 m²) in single-channel mode when using the RX22-4 or RX12-4 Receiver

18,000 ft² (1,700 m²) in two-channel mode when using the RX22-4 or RX12-4 Receiver

3,500 ft² (325 m²) in single-channel mode when using the RX14-2 Receiver 3,063 ft² (285 m²) in single-channel mode when using the RX16 Receiver

(See coverage area diagrams)

Baseband Input: BNC, 100 mV per carrier, 50 Ω , for use with WIR TX9 or MOD 232 only

Baseband Output: BNC, 50 Ω , for use with TX9 only

Baseband Cable: RG 58 Coax, BNC connectors, maximum 1000' (300 m) length

Operating Requirements: 0-50° C ambient temperature, non-condensing, non-corrosive atmosphere

Mounting Kits: Wall or Ceiling Mount: BKT 024 Omnidirectional mount, Mic Stand Kit: SS-11 or SS-6
Warranty: 5 years on Emitter, 90 days on accessories

Approvals: CE, FCC, RoHS, WEEE

Compatible Receivers: WIR RX22-4 Four-Channel Receiver, RX14-2 Two-Channel Receiver,

RX16 Two-Channel Receiver





Fig. 3: WIR TX9 Rear Panel


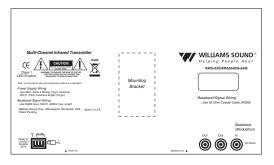


Fig. 4: Receiver Coverage Area with TX9 Emitter in Single Channel Mode

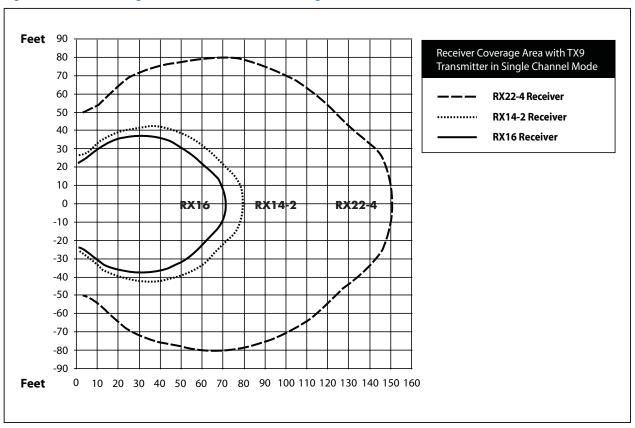


The coverage area for the TX9 will vary depending on the receiver being used. The diagram above demonstrates the receiver coverage when operating a single TX9 emitter in single channel mode. Patterns are direct radiation patterns.

Note: Reflections of the infrared light from walls, ceilings and floors may change these patterns.





Fig. 5: 3-Dimension Foot Pattern

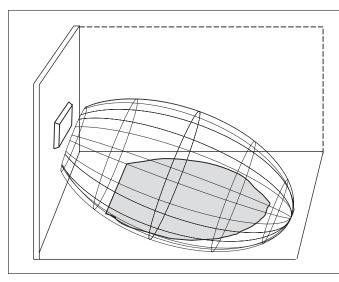


The TX9 floods the listening audience with a cone shape light pattern as shown here.

The path of the cone shape light leaves a pattern on the ground, or "foot print, " and indicates where the strongest receiver reception will occur.

The actual coverage area will vary depending on the sensitivity of the receiver being used. Refer to Figures 4 and 7 to determine how many emitters are required for 100% coverage of the listening area.

To determine the best location for the emitter, it helps to think of the IR emitter as an invisible flood-light. You'll want to aim it so the listeners are "flooded" with the infrared light. The emitter should also be positioned high enough so it won't be blocked by people and other physical obstructions. See Figure 6 below. **Mount the emitter at least 2 ft. (.61 m) above the audience.** Position the emitter to face in a slightly downward angle, 20°, that will increase the "throw" of the infrared beam.

Fig. 6: Vertical Beam Spread

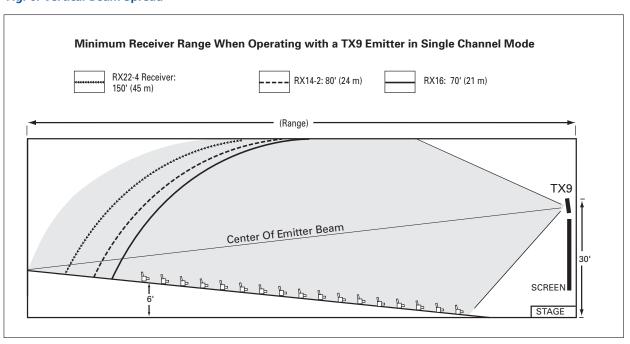






















Maximum Range When Using the RX22-4 Infrared Receiver

Fig. 7: Horizontal and Vertical Radiation Polar Plots

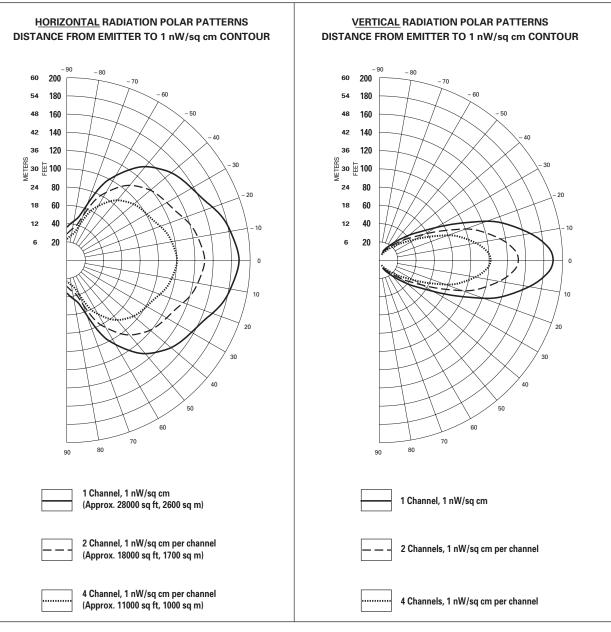

Reflections of the infrared light from walls, ceilings, and floors may change these patterns. Important: Remember to point the emitter towards the listening audience!

If you're not getting sufficient coverage with a single, properly installed TX9 Emitter, you may need to add additional WIR TX9 Emitters to achieve full coverage of your listening area. Figures 8a and 8b illustrate how multiple emitters can be used for large room installations.





















Multiple Emitters Installed to Maximize Coverage

Fig. 8a: Overlapping Illumination Patterns to Cover Larger Listening Areas


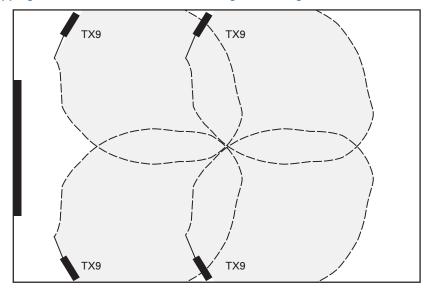


Fig. 8a above is a typical example of how multiple emitters are used to cover larger listening areas. Generally it is desirable for the illumination patterns to overlap. Note: The coverage area will vary depending on the infrared receiver being used; refer to Figures 4 and 7 to determine how many emitters are required to achieve full coverage of a listening area.

Fig. 8b: Overlapping Illumination Patterns to Cover Larger Listening Areas



When two emitters are used at the same emission point in single channel mode, the overall coverage area increases 50%. When using an RX22-4 receiver, as a result, the coverage area will increase to an estimated 42,000 ft² (3,902 m²); the RX14-2 will increase to 5,250 ft² (488 m²); the RX16 will increase to 4,590 ft² (426 m²).

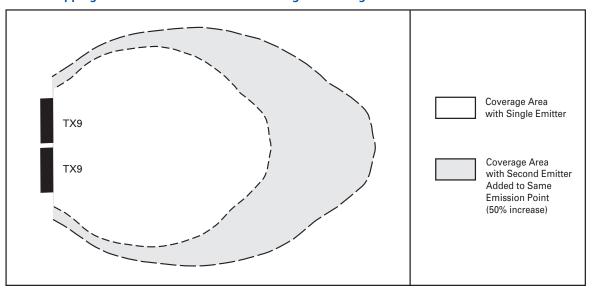



Weight:

















WIR RX22-4 Receiver:

Color and Material:

Receiver Style: Body-Pack, dual-lens detector, lanyard Size:

4.5" L x 2.85" W x 1.2" H (114.3 mm x 72.4 mm x

30.4 mm)

4.6 oz (130 g) with batteries

Black

Lanyard: 3 ft (.91 m), allows receiver to be worn around the neck

Operating Temperature: -10° C to +50° C

Battery Type: 2 x AA, alkaline (BAT 001) or NiMH (BAT 026) Alkaline: 60 hours, NiMH: 30 hours/charge

Battery Life: **Battery Drain:** 25 mA, nominal

For use only with CHG 3512 **Charging Contacts:**

Channel 1: 2.3 MHz, Channel 2: 2.8 MHz Carrier Frequency: Channel 3: 3.3 MHz, Channel 4: 3.8 MHz

De-Emphasis: 50 uS FM Deviation: ±50 kHz Signal-to-Noise Ratio: 60dB min.

Receiver squelches (mutes) at 40 dB S/N ratio

Squelch:

Frequency Response: 25 Hz to 16 KHz, +1 dB, -3 dB, electrical response

Total Harmonic Distortion: Less than 1%, electrical response

Controls: ON/OFF/VOLUME: combination thumbwheel knob Channel Selector: four-position rotary switch

Red LED "ON" indicator, flashes to indicate Low battery

Indicators: Audio Output Jacks: 3.5 mm stereo mini phone jack

Accepts 3.5 mm mono or stereo phone plug

Audio Output Power: 15 mW max at 32 Ω 110 dB SSPL90 w/ EAR 013 **Acoustic Output:**


Better than 1 nW/cm² for 40 dB signal-to-noise ratio Sensitivity:

Approvals: CE, FCC, RoHS, WEEE

Warranty: 5 years on receiver, 90 days on accessories

Compatible Headphones/Earphones: Mono or stereo, 8-32 ohms, 3.5 mm mini phone plug, HED 021, HED 026, EAR 013, EAR 014, EAR 022, NKL 001

Fig. 9 WIR RX22-4 Receiver

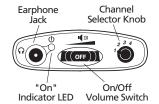

RX22-4 Top



RX22-4 Front

2.5" (6.5 cm) diameter, adjustable headband













Headset







WIR RX14-2 Receiver:

Receiver Style: Earpad Size: Weight: Color and Material: Operating Range: Battery Type: Battery Life: Battery Drain: Controls:

6.7 oz (191 g) without batteries Black, plastic Up to 3,500 ft² (325 m²) when using a single WIR TX9 Emitter AAA Alkaline batteries (BAT 010). AAA NiMH (BAT 022) optional Alkaline: 50 hours, NiMH: 8 hours/charge 25 mA, nominal

ON/OFF switch (2) Thumbwheel volume control knob, left and right (1) Frequency push-button selector, 2.3 MHz or 2.8 MHz

Frequency Response: Signal-to-Noise Ratio: Deviation: **Acoustic Output:** Approvals: Warranty:

50 Hz – 12 kHz 58 dB (IEC weighted) FM wideband modulation

118 dB MAX SSPL90, +/- 1 dB with 6 cc coupler


CE, RoHS, WEEE

1-year warranty (excludes physical damage)

Fig. 10: WIR RX14-2 Side View



Left Earphone

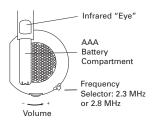

CHA

Audio In


CHB

Audio In

SOUND SYSTEM

SoundPlus® Language Interpretation System • Model WIR SYS 4



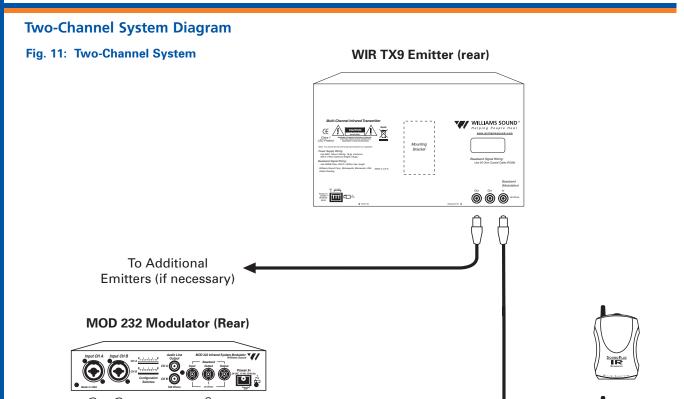

Baseband Signal

NOTE: SPECIFICATIONS SUBJECT TO CHANGE WITHOUT NOTICE!



RX22-4 Receiver (wireless)



















Bid Specs

Modulator, Model MOD 232

The infrared system shall consist of separate modulator and emitter units, with portable receivers. The modulator unit shall be a half-rack style, metal enclosure. A rack panel shall be available to mount one or two modulator units within a single EIA rack space. An adjustable floor stand and mounting bracket shall be available to mount the modulator and emitter together for portable operation.

The modulator shall provide two channels of selectable FM carrier signals; 2.3/2.8/3.3/3.8 MHz, so that a single modulator can be used to simultaneously transmit up to two channels, and two modulators can be ganged together to transmit up to four channels simultaneously. The carrier signals shall use 50 kHz deviation and 50 µS pre-emphasis. The carrier signals (baseband) shall be transmitted to one or more emitters by 50 ohm RG58 coaxial cable with BNC-type connectors. A BNC-type baseband input jack and baseband output jack shall be provided on the modulator. The modulator shall be powered by an external 24 VAC, 10 VA, 50-60 Hz power supply, connected via a three-pin Molex power connector.

It shall have a rocker-type power switch, power LED indicator, four carrier indicator LEDs and two bar graph-type LED audio indicators. The modulator shall have a modulated IR LED on the front panel for testing purposes, and a headphone jack that accommodates mono and stereo 1/4" headphones, and channel monitoring switch. The modulator shall have two rotary audio input level controls, and a screwdriver adjustable control for varying the input compression from 1:1 to 4:1. The modulator shall have two timers that automatically shut off the carriers when there is no audio signal present for 15 minutes. The modulator shall have two combination input jacks that accept 3-pin XLR plugs for balanced microphone input or 1/4" TRS plugs for balanced or unbalanced line-level inputs. The XLR inputs shall be low impedance, accept signal levels from 100 µV to 90 mV and supply 15 V simplex power per DIN45596. The TRS jacks shall accept balanced or unbalanced audio signal levels from 21 mV to 10 V. The modulator shall have CE, FCC, RoHS, and WEEE approval and carry a five-year parts and labor warranty.

The modulator shall be the Williams Sound Corp. model MOD 232.

Emitter, Model TX9

The emitter shall be contained in a metal enclosure with a shatter-resistant lens. The emitter shall include an omni-directional mounting bracket for permanent installation and a bracket shall be available for mounting on a floor stand for portable installations. Each emitter shall be powered by a 24 VAC, 50 VA, 50-60 Hz power supply. The power connector shall be a 3-pin Molex-type. The emitter shall have a BNC-type 50 ohm baseband input and a BNC-type baseband 50 ohm output jack. The emitter shall have a repeater circuit to allow multiple numbers of emitters to operate from the baseband signal. The emitter shall have a visible LED indicator for power and for baseband signal. Carrier frequency is 50KHz to 8 MHz. The emitter shall shut off when the baseband signal is not present. The emitter shall provide an effective coverage area of 28,000 sq ft (2,600 sq m) in single channel mode and 18,000 sq ft (1,700 sq m) in two channel mode when using the RX22-4 receiver. The emitter shall be convection-cooled, without fans. The emitter shall have CE, FCC, RoHS, and WEEE approval and carry a five-year warranty on parts and labor.

The emitter shall be Williams Sound Corp. model WIR TX9.

WIR RX14-2 Receiver

The receiver shall be a headset style with IR detectors positioned on top of each individual earphone. The receiver shall operate on 2.3 MHz or 2.8 MHz frequency. The receiver shall have an individual rotary-type volume control on each individual earphone and a on/off selection switch on the right earphone. The receiver shall have a push button frequency selector to choose between 2.3 MHz or 2.8 MHz operation. The receiver shall provide 118 dB SSPL90 output, +/- 1 dB, and a signal-to-noise ratio of 58 dB. The receiver shall operate up to 50 hours when using AAA alkaline non-rechargeable batteries and 8 hours per charge when using AAA NiMH rechargeable batteries. The receiver shall be encased in a black, plastic case. The receiver shall operate up to 3,500 ft² (325 m²) when using a single WIR TX9 Emitter. The receiver shall have CE, RoHS, and WEEE approval and be covered by a one-year parts and labor warranty, not including batteries or accessories.

The receiver shall be the Williams Sound Corp. model WIR RX14-2.

























Four-Channel Receiver, Model RX22-4

The receiver shall be a body-pack type with IR detector lens behind face of the unit. The unit shall have a lanyard for hands-free operation. The receiver shall have a rotary-type volume control. The receiver shall operate for 60 hours with two AA alkaline batteries and for 30 hours per charge with NiMH AA batteries. The receiver shall be charged without battery removal via charger contacts in the case. A drop-in charger accessory shall recharge the batteries in 8 hours when used with CHG 3512 charger. The receiver shall be housed in an impact resistant plastic case with a hinged battery door that does not separate from the receiver. The receiver shall receive 2.3 MHz, 2.8 MHz, 3.3 MHz or 3.8 MHz modulated IR signals with 50 µS de-emphasis. The receiver shall have a 3.5 mm stereo phone jack and accommodate low-impedance mono or stereo earphones and headphones. The receiver shall accommodate neckloop telecoil couplers. The receivers shall provide 110 dB SSPL90 output with EAR 013 earbud-type earphone.

The system electrical frequency response shall be 25 Hz to 16 kHz, +1, -3 dB and the signal-to-noise ratio shall be 60 dB. The receiver shall have CE, FCC, RoHS, and WEEE approval. The receiver shall be covered by a five-year parts and labor warranty, excluding earphones, headphones, batteries and chargers.

The receiver shall be the Williams Sound Corp. model WIR RX22-4.





















Contact:

United States

Williams Sound Corp. 10321 W. 70th Street Eden Prairie, MN 55344

Phone: 800-328-6190 or 952-943-2252

Fax: 952-943-2174

Web: www.williamssound.com Email: info@williamssound.com

Canada

Thorvin Electronics

2861 Sherwood Heights Dr. Units 36-37

Oakville, ON L6J-7K1 Canada

Phone: 800-323-6634 or 905-829-3040

Fax: 905-829-4196

Web: www.thorvinelectronics.com

United Kingdom

Sound Associates

Keeble House, 81 Island Farm Road

West Molesey, Surrey KT 2SA

United Kingdom

Phone: (44) 020 8939 5900 Fax: (44) 020 8939 5901

Web: www.soundassociates.co.uk Email: jmurdoch@soundassociates.com

Asia, Australia, Europe, Latin America, South America, South Africa

International Sales Department

Williams Sound Corp. 10321 W. 70th Street

Eden Prairie, MN 55344 USA Phone: +1 952 224 7791 Fax: +1 952 943 2174

Email: info@williamssound.com Web: www.williamssound.com



Free Manuals Download Website

http://myh66.com

http://usermanuals.us

http://www.somanuals.com

http://www.4manuals.cc

http://www.manual-lib.com

http://www.404manual.com

http://www.luxmanual.com

http://aubethermostatmanual.com

Golf course search by state

http://golfingnear.com

Email search by domain

http://emailbydomain.com

Auto manuals search

http://auto.somanuals.com

TV manuals search

http://tv.somanuals.com