

PRO·FORM[®] RESTORATION[™]


2-IN-1 MASSAGE CHAIR

SEARS

Model No. 831.270000

Serial No. _____

The serial number is found in the location shown below. Write the serial number in the space above for reference.

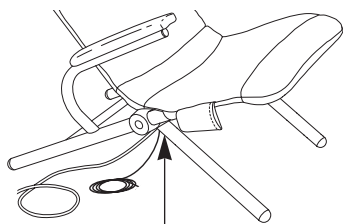

Serial Number Decal (on frame)

EXERCISE
EQUIPMENT

QUESTIONS

HELPLINE!

1-800-736-6879

⚠ CAUTION

Read all precautions and instructions in this manual before using this equipment. Save this manual for future reference.



Patent Pending

USER'S MANUAL

SEARS, ROEBUCK AND CO., HOFFMAN ESTATES, IL 60179

PRO·FORM[®] RESTORATION[™]

2-IN-1 MASSAGE CHAIR

TABLE OF CONTENTS

IMPORTANT PRECAUTIONS3
BEFORE YOU BEGIN4
ASSEMBLY5
HOW TO OPERATE THE MASSAGE CHAIR6
MAINTENANCE AND STORAGE9
PART LIST10
EXPLODED DRAWING11
HOW TO ORDER REPLACEMENT PARTSBack Cover
FULL 90-DAY WARRANTYBack Cover

IMPORTANT PRECAUTIONS

WARNING: To reduce the risk of burns, fire, electric shock, or serious injury to persons, read all important precautions before using the massage chair.

READ AND FOLLOW ALL INSTRUCTIONS

1. Read all precautions and instructions in this manual before using the massage chair.
2. It is the responsibility of the owner to ensure that all users of the massage chair are adequately informed of all precautions.

DANGER

To reduce the risk of electric shock:

3. Always turn off the hand control and unplug the power cord from the outlet immediately after using and before cleaning the massage chair.

WARNING

To reduce the risk of burns, fire, electric shock, or injury to persons:

4. Never leave the massage chair unattended while it is plugged in.
5. Do not use the massage chair under a blanket or pillow. Excessive heating can occur and cause fire, electric shock, or injury to persons.
6. Close supervision is necessary when the massage chair is used by, on, or near invalids or disabled persons.
7. Keep children away from the massage chair.
8. Use the massage chair only for its intended use as described in this manual.
9. Never operate the massage chair if it is not working properly, if it is damaged, or if the power cord or plug is damaged. If any of

WARNING: Read all instructions before using this product. SEARS assumes no responsibility for personal injury or property damage sustained by or through the use of this product.

these conditions exists, call the toll-free telephone number on the front of this manual.

10. Keep the power cord away from heated surfaces.
11. Use the massage chair indoors. Never use the massage chair outdoors.
12. The massage chair is intended for home use only. Do not use the massage chair in a commercial, rental, or institutional setting.

DANGER

13. Keep pins and other sharp objects away from the massage chair.
14. Carefully examine the massage chair before each use. If there are any signs of deterioration, call the toll-free telephone number on the front of this manual.
15. Keep the massage chair dry, away from water and moisture.

WARNING

16. Temperatures sufficiently high to cause burns may occur. Do not use on an infant or invalid or on a sleeping or unconscious person. Do not use on insensitive skin or on a person with poor blood circulation. Check the skin in contact with the heated area of the massage chair frequently to reduce the risk of blistering.
17. Do not crush the massage chair. Avoid sharp folds in the massage pad.


SAVE THESE INSTRUCTIONS

BEFORE YOU BEGIN


Congratulations for selecting the PROFORM® RESTORATION 2-IN-1 massage chair. The innovative RESTORATION 2-IN-1 massage chair is designed to melt away stress—relaxing you, stimulating circulation, and reducing muscle fatigue—any time you choose. Using the convenient hand control, you can choose upper-back, lower-back, or thigh massage, wave or pulse motion, or the built-in heater. You can even remove the massage pad from the frame and use it on the floor or a bed. And when the massage chair is not in use, it can be folded for compact storage, so you can take it wherever there is an electrical outlet!

For your safety and benefit, read this manual carefully before using the massage chair. If you have additional questions, please call our toll-free HELPLINE at **1-800-736-6879**, Monday through Saturday, 7 a.m. until 7 p.m. Central Time (excluding holidays). Before calling, please note the product model number and serial number. The model number is 831.270000. The serial number can be found on a decal attached to the massage chair (see the front cover of this manual for the location).

Before assembling the massage chair, please look at the drawing below and familiarize yourself with the parts that are labeled.

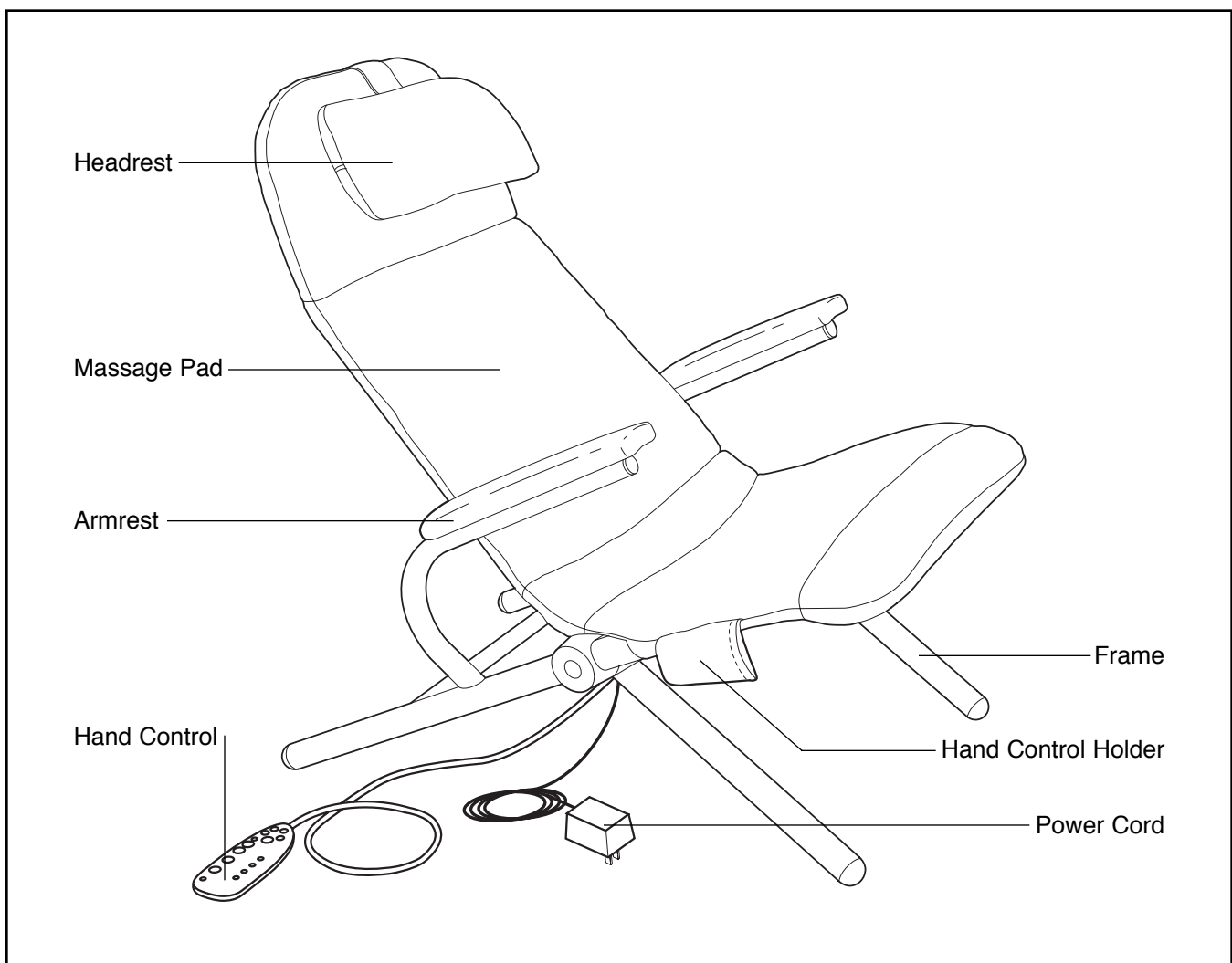


ASSEMBLY

Place all parts of the massage chair in a cleared area and remove the packing materials. Do not dispose of the packing materials until the massage chair is fully assembled. For your convenience, the massage chair can be assembled in three simple steps. **An adjustable wrench**  **(not included) is required.**


1. Unfold the massage chair to the position shown at the right. Make sure that the Massage Pad (12) is correctly positioned on the Seat Frame (10) and the Back Frame (15).

Attach one of the Arms (9) to the Seat Frame (10) with two 5/16" x 2" Carriage Bolts (1), two 5/16" Washers (4), and two 5/16" Nylon Locknuts (5) as shown.


Attach the other Arm (9) to the Seat Frame (10) in the same manner.

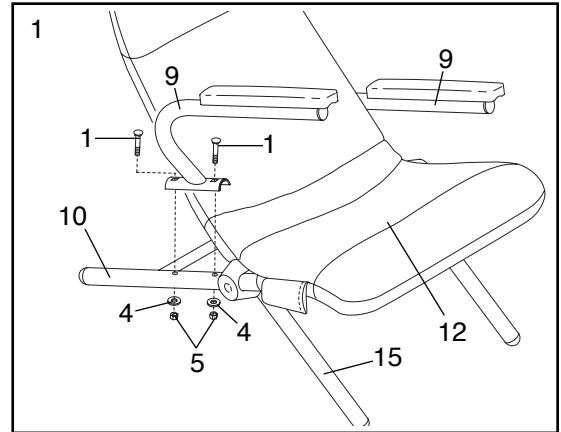

2. Connect the wire harness on the Hand Control (14) to the wire harness on the Massage Pad (12). Refer to the inset drawing. Make sure that the indicated latch on the wire harness is engaged. **If the wire harnesses do not fit together easily, turn them. Do not force the wire harnesses together.**

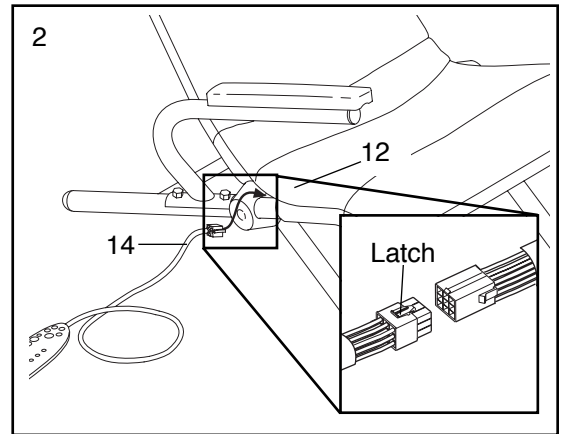


3. Plug the Power Cord (16) into the jack on the Massage Pad (12).

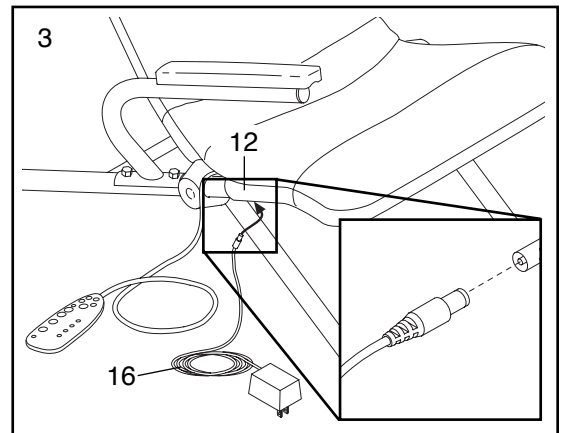

4. Make sure that all parts of the massage chair are properly tightened.

HOW TO OPERATE THE MASSAGE CHAIR

DIAGRAM OF THE HAND CONTROL

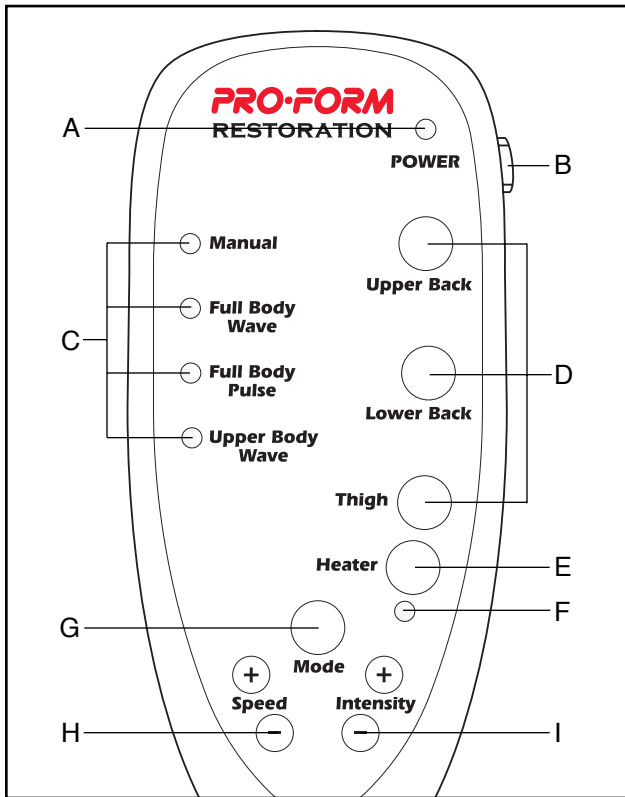


- A. Power Indicator—This indicator lights when the power is on.
- B. Power Switch—This switch turns the power on and off.
- C. Mode Indicators—These indicators show which mode is selected.
- D. Upper Back, Lower Back, and Thigh Buttons—When the Manual mode is selected, these buttons activate the three massage zones.
- E. Heater Button—This button turns the built-in heater on and off.
- F. Heater Indicator—This indicator lights when the built-in heater is on.
- G. Mode Button—This button selects the four modes.
- H. Speed Buttons—These buttons control the speed of the massage action when one of the three automatic modes is selected.
- I. Intensity Buttons—These buttons control the intensity of the massage action in all modes.

FEATURES OF THE MASSAGE CHAIR

The convenient hand control lets you customize the type of massage you get each time you use the massage chair. With the manual mode selected, you can choose any combination of upper-back, lower-back, and thigh massage action. If desired, the built-in heater can be turned on. The control also offers three automatic modes: Full Body Wave, Full Body Pulse, and Upper Body Wave. Whichever mode you choose, the speed and intensity of the massage action can be adjusted to the level that's just right for you.

HOW TO USE THE HAND CONTROL


To use the massage chair, first plug the transformer on the power cord into a 120-volt outlet. Keep the power cord away from walkways and heated surfaces.

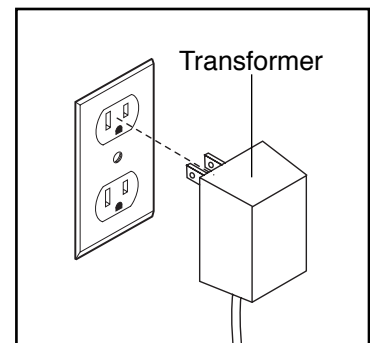

Next, sit on the massage chair and relax. Follow the steps below to use the hand control.


HOW TO TURN ON THE POWER

Slide the power switch to the "on" position; the power indicator will light. Note: If there is a thin sheet of clear plastic on the face of the hand control, remove it.

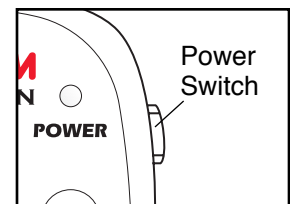


HOW TO USE THE MANUAL MODE


1. When the power is turned on, the Manual mode will be selected and the Manual mode indicator will light. Note: If a different mode is selected, press the Mode button repeatedly until the Manual mode indicator lights.

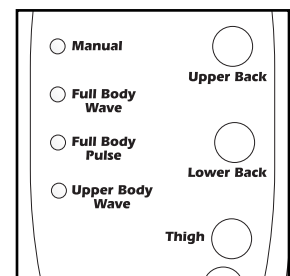

2. The massage chair features three separate massage zones: the upper back, lower back, and thigh zones. The massage zones are controlled with the Upper Back, Lower Back, and Thigh buttons. Pressing a button once will activate a massage zone. Pressing the button again will turn off the massage zone. Press the buttons to activate the desired massage zones.

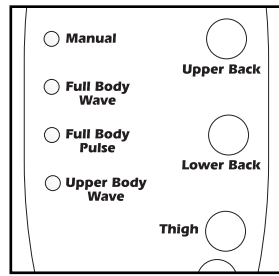


3. After you have activated the desired massage zones, press the Intensity + or - buttons to adjust the intensity of the vibrating massage action.

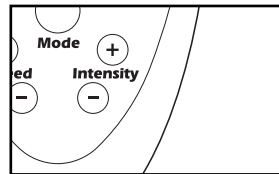

Note: If no buttons on the hand control are pressed for twenty minutes, the vibrating massage action will automatically stop. To restart the massage action, follow steps 1 through 3 above.


HOW TO USE THE AUTOMATIC MODES (FULL BODY WAVE, FULL BODY PULSE, AND UPPER BODY WAVE)

1. Press the Mode button repeatedly to select the desired mode. A mode indicator will light to show which mode is selected.

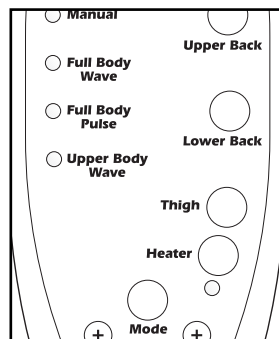

When Full Body Wave is selected, the upper body, lower body, and thigh zones will be activated in sequence, producing a wave-like massage motion. When Full Body Pulse is selected, the upper body, lower body, and thigh zones will be activated together with a pulsing motion. The Upper Body Wave is similar to the Full Body Wave, but only the upper body and lower body zones will be activated.


2. After you have selected the desired mode, press the Speed and Intensity buttons to adjust the speed and intensity of the vibrating massage action.

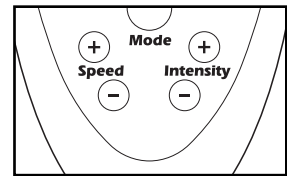

Note: If no buttons on the hand control are pressed for twenty minutes, the vibrating massage action will automatically stop. To restart the massage action, follow steps 1 and 2 above.

HOW TO ACTIVATE THE BUILT-IN HEATER

Whether the Manual mode or one of the automatic modes is selected, you can add soothing heat to your massage, if desired. To turn on the heater, press the Heater button. The Heater indicator will light. Press the button again when you want to turn off the heater. Note: When the heater is on, the maximum intensity setting of the vibrating massage action will be lower. Therefore, the intensity may decrease slightly when the heater is turned on.

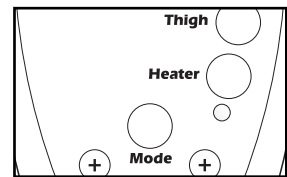

Note: If no buttons on the hand control are pressed for twenty minutes, the heater will automatically turn off. To turn on the heater, follow the instructions above.

HOW TO TURN OFF THE POWER

To turn off the power, slide the power switch to the "off" position. The power indicator will darken.


HAND CONTROL TROUBLE-SHOOTING

If the hand control does not function properly, make sure that the transformer on the power cord is plugged into a 120-volt outlet. In addition, make sure that the hand control is properly connected to the massage pad (see assembly step 2 on page 5).


HOW TO REMOVE THE MASSAGE PAD FROM THE FRAME

If desired, the massage pad can be removed from the frame and used on the floor or a bed. Follow the steps below to remove the massage pad.


1. Unplug the power cord from the outlet. It is recommended that you also disconnect the power cord from the jack on the massage pad.

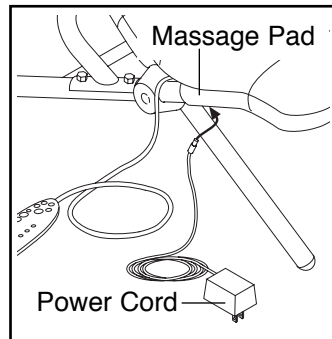

2. Next, it is recommended that you disconnect the wire harness on the hand control from the wire harness on the massage pad. To disconnect the wire harnesses, first press down the indicated small latch.

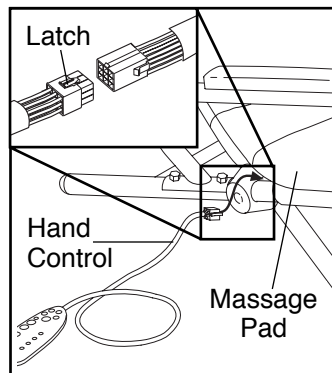


3. Fold the massage chair into the storage position. Slide the massage pad off the seat frame and the back frame. Note: There is a sleeve between the massage pad and the frame. Be careful not to remove the sleeve as you remove the massage pad.

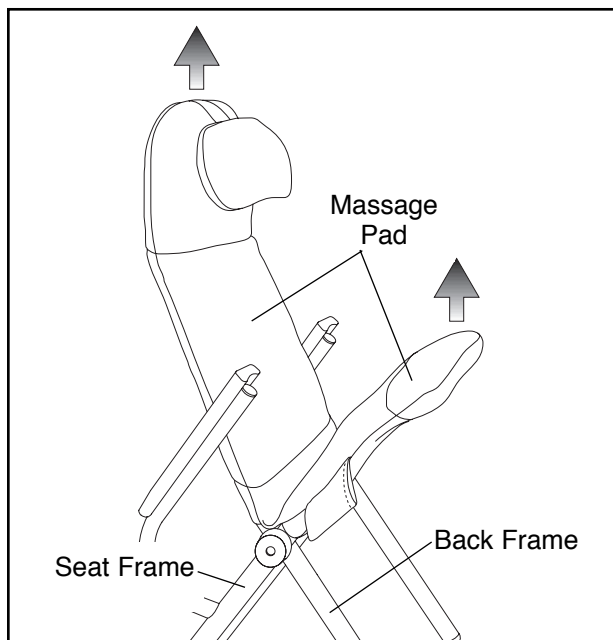

HOW TO USE THE MASSAGE PAD ON A FLOOR OR BED

After the massage pad is removed from the frame, reconnect the hand control and the power cord to the massage pad (see the drawings at the left).

Next, spread the massage pad flat on the floor or bed. If desired, pillows can be placed under the massage pad for added comfort. **WARNING: Do not place a blanket or pillow on top of the massage pad. Excessive heating can occur and cause fire, electric shock, or injury to persons.**

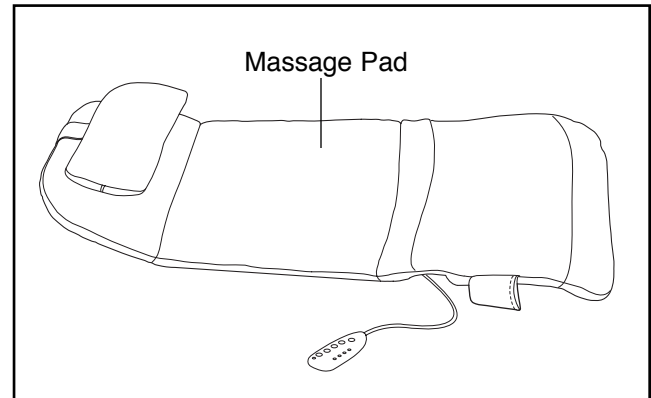

To operate the massage pad, refer to pages 6 and 7 of this manual.

HOW TO ATTACH THE MASSAGE PAD TO THE FRAME

To use the massage pad on the frame, it is recommended that you first disconnect the power cord and the hand control from the massage pad (see the drawings at upper left).

Next, fold the massage chair into the storage position. Slide the massage pad onto the frame (see the drawing at the left). Make sure that the sleeve is properly positioned between the massage pad and the frame.

Finally, reconnect the hand control and the power cord to the massage pad.

MAINTENANCE AND STORAGE

Check the massage chair periodically to make sure that all parts are properly tightened.

The massage chair can be cleaned with a low-power hand-held vacuum or a dry cloth. **IMPORTANT: Never use water or solvents to clean the massage chair.**

If the hand control does not function properly, make sure that the transformer on the power cord is plugged into a 120-volt outlet. In addition, make sure

that the hand control is properly connected to the massage pad (see assembly step 2 on page 5).

When the massage chair is not in use, it can be folded for compact storage. **IMPORTANT: Avoid sharp folds in the massage pad when storing the massage chair. Never place objects on top of the massage chair during storage.**

PART LIST—Model No. 831.270000


R0497A

Key No.	Part No.	Qty.	Description	Key No.	Part No.	Qty.	Description
1	013278	4	5/16" x 2" Carriage Bolt	11	133350	4	1/4" x 1 3/4" Screw
2	138565	2	Arm Cap	12	139279	1	Massage Pad
3	124916	2	5/16" x 3 3/4" Bolt	13	139284	1	Headrest
4	014073	4	5/16" Washer	14	137817	1	Hand Control
5	012056	4	5/16" Nylon Locknut	15	138570	1	Back Frame
6	138566	4	Frame Cap	16	138278	1	Power Cord w/Transformer
7	138567	4	Pivot Cover	17	138031	1	Sleeve
8	139285	2	Armrest	18	014063	4	1/4" Washer
9	138568	2	Arm	19	012082	2	5/16" Nylon Jam Nut
10	138569	1	Seat Frame	#	138564	1	User's Manual

Note: “#” indicates a non-illustrated part. Specifications are subject to change without notice. To order replacement parts, see the back cover of this manual.

EXPLODED DRAWING—Model No. 831.270000

R0497A

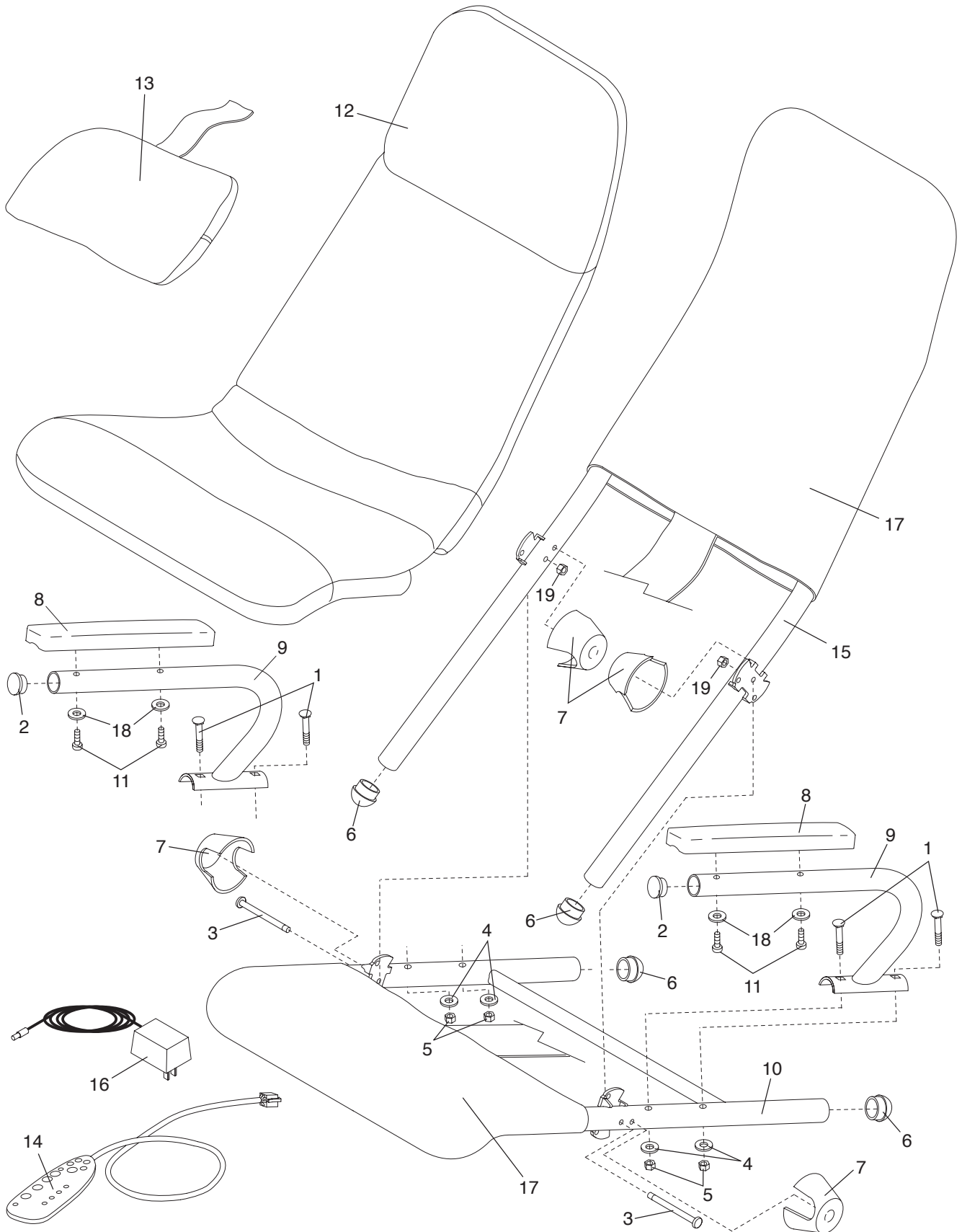



Model No. 831.270000

QUESTIONS?

If you find that:

- you need help assembling or operating the massage chair
- a part is missing
- you need to schedule repair service

call our toll-free HELPLINE

1-800-736-6879

Monday–Saturday, 7 am–7 pm
Central Time (excluding holidays)

REPLACEMENT PARTS

If parts become worn and need to be replaced, call the following toll-free number

**1-800-FON-PART
(1-800-366-7278)**

The model number and serial number of your PROFORM® RESTORATION 2-IN-1 massage chair are listed on a decal attached to the frame. See the front cover of this manual to find the location of the decal.

All replacement parts are available for immediate purchase or special order when you visit your nearest SEARS Service Center. To request service or to order parts by telephone, call the toll-free numbers listed at the left.

When requesting help or service, or ordering parts, please be prepared to provide the following information:

- The NAME OF THE PRODUCT (PROFORM® RESTORATION 2-IN-1 massage chair)
- The MODEL NUMBER OF THE PRODUCT (831.270000)
- The PART NUMBER OF THE PART (see the PART LIST and EXPLODED DRAWING on pages 10 and 11 of this manual)
- The DESCRIPTION OF THE PART (see the PART LIST and EXPLODED DRAWING on pages 10 and 11 of this manual).

FULL 90 DAY WARRANTY

For 90 days from the date of purchase, if failure occurs due to defect in material or workmanship in this SEARS MASSAGE CHAIR, contact the nearest SEARS Service Center throughout the United States and SEARS will repair or replace the MASSAGE CHAIR, free of charge.

This warranty does not apply when the MASSAGE CHAIR is used commercially or for rental purposes.

This warranty gives you specific legal rights, and you may also have other rights which vary from state to state.

SEARS, ROEBUCK AND CO., DEPT. 817WA, HOFFMAN ESTATES, IL 60179

Free Manuals Download Website

<http://myh66.com>

<http://usermanuals.us>

<http://www.somanuals.com>

<http://www.4manuals.cc>

<http://www.manual-lib.com>

<http://www.404manual.com>

<http://www.luxmanual.com>

<http://aubethermostatmanual.com>

Golf course search by state

<http://golfingnear.com>

Email search by domain

<http://emailbydomain.com>

Auto manuals search

<http://auto.somanuals.com>

TV manuals search

<http://tv.somanuals.com>