

# ENDOSCOPE DISINFECTION GUIDE


## VISERA™ PEF-V Transnasal Esophagovideoscope

— Refer to companion *ENDOSCOPE CLEANING GUIDE* for cleaning information —

**WARNING:** This guide is only a summary of the steps necessary to properly disinfect your endoscope. Be sure to follow the detailed steps outlined in the *INSTRUCTIONS* manual that was included with your endoscope when purchased.


### HIGH-LEVEL DISINFECTION

**1**

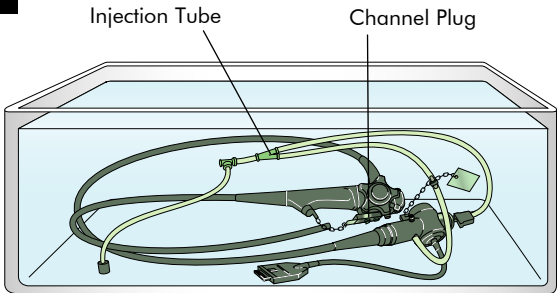


Fill basin with disinfectant. Attach channel plug and injection tube to previously cleaned endoscope. Immerse completely in disinfectant solution.

**2**

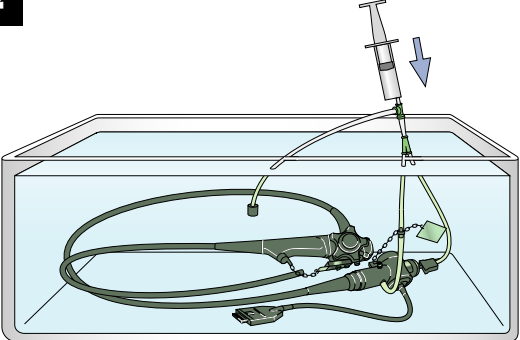


Use syringe to completely fill air, water and suction channels with disinfectant solution.

**3**

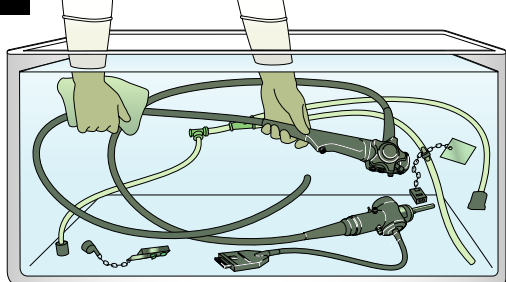


While the endoscope is immersed, disconnect cleaning accessories and use a clean lint-free cloth to wipe away any bubbles clinging to external surfaces.

**4**

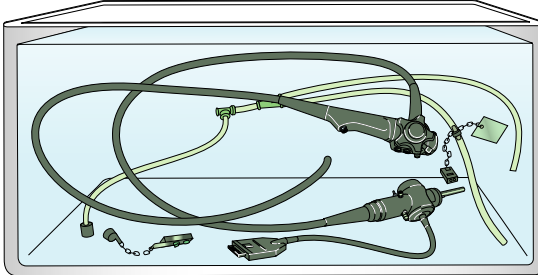

Soak all equipment according to the disinfectant manufacturer's recommended time and temperature. Use a timer to measure exact contact time.


**5**

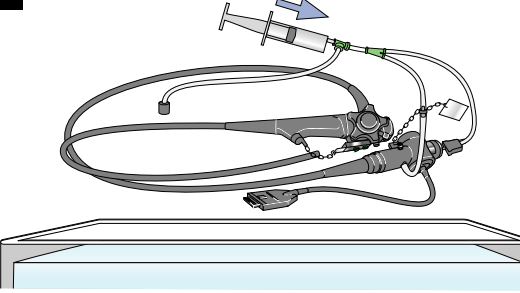

Connect channel plug and injection tube. Inject air to remove disinfectant from air, water and suction channels. Remove endoscope from disinfectant.


**6**

**CAUTION:** Follow disinfectant manufacturer's rinsing instructions.

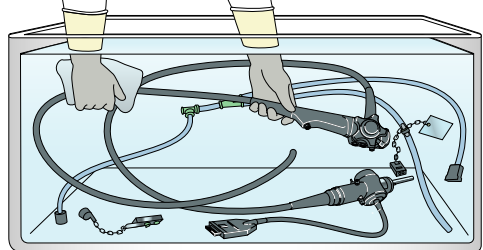


Disconnect channel plug and injection tube. Immerse endoscope and detached parts in water. Thoroughly rinse all external surfaces.

**7**

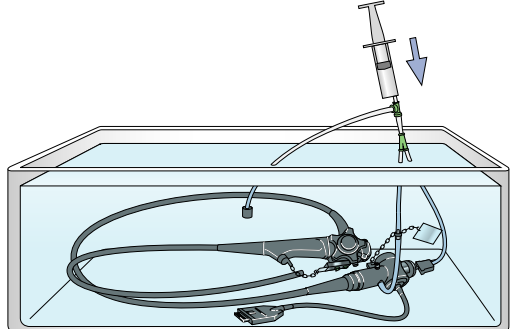


Connect channel plug and injection tube to endoscope. Use a syringe to flush water through air, water and suction channels.

**8**

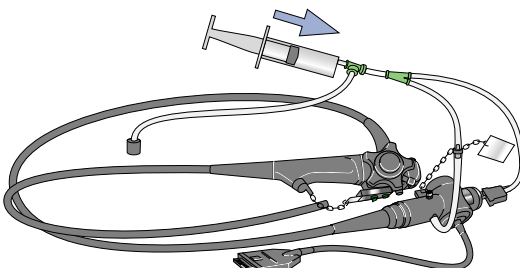


Remove endoscope from rinse water. Use syringe to inject air through air, water and suction channels, expelling the rinse water.

**9**

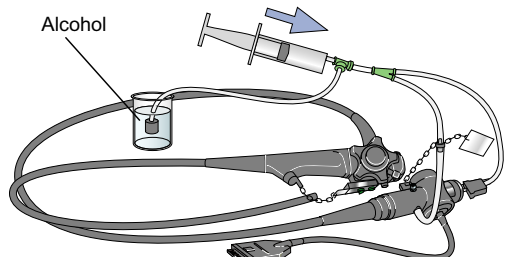


Use syringe to flush 70% alcohol through air, water & suction channels. Inject air to remove retained alcohol.

**10**

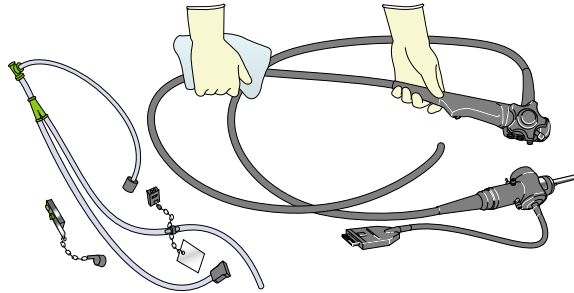

Disconnect all accessories from the endoscope. Dry endoscope and accessories with an alcohol-moistened cloth. Make sure equipment is completely dry prior to storage.

**TRANSPORTATION**

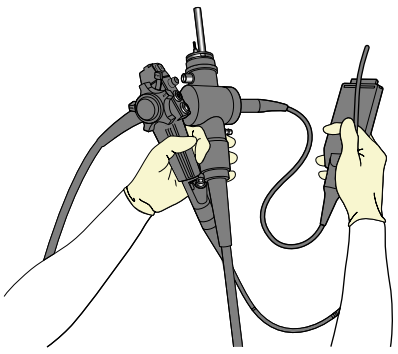

Hold all parts of the endoscope when carrying. Do not let the ends of the endoscope swing freely.

**IMPORTANT**

- Use a compatible disinfectant. Test for proper concentration per manufacturer's instructions.
- Autoclave all autoclavable accessories.
- Store endoscope hanging vertically with valves and caps removed, and with angulation lock in a "free" position.

**Clean endoscope meticulously prior to disinfection.**

## Free Manuals Download Website

<http://myh66.com>

<http://usermanuals.us>

<http://www.somanuals.com>

<http://www.4manuals.cc>

<http://www.manual-lib.com>

<http://www.404manual.com>

<http://www.luxmanual.com>

<http://aubethermostatmanual.com>

Golf course search by state

<http://golfingnear.com>

Email search by domain

<http://emailbydomain.com>

Auto manuals search

<http://auto.somanuals.com>

TV manuals search

<http://tv.somanuals.com>