BRAUN

BP 1650

BP 1600

VitalScan Plus

Type 6073

Type 6071

Downloaded from www.Somanuals.com. All Manuals Search And Download.

Braun Infoline



English	4	Thank you for purchasing a Braun product. We hope you are completely satisfied with your new Braun VitalScan Plus. If you have any questions, please call: 1-888-848-2572
Français	14	Merci d'avoir acheté un produit Braun. Nous espérons que votre nouveau Braun VitalScan Plus vous donnera entière satisfaction. Si vous avez des questions, veuillez composer le : 1 888 848-2572
Español	25	Gracias por comprar un producto Braun. Esperamos que se encuentre completamente satisfecho con su nuevo VitalScan Plus. Si Ud. tiene preguntas, por favor en EEUU llame al: 1-888-848-2572

Internet:

www.braun.com

BP 1650

BP 1600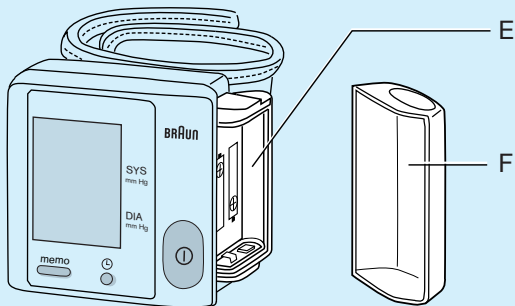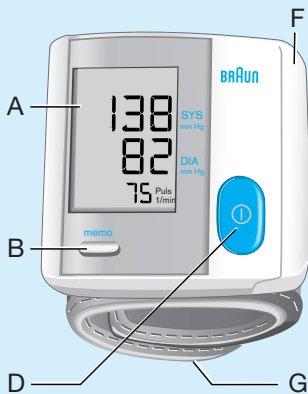


English

Braun VitalScan Plus is a blood pressure monitor that has been developed for easy and accurate blood pressure measurements from the wrist. The measuring accuracy of Braun VitalScan Plus was tested at the time of manufacture and was proven by clinical research in accordance with AAMI/ANSI-SP10.

What you should know about blood pressure

Blood pressure constantly changes throughout the day. It rises sharply in the early morning and declines during the late morning. Blood pressure rises again in the afternoon and finally drops to a low level at night. Also, it may vary in a short period of time. Therefore, readings from successive measurements can fluctuate.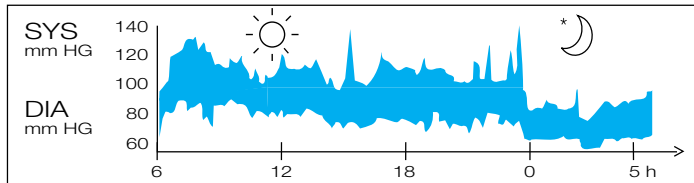

Blood pressure readings from a healthy 31-year old male, measured at 5-minute intervals

Blood pressure measured in a doctor's office only provides a momentary value. Repeated measurements at home better reflect one's actual blood pressure values under everyday conditions.

Moreover, many people have a different blood pressure when they measure at home, because they tend to be more relaxed than when in the doctor's office. Regular blood pressure measurements taken at home can provide your doctor with valuable information on your normal blood pressure values under actual «everyday» conditions.

The World Health Organisation (WHO) has set up the following standard blood pressure values when measured at resting pulse.

Blood pressure (mmHg)	Normal values	Mild hypertension	Severe hypertension
SYS = systole = upper value	up to 140	140 – 180	over 180
DIA = diastole = lower value	up to 90	90 – 110	over 110


- To ensure accurate measuring results, carefully read these use instructions.
- This product is intended for household use only. Keep product and batteries away from children.
- People suffering from cardiac arrhythmia, vascular constriction, arteriosclerosis in extremities, diabetes or users of cardiac pacemakers should consult their doctor before measuring their blood pressure themselves, since deviations in blood pressure values may occur in such cases.
- If you are under medical treatment or taking any medication, please consult your doctor first.
- The use of this blood pressure monitor is not intended as a substitute for consultation with your doctor.

Product description

- A Display
- B Memory button («memo»)
- C Time/date button (BP 1650 only)
- D Start/stop button
- E Battery compartment
- F Battery compartment cover
- G Cuff

Inserting batteries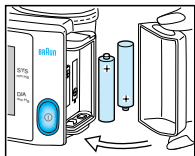


- Use alkaline batteries, type LR 03 (AAA) as supplied with this product.
- Remove the battery compartment cover (F) and insert two batteries with correct polarity (see symbol in the battery compartment (E)).
- New batteries will provide approximately 200 measurements.
- Only discard empty batteries. They should not be disposed of in the household waste, but at appropriate collection sites or at your retailer.

Key rules for accurate blood pressure measurement


- Always take readings at the same time of day, ideally in the morning, under the same conditions.
- Do not measure within 30 minutes after smoking, coffee or tea consumption.
- Take off wrist watch and jewelry before fitting the monitor on the measuring arm.
- While taking a measurement, sit down, relax, keep still, especially the hand of the measuring arm, do not move or speak.

- Hold the monitor at heart level.
- Wait approximately 3 minutes before repeating a measurement.


Taking a measurement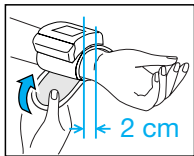


1. Fasten the monitor on the palm side of your wrist, roughly 2 cm below the palm line. Make sure that the cuff (G) fits tightly.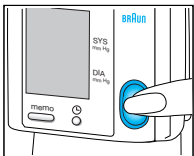

2. Press the red start/stop button (D).

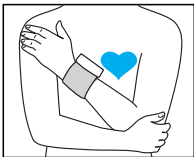
3. Hold the monitor at heart level. The measurement starts automatically. While measurement is in progress, do not move or speak.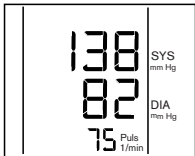

4. At the end of the measurement, a beep sounds and the reading is displayed:


Upper (systolic) value
Lower (diastolic) value
Pulse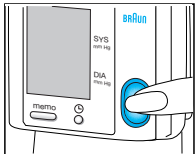

5. Press the start/stop button to turn the product off. Otherwise the product will turn off automatically after 2 minutes.
If at any time, you want to stop the measurement, press the red start/stop button.

Memory function

After each measurement, the blood pressure monitor automatically stores the reading in memory. Please note that stored data will be erased if changing batteries takes longer than 15 seconds.
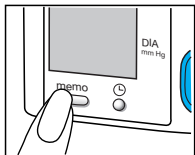


Recalling stored readings

- To recall stored readings, first turn the device off.
- Then press the memory button («memo») (B).
The latest reading will be displayed.
- Repeatedly pressing the memory button will call up one by one all stored readings.
- To exit the memory mode, turn the device off by pressing the start/stop button.



BP 1600: The readings of the last 10 measurements are stored in memory.

BP 1650: The readings of the last 20 measurements with time and date are stored in memory. The lower display line successively shows date, time and pulse of the respective measurement.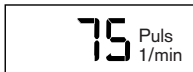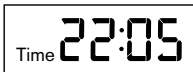

Setting time and date (BP 1650 only)

- Push the time/date button (C). The lower display line shows «Time» and flashing digits in the «hour» position. To adjust the hours, repeatedly push the «memo» button.

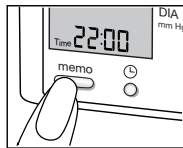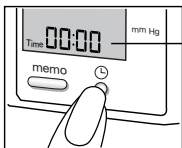- Push button (C) again: The minutes digits will flash. Again, repeatedly push the «memo» button to adjust the minutes.
- Push button (C) again to adjust the date: The lower display line shows «Date» and flashing digits in the month position. Adjust the month by repeatedly pushing the «memo» button.
- Push button (C) again: The day digits will flash. Again, repeatedly push the «memo» button to adjust the day.
- To exit the time and date setting mode, push button (C).
- Otherwise the device automatically turns off after 2 minutes and activates the settings made so far.

Storage and cleaning

Clean the product with a soft, slightly damp cloth. Stains on the cuff (G) can be carefully removed with a damp cloth. Do not wash or dry clean the cuff. Do not store the product in a place exposed to direct sunlight or high humidity.

Calibration

This product has been calibrated at the time of manufacture. If used according to the use instructions, periodic re-calibration is not required.

If at any time you question the measuring accuracy, please call at 1-888-848-2572.

What to do if ...

Problem	Reason	Solution
	Batteries are flat.	Insert new batteries.
ERR	Measurement / handling error	Repeat measurement as described under «Taking a measurement». If problem persists, contact a Braun Service Centre or call the Braun Helpline.
---	Inflation above 320 mmHg	Repeat measurement.
While measuring (during deflation of the cuff), you can hear a clicking sound with every detected heartbeat.	The device works with a special deflation procedure (stepwise deflation).	none

Problem	Reason	Solution
Readings from repeated measurements differ considerably.	Blood pressure is a fluctuating value. For healthy adults, deviations of 10 to 20 mmHg are possible.	none
	Cuff is not properly fitted.	Ensure that the monitor is fitted as follows: - on the palm side of your wrist - approx. 2 cm away from the palm line.
	Readings were not consistently taken at heart level.	For each measurement, hold the monitor at heart level.
	Talking, coughing, laughing, moving etc. when measuring will influence the reading.	While measuring, relax, keep still, do not move or speak.

Specifications

Model:	BP1650 / BP1600
Operation principle:	Oscillometric method
Display:	Liquid Crystal Display
Measuring range:	0–320 mmHg (cuff pressure) 40–199 / min (pulse rate)
Blood pressure measurement:	20 mmHg (minimum diastolic value) 300 mmHg (maximum systolic value)
Laboratory accuracy:	± 3 mmHg (cuff pressure) ± 5 % of reading (pulse rate)

Clinical accuracy:	According to AAMI-SP10 with auscultatory reference: < 5 mmHg systematic offset < 8 mmHg standard deviation
Inflation:	Automatic
Deflation:	Step valve
Power supply:	2 batteries, type LR 03 (AAA)
Operating temperature:	+ 10 °C to + 40 °C
Storage temperature:	- 20 °C to + 60 °C
Humidity:	Up to 85% RH
Cuff:	Suitable for wrist circumference of 13 to 21 cm
Clinical validation:	According to AAMI-SP10 with auscultatory reference. (Please note that a validation against intra-arterial measurements can lead to different results for devices which are calibrated against auscultatory reference.)

If device is not used within specified temperature and humidity ranges the technical accuracy can not be ensured.

Type BF equipment

Blood pressure measurements determined with this device are equivalent to those obtained by a trained observer using cuff/stethoscope auscultation method, within the limits prescribed by the American National Standard, Electronic or automated sphygmomanometers.

Subject to change without notice.

This device conforms to the following standards:

DIN EN 60601-1: 3/96 «Medical electrical equipment» –

Part 1: General requirements for safety

DIN EN 1060-1: 12/95, AMD 1 09/02 «Non-invasive sphygmomanometers» –

Part 1: General requirements

DIN EN 1060-3: 9/97 «Non-invasive sphygmomanometers» –

Part 3: Supplementary requirements for electro-mechanical blood pressure measuring systems

This product conforms to the provisions of the EC directive 93 /42 / EEC (Medical Device Directive).

Manufactured by:
Sein Electronics Co. Ltd.
Anyang-City, Kyunggi-Do,
Korea
Made in Korea

Distributed by
Gillette Commercial
Operations North America
One Gillette Park, Boston,
MA 02127-1096
Toll-free number
1-888-848-2572
(US, CDN)

Limited two-year warranty

Braun warrants this product against any defects that are due to faulty material or workmanships for a time of two years from the original date of consumer purchase. This warranty applies when used for normal household use in accordance with the use instructions and excludes the battery and damage to the product resulting from accident or misuse. This product is not warranted when used in a professional environment.

In no event shall Braun be liable for any special, incidental, indirect or consequential damages in connection with the purchase or use of this product or costs over the original cost of the product.

If the product should develop a persistent problem within the warranty period, contact Braun's Technical Support Center at 1-888-848-2572 for repair or replacement at no charge. Braun reserves the right to replace your product with the most comparable product currently available.

While this warranty gives you specific legal rights, you may also have other rights which vary from state to state.

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