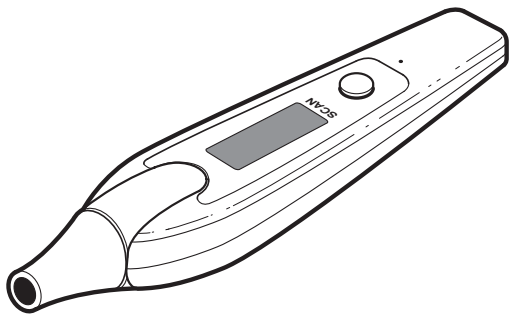
**Healthometer®**

*Your Measure for Good Health™*

COMPACT INFRARED

# Ear Thermometer

Model 7605

## IMPORTANT NOTES

Please read this entire manual carefully

before using this device.

## Compact Infrared Ear Thermometer

# IMPORTANT SAFETY INSTRUCTIONS

**DO NOT DESTROY**

### Features:

- Instant 0.1 Second Scanning for Quick, Accurate Temperature Display
- Easy-To-Clean, Waterproof Probe Tip Does Not Require Probe Covers
- Simple Press, Hold and Release Operation
- Automatic "OFF" Feature
- Convenient Storage Case

---

## Table of Contents

Introduction .....	3
Important Safeguards .....	4
About Body Temperature .....	4-5
Product Description .....	5
Battery Replacement .....	6
How to Use the Thermometer .....	6-9
Performing an Ear Tug .....	9
Other Display Readings .....	9
Display Messages .....	10
Thermometer Care and Cleaning .....	10
Troubleshooting .....	11
Technical Specifications .....	12
Warranty Information .....	13

---

## Introduction

Congratulations on the purchase of your new Health o meter® Infrared Ear Thermometer. This thermometer incorporates an innovative detection technology that offers instant, accurate temperature measurement. The compact design is very lightweight, making it ideal for both home and travel usage.

The newly-designed ear probe is ergonomically shaped – offering comfort to the user. The ear probe is waterproof and can easily be cleaned (please see Thermometer Care page 10). The one-button operation and easy-to-read LCD make using this thermometer a pleasure.

In order to obtain optimum performance from your new thermometer, please read and follow all the instructions carefully before using the product. This is a highly sensitive monitoring device; please handle it carefully and store it in the storage case (included) when not in use.

The temperature scale for your Health o meter® thermometer has been preset at the factory.

*This thermometer is intended for "in-home" use only and is not intended to be used as a substitute for consultation with your family physician.*

## Important Safeguards

1. The thermometer body is NOT waterproof. Please do not immerse the thermometer body in water or other liquids. Please use care when cleaning the ear probe. The ear probe tip of your Health o meter® Infrared thermometer IS waterproof. (Please see Thermometer Care section of this user guide, Page 10)
2. Do not expose the thermometer to prolonged, direct sunlight. The heat may damage the internal electronics or the thermometer housing.
3. When not in use, please keep the thermometer in the storage case and out of the reach of children. The optimal storage temperature for the device is -4°F to 122°F (-20°C to 50°C) and relative humidity up to 95% (non-condensing).
4. Your Health o meter® Infrared Ear Thermometer is designed for taking human body temperatures from the ear canal only. DO NOT use the thermometer to take rectal, oral or skin surface temperatures.

## About Body Temperature

Your Health o meter® thermometer measures the infrared heat of the ear drum and provides the user with an instant, accurate, digital display of the body temperature. With the “scan” button depressed, the thermometer performs 512 scanning sequences per second to “search out” and display the highest temperature detected.

It is normal that body temperature measured at different locations of the body will vary (oral, rectal or ear). As well, body temperature from one individual to another will vary. Body temperature will also fluctuate within an individual throughout the day. Please see the accompanying charts for temperature variation based on site measurement<sup>1</sup> and “normal” body temperature ranges by age.

## Detection Location

Axillary	94.5°F - 99.1°F	34.7°C - 37.3°C
Oral	95.9°F - 99.5°F	35.5°C - 37.5°C
Rectal	97.9°F - 100.4°F	36.6°C - 38.0°C
Ear	96.4°F - 100.4°F	35.8°C - 38.0°C

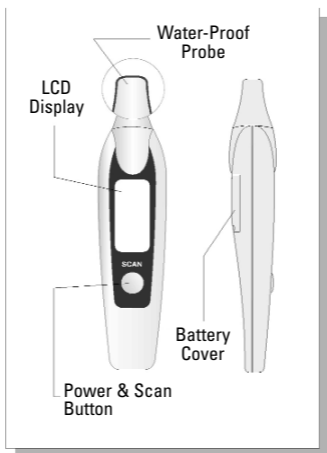
## Normal Temperature Ranges<sup>2</sup>

0 - 2 years	97.5°F - 100.4°F	36.4°C - 38.0°C
3 - 10 years	97.0°F - 100.0°F	36.1°C - 37.8°C
11 - 65 years	96.6°F - 99.7°F	35.9°C - 37.6°C
65 and over	96.4°F - 99.5°F	35.8°C - 37.5°C

1 Chamberlain, J.M., Tendrup, T.E., *New Light on Ear Thermometer Readings, Contemporary Pediatrics*, March 1994

2 Chamberlain, J.M., et al., *Determination of Normal Ear Temperature with an Infrared Emission Detection Thermometer, Annals of Emergency Medicine*, January 1995


## Product Component Description

## Battery Replacement


1. Replace the battery when the (  ) symbol appears on the display.
2. To replace the battery in your thermometer, remove the screw on the backside of the thermometer that holds the battery cover in place.
3. Remove the used battery from the compartment.
4. Reinstall a new CR-2032 lithium battery. Insert the battery into the compartment with the "+" pointing upward ("+" should be able to be seen prior to reinstalling the battery cover).
5. Replace the battery cover and fasten it to the thermometer with the screw.

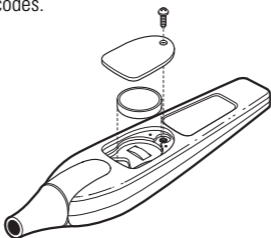Please discard the used batteries as directed by your local environmental codes.


## How To Use the Health o meter<sup>®</sup> Thermometer

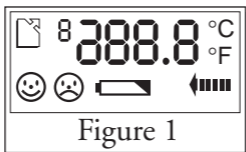1. Depress and release the "SCAN" button (about 1 second) to power ON the thermometer. The unit will perform a 1 second self test where all the LCD segments are displayed. (See Fig. 1)

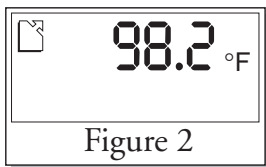
- Following the self-test, the thermometer will then display the last temperature taken. This indicates that the thermometer is ready for use. (See Fig. 2)


- Insert the probe snugly against the ear canal. Press and HOLD the "SCAN" button in order to perform the automatic scanning sequences. Please be sure to keep the "SCAN" button depressed and to position the probe snugly against the ear canal to allow the probe tip to continuously scan around the eardrum. The thermometer will continue to scan for new, higher temperatures as long as the "SCAN" button is depressed. Although the thermometer will instantly take a temperature reading, you can "SCAN" for as long as you choose. ONLY the highest temperature detected will be displayed. (See Fig. 3)


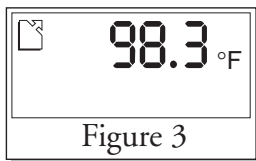
*Note: With the probe tip against the ear canal and the "SCAN" button depressed, you can slightly move the probe tip within the ear canal to insure that the temperature of the ear drum is recorded. Remember, ONLY the highest temperature will be recorded and shown on the display. To ensure that the ear drum has been scanned, you may perform an "ear tug" as described in this manual on Page 9.*

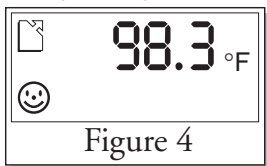4. Release the "SCAN" button and remove the thermometer from the ear. Once the button is released, you will hear an audible "beep" to indicate the end of the temperature measurement. The temperature reading will be indicated on the thermometer display. An accompanying face will also be displayed with the temperature reading. (Please remember, since the thermometer displays only the highest temperature detected, you can remove the thermometer probe from your ear with the "SCAN" button still depressed. This will NOT result in a different temperature reading.) (See Fig. 4)

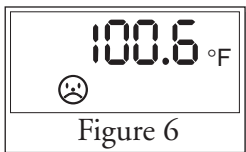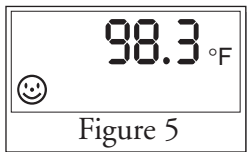5. Depending on the temperature detected, a smiling face on the display will indicate a temperature below 100.4°F (38°C). A frowning face displayed, accompanied by a long beep followed by two short beeps, will indicate a temperature above 100.4°F (38°C). (See Figures 5 & 6)

6. Upon completion of a temperature reading (performing Steps 1-5) you can start the process again and take a second temperature reading by depressing and holding the "SCAN" button again. To obtain the most accurate readings, please wait approximately 5 seconds between each separate measurement. The thermometer will turn "OFF" after approximately 1 minute without use.

## Performing an “Ear Tug”

An “Ear Tug” may be performed to help straighten the ear canal in order to provide the thermometer probe with a clear “view” of the eardrum.

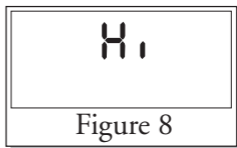
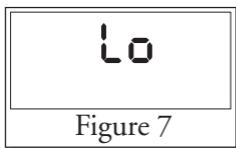
To perform the “Ear Tug”, grasp the ear and pull upward and backward simultaneously. Insert the ear probe snugly into the ear and depress the “SCAN” button to take a reading. (Please refer to the section in this manual entitled “How To Use the Health o meter<sup>®</sup> Thermometer” for complete use instruction.)

When performing an “Ear Tug” on children less than 12 months old, the ear should only be pulled straight back.


## Other Display Readings

Depending on use situations, you may encounter display readings other than temperature readings. The following descriptions pertain to these readings.

1. A “Lo” or “Hi” symbol (See Figures 7 and 8) will appear when the temperature detected is outside the measuring range of 93.2°F – 109.4°F (34°C – 43°C). (See Technical Specifications on Page 12).

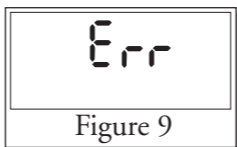2. An “Err” symbol is displayed when the thermometer is used in an environment outside of the thermometer operating range of 60.8°F – 104°F (16°C – 40°C). (See Fig. 9)

## Display Messages

The LCD of your Health o meter<sup>®</sup> thermometer has the capability to display a variety of images to help indicate status, function and information important to the use of the device.

Below is a description of these various display icons.

Temperature Digital Display: 

Fahrenheit Temperature Scale: 

Temp. Scanning In Progress: 

Low Battery Warning: 

Smiling Face Indicating Temperature Less Than 100.4°F (38°C): 

Frowning Face Indicating Temperature Above 100.4°F (38°C): 

## Thermometer Care and Cleaning

Be careful not to expose the ear thermometer to extreme temperatures, excessive humidity or direct sunlight. When not in use, store the thermometer in the storage container for protection. Also, please avoid sudden shock or accidental dropping of the device.

The ear probe is a very sensitive component of the thermometer and should be carefully cleaned after each use. (Avoid touching the probe tip with your finger.) The ear probe has a waterproof design that allows for easy cleaning. Gently clean the device with a soft cloth or cotton swab that has been moistened with alcohol. Carefully dry the probe tip with a soft cloth before storage.

The thermometer body can be cleaned with a soft dry cloth. Never use an abrasive cleanser or submerge the device in water or other liquids.

## Troubleshooting

<i>Situation</i>	<i>Reason/Solution</i>
1. Consistently Low Temperatures	<p>A. The probe is not positioned properly against the ear canal</p> <ul style="list-style-type: none"><li>– The tip of the probe must be inserted snugly against the opening of the ear canal</li></ul> <p>B. The waterproof lens of the ear probe is dirty</p> <ul style="list-style-type: none"><li>– Clean the lens with an alcohol-moistened, soft cloth</li></ul>
2. Temperature taken is not with the measuring range	<p>A. Scanning sequences are not operated in the ear canal</p> <ul style="list-style-type: none"><li>– It may occur if you operate the device before placing the ear probe snugly against your ear canal. The thermometer will read the ambient temperature which will most likely be below the measuring range of 93.2°F (34°C)</li><li>– Place ear probe against the ear canal to gain an accurate temperature reading</li></ul>
3. Low Battery Warning	<p>A. Battery is too low to take a measurement</p> <ul style="list-style-type: none"><li>– Replace the batteries (See “Installing Batteries”)</li></ul>

## Technical Specifications:

Measuring Range:

Model 7605: 93.2°F – 109.4°F (34°C – 43°C)

Measuring Accuracy: +/- 0.4°F (0.2°C): From 96.8°F to 102.2°F (36°C – 39°C)

+/- 0.5°F (0.3°C): Less than 96.8°F (36°C)

Greater than 102.2°F (39°C)

Display Resolution: 0.1 (°F or °C)

Response Time: 0.1 Seconds (for temperature reading)

Memory Function: Displays the last temperature taken

Battery Life: 5000 measurements

Operating Environment: 60.8°F – 104.0°F (16°C – 40°C) with relative humidity up to 95% (non-condensing)

Storage Environment: -4°F – 122°F (-20°C – 50°C) with relative humidity up to 95% (non-condensing)

## Applied Standards

This product conforms to the provisions of the EC directive MDD (93/42/EEC). The following standards apply to design and/or manufacturing of the product:

- ASTM E 1965-98
  - Standard Specification for infrared thermometers for intermittent determination of patient temperature
- EN 980: 1997
  - Graphic symbol for use in the labeling of medical devices
- EN 60601-1
  - Medical electrical equipment-Part 1: General requirement for safety.
- EN 60601-1-2
  - Medical electrical equipment-Part 2: Collateral Standard: Electromagnetic compatibility – Requirements and tests.

---

## Warranty Information

Your Health o meter<sup>®</sup> Thermometer is covered under the limited Warranty offered by Sunbeam Products, Inc. ("Sunbeam"). Sunbeam warrants that for a period of 2 years from the date of purchase, this product will be free from defects in material and workmanship. Sunbeam, at its option, will repair or replace this product or any component of the product found to be defective during the warranty period. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. This is your exclusive warranty. This warranty is valid for the original retail purchaser from the date of initial retail purchase and is not transferable. Keep the original sales receipt. Proof of purchase is required to obtain warranty performance. Sunbeam dealers, service centers or retail stores selling Sunbeam Products do not have the right to alter, modify or in any way change the terms and conditions of this warranty.

This warranty does not cover normal wear of parts or damage resulting from any of the following: negligent use or misuse of the product, use on improper voltage or current, use contrary to the operating instructions, disassembly, repair or alteration by anyone other than Sunbeam or an authorized service center. Further, the warranty does not cover Acts of God or Nature, such as fire, flood, hurricane and/or tornado.

Sunbeam shall not be liable for any incidental or consequential damages caused by the breach of any express or implied warranty. Except to the extent prohibited by applicable law, any implied warranty of merchantability or fitness for a particular purpose is limited in duration to the duration of the above warranty. Some states, provinces or jurisdictions do not allow the exclusion or limitation of incidental or consequential damages or limitations on how long an implied warranty last, so the above limitations or exclusion may not apply to you. This warranty gives you specific legal rights, and you may also have other rights that vary from state to state or province to province.

©2002 Sunbeam Products, Inc. All rights reserved.

Health o meter<sup>®</sup> is a registered trademark of Sunbeam Products, Inc.

Distributed by Sunbeam Products, Inc., Boca Raton, FL 33431.

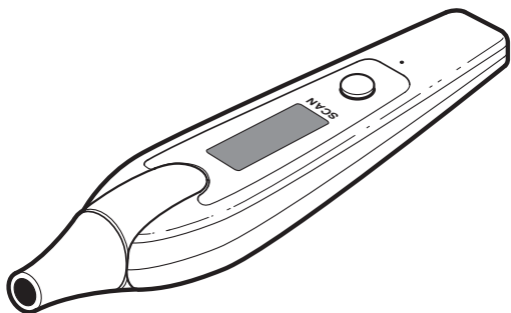
For more information, or to obtain **Warranty Service**, contact Sunbeam Products, Inc. at 1 800 672-5625 or visit the Health o meter<sup>®</sup> website at [www.healthometer.com](http://www.healthometer.com).

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©2002 Sunbeam Products, Inc. Todos los derechos reservados.
Health o meter® es una marca registrada de Sunbeam Products, Inc.
Distribuido por Sunbeam Products, Inc., Boca Raton, FL 33431.